making work easy

- FAQ - Deutsch
- FAQ - English
- FAQ - Français
- FAQ - Italiano
- FAQ - Español
- FAQ - Português
- FAQ - Türkçe
- FAQ - Русский
- FAQ - Polski
- FAQ - Dansk
- FAQ - Українська
- FAQ - 中文
- FAQ - 日本語
- FAQ - 한국어
- FAQ - Nederlands
- FAQ - Suomalainen
- FAQ - Norsk
- FAQ - Svenska
- FAQ - Eesti keel
- FAQ - Latviski
- FAQ - Lietuvių
- FAQ - čeština
- FAQ - Magyar
- FAQ - Slovenský
- FAQ - български
- FAQ - Ελληνικά
- FAQ - Hrvatski
- FAQ - Română
- FAQ - Slovenščina

SYMPRO

FAQ - frequently asked questions

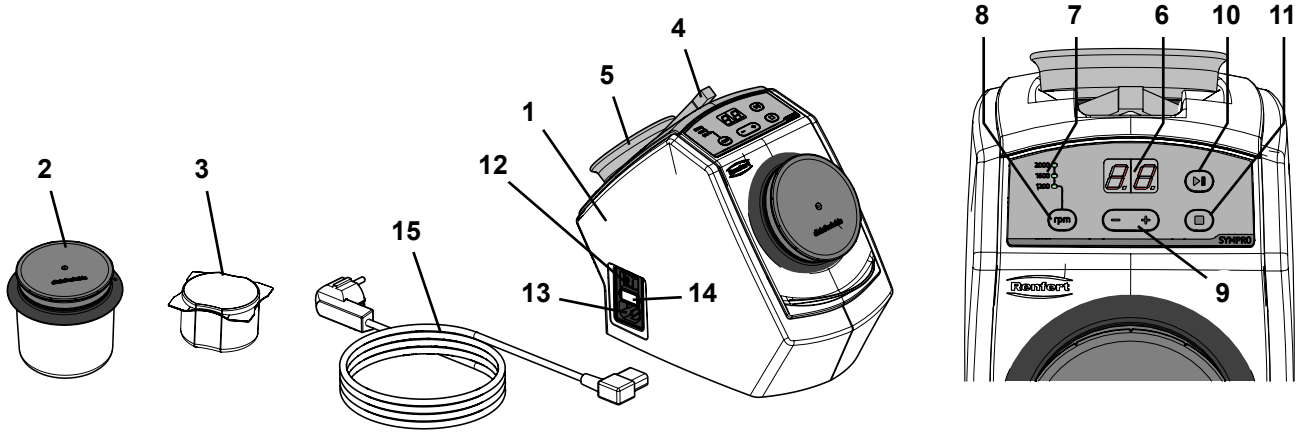
Fig. 1

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FAQ - English

For information on operating elements, see [Fig. 1](#) on page 2 of this document.

For chapter indications, refer to the complete instruction manual on the internet.

Trouble	Possible Cause	Corrective Action
No display.	<ul style="list-style-type: none"> • Appliance not connected to mains. • Appliance not turned on. • No power from the socket. • Faulty fuse. 	<ul style="list-style-type: none"> • Check correct fit of power cable in the appliance socket and wall socket. • Turn on appliance at the mains switch (12. Fig. 1). • Check power socket, for example, with another appliance. • Replace the fuse (see section 6.8). If the fuse blows repeatedly, have the appliance inspected by a service technician.
Cannot change cleaning time.	<ul style="list-style-type: none"> • Appliance still running. 	<ul style="list-style-type: none"> • Interrupt the cleaning process with the Start / Pause button, and then adjust cleaning time.
Cannot save cleaning time.	<ul style="list-style-type: none"> • Cleaning time was entered during an interrupted cleaning process. 	<ul style="list-style-type: none"> • Stop the cleaning process with the Stop button, and then enter new cleaning time.
Insufficient cleaning performance.	<ul style="list-style-type: none"> • Cleaning time too short. • Active ingredients of the cleaning fluid spent. • Insufficient number of cleaning needles. • Cleaning needles spent. 	<ul style="list-style-type: none"> • See chapt. 5.4. • Prepare fresh cleaning fluid after each application. • Fill up the cleaning needles (see section 6.5). • Replace the cleaning needles (see section 6.6).
Cleaning needles have corroded.	<ul style="list-style-type: none"> • Cleaning needles have been insufficiently rinsed. • Cleaning needles have been stored incorrectly. • Cleaning needles in use for longer period of time. 	<ul style="list-style-type: none"> • Rinsing and storage see chapter 6.1. • Removing the corrosion: Clean the needles with 100 ml SYMPROfluid Universal, without contents, for 5 minutes at 2000 rpm. Dispose of cleaning liquid. • The needles are so-called consumable goods. If it is not possible to regenerate the needles, or the needles' performance deteriorates, see chapter 6.6.

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