making work easy

- FAQ - Deutsch
- FAQ - English
- FAQ - Français
- FAQ - Italiano
- FAQ - Español
- FAQ - Português
- FAQ - Türkçe
- FAQ - Русский
- FAQ - Polski
- FAQ - Čeština
- FAQ - 中文
- FAQ - 日本語
- FAQ - 한국어
- FAQ - اللغة العربية
- FAQ - Українська
- FAQ - Română

# SILENT compactCAM

FAQ - frequently asked questions

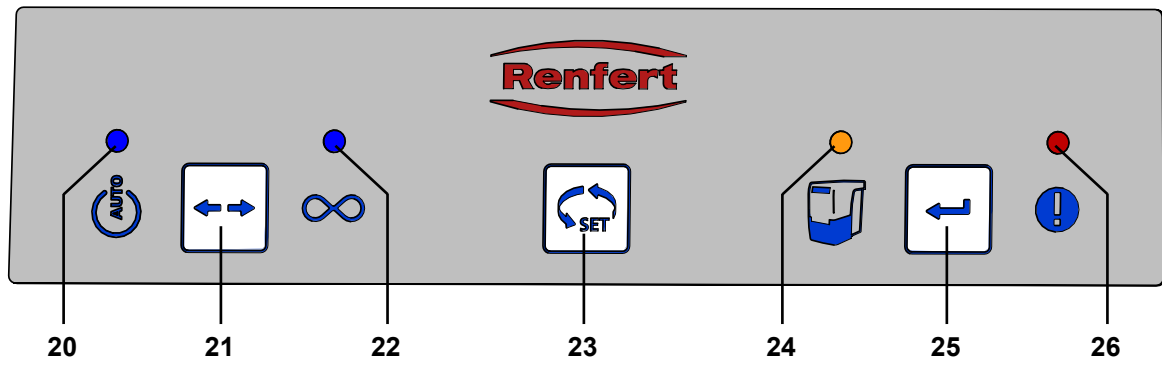
Fig. 1 / 2

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## FAQ - English

**i** For information on operating elements, see [Fig. 1](#) and [Fig. 2](#) on page 2 of this document.

**i** For chapter indications, refer to the complete instruction manual on the internet.

EN

Trouble	Possible Cause	Corrective Action
The error message (26. Fig. 2) blinks.	<ul style="list-style-type: none"> <li>The electronic device has become too hot.</li> </ul>	<ul style="list-style-type: none"> <li>Switch the device off and allow it to cool down.</li> <li>Ensure sufficient cooling, e.g. with:                             <ul style="list-style-type: none"> <li>- Chap. 4.2 Setup.</li> <li>- External ventilation (see chap. 4.6).</li> <li>- Change fine filter.</li> </ul> </li> </ul>
The suction performance is insufficient.	<ul style="list-style-type: none"> <li>There is a blockage or leak in the suction tube.</li> <li>Dust drawer is not airtight.</li> <li>The fine filter is full.</li> </ul>	<ul style="list-style-type: none"> <li>Check suction tube.</li> <li>Please observe the points in chapter 4.4.</li> <li>Check the dust drawer is in the correct position (see chap. 6.2).</li> <li>Switch the unit off and back on in order to initiate the filter cleaning function.</li> <li>Change the fine filter (see chap. 6.3).</li> </ul>
Dust drawer is over full.	<ul style="list-style-type: none"> <li>The time period for "empty dust drawer" has been set too high.</li> </ul>	<ul style="list-style-type: none"> <li>Adjust to a lower time period (see chap. 6.2.1).</li> </ul>
The signal to empty the dust drawer shows even though the dust drawer is not yet full.	<ul style="list-style-type: none"> <li>The time period for "empty dust drawer" has been set too low.</li> </ul>	<ul style="list-style-type: none"> <li>Adjust to a higher time period (see chap. 6.2.1).</li> </ul>
The display "empty dust drawer" blinks and an acoustic signal occurs 3 times.	<ul style="list-style-type: none"> <li>The flow velocity is insufficient; the filter cleaning function is carried out.</li> </ul>	<ul style="list-style-type: none"> <li>Wait until the filter cleaning has finished.</li> </ul>
The error message lights up (26. Fig. 2).	<ul style="list-style-type: none"> <li>The fine filter is so full, that the filter cleaning function alone cannot solve the problem.</li> </ul>	<ul style="list-style-type: none"> <li>Change the fine filter (see accessories and chap. 6.3).</li> </ul>
The error message lights up (26. Fig. 2) and an acoustic signal occurs 3 x again.	<ul style="list-style-type: none"> <li>The fine filter is so full, that the filter cleaning function alone cannot solve the problem.</li> </ul>	<ul style="list-style-type: none"> <li>Change the fine filter (see accessories and chap. 6.3).</li> </ul>
The suction process in the continuous operation or automatic operation stops abruptly and the relevant displays are still on (20 / 22. Fig. 2).	<ul style="list-style-type: none"> <li>The suction turbine is overheated.</li> <li>The suction turbine is defect.</li> </ul>	<ul style="list-style-type: none"> <li>Switch the unit off and allow to cool down for at least 60 min.</li> <li>Check whether the suction tube is blocked. Eliminate blockage.</li> <li>Switch the unit off and then on again, so that the filter cleaning process can be carried out. Change the fine filter (see chap. 6.3) (if the filter cleaning function does not improve the suction performance).</li> <li>Change the suction turbine.</li> </ul>
When the unit is switched on, not all of the displays light up.	<ul style="list-style-type: none"> <li>The display which does not light up is defect.</li> </ul>	<ul style="list-style-type: none"> <li>Contact Renfert / Service.</li> </ul>

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