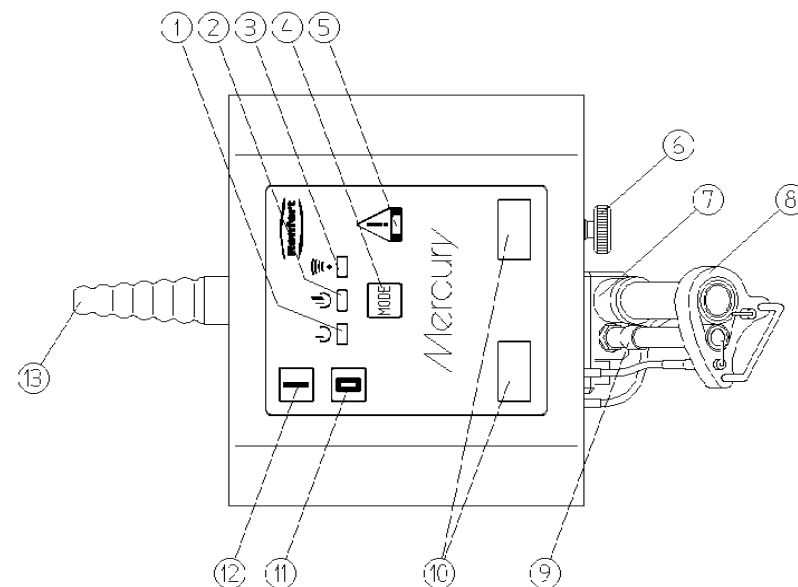
Seriennummer und Herstelldatum befinden sich auf dem Geräte-Typenschild.
 Serial number and date of manufacturing are shown on the type plate of the unit.
 Le numéro de série et la date de fabrication se trouvent sur la plaque signalétique de l'appareil.
 Il numero di serie e la data di costruzione si trovano sulla targhetta dell'apparecchio.
 El número de serie y la fecha de fabricación están indicados en la placa identificadora del aparato.
 Seriennummer en produktiedatum staan op het apparaat-type plaatje.

Mercury Laborbrenner

Nr. 900-0000 / -1000 / -2000

DEUTSCH

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Fault	Cause	Remedy
Loud burning sound of the pilot of the gas burner.	<ul style="list-style-type: none"> Pilot flame is adjusted too hard. 	<ul style="list-style-type: none"> The air feeds of the small tube are to be closed by using the sliding ring.
Smell of gas when the burner is lit.	<ul style="list-style-type: none"> Connections are leaking. The burner is not sealed correctly. Flame is burning in the pipe. 	<ul style="list-style-type: none"> Check the connections. In this case do not carry out repairs yourself! Send the burner for inspection. Select a sufficiently large flame via the gas volume valve (6).
Burner switches off during use.	<ul style="list-style-type: none"> The burner is overheating. The supply to the burner pipe has been changed / is not connected properly. Sliding ring of the small tube closes the air feeds. 	<ul style="list-style-type: none"> Check conditions of use. Re-start after a cooling down period (approx. 15 min.). Check connections to ignition electrodes and flame monitor electrodes (see Fig. 1-3). Air feeds can be opened by sliding the ring.

6. Cleaning / Maintenance

The *Mercury* burner requires no maintenance. It should, however, be cleaned from time to time and the following instructions should be observed:

- The pipe unit can be boiled out after removing it from burner. To do this remove both connections to the ignition and flame monitor electrodes and loosen both the screws on the pipe unit. Take care not to remove any seals from the burner. Do not remove the electrode board (8). The pipe unit can now be boiled out (Boiling out time: max. 5 minutes). To re-build the burner follow this process in reverse order. Make sure that any connections are replaced correctly. Moreover please see the page before last of this instructions.

Caution. The burner consists of sensitive electronic components which should not be brought into contact with water!

- To clean the outer casing and the touch elements use a soft cloth. Never clean using a temperature of more than 50°C!
- Do not use cleaning agents as chemical reactions may occur.
- In particular keep the sensor window clean (10) otherwise the Sensor Function mode cannot be guaranteed.
- The lab burner must not be dismantled!
- Never drill out the burner's nozzles.
- Do not blow compressed air down the inlet nozzle (13)!
- The burner's nozzles must not be unscrewed during cleaning. The seals in the burner may be damaged and safe usage could no longer be guaranteed.

7. Spare Parts

You will find the order numbers for spare parts in the attached list.

8. Switching to a different type of gas

The *Mercury* burner conforms to DIN 30665 Part 1 regulations and enables a switch to another gas type by changing the nozzle inserts (see following page).

- The natural gas H versions is marked with the sticker "Natural Gas H (E) 20 mbar".
- The natural gas L version is marked with the sticker "Natural gas L (LL) 20 mbar".
- The liquid gas version is marked with the sticker "Liquid gas 50 mbar".


The pre-set lab burner can be switched over to natural gas H (E), natural gas L (LL) and liquid gas by changing the nozzle inserts and their respective stickers. The relevant conversion kits are given in the following table. They consist of a small nozzle, a large nozzle and a sticker naming the respective gas type. The conversion kits are listed in the spare parts list.

The changeover of nozzles must only be carried out by qualified specialists in accordance with DVGW regulations (German legislation - check local regulations).

Mercury Burner

No. 900-0000 / -1000 / -2000

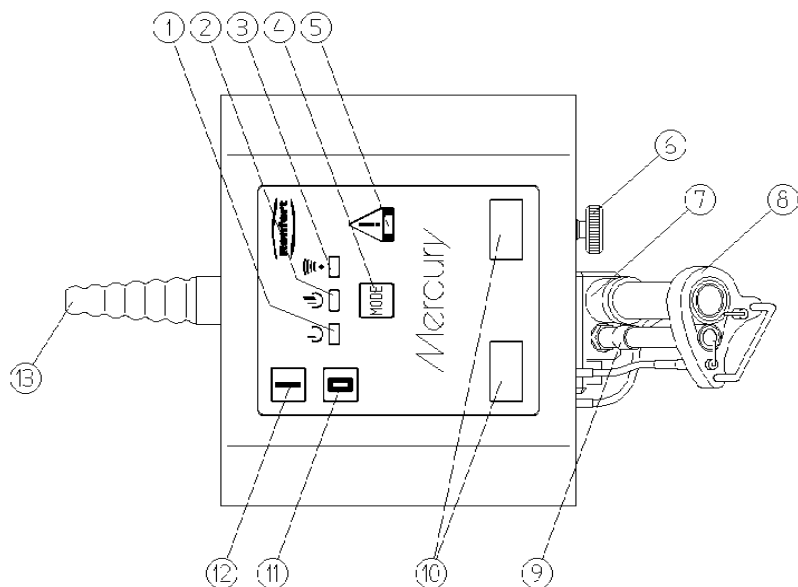ENGLISH

The *Mercury* burner is a unit which guarantees comfort at work due to its diverse functions. To take advantage of all the unit's advantages and in order to achieve long-term, trouble-free usage please take note of the following instructions.

1. Description / Applications

The *Mercury* burner is fitted with a flame monitor which quickly blocks the gas supply to a dying flame. The ignition function allows you to start up the burner very simply. The integrated sensors enable hands-free activation of the main flame which means optimum usage of gas and quick, efficient work using the burner flame.

The *Mercury* burner is suitable for left and right-handed users.

The burner is available in the following versions:

No. 900-0000 > Burner for liquid gas
(propane / butane)

No. 900-1000 > Burner for natural gas E (H)

No. 900-2000 > Burner for natural gas LL (L)

The *Mercury* burner is type-tested by the DVGW

(German Association of Gas and Water Installation).

**Type Mercury 900-9999,
Reg. No. NG-2211AU0076**

2. Installation / Operation

The burner may only be installed by qualified staff in accordance with DVGW regulations.

1. Check whether the type of gas and operating pressure required for the burner are compatible with your gas supply.
2. Check whether the electrical unit supplied is suitable for your electricity supply (details can be found on the identification plate on the mains connection unit). First plug the voltage reduction plug into the socket on the *Mercury* burner.
3. Next plug the mains connection unit into the intended mains socket.
4. Inspect the gas connection pipe to ensure it is correctly positioned and is not damaged.

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- Close the gas tap on the lab bench.
- Push the gas hose (DIN 30 664) onto the inlet nozzle of the *Mercury* burner right up to the last groove. If necessary secure the tube with a circular clip to prevent its slipping off (see DVGW worksheet G 621).

Your *Mercury* burner is now ready to use.

3. Areas of Caution

- **Lab burners should only be used under constant supervision. After finishing work with the burner the gas supply to the burner should be immediately cut off either via the shut-off valve or by disconnecting the gas from the mains. When used in classrooms, laboratory burners may only be used after first switching on the connected series of safety shut-off devices (e.g., central shut-off device for the whole room, group or general shut-off device). For installation and operation of gas systems in labs and science lecture rooms please heed the regulations in DVGW worksheet G 621.**
- If there is a strong smell of gas close the gas shut-off valve immediately and initiate the relevant safety precautions.
- Also observe the accident prevention regulations of the relevant trade association!
- More information relating to connection and operating regulations can be obtained from your local gas supply firm as well as from your installer.
- **The unit must only be serviced by authorised and qualified personnel.**
- **The user must avoid all work modes which endanger personal safety and hinder the perfect functioning of the unit.**
- **The user must ensure that no unauthorised personnel use the unit.**

4. Handling

The burner can be used in three ways depending on which mode you select on the button (4).

I) **Sensor function (3):** The pilot flame burns but the main flame will be ignited via the unit's in-built sensors as soon as an instrument or hand comes in range of the sensors.

The main flame burns for as long as the object is in range of the sensors. This means quick, effective work with minimum gas usage.

II) **Constant function (2):** The pilot and main flames burn. The sensors are not active.

Intended for work requiring great heat for a long time.

III) **Stand-by function (1):** The pilot flame only burns. The sensors are not active. Intended for work requiring little heat.

• ON/OFF function

Before commencing, open the shut-off device to the lab bench. The air valves of the large burner pipe should not be fully closed. The gas volume valve (6) on the burner should be open.

Make sure that there are no objects in range of where the flame comes out. Never touch the electrodes whilst igniting!

Attention: During the ignition process sparks may occur on the ignition electrodes which could lead to injury if touched.

By pressing the ON button (12) you start the ignition process which opens the valves for the pilot and main flame. The flame will be lit by the ignition electrodes. The unit recognises the flame and switches to sensor function. You can now work in sensor mode or select other modes via the mode selector button (4).

The burner and the casing become heated due to thermal conduction of the flame. This will not inhibit the functioning of the burner.

Switch off the unit via button (11).

After you have finished using the burner close the shut-off device to the lab bench.

Tip:

The ignition process is repeated up to a maximum limit of 30 seconds. When first connecting up the burner, or after long periods during which the burner has not been used, it may take a while before the gas hose has filled with gas.

If the ignition time of 30 seconds is not sufficient the unit automatically switches off (see "Fault Detection").

To re-start the ignition process switch off the unit via button (11) and switch it on again via button (12).

• Mode Selection

After switching on the unit it is always in sensor mode. You can select the desired function via the function button (4).

• Fault Detection

The lab burner is constructed so that if the flame is extinguished or overheating the gas supply is shut off. This will be indicated by the warning signal (5). You must switch off the burner using the OFF switch (11) and then switch it on again. If the unit has switched off due to overheating you must wait at least 15 minutes before the burner can be switched on again.

• Flame Settings

You control the size of the flame via the gas volume valve (6) at the front of the unit.

Caution: The flame must always be large enough so as not to burn in the pipe. Otherwise the burner will overheat very quickly which may lead to severe damage to the unit!

The flame can be set via the knurled ring on the main burner at the front of the unit.

A gentle flame is produced by turning the ring upwards thus closing the air feed. This reduces the amount of air to the flame.

Caution: The air feeds must not be closed during the electrical ignition process.

A harsh flame is produced by opening the air feed over the ring.

5. Troubleshooting Guide

Fault	Cause	Remedy
The burner will not ignite despite pressing the ON switch (12). There is no sound of gas escaping.	<ul style="list-style-type: none"> Central gas supply has not been turned on / gas cylinder is empty. The nozzles are blocked. 	<ul style="list-style-type: none"> Operating failure. Disconnect the supplies to the ignition and flame monitors, unscrew the burner pipe section (see Fig. 1-3), boil out, blow out and re-assemble. Take care to reconnect the supplies correctly. Open the gas volume valve (6).
	<ul style="list-style-type: none"> The gas volume valve (6) on the burner is closed. 	<ul style="list-style-type: none"> Open the gas volume valve (6).
The burner will not light despite the sound of gas escaping.	<ul style="list-style-type: none"> The ignition electrodes or flame monitor electrodes are not connected or have not been connected properly. Danger: Faulty or broken pressure reduction unit. Gas volume valve (6) on the burner is closed. The knurled ring (7) is blocking the air feed vent of the large pipe. Air in the gas supply pipe. 	<ul style="list-style-type: none"> Re-connect the ignition electrodes and flame monitor electrodes (see Fig. 1-3). Replace the pressure reduction unit. Open the gas volume valve (6). Air feed vent must be at least half open. Switch off the unit and re-start the ignition process.
	<ul style="list-style-type: none"> The sensor window (10) is dirty. Surroundings are too bright (sun, UV, lighting). 	<ul style="list-style-type: none"> Clean the sensor window (10). Darken the surroundings.
The Sensor Function mode does not work.	<ul style="list-style-type: none"> The sensor window (10) is dirty. Surroundings are too bright (sun, UV, lighting). 	<ul style="list-style-type: none"> Clean the sensor window (10). Darken the surroundings.

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9. Garantie

All *Mercury* burners are examined carefully by us before dispatch. Should any malfunctions occur follow the "Troubleshooting Guide" in Point 5. If the problem cannot be remedied send the burner in for repair together with a description of the problem. Do not try and repair the unit your-self. The burner's nozzles are harmonised and should not be changed as they will no longer comply with regulations. When used properly Renfert guarantees all *Mercury* parts for 3 years. The guarantee does not include parts prone to natural wear and tear e.g. ignition and flame monitor electrodes. The guarantee is void for damage caused by misuse; by disregard of the instructions for use, cleaning, maintenance and connection; by self-repaired units or repair carried out by unqualified personnel; by use of spare parts from other manufacturers; and by influences outside the normal practice or not authorised in the instruction manual. Repairs made under guarantee to not extend the guarantee.

10. Technical Data

	Type		
	900-0000 Liquid Gas	900-1000 Natural Gas E/H	900-2000 Natural Gas L/LL
Height	approx. 90 mm		
Weight	1500 g (without power unit)		
Connection Hoses	in line with DIN 30664		
Electrical Connection	<u>Primary:</u> depends on version (see identification plate) <u>Secondary:</u> 12 V DC		
Pressure Supply	50 mbar	20 mbar	20 mbar
Consumption	47 g / h	61 l / h	71 l / h
Output (gas)	600 W	600 W	600 W
Output (electrical)	20 W	20 W	20 W
Mode of Operation	Continuous operation		
Ambient Temperature	max. 40°C		

11. Parts supplied

- 1 *Mercury* burner
- 1 Power unit
- 1 Spare parts list
- 1 Instruction manual with attachments

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