Achtung | Warning | Attention
Attenzione | Atención
Atenção | Осторожно

UFI: NQ00-7098-E00C-4SDQ

2L | 10L

Renfert GmbH
Untere Giesswiesen 2 | 78247 Hilzingen
Germany | Tel. +497731 8208-0
www.renfert.com | Made in Germany

making work easy

Elektrolyt

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INSTRUCTIONS FOR USE ENGLISH

Description:

Polishing solution for CoCr alloys.
Ready for use, suitable for all polishing units.

Use:

The object to be polished must be completely finished before it is placed into the solution. Oxide deposits could leave dark areas on the framework. In order to avoid this, sandblast and steam clean the entire framework beforehand.

Recommended settings for the polishing unit:

- ▶ Current: 5 Amps
- ▶ Time: 3–5 min
- ▶ Ideal working temperature: 52–60 °C (125–140 °F)

Caution:

During the electrolytic polishing a layer of material is removed from the framework. Thin areas on the framework i.e. clasps, should therefore be covered with a layer of wax beforehand.

Constituents:

Contains ethanediol and sulphuric acid.

Storage:

Store the container in an upright position with the lid tightly sealed. Only store in the original container. Do not store together with alkaline substances.

Disposal:

Dispose of the electrolyte according to the statutory regulations for waste disposal within the country of use. Waste key: 06 01 01 sulphuric acid and sulphurous acid.

Further information can be found on the safety data sheets.

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Hazard statements/Precautionary statements:

Warning. Contains: ethanediol.

Harmful if swallowed. Causes skin irritation. Causes serious eye irritation. May cause damage to kidneys through prolonged or repeated exposure if swallowed.

Wear protective gloves/protective clothing/eye protection/face protection. If eye irritation persists: Get medical advice/attention. Get medical advice/attention if you feel unwell.

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