making work easy

- de
- en
- fr
- it
- es
- pt
- tr
- ru
  
- zh
- ja
- ko
- ar

# SILENT flow sensor

Quick Start Guide

Made in Germany

21-6035 26012024



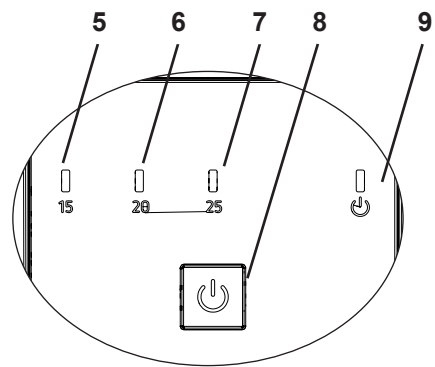
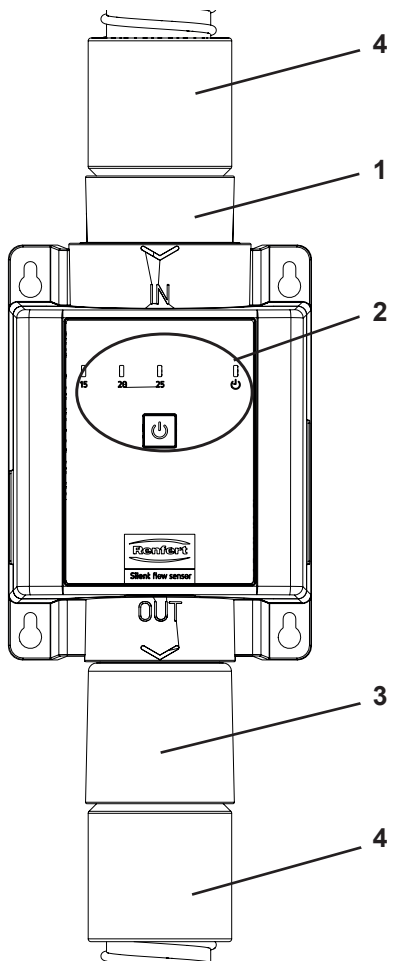
Fig. 1

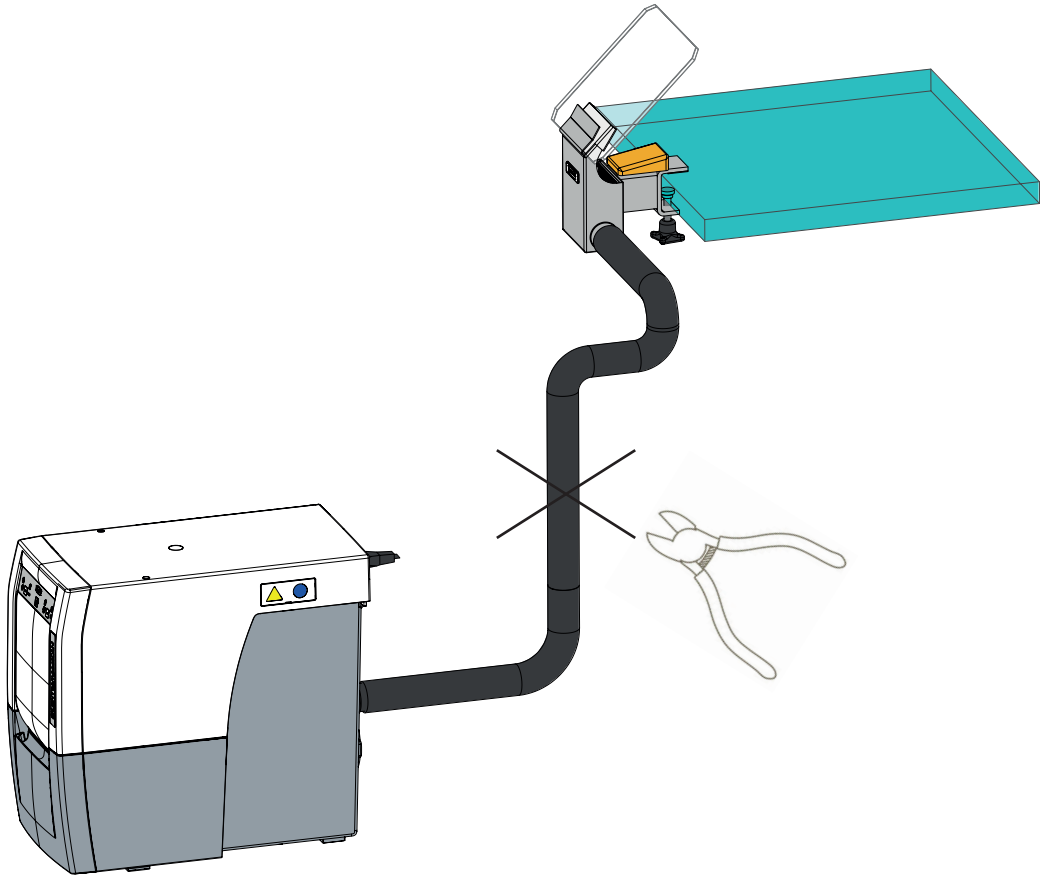
1.

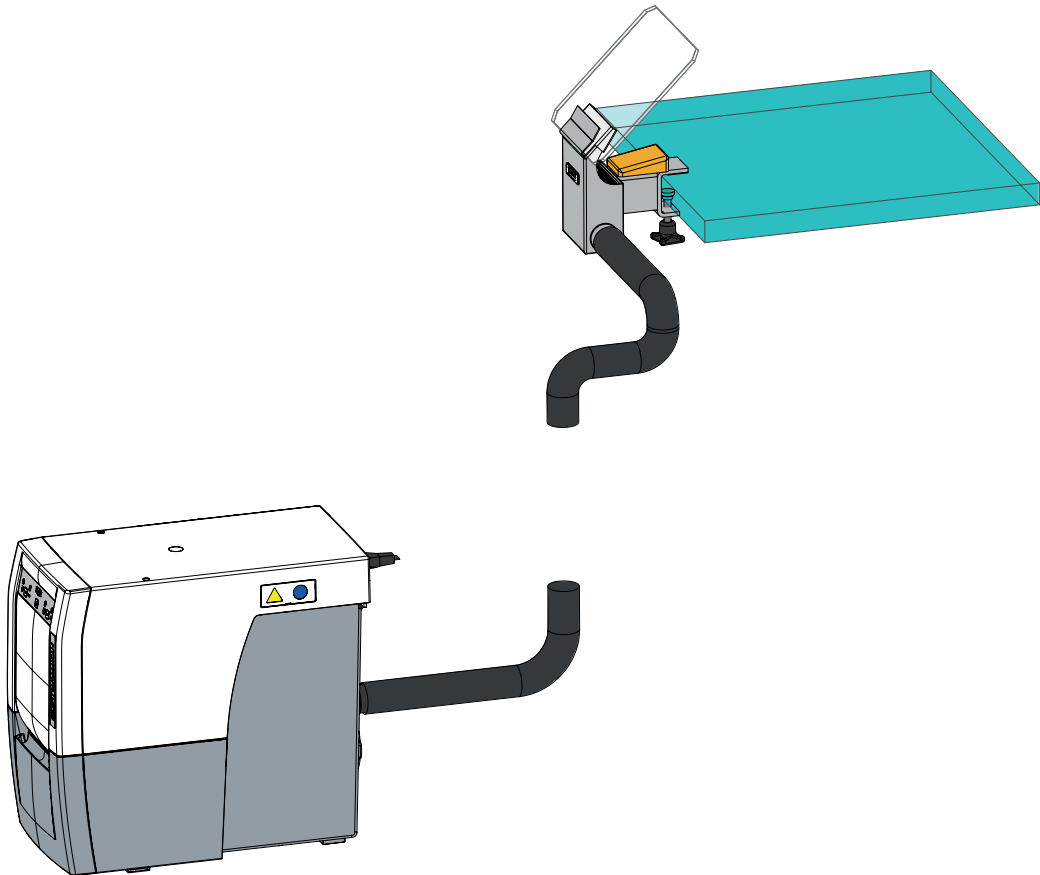
2.




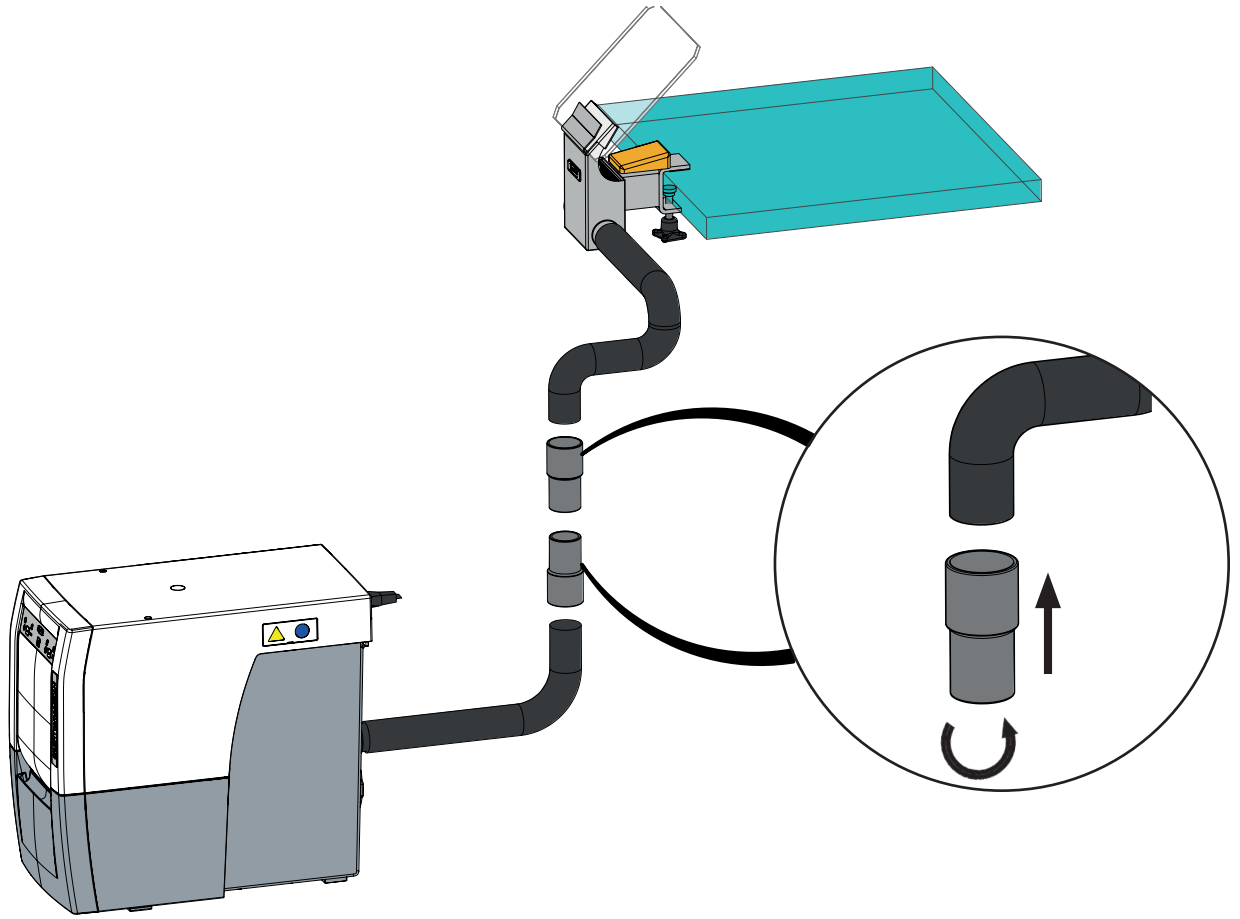
3.


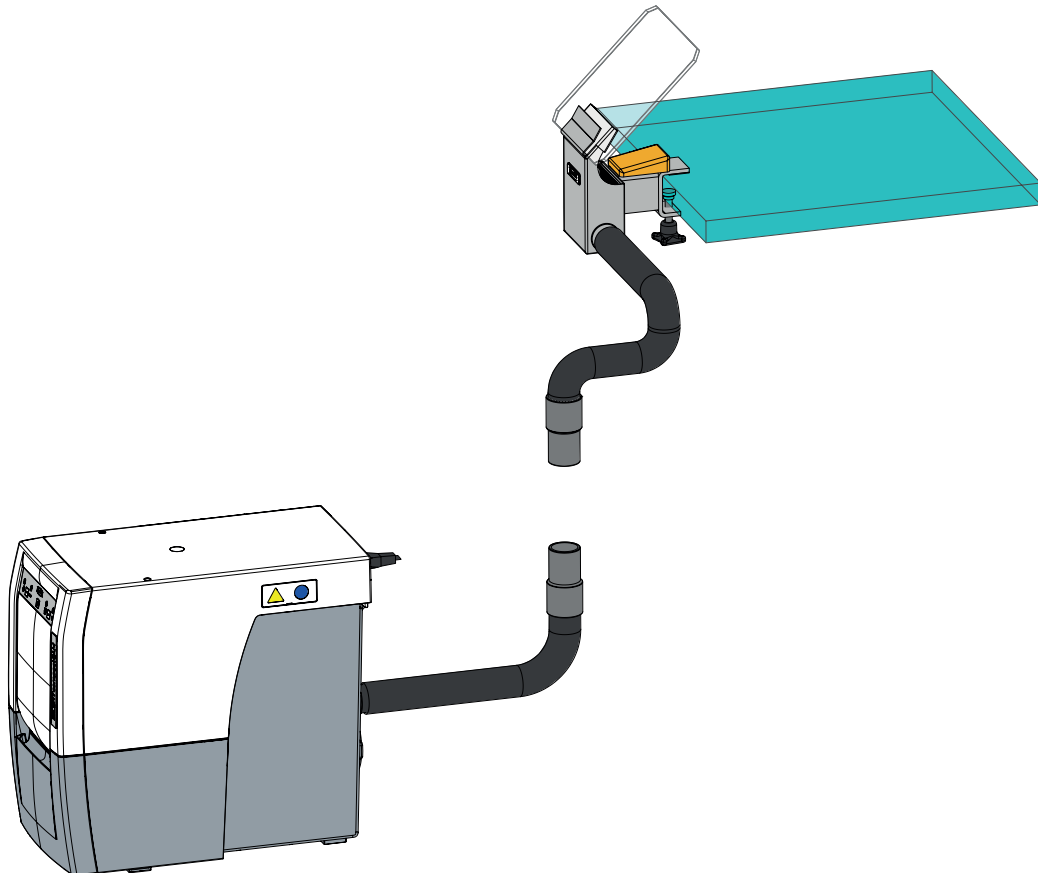
4.

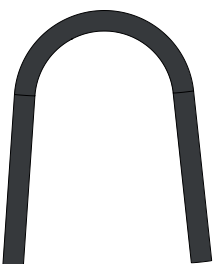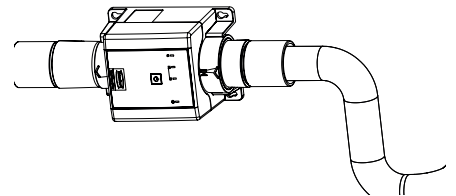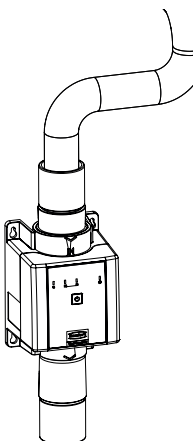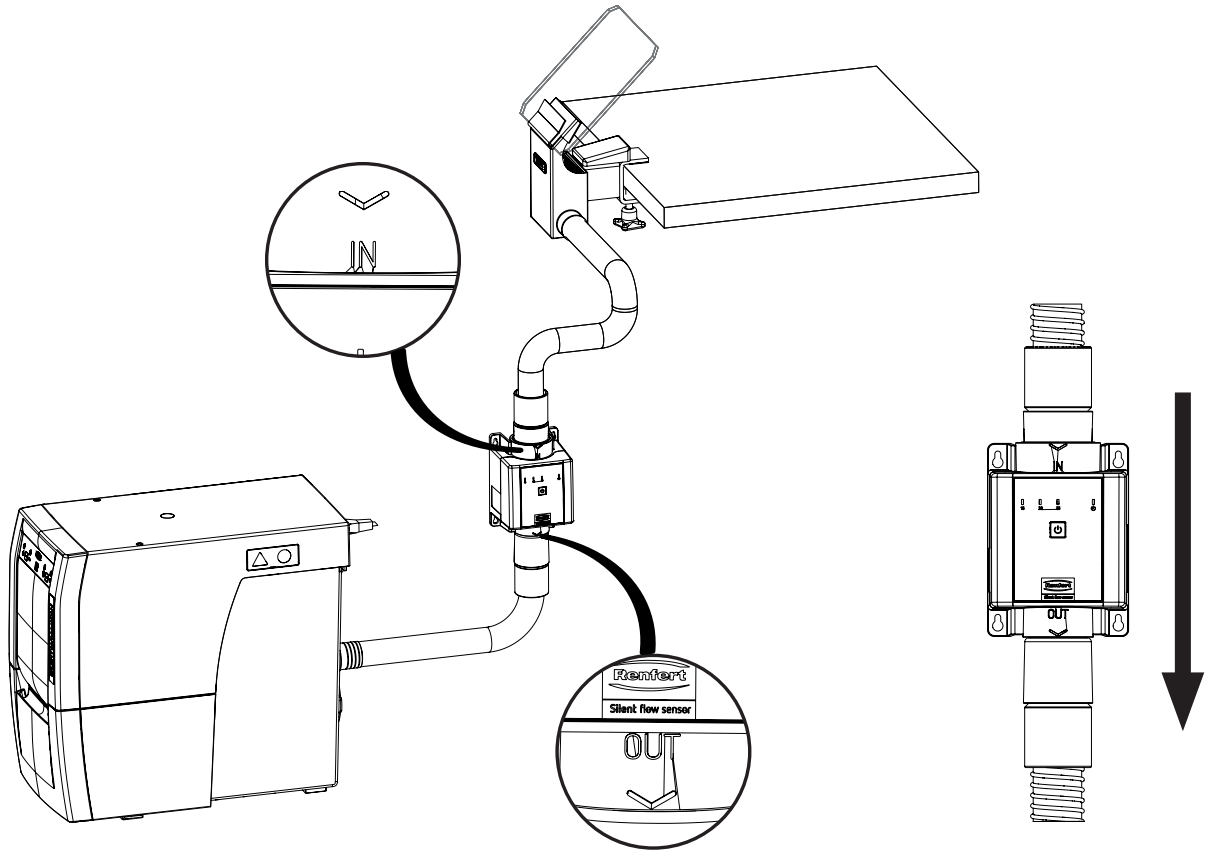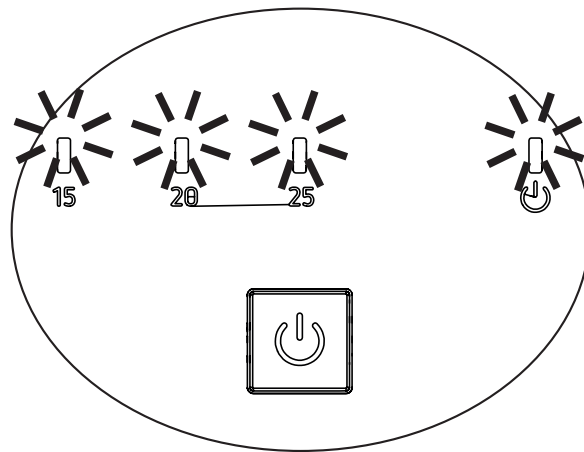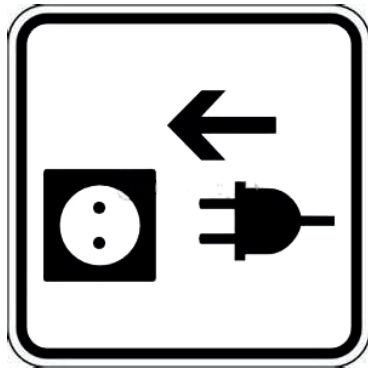5.




~ 10 sec.

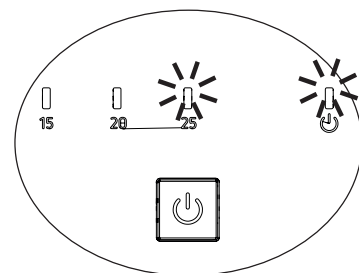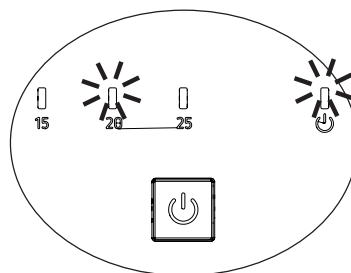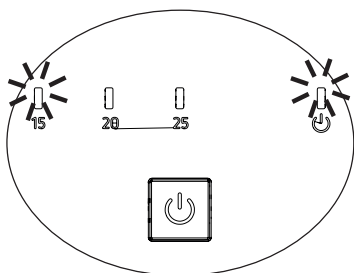Beep

Ready

# Einleitung

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# Introduction

This Quick Start Guide contains instructions on the operation of the unit and relevant safety information.

Detailed information about the unit is provided in the instructions for use available online at [www.renfert.com/p918](http://www.renfert.com/p918).

Enter the following item number: 29140000.

## Symbols used

Symbols with the following meanings are used in these instructions or on the equipment:

### Danger

**Immediate risk of injury. Consult accompanying documents!**

### Electrical voltage

**Danger due to electrical voltage.**

### Caution

**Risk of damage to the equipment if the instruction is not followed.**

### Notice

**Provides the operator with useful information that eases use.**

### Protection class II

**The equipment complies with applicable EU directives.**

**This product complies with the relevant UK legislation.**

**See the UKCA declaration of conformity available online at [www.renfert.com](http://www.renfert.com).**

**The equipment is subject to the provisions of the EU directive 2002/96/EC (WEEE directive).**

Other symbols are explained as they occur.

## Safety

### Intended use

The area of application of the SILENT flow sensor is with extraction systems that are installed at dental technician workbenches where restorations are processed using dental handpieces.

The SILENT flow sensor can be used together with one of the following extraction units in the SILENT range:

- SILENT compact
- SILENT TS / TS2
- SILENT TC / TC2 / EC2

within the scope of measures to protect against exposure to dust \*) as defined by the German ordinance on hazardous substances ("GefStoffV") / respective employers' liability insurance association.

**In doing so, both the instructions for use of the SILENT flow sensor as well as those of the extraction unit must be observed, particularly with regard to low-dust disposal of the extracted material and filter replacement.**

\*) Note for customers in Germany: A validated and recognized dust protection measure in accordance with testing principle GS-IFA-M20 (issued by the German Institute for Work and Health IFA) shall only be deemed to have been provided when all components, i.e. the flow rate monitoring unit such as the SILENT flow sensor, the extraction unit, and the detection unit, are validated and recognized.

In the case of multi-workbench extraction units and the use of Y-junctions or extractor switches, every suction channel / every suction point must be fitted with a flow rate monitoring unit.

### Improper use

The SILENT flow sensor may not be operated in combination with equipment that generates a significant amount of dust such as sandblasting units, trimmers, or CAM milling systems!

Only spare parts and accessories supplied or authorized by Renfert GmbH may be used with this product. If other spare parts or accessories are used, this could have a detrimental effect on the safety of the equipment, increase the risk of serious injury and lead to damage to the environment or the product itself.

## Hazard and warning information

### General information

- ▶ If the unit is not operated in accordance with the supplied instructions for use, the intended safety can no longer be guaranteed.
- ▶ The unit may only be operated using a power cord equipped with a plug appropriate to the local power supply. Any necessary modifications may only be carried out by a qualified electrician.
- ▶ Only operate the unit if the information on the nameplate corresponds to the specifications of your local mains power supply.
- ▶ The unit may only be plugged into electrical sockets that are connected to the protective conductor system.
- ▶ The power plug must be easily accessible.
- ▶ Disconnect the unit from the mains power supply before carrying out work on the electrical parts.
- ▶ It is the responsibility of the operator to ensure that national regulations are complied with during operation and regarding recurring safety inspection of electrical equipment. In Germany, these are DGUV Regulation 3 in conjunction with VDE 0701-0702.
- ▶ Check connection cables (such as power cords), tubes and housing (such as the key pad) regularly for damage (such as kinks, cracks and porosity) or signs of ageing.  
Do not operate equipment with damaged connection cables, hoses or housing parts or other defects!
- ▶ Defective equipment must be taken out of service immediately. Disconnect the power plug and secure the equipment against further use. Send the equipment in for repair!
- ▶ Only operate the equipment under supervision.
- ▶ Please observe the national accident prevention regulations!
- ▶ Information on REACH and SVHC is available on our website at [www.renfert.com](http://www.renfert.com), in the Support area.

### Specific information

- ▶ Only the adapter provided on delivery may be used for adaption to the local power supply.
- ▶ The German ordinance on hazardous substances ("GefStoffV") or equivalent national regulations must be observed during operation, cleaning, and maintenance.

## Authorized persons

Operation and maintenance of the equipment may only be performed by trained personnel.

Any repairs not described in this user information may only be performed by a qualified electrician.

## Product description

### General description

The SILENT flow sensor measures the flow rate that flows through it. The measured flow rate is compared electronically with a threshold value defined in the unit. If the flow rate is less than the defined threshold value for a certain period, this is indicated by an acoustic as well as a visual signal.

The SILENT flow sensor meets the requirements of the section on flow rate monitoring units in Annex A of the testing principles GS-IFA-M20-2012-12 issued by the IFA (German Institute for Work and Health).

### Assemblies and functional elements

See Fig. 1 (on inside cover)

- |   |   |   |   |
|---|---|---|---|
| 1 | Inlet nozzle (IN)                             | 6 | Indicator for threshold value 20 l/s (yellow) |
| 2 | Indicators and operation                      | 7 | Indicator for threshold value 25 l/s (yellow) |
| 3 | Outlet nozzle (OUT)                           | 8 | Button  |
| 4 | End bushing                                   | 9 | Operating indicator (green)                   |
| 5 | Indicator for threshold value 15 l/s (yellow) |   |   |

## Scope of delivery

- 1 SILENT flow sensor
- 2 End bushing
- 1 Universal power plug set (only for item number 2914 1000)
- 4 Cable ties
- 1 Operating instructions

## Startup

### Unpacking

⇒ Check the delivery for completeness (compare with the "Scope of delivery").

### Connection

⇒ Determine a location where you can install the SILENT flow sensor.

**See the sequence of images at the beginning of this Quick Start Guide:**

<b>1. / 2.</b>	⇒ Cut the suction hose at an appropriate location.
----------------	--

**When using extractor switches, the SILENT flow sensor must always be placed between the switch and the suction mouth.**

**Before installation, verify that the inlet nozzle IN (1) and the outlet nozzle OUT (3) are screwed in fully until stop into the SILENT flow sensor.**

***If the inlet nozzle and outlet nozzle are not fully screwed in, this could result in leakage that could distort the measurement result and cause unwanted noise.***

<b>3. / 4.</b>	⇒ Screw the end bushing on to the end of the tube. Direction of rotation: Counter-clockwise!
----------------	---

<b>5.</b>	⇒ By rotating, insert the end bushing until stop into the inlet nozzle IN and the outlet nozzle OUT (3).
-----------	--

**The SILENT flow sensor must be operated in an upright position.  
The suction current must flow through the SILENT flow sensor from top to bottom.  
If installed on its side, dust could accumulate and distort the measurement result.**

**When routing the tube, avoid sharp bends and kinks.**

⇒ Install the SILENT flow sensor using appropriate attachment material, for example the cable ties provided or screws (screws are not included with the delivery).

### Electrical connection

***If required, use one of the mains power adapters provided for connection to a building electrical socket.***

⇒ Insert the power plug into a building electrical socket.

- ◆ All 4 LEDs light up, the measurement system initializes. This can take approx. 10 seconds.
- ◆ A short acoustic signal is output when initialization is complete.

Once initialization is complete:

- ◆ The green operating indicator (9) lights up.
- ◆ The yellow indicator for the defined threshold value lights up (5, 6, 7).

## Setting the warning threshold (minimum flow rate)

For the Renfert suction mouth (REF: 2925 0000), a threshold value of "20" must be defined.

For threshold values for other certified dust detection units, contact the respective manufacturer.

If a threshold value cannot be obtained, set the largest threshold value of "25" on the SILENT flow sensor and select an appropriately adjusted, higher suction level on the extraction unit.

⇒ Press the button until a confirmation tone is output.

- ◆ The green operating indicator (9) flashes quickly.
- ◆ The yellow indicator of the last threshold value defined lights up (5, 6, 7).

⇒ Press the button briefly.

- ◆ Every time you press the button, a confirmation tone is output, and the next threshold value is selected.

To save the selected threshold value:

⇒ Press the button until a confirmation tone is output.

- ◆ The green operating indicator (9) lights up.
- ◆ The yellow indicator for the defined threshold value lights up (5, 6, 7).

***If a button is not pressed for 5 seconds, the threshold value defined at that time is automatically saved; a short warning tone is output and the unit exits setting mode.***

## Zero Adjustment (Taring)

**To ensure a correct measurement, a zero adjustment (taring) must be performed on the SILENT flow sensor during commissioning and at regular intervals.**

The zero adjustment is performed if no flow rate passes through the SILENT flow sensor.

⇒ The SILENT flow sensor is installed and electrically connected.

⇒ Extraction is not in operation!

⇒ Press the key for approx. 5 sec. until a double acoustic signal sounds.

- ◆ After approx. 2 sec., a single acoustic signal sounds for setting the warning threshold.

⇒ Continue to hold down the key.

- ◆ After approx. 5 sec., a double acoustic signal sounds to perform the zero adjustment.

⇒ Release the key.

- ◆ The displays flash alternately.
- ◆ The zero adjustment is performed.
- ◆ After approx. 5 sec. the SILENT flow sensor returns automatically into operating mode; the zero adjustment is completed.

## Checking leak-tightness

- ◆ The SILENT flow sensor is connected to a socket.
- ◆ The green operating indicator (9) lights up.
- ◆ The yellow indicator for the defined threshold value lights up (5, 6, 7).

⇒ Switch suction on.

- ◆ The green operating indicator begins to flash.
- ◆ The yellow indicator for the defined threshold value lights up.

⇒ Close the suction mouth with your hand so that air is no longer drawn in through the suction mouth.

⇒ Pay attention to noises that could indicate leakage.

Green indicator	Yellow indicator	Result / measure
Light is on	Light is on	Leak-tight / no leakage
Flashing	Light is on or flashing	Leak between the suction mouth and the SILENT flow sensor / check the suction route between the suction mouth and the SILENT flow sensor, in particular whether the inlet nozzle IN (1) has been fully and securely screwed into the sensor.

## Checking function

After assembly, check the leak-tightness of the suction route and the function of the SILENT flow sensor.

- ◆ The SILENT flow sensor is connected to a socket.
  - ◆ The green operating indicator (9) lights up.
  - ◆ The yellow indicator for the defined threshold value lights up (5, 6, 7).
- ⇒ Switch suction on.
- ◆ The green operating indicator begins to flash.
  - ◆ The yellow indicator for the defined threshold value lights up.
- ⇒ Close part of the suction mouth with your hand so that the flow rate drops below the defined threshold value.
- ◆ After a short time, the indicator for the corresponding threshold value starts to flash.
  - ◆ The warning tone is output after approx. 30 seconds.
- ⇒ Pay attention to noises that could indicate leakage.
- ⇒ Release the suction mouth again and continue to suction for approx. 30 seconds.
- ◆ The indicator for the corresponding threshold value lights up steadily.
  - ◆ The warning tone is no longer output.
- ⇒ Switch off the extraction unit.

## Operation

### Operation

If the flow rate is less than the defined threshold value for a certain period, the corresponding indicator begins to flash and a warning tone is output every 30 seconds.

**If the flow rate is too low at the suction point being monitored, the level of exposure to dust is harmful to health!**

- ⇒ Stop working at the suction point.
- ⇒ Stop working at all suction points of the affected extraction unit. This applies to the individual suction channels of any multi-workbench extraction unit as well as to all suction points operated via a Y-junction or via extractor switches.
- ⇒ Determine why the flow rate is too low and resolve the issue.

Cause	Measure
<b>Suction level is too low.</b>	• Choose a higher suction level.
<b>In the case of multi-workbench extraction units: the suction power is too low because an additional suction channel has been connected.</b>	• Stop working with the additional suction channel.
<b>Filter blockage</b>	• In the case of extraction units with a pre-filter (filter bag): change the filter bag. • In the case of extraction units without a filter bag: clean the filter.
<b>Changing the filter bag / cleaning the filter do not result in the expected improvement.</b>	• Change the fine filter. • When doing so, observe the instructions and the safety information for the corresponding extraction unit in order to ensure that changing the fine filter results in as little dust as possible.
<b>Blockage in the suction point (suction mouth).</b>	• Disconnect the suction hose at the suction point and check whether there is a blockage in the suction point; remove where appropriate.
<b>Blockage in the suction line.</b>	• Check the suction line at all disconnection points / service openings section by section for blockages.
<b>Leakage in the section of the suction line between the SILENT flow sensor and the extraction unit.</b>	• Check that the outlet nozzle of the SILENT flow sensor has been fully screwed into the SILENT flow sensor until stop and that the end bushing of the suction tube is fully inserted into the outlet nozzle.

Determining the cause and measures can be carried out in the order shown.

After every measure, you should switch on the extraction unit again and check whether the minimum flow rate is again exceeded so that correct operation is possible again.

⇒ Work should only resume after the fault has been rectified and the minimum flow rate is exceeded again. It is also the case with multi-workbench extraction units that work must be interrupted at all connected suction points until the fault is rectified.

## Checking function

Every day when beginning your work, check the function of the SILENT flow sensor as described in the sections "Zero Adjustment (Taring)", "Checking leak-tightness" and "Checking function".

## Cleaning / Maintenance

The unit does not contain any internal parts that require maintenance.

**Do not use compressed air!**

**Compressed air irreparably damages the sensitive measurement equipment. Using compressed air to blow off deposits of dust is generally not permitted.**

To clean the unit, only wipe the outside with a damp cloth.

Do not use any solvent-based or abrasive cleaning agents.

## Effectiveness of the measurement equipment

The effectiveness of the measurement equipment must be checked annually as described in the sections "Checking leak-tightness" and "Checking function".

## Disposal information

**For this reason, equipment with this symbol may not be disposed of in unsorted domestic waste within the European Union.**

Please contact your local authorities for more information on proper disposal.

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declare under our sole responsibility, that the product(s)

Name	Part No.
SILENT flow sensor 100-240 V 50/60 Hz Steckertyp 1	29140000
SILENT flow sensor 100-240 V 50/60 Hz Universaladapter	29141000

is (are) in conformity with the relevant regulatory requirements by compliance with the UK designated standards.

## UK legislation

*Electromagnetic Compatibility Regulations 2016*

*Electrical Equipment (Safety) Regulations 2016*

*The Restriction of the Use of Certain Hazardous Substances in Electrical and Electronic Equipment Regulations 2012*

## Designated standards

EN 61010-1: 2010 + A1:2019 + A1:2019/ AC:2019

EN 61326-1: 2013

EN IEC 63000:2018

Tilo Burgbacher,  
Engineering Director

Hilzingen, 09.03.2023

The Technical Documentation has been retained by Renfert GmbH

# Technical Data

<b>Item number:</b>	<b>2914 0000 / 2914 1000</b>
<b>Nominal voltage / V:</b>	<b>230</b>
Permissible mains voltage / V:	100 - 240
Mains frequency / Hz:	50 / 60
Power consumption / W:	0,6
Dimensions (width x height x depth) / mm [inch]:	108 x 223 x 94 [4.25 x 8.78 x 3.70]
Weight / kg [lbs]:	0,8 [1,76]

**Hochaktuell und ausführlich auf ...  
Up to date and in detail at ...  
Actualisé et détaillé sous ...  
Aggiornato e dettagliato su ...  
La máxima actualidad y detalle en ...  
Актуально и подробно на ...**

**[www.renfert.com](http://www.renfert.com)**

Renfert GmbH • Untere Gießwiesen 2 • 78247 Hilzingen/Germany  
Tel.: +49 7731 82 08-0 • Fax: +49 7731 82 08-70  
[www.renfert.com](http://www.renfert.com) • [info@renfert.com](mailto:info@renfert.com)

Renfert USA • 3718 Illinois Avenue • St. Charles IL 60174/USA  
Tel.: +1 6307 62 18 03 • Fax: +1 6307 62 97 87  
[www.renfert.com](http://www.renfert.com) • [info@renfertusa.com](mailto:info@renfertusa.com)  
USA: Free call 800 336 7422

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**Renfert GmbH, Industriegebiet, 78247 Hilzingen / Germany**