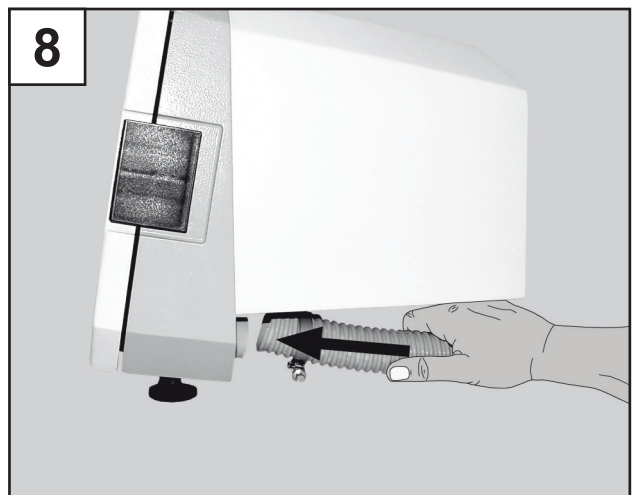
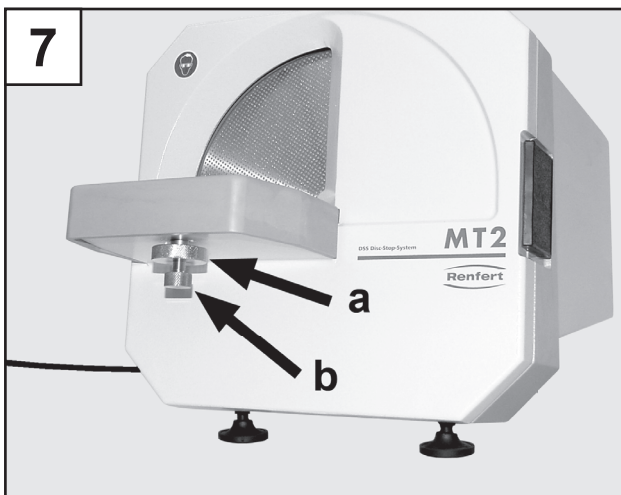
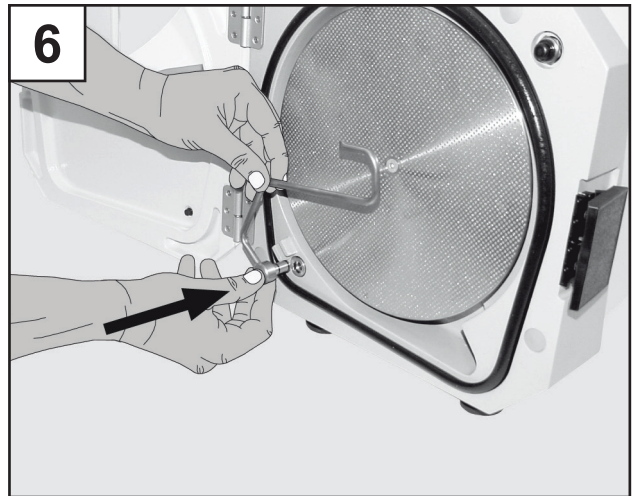
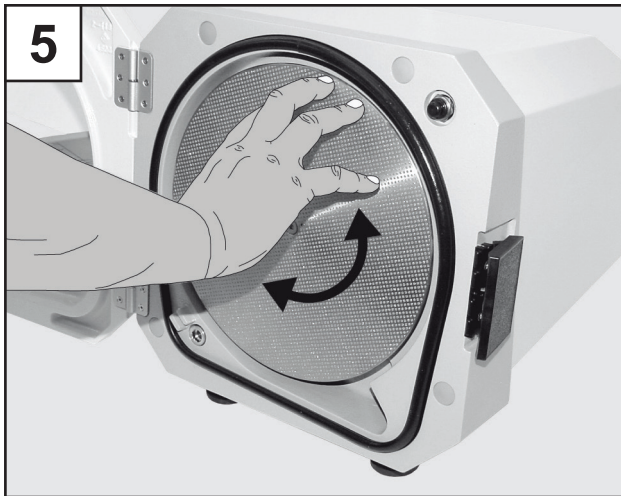
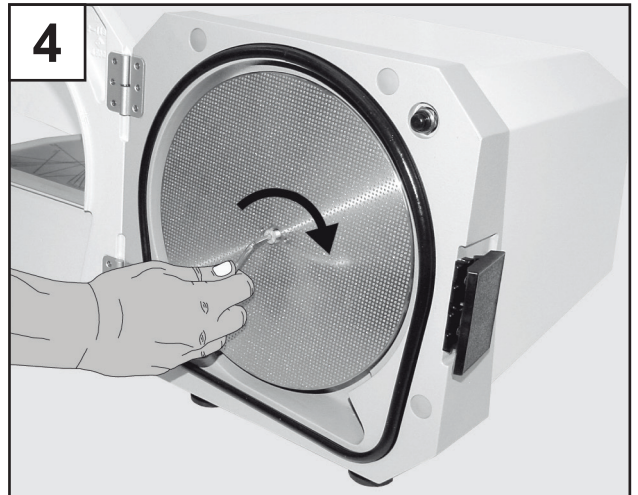
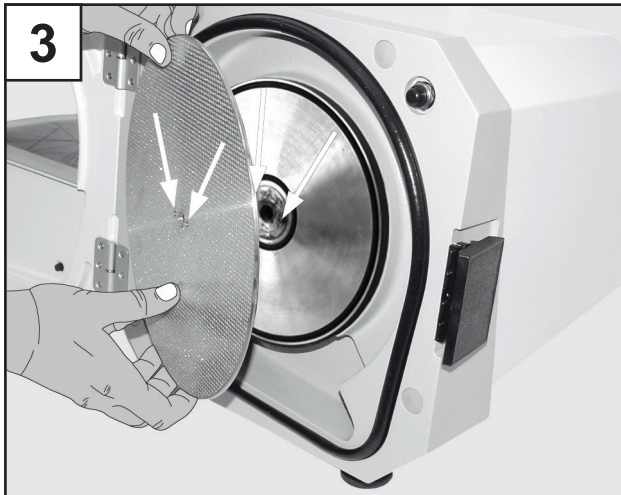
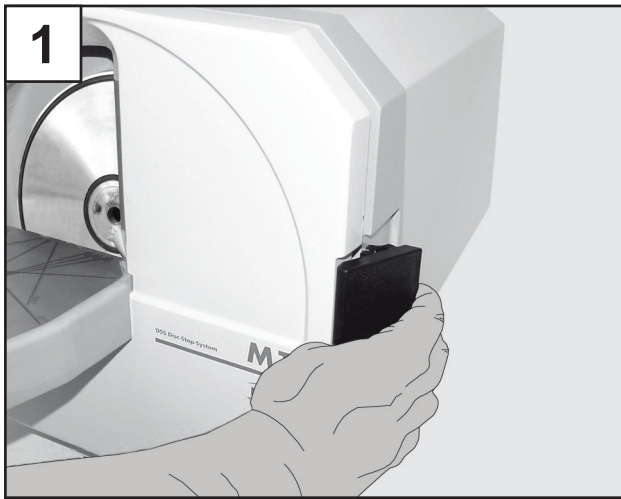
MT 2

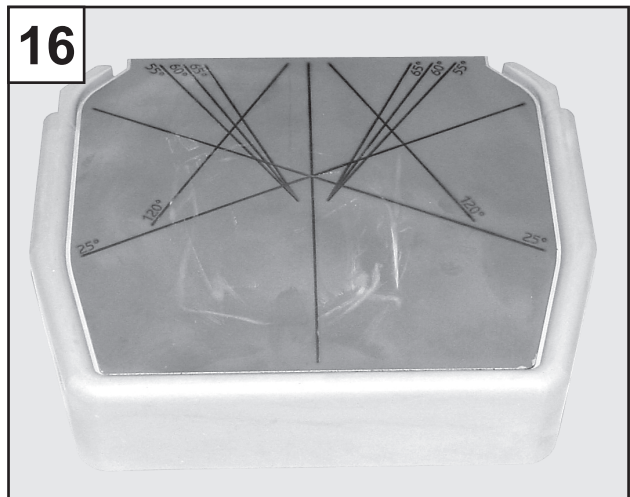
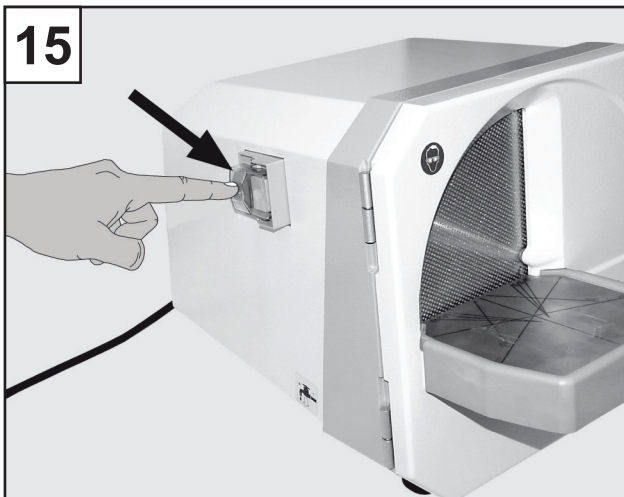
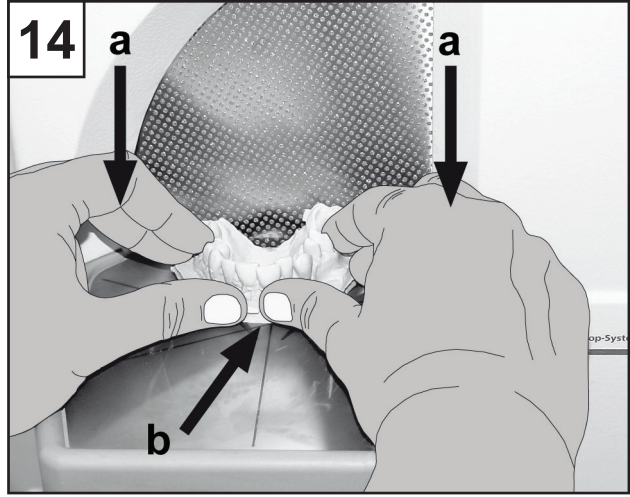
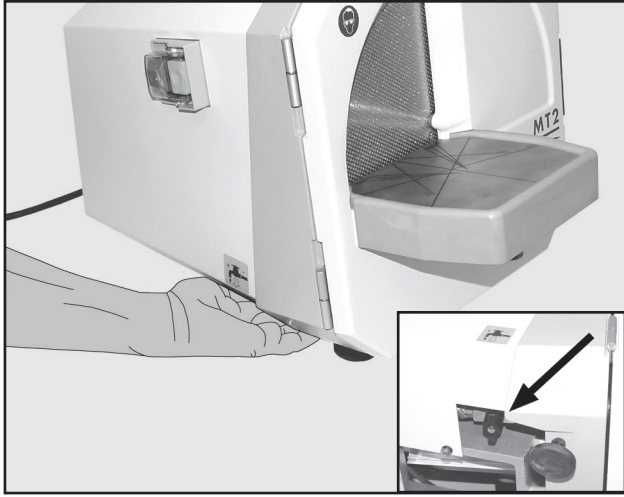
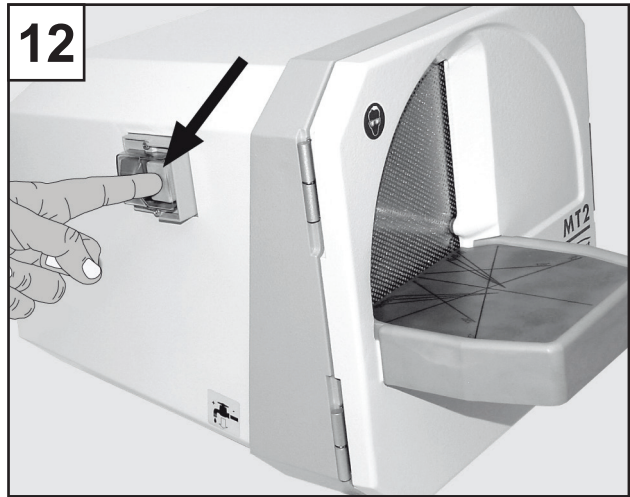
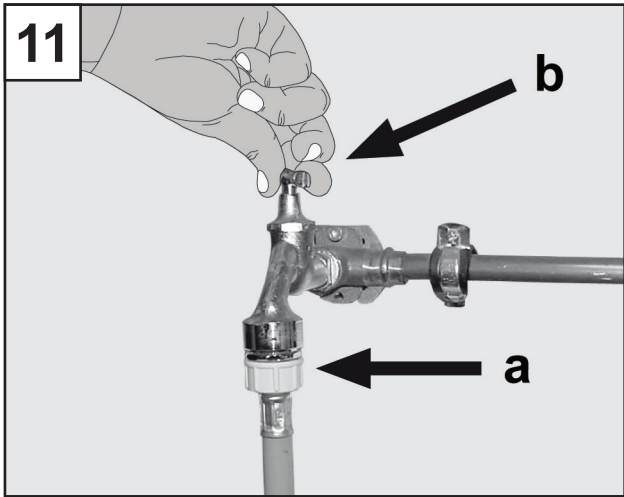
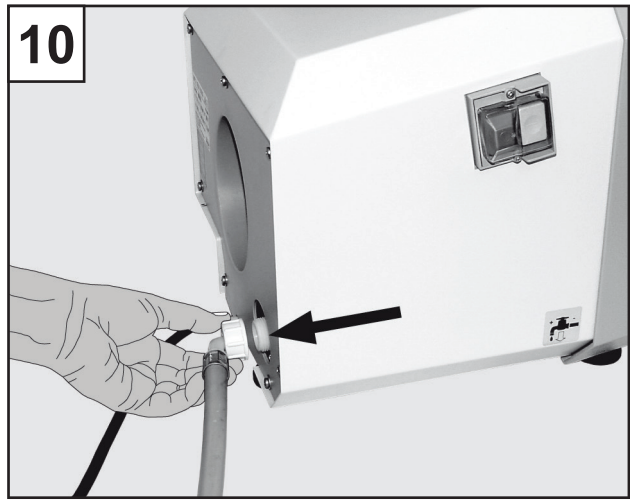
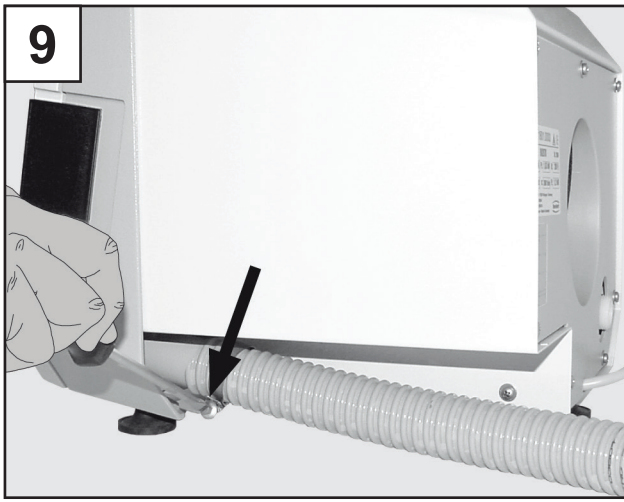
No. 1801-0000

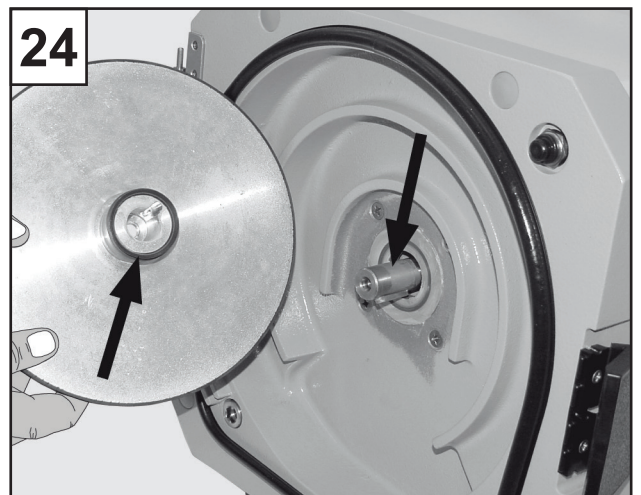
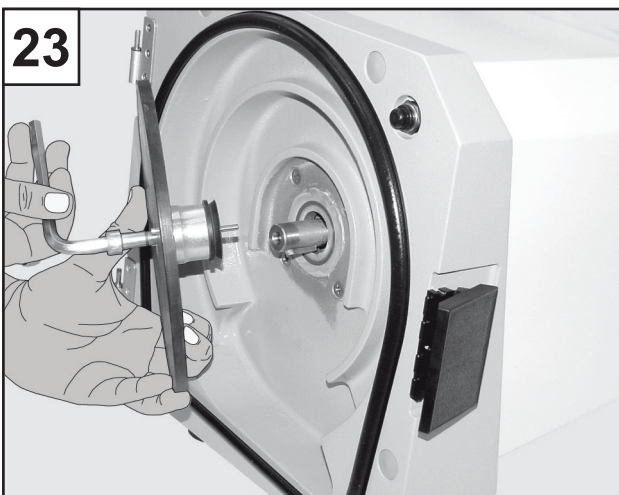
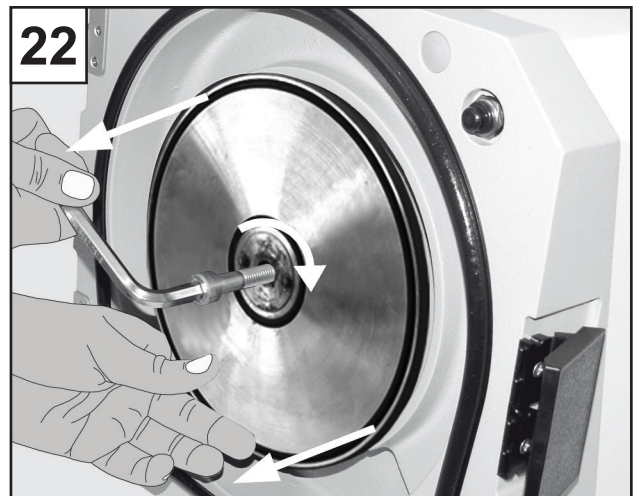
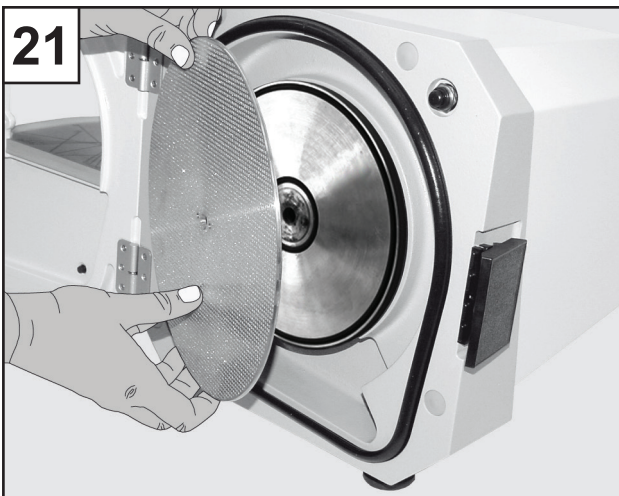
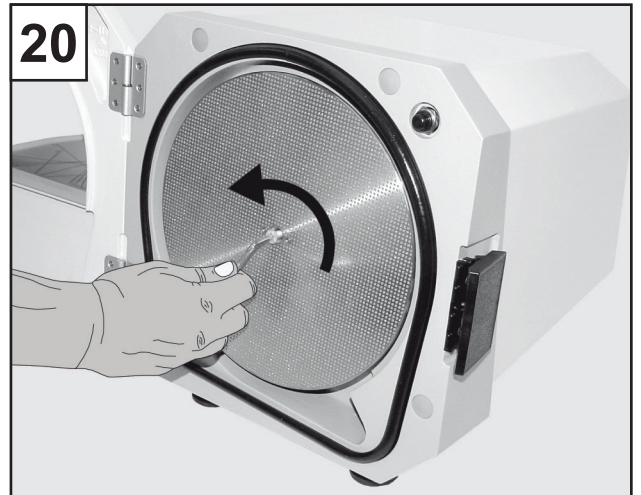
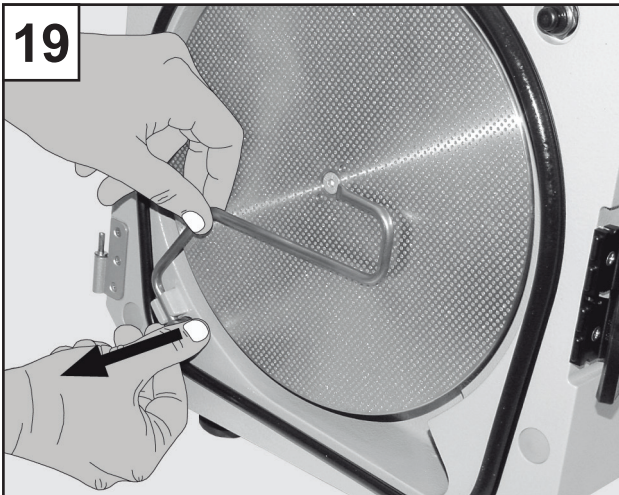
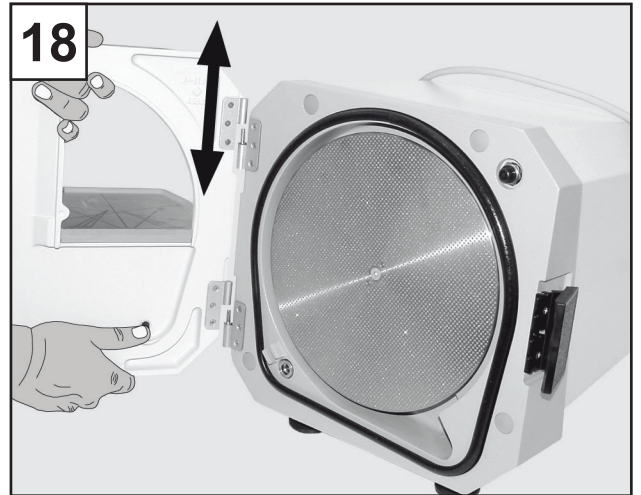
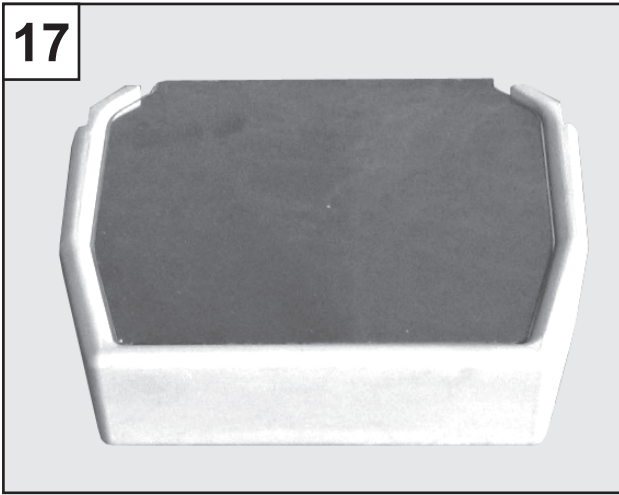
Bedienungsanleitung
Instruction manual • Mode d'emploi
Istruzioni d'uso • Instrucciones para el servicio
Инструкция по эксплуатации

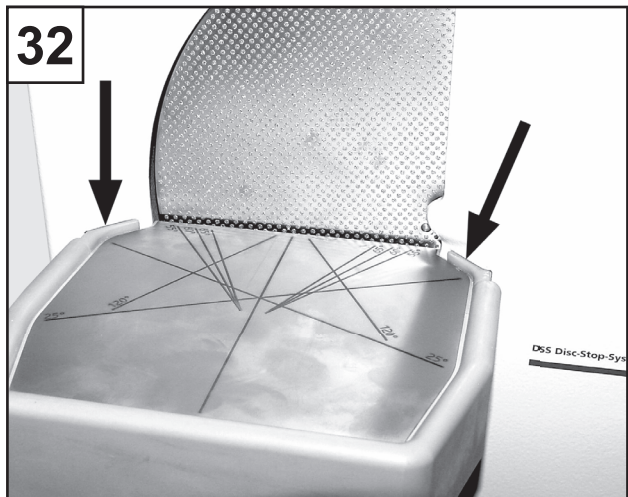
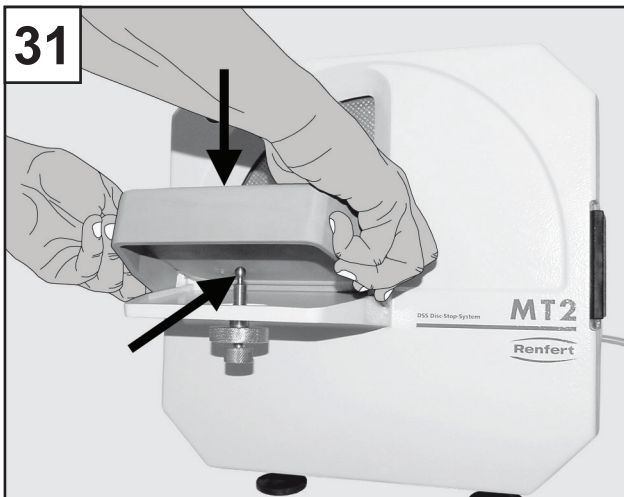
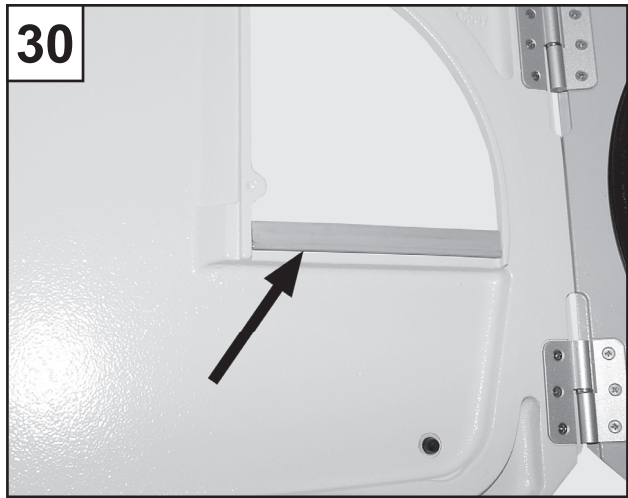
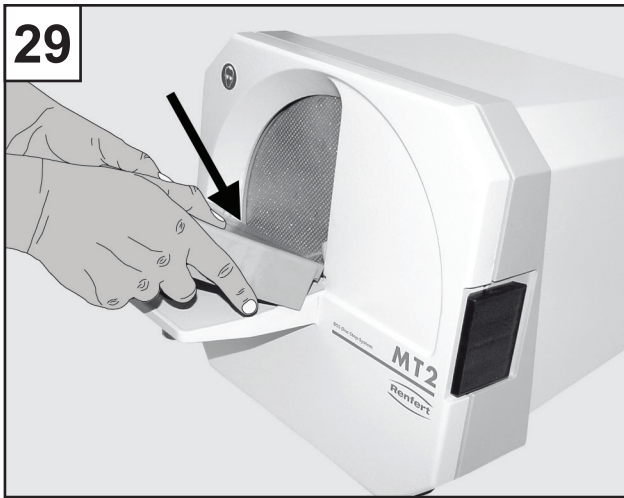
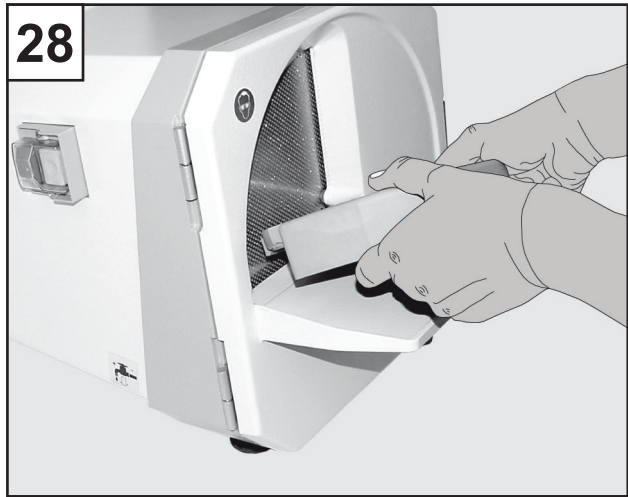
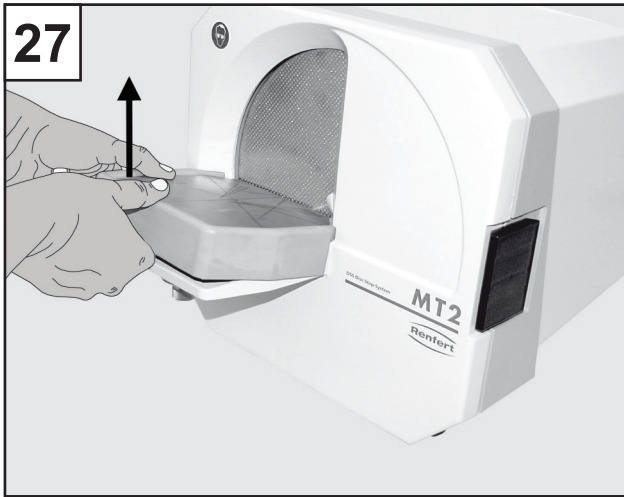
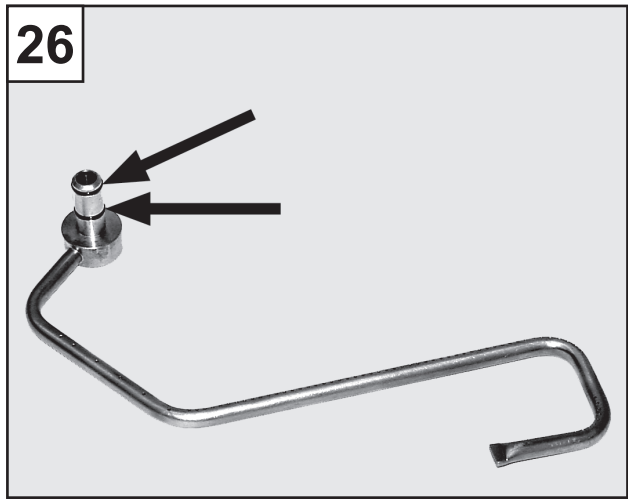
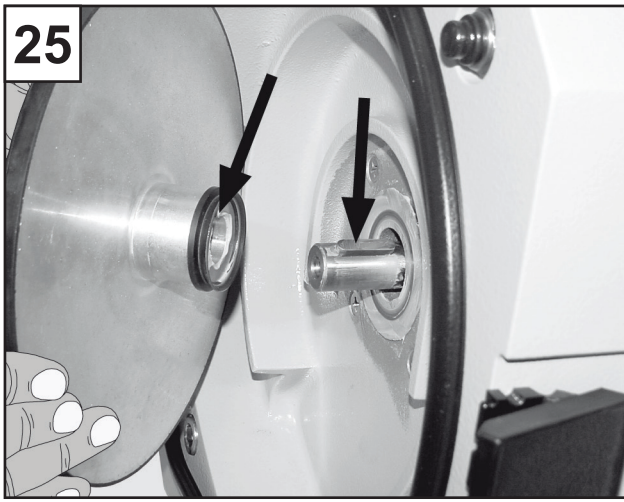
Seriennummer, Herstelldatum und Geräte-Version befinden sich auf dem Geräte-Typenschild.
Serial number and date of manufacturing are shown on the type plate of the unit.
Le numéro de série et la date de fabrication se trouvent sur la plaque signalétique de l'appareil.
Il numero di serie e la data costruzione si trovano sulla targhetta dell'apparecchio.
El número de serie y la fecha de fabricación están indicados en la placa identificadora del aparato.
Номер серии и дата изготовления указаны на фирменной табличке прибора.

MT 2 Gipstrimmer

Nr. 1801-0000

DEUTSCH

1. Einleitung

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ENGLISH

1. Introduction

We are pleased with your decision to purchase the MT2 - Trimmer which sets new standards with respect to function, performance, safety, and styling.

DSS Disc Stop System:

This trimmer is equipped with a maintenance-free electronic motor brake. After the unit is switched off or if the front cover is opened, the trimmer disc is stopped within 5 seconds. The resulting buzzing sound emanating from the motor area is perfectly normal.

Please read the following operating instructions carefully and observe all the information they contain in order to ensure long and trouble-free service.

2. Application Area

The MT2 is a wet trimmer used for grinding plaster or acrylic models for dental technology applications.

2.1 Ambient Conditions

(in accordance with DIN EN 61010-1)

The unit may only be operated:

- indoors;
- up to an altitude of 2,000 m [6,500 ft] above sea level;
- at an ambient temperature range of 5 - 40°C [41 - 104°F] *);
- at a maximum relative humidity of 80% at 31°C [87.8°F], dropping linearly to 50% relative humidity at 40°C [104°F] *);
- with a mains power supply whose voltage fluctuations do not exceed 10% of the rated value;
- under contamination level 2 conditions;
- under over-voltage category II conditions.

*) Between 5 - 30°C [41 - 86°F] the unit can be operated at a relative humidity of up to 80%. At temperatures between 31 - 40°C [87.8 - 104°F] the humidity must decrease proportionally in order to ensure proper operability (e.g., at 35°C [95°F] = 65% relative humidity, at 40°C [104°F] = 50% relative humidity). The unit may not be operated at temperatures above 40°C [104°F].

3. Hazard Information

3.1 Employed Symbols

The following symbols are used in these instructions and on the unit:

Danger
This indicates an imminent risk of injury.

Electrical hazard
There is a risk of electrical shock.

Attention
Failure to observe the associated information can result in damage to the unit.

Note
The operating instructions contain useful information to make handling easier.

Wear eye protection.

For indoor use only.

Before opening the unit, disconnect it from the mains power supply by pulling the wall plug.

Note the operating instructions.

3.2 Hazard Information

During trimming, please observe the accident prevention regulations of the relevant professional association as well as the following safety information:

- **The MT2 is an electrical device with inherent hazard potential. This device may only be installed and operated by authorized specialists after examination of compliance with locally applicable standards.**
- **The unit may only be connected to a properly grounded power supply.**
- **Prior to commissioning, compare the voltage information on the nameplate with the specifications of the local mains power network.**
- **Never touch the trimmer disc while the unit is connected to the mains power supply. Serious injury could result should the unit accidentally start!**
- **Never wear loose clothing when working with the trimmer and always tie long hair up or back. Clothing or hair may become tangled or wound up in the unit.**
- **The unit is only designed to grind plaster or acrylic.**
- **Never open the front cover while the motor is running. The motor automatically switches off if the front cover is opened, but the disc will continue to spin for approx. 5 seconds.**

- Do not open the front cover until the trimmer disc has come to a complete stop.
- Never touch the trimmer disc while it is turning.
- Never work without the trimmer table installed. The wider trimming gap represents a potential source of injury due to the risk of small objects or fingers being drawn into the trimmer.
- Always wear eye protection when trimming.
- Regularly inspect all connecting lines and hoses (e.g., power cord and water hoses) for damage (e.g., kinks, cracks, porosity).
- Units with damaged connecting lines or hoses, or those which exhibit other defects may not be used.
- Always disconnect the unit from the mains power supply prior to performing any work on electrical components.
- Only spare parts and accessories supplied or authorized by Renfert GmbH may be used with this product. If other spare parts or accessories are used, this could have a detrimental effect on the safety of the device, increase the risk of serious injury and lead to damage to the environment or the device itself.
- It is the responsibility of the operator that national regulations during operation and regarding a repeated safety inspection of electrical equipment are complied with. For Germany these are BGV A3 in relation with VDE 0701-0702.

3.3 Liability Exclusion

Renfert GmbH shall be absolved from all claims for damages or warranty if:

- The product is employed for any purposes other than those cited in the operating instructions.
- The product is altered in any way other than those alterations described in the operating instructions.
- The product is repaired by other than an authorized facility or if any but Renfert OEM parts are employed.
- The product continues to be employed, despite obvious safety faults or damage.
- The product is subjected to mechanical impacts or is dropped.

4. Commissioning

4.1 Setting Up and Connecting the Unit

1. Set the trimmer up on a level surface near the sink / plaster separator.
2. Open the front door (Figure 1 + 2).
3. Install the trimmer disc.
 - Mount the trimmer disc (Figure 3).
 - Make sure the locking pins are properly seated in the retention holes (Figure 3).

- Using the Allen key, turn the centring screw clockwise to tighten it (Figure 4).
 - Turn the disc by hand to make sure it is running true (Figure 5). If the disc exhibits large variations in height:
 - remove it,
 - turn it 180°,
 - reinstall.
 - Insert the M5 covering cap (from the mounting hardware set) into the Allen screw to protect the screw against contamination.
4. Mount the spray pipe on the water inlet (Figure 6).
 5. Close the front door.
 6. Adjust the trimmer table.
 - Loosen the locknut (Figure 7-a).
 - Precise adjustment (Figure 7-b):
 - Clockwise – Increases the table angle;
 - Counter clockwise – Decreases the table angle.
 - Retighten the locknut (Figure 7-a).
 7. Connect the drain hose (Figure 8 + 9).
 8. Connect the drain hose to the waste water removal system. Make sure the drain hose has an adequate decline.
 9. Connect the water supply hose (Figure 10).
 10. Connect the water supply hose to the water supply (with a shut-off tap) (Figure 11-a).
 11. Turn the water supply on (Figure 11-b).
 12. Check all water connections for leaks.

Before connecting the unit to the mains power supply, compare the voltage information on the nameplate with the local power supply specifications.

13. Plug the unit's power cord into the wall outlet.

The MT2 is now ready for use

5. Operation

5.1 Model Trimming

Never trim without water (risk of overheating)!

Always wear eye protection when trimming!

1. Turn the water supply on (Figure 11-b).
2. Switch the trimmer on (Figure 12). Water is automatically supplied via the valve.
3. The water volume can be adjusted at the reduction valve (Figure 13) located on the bottom left-hand side of the unit.
4. Use both hands to hold the model securely on the trimmer table (Figure 14-a).
5. Gently push the model against the trimmer disc (Figure 14-b).
6. Always switch the unit off when you are done trimming (Figure 15).
7. Turn the water supply off if the unit is to remain out of service for an extended period of time (Figure 11-b).

Various factors influence the service life of trimmer discs. Particularly in the case of diamond-coated discs, an adequate water supply is very important. Overheating will result in damage to the trimmer disc.

Never use the trimmer discs to grind metal. Muffle rings, pins, and retention rings, etc. can also damage trimmer discs. Investments contain extremely abrasive components that will significantly shorten the disc's service life.

5.2. Adjusting the Water Volume

The water volume can be individually adjusted at the reduction valve (Figure 13).

5.3. Adjusting the Trimmer Table

The standard delivery includes two tables:

- Table 1 = 90° (Figure 16);
- Table 2 = 98° (Figure 17).

To adjust the trimmer table:

1. Loosen the locknut (Figure 7-a).
2. Precise adjustment (Figure 7-b):
 - Clockwise – Increases the table angle;
 - Counter clockwise – Decreases the table angle.
3. Retighten the locknut (Figure 7-a).

6. Cleaning / Maintenance

Always unplug the unit from the mains power supply before performing any cleaning or maintenance tasks.

Regularly inspect the power cord and water hoses for damage or aging. Damaged power cords or hoses must be replaced before the unit can be used again.

6.1 Weekly Cleaning Tasks

A clean drain line and clean water supply are vital to the trimmer's proper function. The unit should therefore be cleaned at least once every week.

1. Switch the unit off (Figure 15), unplug the power cord, and turn off the water (Figure 11-b).
2. Open the front door (Figure 1 + 2) and take it off its hinges (Figure 18).
3. Using a brush, clean under running water.
4. Reinstall the front door, making sure the spacers on the hinge bolt are installed (Figure 18).
5. Close the front door.
6. Plug the power cord back into the wall outlet.

6.2 Replacing the Trimmer Disc

1. Switch the unit off (Figure 15), unplug the power cord, and turn off the water (Figure 11-b).
2. Open the front door (Figure 1 + 2).
3. Remove the spray pipe (Figure 19).
4. Remove the protective cap covering the centring screw and use the SW4 Allen key included in the accessories pack to turn the screw counter clockwise and loosen it, (Figure 20).
5. Remove the trimmer disc (Figure 21).
6. Clean the retainer flange as required (refer to the following section).
7. Install the new trimmer disc (Figure 3).
8. Make sure the locking pins are properly seated in the retention holes (Figure 3).
9. Using the Allen key, turn the centring screw clockwise to tighten it (Figure 4).
10. Turn the disc by hand to make sure it is running true (Figure 5); If the disc exhibits large variations in height:
 - remove it;
 - turn it 180°;
 - reinstall it.
11. Install the covering cap on the centring screw (protection against contamination).
12. Mount the spray pipe on the water inlet (Figure 6).
13. Close the front door.
14. Plug the unit's power cord into the wall outlet.

6.3 Replacing / Cleaning the Retainer Flange

1. Switch the unit off (Figure 15), unplug the power cord, and turn off the water (Figure 11-b).
 2. Open the front door (Figure 1 + 2).
 3. Remove the spray pipe (Figure 19).
 4. Remove the protective cap covering the centring screw and use the SW4 Allen key included in the accessories pack to turn the screw counter clockwise and loosen it (Figure 20).
 5. Remove the trimmer disc (Figure 21).
 6. Using the SW8 Allen key included in the accessories pack, turn the setbolt clockwise (Figure 22).
 7. The retainer flange is pushed off the motor shaft (Figure 22).
 8. Remove the retainer flange (Figure 23).
 9. Clean the retainer flange and the trimmer chamber.
- ! Do not allow any water to enter the motor through the motor shaft opening.**
10. Remove the setbolt from the retainer flange thread.
 11. Apply a thin coating of grease (refer to the accessories) to the motor shaft and rubber seal (Figure 24).
 12. Push the setbolt onto the motor shaft, making sure the carrier aligns with the carrier groove (Figure 25).
 13. Reinstall the trimmer disc as previously described.

6.4 Changing / Replacing the Spray Pipe

1. Switch the unit off (Figure 15), unplug the power cord, and turn off the water (Figure 11-b).
2. Open the front door (Figure 1 + 2).
3. Gently move the spray pipe back and forth and pull it off the water inlet (Figure 19).
4. Use a suitable delimiting solution to clean the spray pipe.
5. Apply a thin coating of grease (refer to the accessories) to the seal rings or replace them as required (Figure 26).
6. Mount the clean spray pipe on the water inlet (Figure 6).
7. Close the front door.
8. Plug the unit's power cord into the wall outlet.
9. Open the shut-off valve (Figure 11-b) and readjust the water volume (Figure 13).

6.5 Replacing / Cleaning the Trimmer Table

1. Switch the unit off (Figure 15), unplug the power cord.
2. Lift the trimmer table up and off the adjusting screw then pull it towards the side and front (Figure 27 + 28).
3. Clean the trimmer table under running water.
4. Apply a thin coating of silicon-based oil to the inner side of the table to make reinstallation easier.
5. Reinstall the trimmer table.
 - Working from above, push the trimmer table into the trimmer opening at an angle (Figure 29).
 - Once the rubber tab (Figure 30) has snapped into place over the edge, the trimmer table is locked in place in the adjusting screw (Figure 31).
 - Press the edges of the trimmer table (Figure 32) into the recesses in the front cover.

Make sure the trimmer table is properly and securely seated on the front cover!

6. Adjust the trimmer table as previously described.

7. Spare Parts

Please refer to the enclosed spare parts list for the numbers of all consumables and spare / replacement parts.

8. Warranty

Provided the unit is properly used, Renfert **warrants** the *MT2* a period of **3 years**.

The original sales receipt from the dealer must be provided when making warranty claims.

Parts which are subject to natural wear and tear (wear parts) and consumables are excluded from the guarantee. These parts are marked in the spare part list.

The warranty is voided in case of improper use; failure to observe the operating, cleaning, maintenance, and connection instructions; in case of independent repairs or repairs by unauthorized personnel; if spare parts from other manufacturers are employed, or; in case of unusual influences or influences not in compliance with the utilization instructions. Warranty service shall not extend the original warranty.

9. Technical Specifications

Power supply:	230 V, 50-60 Hz
Power consumption:	1,630 W
Speed:	2,860 rpm.
Water consumption:	9 l/min., max
Dimensions (WxHxD):	334 x 320 x 456 mm
Weight:	26.5 kg
Sound pressure level, Lp(A), measured under full load in accordance with DIN 45635-01-KL3:	Lp(A) = 74 dB(A) K = 4 dB

10. Standard Delivery

- 1 *MT2*
- 1 Inlet hose
- 1 Drain hose
- 1 Tool set
- 1 KFO 90° trimmer table
- 1 98° trimmer table
- 1 Mounting hardware set
- 1 Operating instructions, incl. spare parts list

11. Accessories

- No.1801-1000 Set-up kit: 5 Klettfix silicon carbide trimmer discs
(3 x 80 grain, 2 x 120 grain)
- No.1801-1100 Refill pack: 5 Klettfix K80 silicon carbide trimmer discs
- No.1801-1200 Refill pack: 5 Klettfix K120 silicon carbide trimmer discs
- No.1801-2000 1 Marathon partially diamond-coated trimmer disc
- No.1801-3000 1 Infinity fully diamond-coated trimmer disc
- No. 2-0973 1 plastic triangle, 90°

12. Troubleshooting

Fault	Cause	Remedy
The unit does not switch on.	<ul style="list-style-type: none"> The electricity supply is not properly connected. The electrical safety device is defect. The front lid is not closed properly. The motor is overheated. 	<ul style="list-style-type: none"> Check the electricity supply. Check the safety device. Close the front lid. Let the motor cool down. Switch on the power switch again.
The disc jams.	<ul style="list-style-type: none"> Not enough water pressure. The quantity of water flowing to unit is incorrectly set. The water supply is not fully opened. The spraypipe is blocked. 	<ul style="list-style-type: none"> Open the water supply. Increase the water supply to the trimmer. Open the water supply. Clean / de-scale the pipe.
Water sprays out of the trimmer opening.	<ul style="list-style-type: none"> Too much water. The water drainage is not functioning properly. 	<ul style="list-style-type: none"> Reduce the water supply to the trimmer. Check if the drainage pipe is attached properly (gradient).
The disc is wobbling.	<ul style="list-style-type: none"> Disc is not lying flat against the joint. Disc has not been tightened sufficiently. 	<ul style="list-style-type: none"> Refit the disc according to the instructions. Hand-tighten the disc.
No water.	<ul style="list-style-type: none"> Water supply is not properly connected. Spraypipe is clogged. 	<ul style="list-style-type: none"> Check the water supply. Clean / de-scale the pipe.
Model tray wobbles.	<ul style="list-style-type: none"> Model tray is not mounted properly. 	<ul style="list-style-type: none"> Refit the model tray according to the instructions.

13. Disposal instructions for countries in the EU

To conserve and protect the environment, prevent environmental pollution and improve the recycling of raw materials, the European Commission adopted a directive that requires the manufacturer to accept the return of electrical and electronic units for proper disposal or recycling.

Within the European Union units with this symbol should not therefore be disposed of in unsorted domestic waste:

For more information regarding proper disposal please apply at your local authorities.

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Renfert GmbH • Industriegebiet • 78247 Hilzingen/Germany