making work easy

- [FAQ - Deutsch](#)
- [FAQ - English](#)
- [FAQ - Français](#)
- [FAQ - Italiano](#)
- [FAQ - Español](#)
- [FAQ - Português](#)
- [FAQ - Türkçe](#)
- [FAQ - Русский](#)
- [FAQ - Polski](#)
- [FAQ - 中文](#)
- [FAQ - 日本語](#)
- [FAQ - 한국어](#)
- [اللغة العربية - FAQ](#)

# MT premium

[FAQ - frequently asked questions](#)

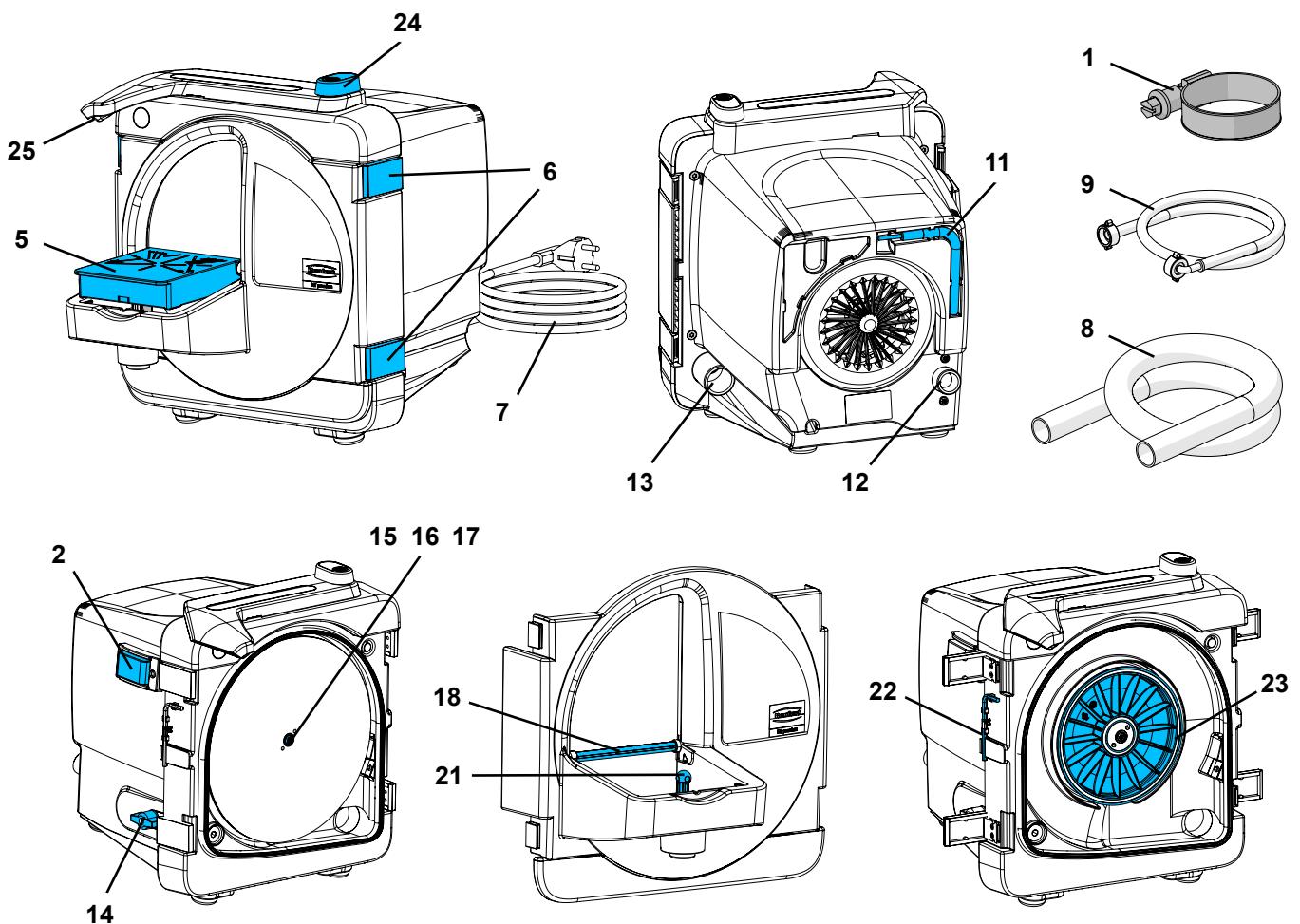
Fig. 1

# FAQ - Deutsch

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# FAQ - English

For information on operating elements, see [Fig. 1](#) on page 2 of this document.

For chapter indications, refer to the complete instruction manual on the internet.

Error	Cause	Corrective action
<b>Unit does not start.</b>	Mains power supply not available.	Check the mains power supply.
	Mains fuse is faulty.	Check the mains fuse.
	The trimmer door is not properly closed.	Close the door locks (6, <a href="#">Fig. 1</a> , 8).
	Motor overheated.	Allow the motor to cool for approx. 60 minutes. Press the power switch again. If necessary, check in the meantime whether the thermal protection has cooled down again.
<b>Trimmer disc wobbles.</b>	The trimmer disc is not placed flat on the flange.	Attach the trimmer disc again in accordance with the instructions (see Chap. 5.4). Pay attention to exact positioning of the trimmer disc with regard to the lock pins (17, Fig. 9!). Remove deposits and dirt from the back of the trimmer disc and the flange.
	Fastening screw (16, <a href="#">Fig. 1</a> , 8) not sufficiently tightened.	Securely tighten the fastening screw (16, <a href="#">Fig. 1</a> , 8).
<b>The trimmer disc makes noise while running.</b>	Fastening screw (16, <a href="#">Fig. 1</a> , 8) not sufficiently tightened.	Tighten the fastening screw (16, <a href="#">Fig. 1</a> , 8) securely and attach again if necessary (Chap. 5.4).
<b>Model table wobbles / is loose.</b>	Model table not correctly attached.	Press the model table fully onto the ball head.
<b>Trimmer disc becomes clogged.</b>	The water quantity for the unit is not correctly regulated.	Increase the amount of water for the trimmer (Chap. 5.2).
	Insufficient water supply.	Open the water shut-off valve of the water supply (Chap. 5.2).
	Aqua stop closed.	Open the aqua stop for the water supply.
	Spray tube is blocked.	Clean / descale the spray tube (Chap. 6.4).
	Water connection sieve is blocked.	Clean the water connection sieve (Chap. 6.5).
	Spray tube seal is missing.	Check whether the spray tube seal is available.
	Spray tube is missing.	Insert the spray tube.
	The water pressure in the supply pipe is too low.	The cross section of the supply pipe is too small or there are too many consumers using a narrow supply pipe. >>> Check whether the water pressure in the supply pipe is at least 1 bar (ask the installation technician). If necessary, increasing pressure artificially by installing a domestic water pressure system (electrical pump with pressure tank) can ensure appropriate pressure in the pipe.
<b>Water sprays from the trimming opening.</b>	Too much water.	Reduce the amount of water for the trimmer (Chap. 5.3).
	Drainage not fully guaranteed.	Check the drain hose, lay at an incline (Chap. 4.2.2).

Error	Cause	Corrective action
<b>No rinsing water.</b>	Water connection is not correctly installed.	Check the water connection (Chap. 4.2.1).
	Aqua stop closed.	Open the aqua stop for the water supply.
	Spray tube is dirty / calcified.	Clean / descale the spray tube (Chap. 6.4).
<b>Water is leaking between the front cover and housing.</b>	Plaster dust or plaster grit on the surface of the seal.	Clean the seal and the surface of the seal.

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