making work easy

- de
- en
- fr
- it
- es
- pt
- tr
- ru
- zh
- ko
- ar

# SILENT TS 2

Quick Start Guide

Made in Germany

21-6729 30072024

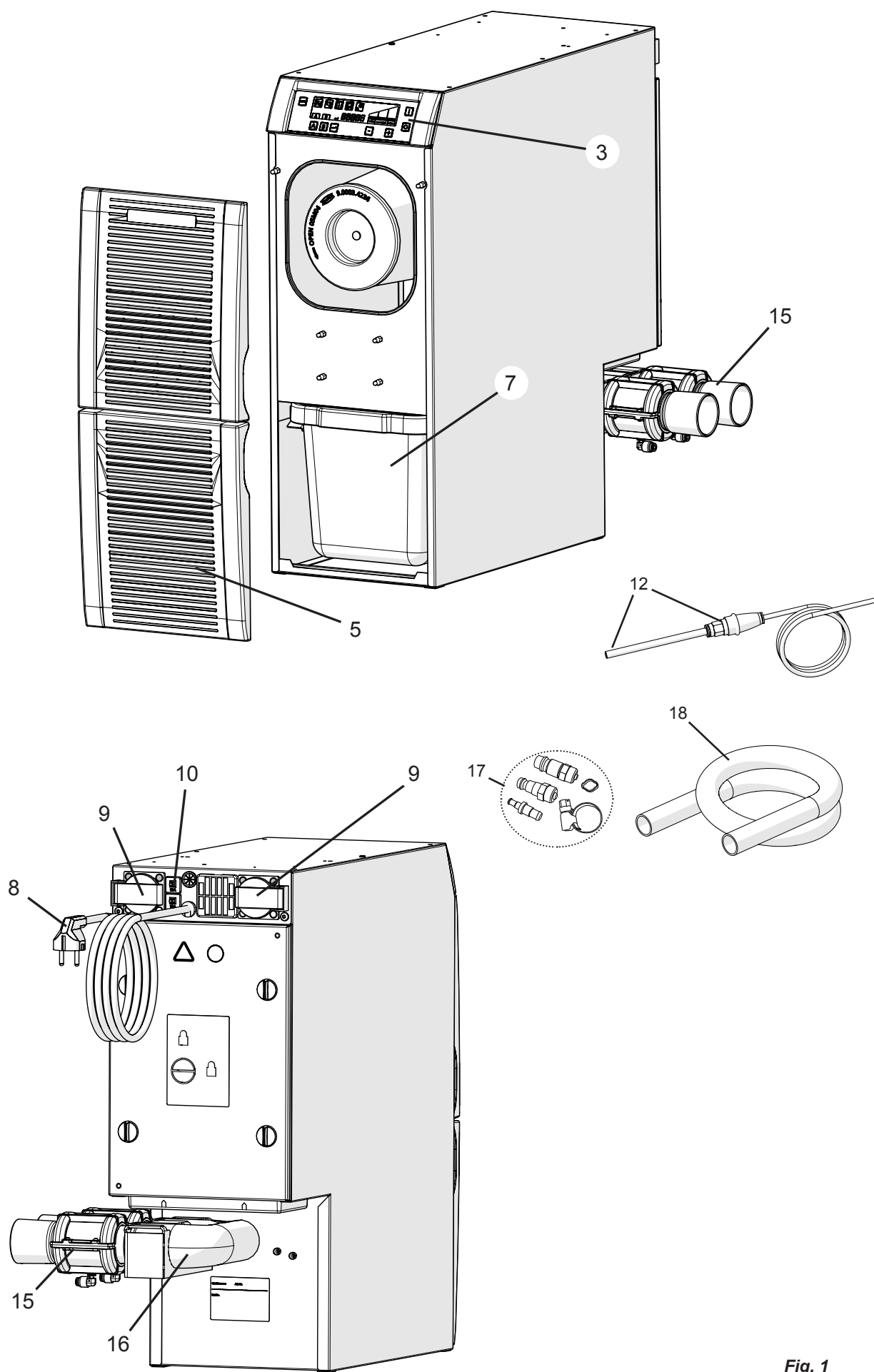
Fig. 1

# Einleitung

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# Introduction

This Quick Start Guide contains instructions on the operation of the product and relevant safety information.

Detailed information on the product can be found in the operating instructions which are available for download on our website at [www.renfert.com/p915](http://www.renfert.com/p915).

Enter the following item number: 29300050

## Symbols

In the instructions for use and on the unit itself you will find these symbols with the following meanings:

### Danger

This indicates a direct risk of injury. Consult accompanying documents!

### Electrical current

This indicates a risk of hazard due to an electrical current.

### Attention

Disregarding this warning may result in damage to equipment.

### Note

This provides the operator with useful information to improve and ease use.

The device complies with the requirements of the applicable EU directives.

This product complies with the relevant UK legislation.

See UKCA Declaration of Conformity in the Internet under [www.renfert.com](http://www.renfert.com).

The device is subject to the EU directive 2002/96/EG (WEEE directive).

Other symbols are explained as they occur.

## Safety

### Intended Use

The product is intended for indoor use for the extraction of dry, non-explosive dust particles. The ambient temperature must be between 5 - 40 °C.

This device is intended solely for commercial use in a dental laboratory.

The intended use also includes compliance with the instructions specified by the manufacturer concerning operation, servicing and maintenance.

The extraction unit can be used in combination with a SILENT flow sensor within the scope of measures to protect against exposure to dust \*) as defined by the German ordinance on hazardous substances ("GefStoffV") / respective employers' liability insurance association.

For this purpose, the information on low-dust disposal of the extracted material (chapter "Emptying the dust drawer") and on low-dust replacement of the fine filter (chapter "Replacing the fine filter") in particular must be observed.

**In doing so, both the instructions for use of the extraction unit as well as those of the SILENT flow sensor must be observed.**

\*) Note for customers in Germany: A validated and recognized dust protection measure in accordance with testing principle GS-IFA-M20 (issued by the German Institute for Work and Health IFA) shall only be deemed to have been provided when all components, i.e. the flow rate monitoring unit such as the SILENT flow sensor, the extraction unit, and the detection unit, are validated and recognized.

In the case of multi-workbench extraction units and the use of Y-junctions or extractor switches, every suction channel / every suction point must be fitted with a flow rate monitoring unit.

### Hazard and Warning Information

#### General Information

- ▶ If the device is not used in compliance with the supplied instructions, the safety of the device can no longer be guaranteed.
- ▶ Only operate the device under supervision.
- ▶ The mains plug must be easily accessible.

- ▶ Disconnect the device from the mains before carrying out work on the electrical parts.
- ▶ The product does not contain any parts inside which require servicing.
- ▶ Defective devices must be put out of service immediately. Remove the mains plug and ensure the device is not used. Send the device for repair!
- ▶ The product may not be opened, except as instructed below.
- ▶ Information on REACH and SVHC is available on our website [www.renfert.com](http://www.renfert.com), in the Support area.

### Specific Information

- ▶ The German ordinance on hazardous substances (“GefStoffV”) or equivalent national regulations must be observed during operation, cleaning, and maintenance.
- ▶ Please observe the national regulations and permitted exposure to dust in a working environment. Please ask the “National Institute for Occupational Safety and Health” or other responsible authority.
- ▶ Please observe the Safety Data Sheets of the materials to be extracted.
- ▶ It is necessary to wear suitable personal protective equipment when emptying the dust drawer or cleaning, depending on the type of extracted material.
- ▶ Do not extract flammable or explosive gasses, fumes or dust.
- ▶ The following applications involve increased risks and are therefore not permitted:  
When cleaning milling chambers and SLM printing systems by suction, a sufficiently high concentration and purity (i.e. unmixed with other dental dusts such as gypsum, resins) can result in spontaneous ignition or the milling dust exploding or printing powder because of a potential exothermic reaction (e.g. due to oxidation).  
An alternative cleaning method must be selected for the following milling or printing media in particular (such as manual cleaning):
  - Wood
  - Titanium / titanium-aluminum
  - Light metals and light metal alloys (e.g. aluminum, magnesium)
  - Cobalt chrome powder (e.g. for use in SLM systems)
 If large quantities of light metals such as titanium alloy are processed (e.g. with sandpaper), producing very fine grinding dust, a sufficiently high concentration and purity may result in spontaneous combustion because of a potential exothermic reaction.
- ▶ Do not extract hot materials.
- ▶ Do not extract liquids.
- ▶ Dispose of extracted material according to local statutory regulations.

### Authorized Persons

Operation and maintenance of the device may only be performed by qualified personnel.

## Product Description

See Fig. 1 (on inside cover)

- 3 On / off switch
- 5 Front panel/ Dust drawer
- 7 Dust drawer
- 8 Mains cable
- 9 Device coupler socket (A, B)
- 10 Device protection switch
- 15 Pinch valve with extractor connection fitting (rotatable)
- 16 Suction pipe
- 17 Connection set
- 18 Suction hose

The other units are covered in the full set of operating instructions available on the internet.

## Scope of Delivery

- 1 Silent TS2
- 1 Operating instructions
- 1 Quick reference card
- 1 Dust bag (ready for us in the dust drawer)
- 1 Compressed air tube (sealed)
- 2 Pinch valves with suction port
- 1 Connection set
- 2 Shockproof plug (only for 2930-0050)
- 1 4 m Suction Hose
- 4 PVC end bushings

## Setting Up

### Unpacking

⇒ Check the delivery for completeness (refer to the “Scope of Delivery” section).

### Setup

**!** If the device is to be installed in a cabinet, an opening for the exhaust air with the following dimensions must be provided, in addition to the openings for the suction hoses:


- Circular opening: Min. 120 mm diameters.
- Rectangular opening: Min. 170 x 65 mm.

### Installing the Pinch Valves

⇒ Push the suction pipe (16) onto the suction support on the extractor and hang the holder (20) onto the holder screws.

⇒ Connect the compressed air tubes to the pinch valves:

- Push the compressed air tube onto the pipe connection in the pinch valve as far as it will go. There will be a slight resistance to overcome when attaching.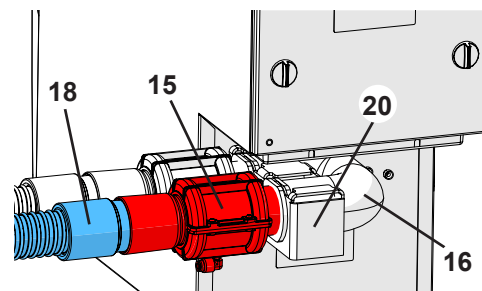

### Connection

⇒ Connect to the compressed air supply system, using a connection kit if necessary (17, Fig. 1).

**i** **Observe the minimal / maximum connection pressure, see technical data!**

**!** **Compressed air should be clean and dry (without condensed water). Wet compressed air can damage the unit.**

⇒ Shorten the suction pipes for both extraction points to the required length with a knife.

⇒ Screw the PVC end bushings onto the suction pipes (left handed thread).

**!** **Caution: risk of injury!**  
**When shortening the suction pipe please ensure to cut at a right angle.**

⇒ Fit suction hose (18) in intake socket (15) and connect to suction point.

**i** **Long suction hoses, tight bends and kinks will considerably reduce the extraction force at the extraction point.**

⇒ Electrical appliances can be connected to the coupler sockets (9, Fig. 1) at the back of the unit. Check the connections are correct - coupler socket (A, B) - suction hose / pinch valve (A, B).

**!** **When connecting electrical appliances to the extraction unit, please ensure that the power consumption of all devices connected to the unit power outlet does not exceed the total maximum power for connected equipment (see “tech. Data”).**

**!** **Before connecting the device, ensure that the voltage information on the identification plate corresponds with your local power supply.**

⇒ Insert the mains cable into the plug socket.

# Operation


Before beginning to work, check that a dust bag has been inserted.

If operated without a dust bag, the intended protection against exposure to dust can no longer be guaranteed.

## Switching the Unit On

The extractor is put into operation at the on / off switch (3).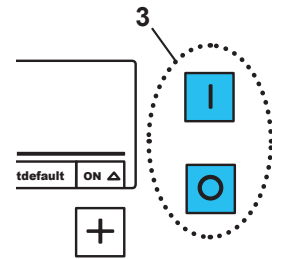

## Stand-By

In the stand-by mode the display is dark.


**Leave stand-by mode:**

- Press any button
- Switch on any electrical appliance connected to the extractor.

## Select vacuum channel

Start / stop suction turbine and open / close corresponding pinch valve valve:


- Automatically:  
The electrical appliance connected to the coupler socket (9, Fig. 1) is activated / deactivated.
- Manually:  
By pressing a suction hose button (39).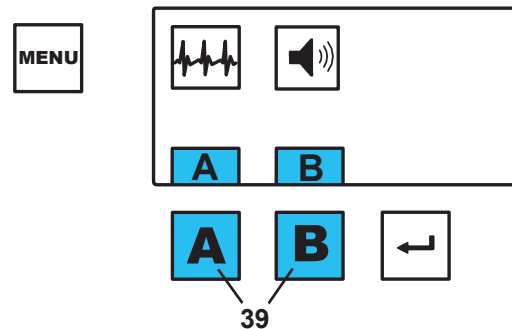

## Adjust suction power

Set the extraction force:

- ⊕ Button (43), increases extraction force.
- ⊖ Button (41), reduces extraction force.

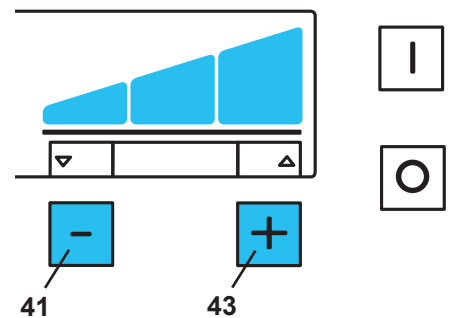## Programming

For the extractor programming instructions, please see enclosed Quick Reference Card and full set of operating instructions on the internet.

## Operation with the SILENT flow sensor

If the extraction unit is operated in combination with a flow rate monitoring unit such as the SILENT flow sensor, the following must be observed:

- ▶ **If the flow rate is less than the defined threshold value for a certain period, this is detected by the SILENT flow sensor, the corresponding indicator begins to flash, and a warning tone is output every 30 seconds.**

**If the flow rate is too low at the suction point being monitored, safe operation is no longer guaranteed and the level of exposure to dust can be harmful to health!**

In this case, the following measures must be taken:

- ⇒ Stop working at all suction points of the affected extraction unit. This applies to the individual suction channels of any multi-workbench extraction unit as well as to all suction points operated via a Y-junction or via extractor switches.
- ⇒ Determine why the flow rate is too low and resolve the issue.

Cause	Measure
Suction level is too low	• Choose a higher suction level
Filter blockage	• Clean the filter.
Cleaning the filter does not result in the expected improvement.	• Replace the fine filter. • For low-dust replacement of the fine filter, observe Chapter 6.3.
Blockage in the suction point (suction mouth)	• Disconnect the suction hose at the suction point and check whether there is a blockage in the suction point; remove where appropriate.
Blockage in the suction line	• Check the suction line at all disconnection points / service openings section by section for blockages.
Leakage in the section of the suction line between the SILENT flow sensor and the extraction unit.	• Check the installation of the SILENT flow sensor in accordance with its instructions.

Determining the cause and measures can be carried out in the order shown.

After every measure, you should switch on the extraction unit again and check whether the minimum flow rate is again exceeded so that correct operation is possible again.

⇒ Work should only resume after the fault has been rectified and the minimum flow rate is exceeded again.

## Cleaning / Maintenance


Wipe the product down with a slightly damp cloth.

### Replacing the Dust Bag

It is essential to change the dust bag when indicated as follows on the product display:


The set dust bag fill level has been reached:

- ◆ The dust drawer symbol blinks.
- ◆ An audible signal will be emitted 3 x.
- ◆ The display blinks “Cod01”.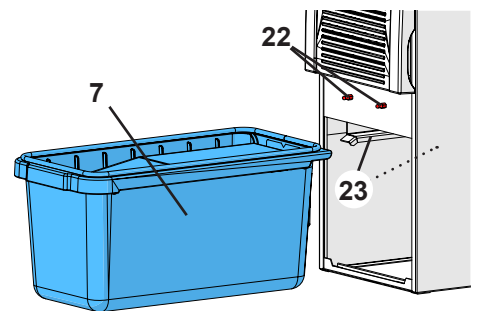


Emptying the dust drawer:

- ⇒ Switch the extractor off.
- ⇒ Pull the front panel forward and off (5, Fig.1).
- ⇒ Pull the dust drawer (7) out towards the front.
- ⇒ Remove the protective film from the sticky tab and seal the dust bag opening adhesively to prevent dust escaping!
- ⇒ Bring the dust bag in the dust drawer to the disposal location.
- ⇒ Dispose of the dust bag properly.
- ⇒ Insert a new dust bag into the dust drawer. Ensure that the new dust bag is properly aligned in the guide grooves (21) and the glue flap faces upwards.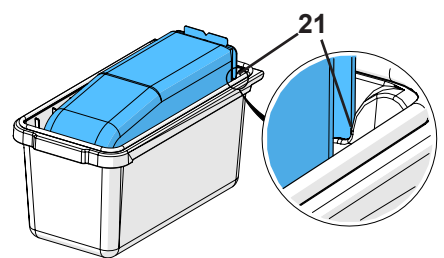

**Use only original Renfert dust bags (see spare parts list).**

- ⇒ Inspect the dust drawer seal for damage and replace it, as required (24).
- ⇒ Insert the dust drawer, making sure it is straight, and push it in as far as it will go. Make sure that the dust drawer is seated correctly on the guide (23).
- ⇒ Install the front panel on the bottom hooks then push it into the upper lock bolts, pressing firmly until it locks in place.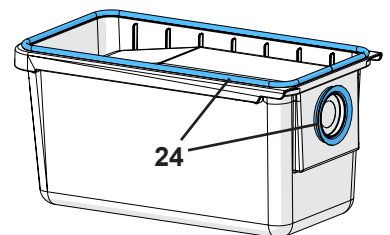


## Replacing the Fine Particle Filter

Examine and replace the fine particle filter (6) on a regular basis, at least once a year. It must always be replaced if the dust drawer symbol (33, Fig. 2) blinks, despite a new dust bag having just been replaced.

There is a risk of increased exposure to dust when replacing the fine filter.

That's why the following measures must be taken:

- ⇒ Keep the plastic bag ready that contains the new filter.
- ⇒ Keep an additional dust extraction device or suction tube from another workbench extraction unit ready.
- ⇒ Wear at least an FFP2 mask for respiratory protection.
- ⇒ Pull the upper front panel (4, Fig. 1) forward and off.
- ⇒ Turn the fine particle filter in an anti-clockwise direction to release it.
- ⇒ Pull the fine particle filter straight forwards.
- ⇒ Place the fine filters immediately in the plastic bag and seal the bag tightly to prevent dust escaping.
- ⇒ Ensure proper disposal.
- ⇒ Install a new fine filter and fully insert it over the holder bar (25).
- ⇒ Turn the fine particle filter in a clockwise direction and hand-tighten it in place.
- ⇒ Inspect the seal on the front panel for damage and replace it, as required.
- ⇒ Snap the front panel back in place.
- ⇒ Use the additional dust extraction device to suction up dust that has been released.

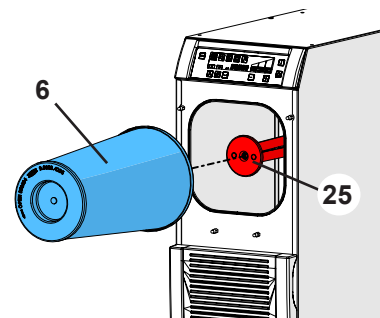## Errorcodes

If an error code is displayed during operation or in diagnostic mode, please consult the full set of operating instructions on the internet for the relevant information on the required course of action.

## Spare parts

You can find components subject to wear and the spare parts on the spare part list in the internet at [www.renfert.com](http://www.renfert.com).

Enter the following item number: 29300050.

The components excluded from the warranty (such as consumables or parts subject to wear and tear) are marked on the spare part list.

Serial number and date of manufacturing are shown on the type plate of the unit.

## Disposal Information

**Within the European Union, units with this symbol should not therefore be disposed of in unsorted domestic waste.**

Please contact your local authorities for more information on proper disposal.

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declare under our sole responsibility, that the product(s)

Name	Part No.
SILENT TS	2921 0050
SILENT TS2	2930 0050
SILENT TC	2935 0000 / 2935 0000 03
SILENT TC2	2936 0000 / 2936 0000 03
SILENT EC2	2937 0000 / 2937 0000 03
SILENT compact	2934 0000 / 2934 0000 03
SILENT compactCAM	2934 2000 / 2934 2000 03 2934 2500 / 2934 2500 03
SILENT powerCAM EC	2939 0000
SILENT powerCAM TC	2938 0000
iVAC silent	36 2934 2000

is (are) in conformity with the relevant regulatory requirements by compliance with the UK designated standards.

## UK legislation

*Electromagnetic Compatibility Regulations 2016*

*Supply of Machinery (Safety) Regulations 2008*

*The Restriction of the Use of Certain Hazardous Substances in Electrical and Electronic Equipment Regulations 2012*

## Designated standards

EN 61326-1:2013

EN ISO12100:2010

EN 61010-1:2010/A1:2019/AC:2019-04

EN IEC 63000:2018

Tilo Burgbacher,  
Engineering Director

Hilzingen, 02.11.2022

The Technical Documentation has been retained by Renfert GmbH

# Technical Data

Nominal Voltage	230 V	120 V
Permissible mains voltage:	220 - 240 V	120 V
Mains frequency:	50 / 60 Hz	60 Hz
Suction turbine power: *)	1400 W	1400 W
max. connecting power: *)	2000 W	360 W
Total connected power: *)	3400 W	1800 W
Mains input fuse:	2 x 15 A (T)	
max. connection pressure:	8 bar [116 psi]	
min. connection pressure:	4.5 bar [65 psi]	
LpA **) (at max. volume flow):	56,9 dB(A)	
Number of suction hoses:	2	
Volume flow, max per suction hose ***):	3300 l/min	
Negative pressure, max. ****):	200 hPa [2.9 psi]	
Fine particle filter:		
Filter surface area, approx.:	0.8 m <sup>2</sup> [1240 sq inch]	
Filter quality:	Class M according to EN60335-2-69	
Fill volume, dust bag, approx.:	7.5 l [2 US gal]	
Dimensions (Width x Height x Depth):	224 x 591 x 584 mm [8.8 x 23.3 x 23 inch]	
Weight (empty), approx.:	27 kg [59.5 lbs]	
Ø Suction fittings:		
internal	45 mm [1.77 inch]	
external	50 mm [1.97 inch]	

- \*) Power consumption at nominal voltage  
 \*\*) Sound pressure levels according to EN ISO 11202  
 \*\*\*) Free-blowing turbine at nominal turbine voltage  
 \*\*\*\*) At nominal turbine voltage

**Hochaktuell und ausführlich auf ...  
Up to date and in detail at ...  
Actualisé et détaillé sous ...  
Aggiornato e dettagliato su ...  
La máxima actualidad y detalle en ...  
Актуально и подробно на ...**

**[www.renfert.com](http://www.renfert.com)**

Renfert GmbH • Untere Gießwiesen 2 • 78247 Hilzingen/Germany  
Tel.: +49 7731 82 08-0 • Fax: +49 7731 82 08-70  
[www.renfert.com](http://www.renfert.com) • [info@renfert.com](mailto:info@renfert.com)

Renfert USA • 3718 Illinois Avenue • St. Charles IL 60174/USA  
Tel.: +1 6307 62 18 03 • Fax: +1 6307 62 97 87  
[www.renfert.com](http://www.renfert.com) • [info@renfertusa.com](mailto:info@renfertusa.com)  
USA: Free call 800 336 7422