# **Vortex compact 2 L**

**No. 2924-4000**

**Operating Instructions  
Instructions de service  
Instrucciones de servicio**

# Index

1	Safety Instructions.....	1-2
	Consignes de sécurité.....	13-14
	Avisos de peligros .....	25-26
2	Description .....	3-4
	Description .....	15-16
	Descripción.....	27-28
3	Assembly / Using for the First Time.....	5-6
	Montage / Mise en service .....	17-18
	Montaje / Puesta en servicio .....	29-30
4	Operation.....	6-7
	Commande.....	18-19
	Manejo.....	30-31
5	Troubleshooting.....	8
	Recherche des dérangements .....	20
	Localización de averías.....	32
6	Cleaning / Maintenance.....	9-10
	Nettoyage / Maintenance .....	21-22
	Limpieza / Mantenimiento .....	33-34
7	Spare Parts .....	11
	Pièces de rechange.....	23
	Piezas de recambio.....	35
8	Guarantee .....	12
	Garantie.....	24
	Garantía .....	36
9	Technical Data.....	12
	Caractéristiques techniques .....	24
	Datos técnicos.....	36
10	Scope of Supply .....	12
	Composition de la livraison .....	24
	Alcance de suministro .....	36

---

# 1. Important Safety Instructions

## For your own safety

Before putting the cleaner

into operation, be sure to read these operating instructions thoroughly and keep them close at hand for future reference.

The cleaner may only be used by persons trained in its use and expressly authorised to use it.

## General

Operation of this cleaner is subject to the applicable national regulations.

In addition to these operating instructions and the binding regulations on accident prevention applicable in the country of use, the generally acknowledged technical rules for safe and correct working must also be observed.

Any method of work endangering health or safety is forbidden!

Use extra care when cleaning on stairs.

### Appliance plug socket

Before connecting any appliance to the appliance plug socket, the cleaner must always be switched off.

If appliances are connected to the appliance plug socket, ensure that these appliances are also switched off when being connected to the appliance plug socket.

### CAUTION!

The operating instructions and the safety precautions contained therein must be observed when using appliances connected to the appliance plug socket of the cleaner.

## Using for the first time

Before using the cleaner for the first time, check that it is in a good and safe condition.

Plugs and connectors of mains leads must be at least splash water-proof. Check the mains connection and mains plug.

Inspect the mains lead at regular intervals for damage or signs of ageing. Use the cleaner only when the mains lead is in a good and safe condition. (Damaged mains leads present a danger of electric shocks!)

Use the cleaner only with an undamaged filter element.

## Fire and explosion hazard

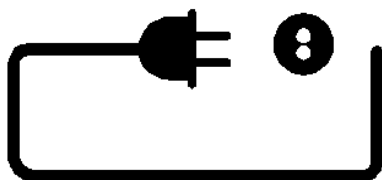
The following materials must not be picked up with the cleaner:

- Hot materials (glowing cigarettes, hot ashes, etc.)
- Inflammable, explosive or aggressive fluids (e.g. petrol, solvents, acids, alkalines, etc.)
- Inflammable, explosive dusts (e.g. magnesium

## During operation

Take care not to damage the mains lead (e.g. by driving over, pinching or dragging). Pull out the mains plug only directly at the plug, i.e. not by pulling at the mains lead.

Before picking up liquids, the filter bag must be removed and the function of the float switch checked. If foaming is observed, stop work immediately and empty the tank.

Connect to a properly grounded outlet only.

## Maintenance and repair

### CAUTION!

Before cleaning and servicing the cleaner, always remove the mains plug.

Carry out only maintenance operations described in these operating instructions.

Use only original spare parts.

Do not make any technical modifications to the cleaner.

### CAUTION!

Your safety could be endangered as a result! For all other maintenance and repair work, please contact Denerica or an authorised specialist workshop!

## Electrical equipment

Check the rated voltage of the cleaner before connecting it to the mains power supply. Ensure that the voltage specified on the rating plate corresponds to the local mains voltage.

When using an extension lead or replacing the mains lead, the lead type specified by the manufacturer must always be used.

We recommend that the power supply to the cleaner be connected to a grounded, GFCI-protected socket.

When using an extension lead, observe the minimum cross-sections of the leads:

---

Cable length	Cross-
feet	section
up to 60	14 AWG
60 to 150	12 AWG

---

Always arrange the current-carrying parts (plug sockets, plugs and connectors) and lay extension leads so that the protection class of the cleaner is maintained.

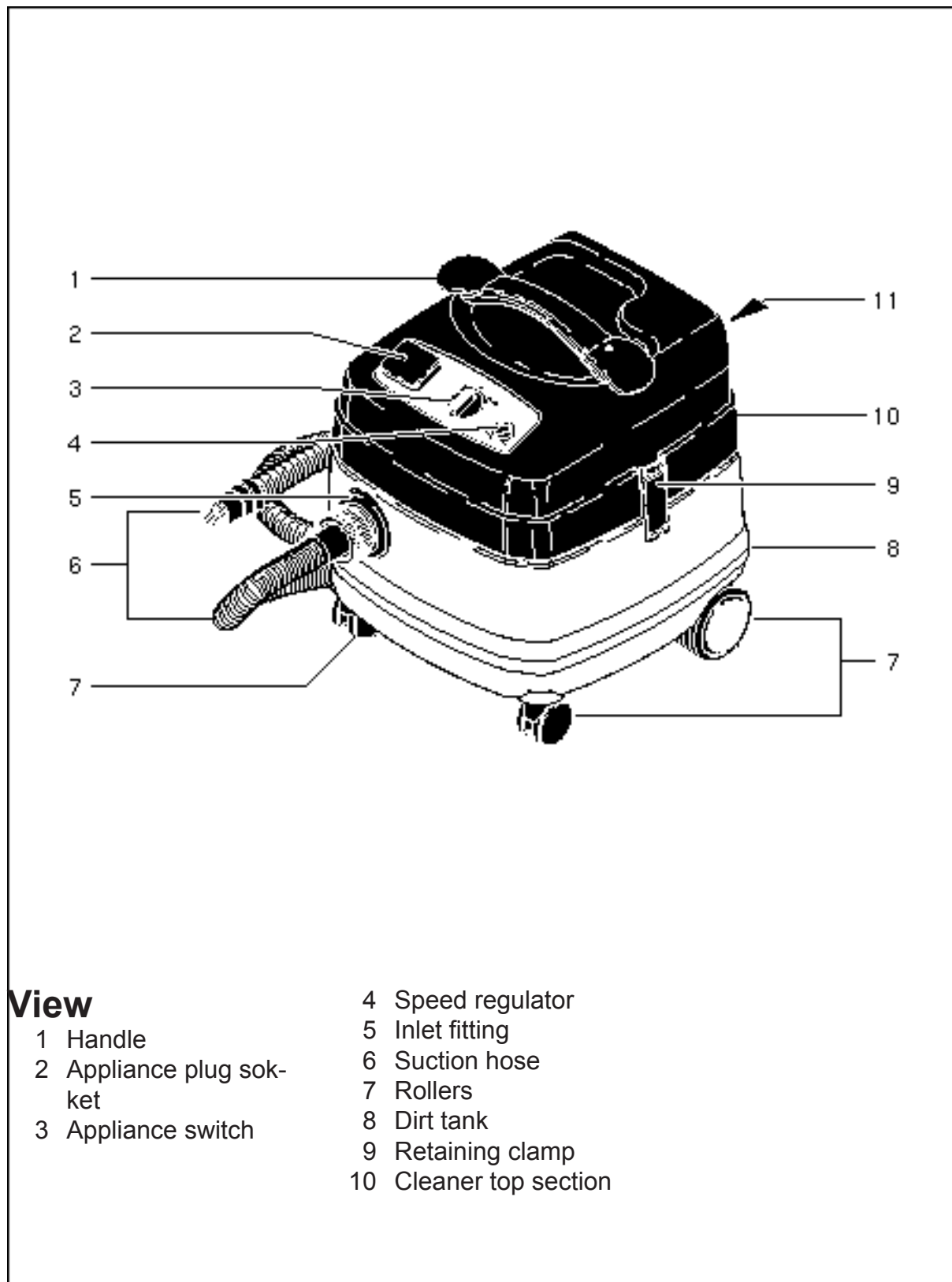
### CAUTION!

Never spray the top section of the cleaner with water: Danger for persons, danger of short circuits.

The latest edition of the IEC Regulations and the

# 2. Description

## 2.1 General

## 2.2

### Intended Use

When used properly, the safety vacuum cleaner described in this manual is suitable for vacuuming up dry, non-flammable dusts and liquids. It is properly suited for the vacuuming up and off of health endangering,

non-flammable dusts with TLV values greater than 1 mg/m<sup>3</sup> in accordance with the dust class 'L' as per IEC 60335-2-69.

## 2.3 Tests and Approvals

Electrotechnically tested

according to test methods/ standards:

IEC 335-1 and  
IEC 335-2-2

The test in accordance with IEC 335-1 and IEC 335-2-2 has showed that the safety requirements with respect to the electrical safety are also satisfied when picking up a water/air mixture.

Protection class     1  
Enclosure             IP X 4

At a filter surface area load of  $\leq 500 \text{ m}^3 \times \text{m}^{-2} \times \text{h}^{-1}$ , the cleaner has a permeability of < 5% and is approved for dust class "L" in accordance with IEC 60335-2-69.

The filter paper of the absolute filter element has been tested by the Berufsgenossenschaftliche Institut für Arbeitssicherheit [Technical Institute for Occupational Safety] (BIA) in accordance with ZH 1/487, paragraph 2 and has a mean degree of penetration reliably < 0.1% measured at an air flow velocity of 0.05 m/s. The two-layer filter material of the bag-type safety filter has been tested by the BIA in accordance

with ZH 1/487 para. 2 and has a mean degree of penetration of reliably < 0.1% measured at an air flow velocity of 0.056 m/s.

Electrical tests must be performed at least once a year and after repair or modification.

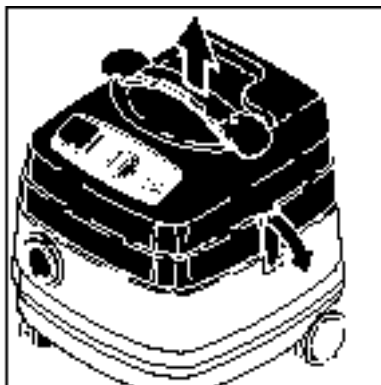
# 3. Assembly / Using for the First Time

## 3.1 Assembling


### the Cleaner

#### CAUTION!


On delivery, the accessories are lying in the tank and must be removed before the cleaner is used for the first time. To remove the accessories, the cleaner top section must be taken off. The mains plug must not yet be inserted into a plug socket. Remove from the plug socket, if necessary.

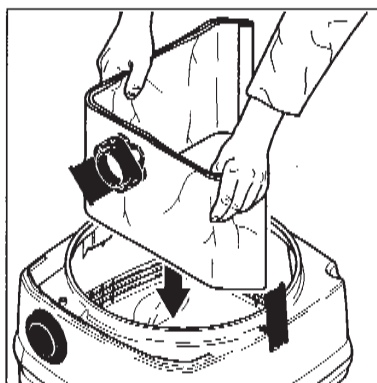
- Open retaining clamps.
- Lift off the cleaner top section.

- Lay a bag-type filter into the tank.

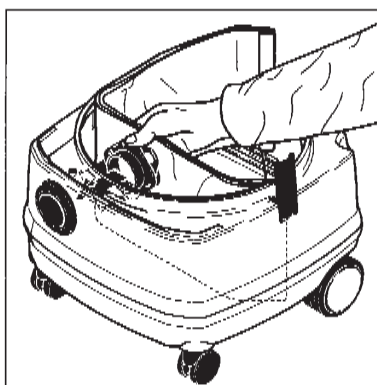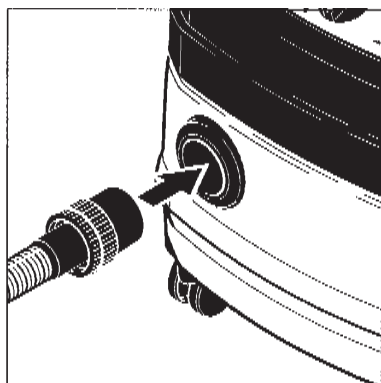- Press the bag-type filter sleeve as far as possible onto the inlet fitting.
- Fit the cleaner top section and close the re-

- Connect the suction hose to the tank.

## 3.2 Electrical Connection

Connecting the Cleaner

properly installed and fused plug socket with ear-

ner to the Power Supply

The operating voltage shown on the rating plate must correspond to the voltage of the mains power supply.

- Ensure that the cleaner is switched off and insert the plug of the mains lead into a pro-

thing contact.

Appliance Plug Socket

- Connect the spray gun to the appliance plug socket on the control panel.

Maximum power consump-

## 4. Operation

Switching ON the Cleaner  
Two operating modes are

possible:

Switch position "1":

The cleaner motor runs continuously.

Switch position "auto":

The cleaner motor starts only when the appliance connected to the appliance plug socket is switched on.

Switching OFF the Clea-

### **CAUTION!**

Before turning the switch to position "auto", ensure that the appliance connected to the appliance plug socket is switched off.

Return the switch to position

After Finishing Work

Picking Up Dry Substances

Before picking up any dry, non-flammable substances, the bag-type filter should always be installed in the tank. It is then easy to dispose of the material picked up. Dry cleaning without bag-type filter is possible for un-hazardous dust ( $MAK > 1 \text{ mg/m}^3$ ) if cloth filter 9.2924.0005 is used. Inflammable substances must not be picked up with the cleaner!  
After picking up liquids,

flammable liquids, the

bag-type filter must always be removed from the tank.  
Inflammable liquids must not be picked up with the cleaner!  
After picking up liquids, the filter element is damp.

The motor speed and thus the suction power can be varied at the speed regulator. This permits a precise adaptation to different substances to be picked up.

“0”.

- Pull the plug of the mains lead out of the plug socket.

the filter element is damp. Damp filter elements can clog more quickly when dry substances are picked up. For this reason, the filter element should be dried or replaced with a dry element before picking up dry substances.  
Picking Up Liquids  
Before picking up non-

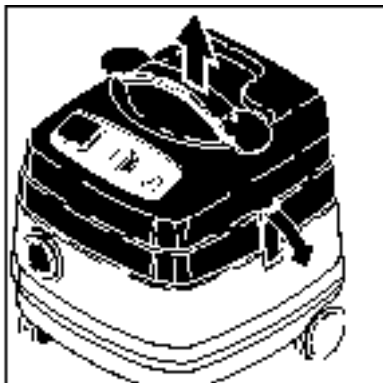
Damp filter elements can clog more quickly when dry substances are picked up. For this reason, the filter element should be dried or replaced with a dry element before picking up dry substances.  
If foaming is observed, stop work immediately and empty the tank.

# 5. Troubleshooting

Problem	Cause	Remedy
≠ Motor does not start	<ul style="list-style-type: none"> <li>&gt; Plug of mains lead not inserted</li> <li>&gt; Fuse of mains plug socket has blown</li> <li>&gt; Overload protection switch has tripped</li> </ul>	<ul style="list-style-type: none"> <li>● Insert main lead plug into plug socket</li> <li>● Reset the fuse</li> <li>● Allow the cleaner to cool down for approx. 5 minutes; if the cleaner does not start again automatically, contact the Service Dept.</li> </ul>
≠ Motor does not run in automatic mode	<ul style="list-style-type: none"> <li>&gt; Electric tool defective or not properly connected</li> </ul>	<ul style="list-style-type: none"> <li>● Check function of electric tool or push in plug firmly</li> </ul>
≠ Reduced suction power	<ul style="list-style-type: none"> <li>&gt; Bag-type filter full</li> <li>&gt; Suction hose clogged</li> <li>&gt; Filter element clogged</li> </ul>	<ul style="list-style-type: none"> <li>● Replace bag-type filter</li> <li>● Clean suction hose</li> <li>● Clean or replace filter element</li> </ul>
≠ No suction power during wet cleaning	<ul style="list-style-type: none"> <li>&gt; Tank full (float switch closing off the suction opening)</li> </ul>	<ul style="list-style-type: none"> <li>● Empty tank</li> </ul>


# 6. Cleaning / Maintenance

Removing the Cleaner Top Section


- Pull the plug of the mains lead out of the plug socket.
- Open the retaining clamps.
- Lift off the cleaner top section.

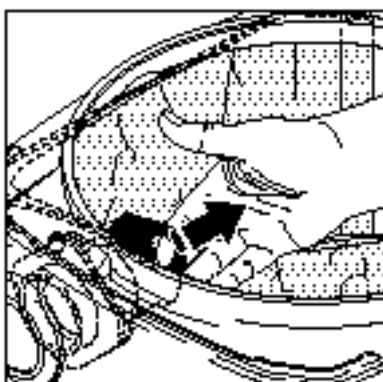
Changing the Bag-Type Filter

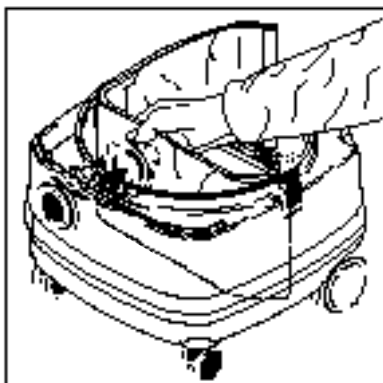- Remove the cleaner top section.
- Close the bag-type filter sleeve with the sleeve.

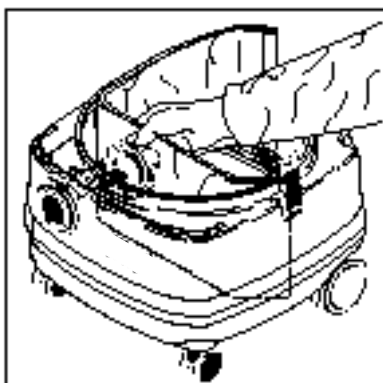
- Pull the bag-type filter sleeve off the inlet fitting.
- Dispose of the used bag-type filter in accordance with the local waste disposal regulations.

- Lay a new bag-type filter into the tank.
- Press the bag-type filter sleeve onto the inlet fitting.
- Fit the cleaner top section and close the retaining clamps.
- Order No. for bag-type filters (5 filters per pack):


## Replacing the Filter Element

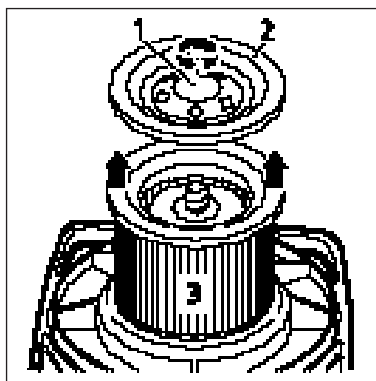- Remove the cleaner top section.
- Place the cleaner top section aside with the filter element facing upwards.
- Unscrew the filter retaining nut (1) and remove together with the retaining washer (2).
- Pull off the filter element (3).
- Push on a new filter element.
- Fit the filter retaining washer and tighten the retaining nut.
- Fit the cleaner top section and close the retaining clamps.
- Dispose of the used filter element in accordance with the local waste disposal regulations.
- Order No. for filter element: Renfert  
No. 9.2924.0002

# 7. Spare Parts

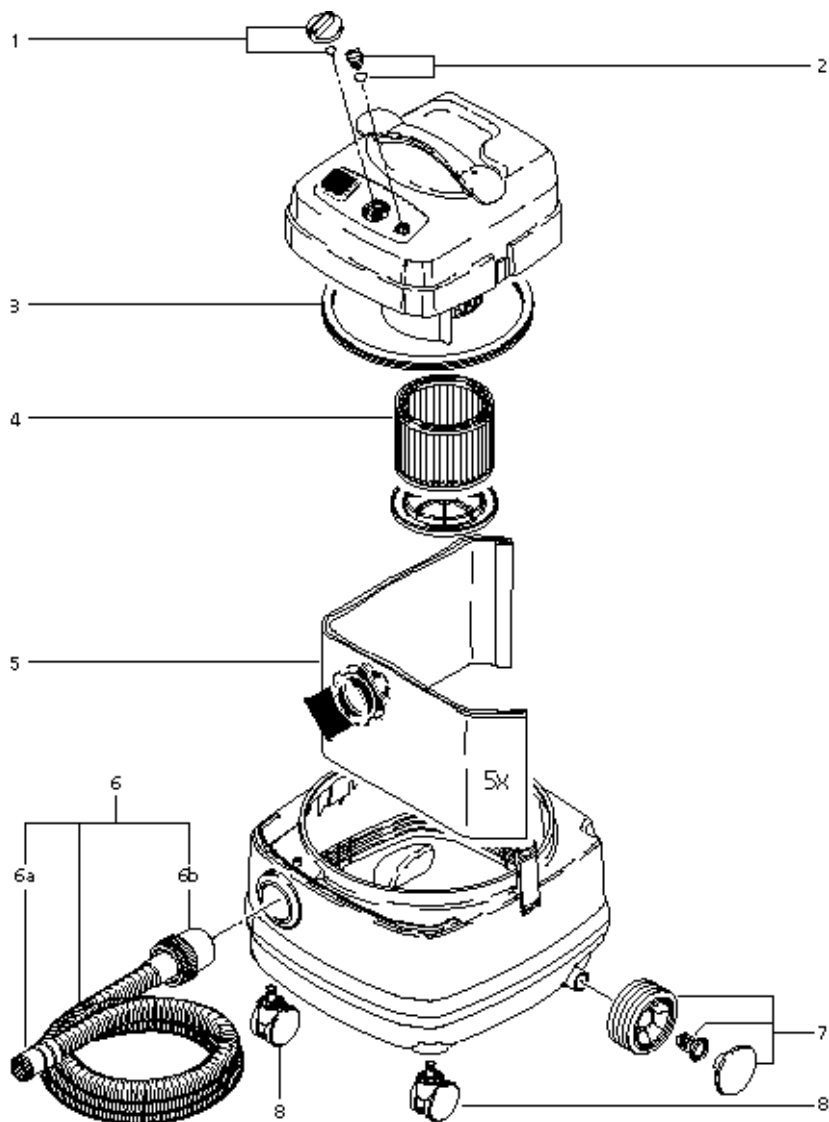Drawing No.: **2924-4000 Vortex compact 2 L, 120 V, 60 Hz**  
 Drawing date: 03.03.01 Status: 03.03.01

ITEM No.	DESIGNATION	ITEM No.	DESIGNATION
1	Complete rotary knob	6	Suction hose assembly, 3 metres
2	Complete rotary knob for potentiometer	6a	Nozzle-side sleeve DN 36
3	Seal ring	6b	Pivot sleeve DN 36
4	Filter element	7	Wheel assembly
5	Bag-type filter	8	Steering casters

When ordering spare parts, always quote the drawing number and drawing date.

## 8. Guarantee

When used for its intended use, Renfert guarantees all parts of the Vortex-Compact 2 L - with the exception of the wear parts (Motor 800 h, Filter material, Hoses)- for a period of 3 years.

### CAUTION!

Use only original Renfert spare parts for all maintenance work.

Installation of parts from other manufacturers or inexperienced attempts at repair will invalidate the guarantee and Renfert will accept no liability for any consequential damage.

Repairs may only be car-

ried out by qualified specialist personnel.

## 9. Technical Data

Voltage	Volt	120
Mains frequency	Hz	60
Power consumption max. at receptacle	Watt	300
Total of unit	Watt	1500
Air flow (max.)	cubic feet per minute	130
Static water lift	inches	90
Filter surface area	square feet	3,5
Sound level	dB(A)	62
Capacity	gallons	10
Power cord length	feet	22
Dimensions:		
Length	inches	18
Width	inches	17
Height	inches	20
Net weight		
pounds		26
Protection class		I
Protection type (splash water-protected)		IP X4

## 10. Scope of Supply

- 1 Renfert *Vortex compact 2 L*
- 1 Suction hose assembly
- 1 Bag-type filter
- 1 Operating manual

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