making work easy

- FAQ - Deutsch
- FAQ - English
- FAQ - Français
- FAQ - Italiano
- FAQ - Español
- FAQ - Português
- FAQ - Türkçe
- FAQ - Русский
- FAQ - Polski
- FAQ - Český
- FAQ - 中文
- FAQ - 日本語
- FAQ - 한국어
- FAQ - اللغة العربية
- FAQ - Українська

AUTO spin

FAQ - frequently asked questions

Made in Germany

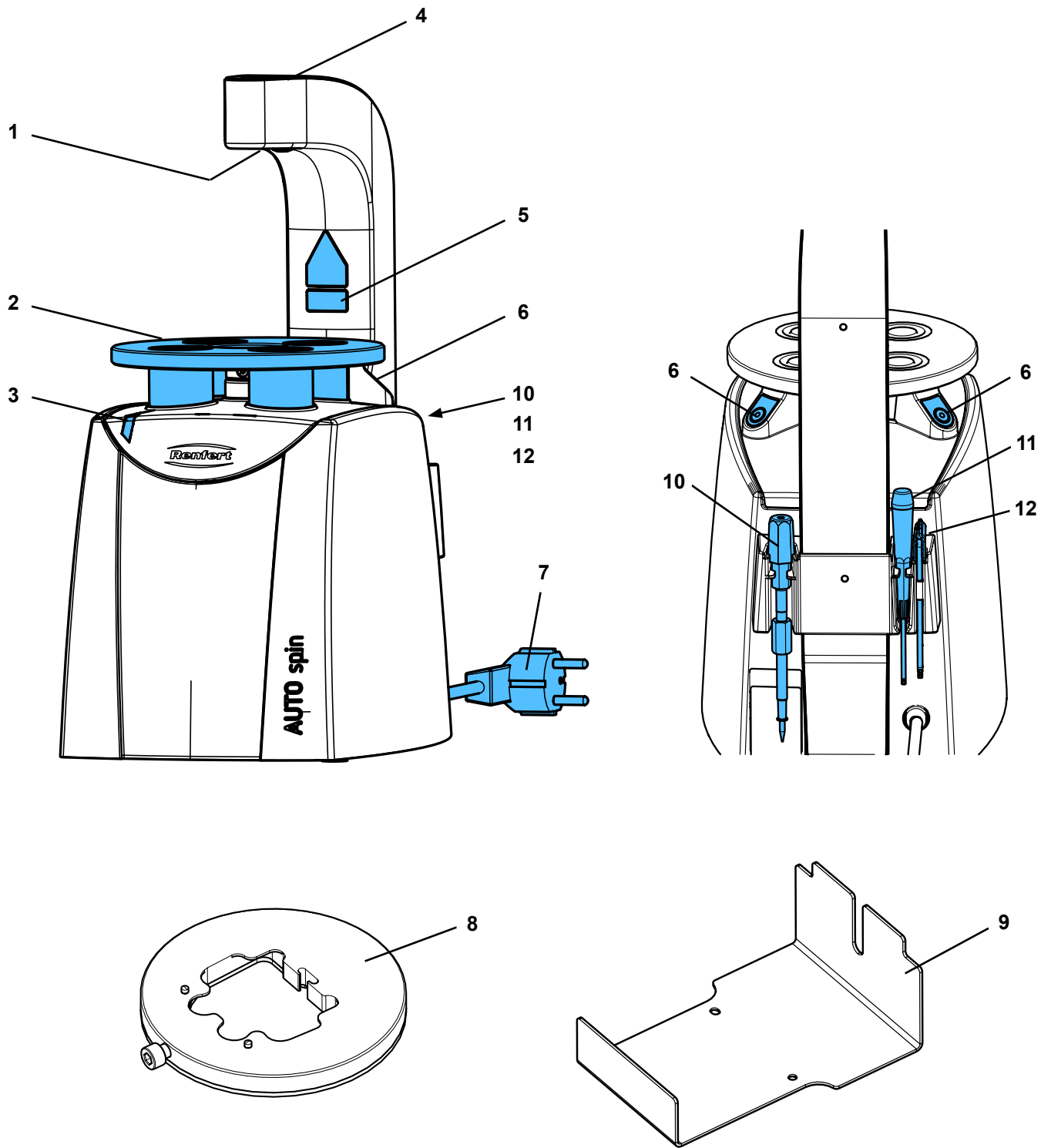
Fig. 1

FAQ - Deutsch

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FAQ - English

i For information on operating elements, see [Fig. 1](#) on page 2 of this document.

i For chapter indications, refer to the complete instruction manual on the internet.

Error	Cause	Solution
The operating indicator (3. Fig. 1) flashes.	<ul style="list-style-type: none"> • The drill motor is not in the starting position. 	<ul style="list-style-type: none"> • Press both operating keys until the drill motor has reached its starting position (see Chapter 4.5).
The drilling process has been interrupted and the operating indicator (3. Fig. 1) flashes.	<ul style="list-style-type: none"> • Drill not adequately tightened. • Drill broken. 	<ul style="list-style-type: none"> • Press both operating keys until the drill motor has reached its starting position (see Chapter 4.5). • Retighten screw of fastening (see Chapter 6.2). • Change the drill.
Plate holder moves during drilling, lifts up.	<ul style="list-style-type: none"> • Insufficient magnetic force due to plastic chips between the magnet and the plate holder. 	<ul style="list-style-type: none"> • Remove plastic chips on the drilling table and under the plate holder.
Drilling depth changes.	<ul style="list-style-type: none"> • Restraint of the stop screw is no longer sufficient. 	<ul style="list-style-type: none"> • Use new stop screw.
After a drill change, the drilling depth deviates significantly from the previous depth, even though a drill of the same length is used.	<ul style="list-style-type: none"> • Restraint of the stop screw is no longer sufficient. 	<ul style="list-style-type: none"> • Use new stop screw.
Drill cannot be removed by hand.	<ul style="list-style-type: none"> • Drill was not sufficiently tightened and seized up in the drill chuck. 	<ul style="list-style-type: none"> • Use a pair of pliers to remove the drill. Do not reinsert drill, discard drill.

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