making work easy

- de
- en
- fr
- it
- es
- pt
- tr
- ru
- zh
- ja
- ko
- ar

BASIC eco

Quick Start Guide

Made in Germany

21-6723 25072023

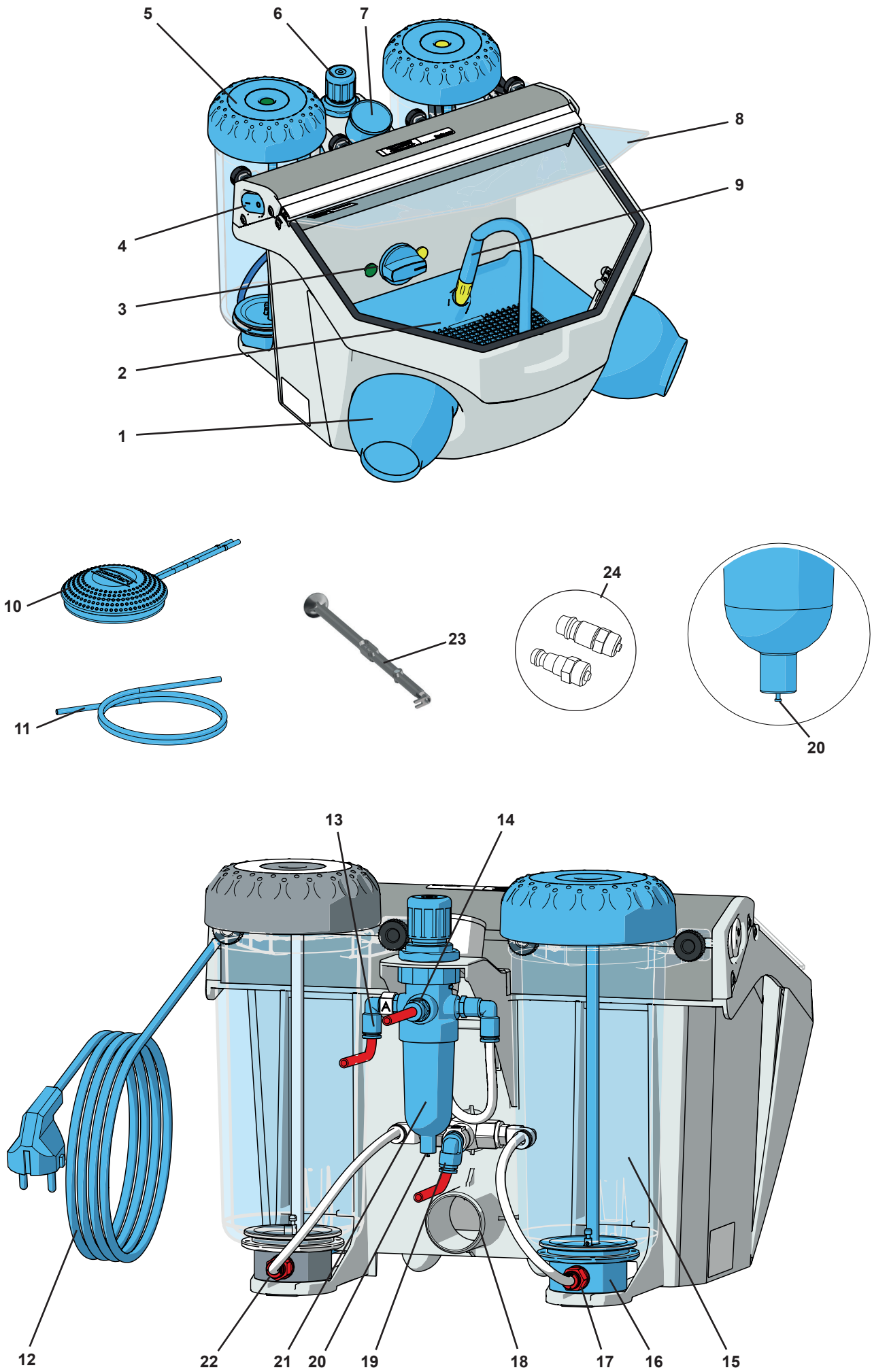
Fig. 1

Einleitung

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Introduction

This Quick Start Guide contains instructions on the operation of the product and relevant safety information. Detailed information on the product can be found in the operating instructions which are available for download on our website at www.renfert.com.

Employed symbols

In these instructions or on the unit itself you will find symbols which have the following meaning:

Danger

This indicates an imminent risk of injury. Please observe the accompanying documents!

Electrical hazard

There is a risk of electrical shock.

Attention

Failure to observe the associated information can result in damage to the unit.

Note

The operating instructions contain useful information to make handling easier.

Attention!

Wear eye protection.

The unit complies with the relevant EC guidelines.

This product complies with the relevant UK legislation.

See UKCA Declaration of Conformity in the Internet under www.renfert.com.

The unit is subject to the EC guidelines 2002/96/EG (WEEE Directive).

Other symbols are explained as they occur.

Safety

Intended use

The Basic sandblaster is intended for commercial use in dental laboratories in accordance with the instructions for use. The unit is intended for use in the following tasks:

- Removal of residual investment material and of oxides on cast components
- Surface treatment
- Devestment of press ceramics
- Sandblasting of ceramic occlusal surfaces

Meaning of these instructions

These instructions for use act as a guide for safe usage of the unit.

If the instructions for use are not observed, accidents may lead to life-threatening electric shock, injury, or damage.

- ⇒ Only use the unit in accordance with these instructions for use.
- ⇒ Keep the instructions for use ready with the unit.
- ⇒ Provide the instructions for use to all subsequent users of the unit.

Requirements that the operator must meet

- ⇒ The product may only be used by persons aged 14 or over who are familiar with the procedures and rules in a dental laboratory.

Safe condition of the unit

Faulty or broken components, or those that are not sealed as required, may cause injuries.

- ⇒ Check the power cord, housing, and other components such as connection cables, tubes, and the key pad for damage, for example such as kinks, cracks, porosity, and signs of aging.
- ⇒ Defective equipment must be taken out of service immediately. Disconnect the power plug. Send the unit in for repair or dispose of it safely. Until this happens, secure the unit to prevent it from being switched on again so that it is not used inadvertently.
- ⇒ Do not expose the unit to mechanical impact. Do not drop it.

Risks as a result of modifications and repairs

Incorrect modifications and repairs to the unit may cause accidents, fire, and electric shock.

⇒ Only have the unit repaired by qualified specialist dealers or return it.

Risk as a result of electric shock and fire

Risk of electric shock and of fire as a result of excessive operating voltage or if there is no protective conductor.

⇒ Ensure that the power plug is easily accessible.

⇒ Only operate the unit using a power cord equipped with a plug appropriate to the local power supply.
Only allow qualified electricians to modify power cords.

⇒ Only start the unit if the information on the nameplate matches the power supply.

⇒ Only connect the unit to electrical sockets that are connected to the protective conductor system.

Risk of injury to the eyes or skin

Injury may occur to the eyes or skin as a result of high pressure or of components fracturing that are subject to high pressure.

⇒ Always wear appropriate protective goggles when carrying out work on the unit.

⇒ Never direct the blasting material towards the eyes or unprotected areas of skin.

⇒ Never perform sandblasting with the viewing panel open.

Risk to health as a result of dust

Dust emissions may result in damage to health.

⇒ Operate the unit with a suitable extraction unit. The extraction unit must be adapted to the dust that occurs.

Product description

Components and functional elements

See Fig. 1 (on inside cover)

- | | |
|--|--|
| 1 Hand sleeves | 13 Foot switch connection (A)
(push-in connection) |
| 2 Base grid | 14 Air connection (push-in connection) |
| 3 Selector switch (only on the 2 tanks version) | 15 Sandblasting tank |
| 4 ON / OFF switch (lighting) | 16 Mixing chamber |
| 5 Tank lid | 17 Filter cartridge (push-in connection) |
| 6 Pressure regulator | 18 Suction tube |
| 7 Pressure gauge | 19 Foot switch connection (B)
(push-in connection) |
| 8 Protection screen | 20 Water trap valve |
| 9 Hand piece | 21 Water trap |
| 10 Foot switch | 22 Foot switch connection (B) (push-in connection)
(only on the 1 tanks version) |
| 11 Compressed air tube | 23 Dosing nipple tool |
| 12 Power cord | 24 Connection set |

Standard delivery

- 1 Sandblasting unit *Basic eco*
- 1 Foot switch
- 1 Connection set
- 1 Dosing nipple tool
- 1 Quick Start Guide

Delivery format

- 1 tank version: 2949 1XXX, 2949 3XXX, 2949 5050
- 2 tank version: 2949 2XXX, 2949 4XXX, 2949 5025

Installation

Risk of injury!

Loose tubes may hit objects around them.

Tubes must be connected first to the unit.

Only press the foot switch after all tubes have been connected.

en

Unpacking

⇒ Check to ensure all delivery parts are complete (compare “Included in Delivery”).

Setting up

⇒ Place the unit on a level and stable base.

Connect foot switch

When inserting the tubes you will notice a slight resistance which must be overcome.

2 Tank version:

⇒ Push the tube marked „A“ as far as it will go into the tube connection point „A“ (13).

⇒ Push the tube marked „B“ as far as it will go into the tube connection point „B“ (19).

1 Tank version:

⇒ Push the tube marked „A“ as far as it will go into the tube connection point „A“ (13).

⇒ Push the tube marked „B“ as far as it will go into the tube connection point „B“ (22) (on sandblasting tank).

Compressed air connection

⇒ Push the compressed air tube (11) onto the tube connection point (14) as far as it will go. In doing so there will be a slight resistance to overcome.

⇒ Select the relevant hose connection piece from the enclosed attachment set and fit it onto the end of the tube.

⇒ Attach to the compressed air network.

Electrical connection

⇒ Insert the power plug into the building installed wall socket.

Connect to extraction unit

⇒ Connect the suction tube from the extraction unit to the suction pipe (18).

For suction devices with automatic activation:

⇒ connect the mains plug from the sandblasting unit into the user socket on the suction device.

Filling the sandblasting tanks

Attention: Never press the foot switch during filling.

⇒ Add abrasive up to the maximum fill level (the point where the tank jacket thickens just below the thread).

⇒ Reinstall the tank lid (5).

Authorised sandblasting material

- Cobra (Renfert GmbH)
- Rolloblast (Renfert GmbH)
- ROCATEC (3M ESPE)

Sandblasting materials from other manufacturers may be used providing the grain size, grain shape and degree of purity comply with the above mentioned materials.

Operation

⇒ Switch the illumination on (4).

⇒ Select the blasting tank at the selection switch (3) (only on the 2 tanks version).

⇒ Take the handpiece out of the holder of the relevant sandblasting tank (see color markings).

⇒ Activate the foot switch (10).

Select blasting pressure

- ⇒ Take the handpiece out of the holder of the relevant sandblasting tank (see color markings).
- ⇒ Activate the foot switch.
- ⇒ Set the sandblasting pressure at the pressure regulator (6) (see „Technical Data“).
 - ◆ The amount of pressure will be shown on the pressure gauge (7).

Cleaning / Maintenance

Sandblasting tank

Check the sandblasting tank or tank lid regularly, at least 1 x a year, by visually inspecting for damage or changes to the material (e.g., cracks ...).

If in doubt, replace!

The sandblasting tank and tank lid are subject to a natural aging process depending on ambient conditions and must be replaced as standard after every 15 years of operation.

Note the year of manufacture on the round stamp engraved on the sandblasting tank and in the tank lid; Our customer service team are happy to advise you.

To clean:

DO NOT use solvent-based cleaning agents or disinfectants.

Solvents and tensides can result in microcracking in the plastic (danger of explosion!).

⇒ Only clean the sandblasting tank and tank lid by rubbing with a dry cloth.

Cleaning the housing

⇒ **DO NOT** use solvent-containing cleaning agents or disinfectants.

⇒ **DO NOT** use aggressive or abrasive cleaning agents.

⇒ Wipe the housing and aluminium frame gently with a slightly damp cloth, do not apply pressure.

Cleaning the Interior

- ▶ **DO NOT** use solvent-containing cleaning agents or disinfectants. (use e.g. soapy water).

⇒ Remove bottom grate and the blast protection mat.

⇒ Vacuum the blasting chamber.

⇒ Check the unit regularly for damage underneath the bottom grate / blast protection mat.

⇒ Exchange the damaged bottom grate / blast protection mat.

Condensation

⇒ Inspect the water separator (21).

⇒ Drain any water off through the valve (20).

Spare parts

Only use accessories and spare parts that have been supplied or approved by Renfert GmbH.

Using other accessories or other spare parts can lead to inadvertent accidents and damage.

You can find components subject to wear and the spare parts on the spare part list in the internet at www.renfert.com/p918.

Obligations regarding information

- ▶ Information on REACH and SVHC is available on our website www.renfert.com, in the Support area.

Disposal instructions

Within the European Union units with this symbol should not therefore be disposed of in unsorted domestic waste.

For more information regarding proper disposal please apply at your local authorities.

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declare under our sole responsibility, that the product(s)

Name	Part No.
Vario basic	2960 0005
Basic quattro IS	2959 0000 / 2959 0000 03
Basic quattro	2958 0000 / 2958 0000 03
Basic master	2948 2000 / 2948 2000 03 2948 2025 / 2948 2025 03 2948 2250 / 2948 2250 03
Basic classic	2947 1050 / 2947 1050 03 2947 1250 / 2947 1250 03 2947 2000 2947 2025 / 2947 2025 03 2947 2250
Basic eco	2949 1050 / 2949 1050 03 2949 1250 / 2949 1250 03 2949 2025 / 2949 2025 03
Basic mobil	2914 3050 2914 3250
Dustex master plus	2626 0105 / 2626 0105 03

is (are) in conformity with the relevant regulatory requirements by compliance with the UK designated standards.

UK legislation

Electromagnetic Compatibility Regulations 2016

Electrical Equipment (Safety) Regulations 2016

The Restriction of the Use of Certain Hazardous Substances in Electrical and Electronic Equipment Regulations 2012

Designated standards

EN 61326-1:2013

EN 61010-1:2010/A1:2019/AC:2019-04

EN IEC 63000:2018

Tilo Burgbacher,
Engineering Director

Hilzingen, 02.11.2022

The Technical Documentation has been retained by Renfert GmbH

Technical Data

Nominal Voltage	230 V	120 V	100 V
Permissible mains voltage:	220 - 240 V	120 V	100 V
Mains frequency:	50 / 60 Hz		
Power consumption:	9 W		
Maximum connection pressure:	6 - 8 bar [87- 116 psi]		
Working pressure:	1 - 6 bar [14.5 - 87 psi]		
Dimensions (Width x Height x Depth):	350 x 275 x 400 mm [13.8 x 10.8 x 15.7 inch]		
Weight, approx.: 1 tank version: 2 tank version:	4.4 kg [9.7 lbs] 5.5 kg [12.1 lbs]		
Ø Compressed air hose: interior: exterior:	4 mm [157 mil] 6 mm [236 mil]		
Ø Size of pipe union for external extraction unit: interior: exterior:	35 mm [1.38 inch] 40 mm [1.57 inch]		
Blasting chamber volume:	10 l [2.64 US gal]		
Tank capacity, each:	1000 ml [34 fl oz]		
This product contains a light source of energy efficiency class:	G		
Ambient conditions for safe operation			
Ambient conditions:	Indoors		
Installation height, max.:	2,000 m [6,500 ft] above sea level		
Ambient temperature:	5 - 40 °C [41 - 104 °F]		
Relative humidity:	Max. 80 % at 31 °C [87.8 °F], decreasing linearly to 50 % at 40 °C [104 °F] *)		
Voltage fluctuations in the mains power supply from the nominal value, max.:	10 %		
Contamination level:	2		
Over-voltage category:	II		
Ambient conditions for storage and transport			
Ambient temperature:	- 20 – + 60 °C [-4 – + 140 °F]		
Maximum relative humidity:	80 %.		

*) Between 5 - 30 °C [41 - 86 °F] the device can be operated at a relative humidity of up to 80 %.
At temperatures between 31 - 40 °C [87.8 - 104 °F] the humidity must decrease proportionally in order to ensure operational readiness (e.g. at 35 °C [95 °F] = 65 % humidity, at 40 °C [104 °F] = 50 % humidity). The device may not be operated at temperatures above 40 °C [104 °F].

**Hochaktuell und ausführlich auf ...
Up to date and in detail at ...
Actualisé et détaillé sous ...
Aggiornato e dettagliato su ...
La máxima actualidad y detalle en ...
Актуально и подробно на ...**

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Renfert GmbH • Untere Gießwiesen 2 • 78247 Hilzingen/Germany