lay:art Anmischplatte crystal

No. 1043 0000

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Instruction manual • Mode d'emploi

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ENGLISH

Instruction Manual

Congratulations, and thank you for choosing the high quality and state-of-the-art porcelain mixing palette system „lay:art“ by Renfert.

EN

1. Description

The system consists of a high quality plastic case with a lid and a glass mixing palette.

1.1 Components and functional elements

- 1 Base
- 2 Lid
- 3 Mixing palette
- 4 Labelling foil
- 5 4 x Case feet
- 6 4 x Rest plugs (tall)
- 7 Non-permanent marker
- 8 Tool compartment

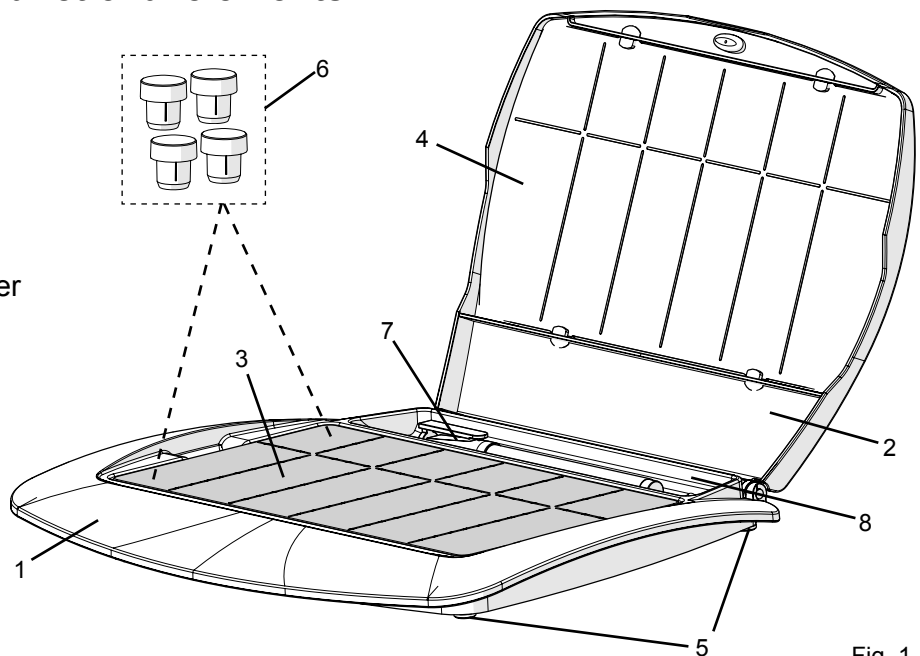
Fig. 1

2. Functional description of the case

When the lid is opened in a 90° angle it can be easily removed from the case base.

Gently press the onto the base (1) and pull the lid (2) upwards.

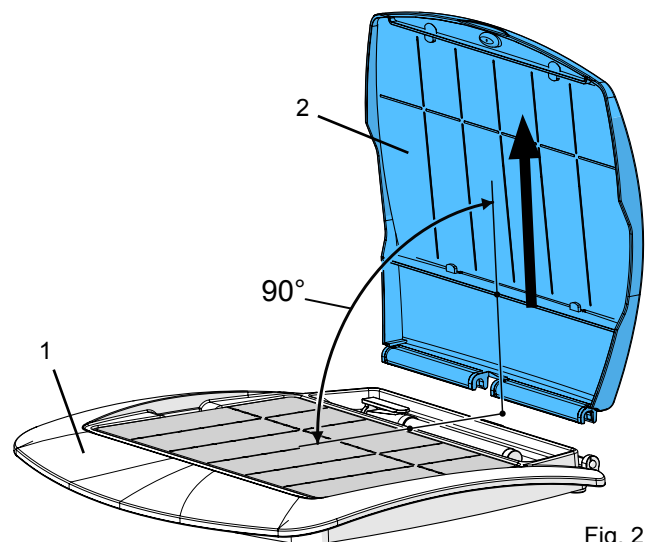
Fig. 2

At an angle of 115° there is a lid locking position, which will hold the lid open in a stable position.

Caution:

If this position is overstretched, the hinges could become damaged.

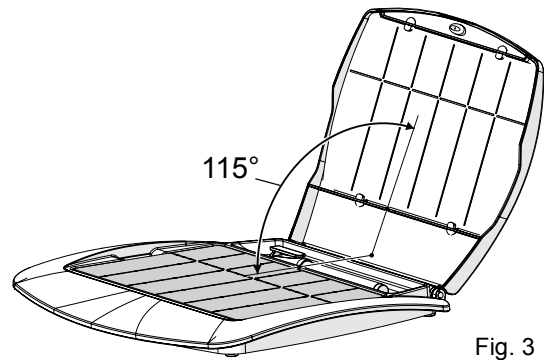
Fig. 3

3. Labelling foil

The non-permanent marker pen provided (7) can be used to write on the labelling foil (4).

The foil can be removed from the lid in order to exchange or inscribe the label.

Use the non-reflective side of the foil to write on.

When replacing the foil, clip behind the 4 mounting points which can be found on the lid. Firstly, bend the retaining flaps towards the matt surface (a).

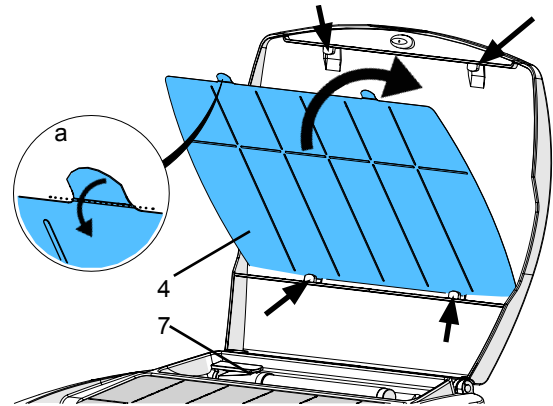
Fig. 4

4. Preparation for using the glass mixing palette

Lay the palette into the case base with the unprinted side towards the top.

5. Care instructions

In order to maintain the consistent quality of the mixing palette, it is essential to clean on a regular basis.

Take the glass mixing palette out of the case base and brush clean under running water or clean carefully using a steam cleaner.

If used inappropriately, the glass could break and cause a risk of injury.

Clean the case using conventional detergent and then rinse thoroughly under running water.

Do not clean using sharp edged, abrasive or solvent based materials. Please only use soft brushes or cleaning cloths.

When using Tricolor stains it may happen that vaporized stains fluid etches the plastic lid. Do not clean the plastic lid with monomers or with liquids containing solvents!

6. Spare parts

1043 0100	Labelling foil crystal, 3 pcs.
9 1044 9991	Housing lay:art big
9 0003 0021	Rest plugs tall lay:art, 4 pcs.
9 0003 0038	Case feet, 4 pcs.

7. REACH

Information on REACH and SVHC is available on our website www.renfert.com, in the Support area.

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Renfert GmbH • Industriegebiet • 78247 Hilzingen/Germany