making work easy

- de
- en
- fr
- it
- es

- tr
- ru
- pl
- zh
- ja
- ko
- ar

# EASY view UV-Filter - Color Calibration

Made in Germany

21-6162\_12072023

Renfert

making work easy

# EASY view UV-Filter - Farbkalibrierung

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```
- color cal. L/R + UV Filter
left = ok, right = cancel
place template "white"
processing ... done
place template "blue"
```

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```
- color cal. L/R + UV Filter
left = ok, right = cancel
place template "white"
processing ... done
place template "blue"
processing ... done
place template "green"
```

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making work easy

# EASY view UV Filters - shade Calibration


**When using the UV filter, if the shade shown on the monitor deviates significantly from the original shade of the object, shade calibration must be performed for the UV filter.**

If you do not yet have the shade calibration card required for shade calibration, it is available to order as a spare part.

900216785 Calibration card EV+3D

## Preparation

- ⇒ Attach the UV filter.
- ⇒ Place the shade calibration card in the EASY view light beam.
- ⇒ Set the distance so that a sharp image is created.
- ⇒ Switch off the head unit using the switch (1).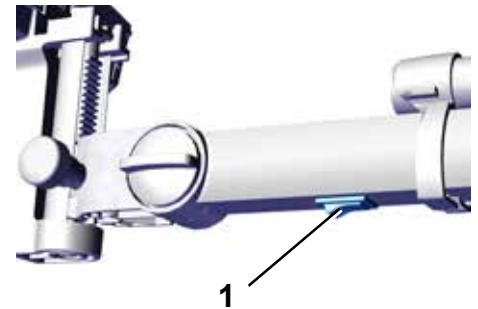


## Activating the service menu


The following procedure can take up to 20 seconds:

- ⇒ While holding down the left and right mouse button, switch on the head unit again. In doing so, keep holding the left and right mouse button down and, at the same time, move the mouse until the service menu appears at the bottom right edge of the monitor.

### EASY view 3D

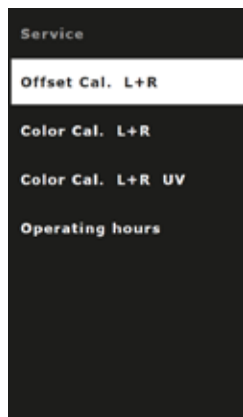### EASY view +
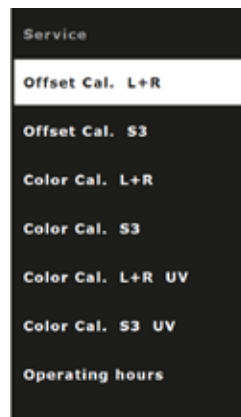
## Selecting shade calibration of the UV filter

- ⇒ Using the mouse wheel, select the menu item for shade calibration of the UV filter.

EASY view 3D	EASY view + 3D	
	Small field of view, 15x and 20x magnification	Large field of view, 4x magnification
Color Cal. L+R UV	Color Cal. L+R UV	Color Cal. S3 UV

**IMPORTANT:**  
Only change the shade calibration of the UV filter.

## Performing shade calibration of the UV filter

The shade calibration procedure is mostly automatic.

You will be prompted to place a certain shade from the shade card in the field of view of the EASY view and to confirm this using the left mouse button in each case.

The following applies for the steps below:

Left mouse button: Confirmation / next step


Right mouse button: Cancellation, shade calibration is canceled and no changes are made.

The shade calibration procedure is the same in all three cases:


- EASY view 3D
- EASY view +, 15x and 20x magnification
- EASY view +, 4x magnification

With EASY view +, the monitor also shows a square inside which the respective shade box must be positioned.


- ⇒ Start shade calibration using the left mouse button.
- ⇒ Place the white box on the shade card in the field of view.
- ⇒ Click the left mouse button to confirm.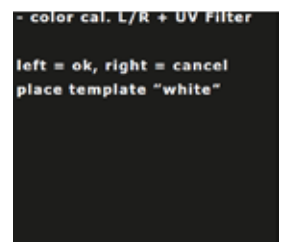

- ◆ Calibration is performed for the color white.
- ◆ You are prompted to place the blue box in the field of view.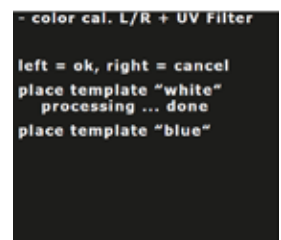

- ⇒ Place the blue box on the shade card in the field of view.
- ⇒ Click the left mouse button to confirm.
  - ◆ Calibration is performed for the color blue.
  - ◆ You are prompted to place the green box in the field of view.

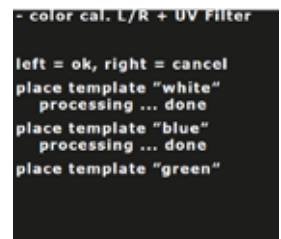- ⇒ Place the required shades step by step in the field of view and confirm in each case using the left mouse button.

- ◆ Once all shades have been calibrated, you are asked whether the new calibration information should be saved.

⇒ Left mouse button:

- ◆ Calibration information is saved.

```
place template "green"  
processing ... done  
place template "red"  
processing ... done  
place template "yellow"  
processing ... done  
place template "magenta"  
processing ... done  
place template "cyan"  
processing ... done  
save calibration data?  
saving data ... done  
Please restart the system !
```

or

⇒ Right mouse button:

- ◆ Discard the new information, the calibration information is not changed.

```
place template "green"  
processing ... done  
place template "red"  
processing ... done  
place template "yellow"  
processing ... done  
place template "magenta"  
processing ... done  
place template "cyan"  
processing ... done  
save calibration data?  
Operation canceled  
Please restart the system !
```

⇒ Following the message "Please restart the system !", switch the head unit off and on again using the switch (1).

- ◆ Even when the UV filter is attached, the shades of the object will be correctly displayed on the monitor.

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Renfert GmbH • Industriegebiet • 78247 Hilzingen/Germany