making work easy

- de
- en
- fr
- it
- es
- pt
- tr
- ru
- zh
- ja
- ko
- ar

EASY view+ 3D

Quick Start Guide

Made in Germany

20-5084 19072023

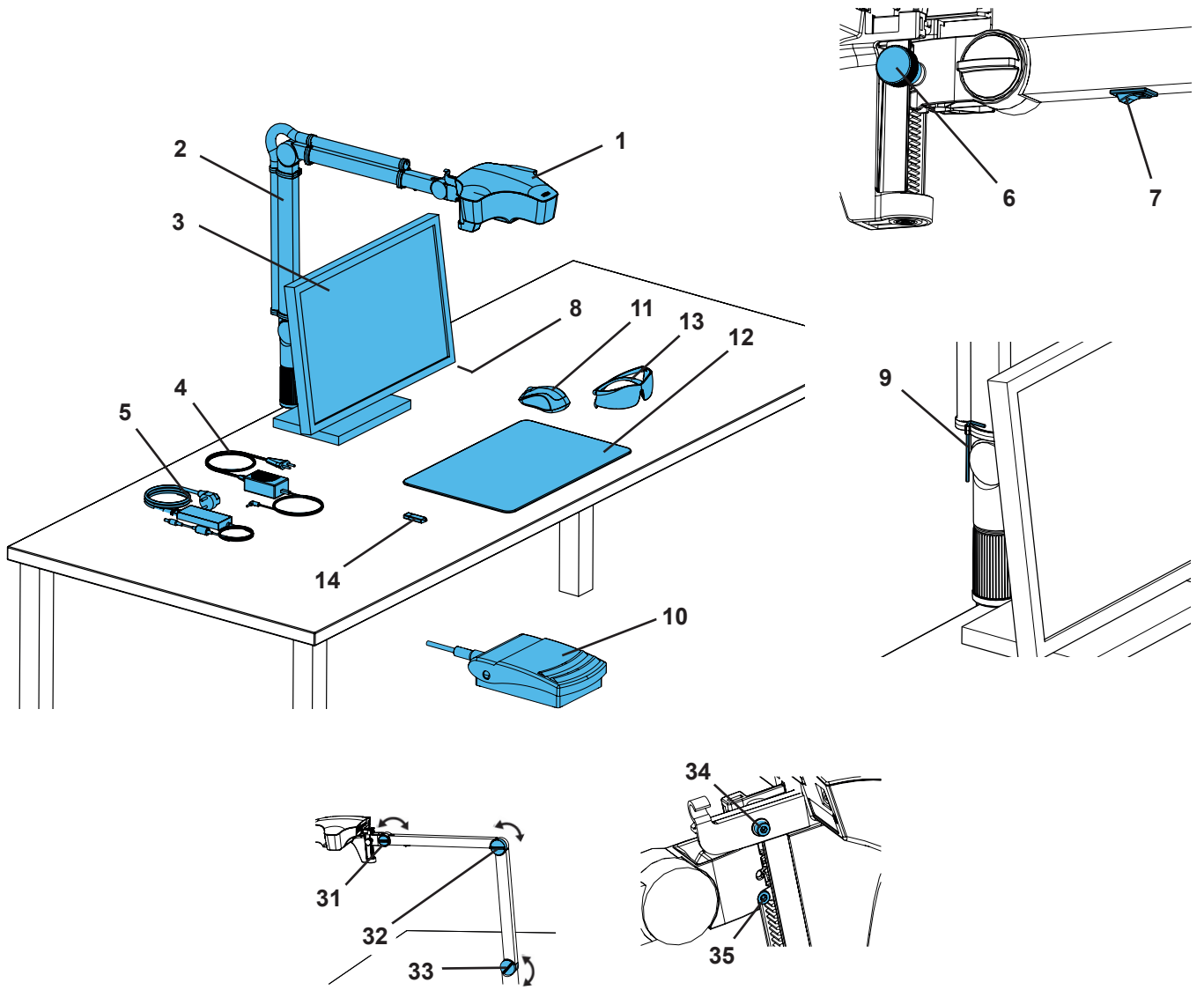
Fig. 1

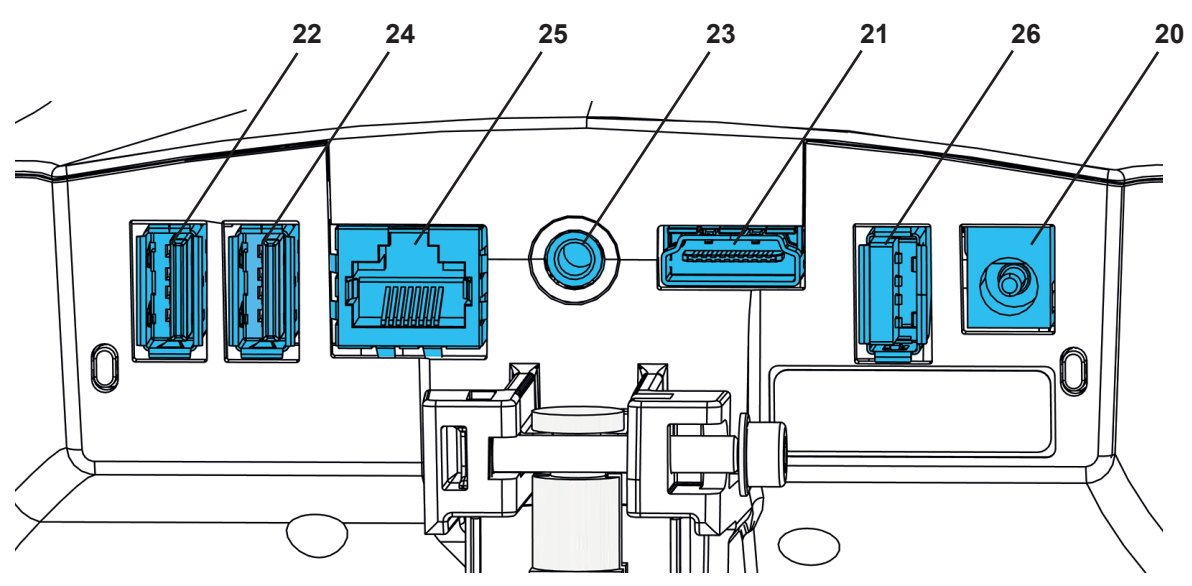
Fig. 2

Einleitung

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Introduction

This Quick Start Guide contains instructions on the operation of the product and relevant safety information.

For detailed information about the device, see the instructions for use which you can download from our website at www.renfert.com/p918.

Enter the following item number: 2400 3000.

After initial start-up, check whether a newer software version is available for the EASY view+.

The version number of the software installed on your EASY view+ is displayed on the bottom right of the screen after switching on (see Chap. 5.1).

Software updates are available on our website at www.renfert.com/p918.

Enter the following item number: 2400 3000.

These instructions describe the range of functions of EASY view + up to and including version V1.021.

Symbols used

This manual contains symbols also found on the appliance itself with the following meaning:

Danger

Immediate risk of injury. Consult accompanying documents!

Electrical current

This indicates a risk of injury due to electrical current.

Caution

Risk of damage to the unit if the instruction is not followed.

Notice

A general notice that improves and eases use.

The appliance complies with applicable EU directives.

This product complies with the relevant UK legislation.

See UKCA Declaration of Conformity in the Internet under www.renfert.com.

Within the EU, this device is subject to the provisions of the directive 2002/96/EC (WEEE directive).

Other symbols are explained when displayed.

Safety

Correct use of the unit

The system is used for digital stereo-magnification in 2D and 3D in real-time and for documentation.

Unapproved Use

Only spare parts and accessories supplied or authorized by Renfert GmbH may be used with this product. If other spare parts or accessories are used, this could have a detrimental effect on the safety of the device, increase the risk of serious injury and lead to damage to the environment or the device itself.

Hazards and Warnings

General information

- ▶ **Granted protection is not given any longer in case the unit is not operated according to the instruction manual on hand.**
- ▶ **The unit should only be operated using a mains cable with a country-specific plug system. Any modification required should only be carried out by an electrician.**
- ▶ **The unit should only be operated if the data on the rating plate corresponds with the data of regional mains voltage.**
- ▶ **The unit should only be plugged into earthed sockets.**
- ▶ **The mains socket should be easily accessible.**
- ▶ **Disconnect the unit from the mains before working on electrical parts.**
- ▶ **Regularly check connecting cables (such as power cables), hoses and housing (for example, the control panel) for damage (for example, kinks, tears, porosity) or deterioration due to aging.**
Do not operate units with damaged connection cables, hoses or housing parts or other defects!
- ▶ **Damaged equipment must be taken out of service immediately. Disconnect from mains and secure against being switched on again. Send the device in for repair!**
- ▶ **Operate the equipment only under supervision.**
- ▶ **Observe national accident prevention regulations!**
- ▶ **It is the responsibility of the operator that national regulations during operation and regarding a repeated safety inspection of electrical equipment are complied with.**
For Germany these are the regulation 3 by DGUV (German Statutory Accident Insurance) in relation with VDE 0701-0702 (Association for Electrical, Electronic and Information Technology).
- ▶ **Information on REACH and SVHC is available on our website www.renfert.com in the Support area.**

Specific information

- ▶ Do not look into the lighting.
- ▶ As with all optical magnification systems, working with the system for long periods can lead to symptoms of fatigue. Take breaks regularly.
- ▶ The 3D goggles do not replace protective eye equipment in any way.

Safety instructions for the monitor can be found in the instructions for use in the internet. When using your own monitor, please follow its safety instructions!

Approved personnel

The appliance may only be operated and serviced by trained personnel.

Product description

General description

The EASY view+ 3D system consists of a head with two cameras, lighting and the image processing unit, as well as a 3D-capable monitor including 3D goggles.

The product contains light sources of energy efficiency class F.

Assemblies and functional elements

See Fig. 1 (on inside cover)

- | | |
|------------------------------|--|
| 1 Head | 8 Monitor power switch |
| 2 Support arm with base | 9 Tool for clamping force adjustment (Allen key) |
| 3 Monitor | 10 Foot switch |
| 4 EASY view 3D power supply | 11 Mouse |
| 5 Monitor power supply | 12 Desk pad |
| 6 Height adjustment (focus) | 13 3D goggles |
| 7 EASY view+ 3D power switch | 14 USB memory stick |

Connections

See Fig. 2 (on inside cover)

- | | |
|----------------------------------|------------------------------|
| 20 Power supply | 24 USB port for mouse |
| 21 HDMI output | 25 Network |
| 22 USB port for USB memory stick | 26 Renfert CONNECT interface |
| 23 Foot switch | |

Accessories

Accessories and their use are described in the instructions for use in the internet.

Commissioning

Setup

⇒ Place the desk pad (12) on the work table.

Mounting the support arm



The support arm should preferably be mounted on the back or side of the table top, at least 400 mm away from the front edge.

This makes it possible to optimally utilize the entire swiveling range of the support arm.

⇒ **A:** Mount the base with the universal clamp on the back or side of the work table.

⇒ **B:** Insert the support arm (2) into the base. Make sure that the red plastic disc (30) is placed between the base and the support arm.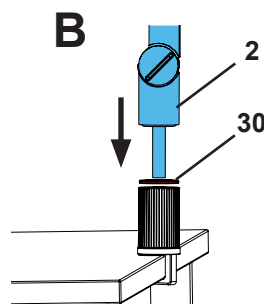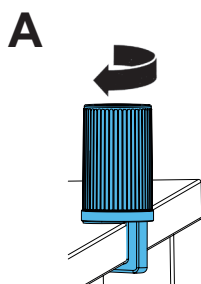

⇒ Place the mouse (11) next to the desk pad.


⇒ Place the foot switch (10) under the table and insert the cable for the foot switch into the foot switch.

Mounting the head

Four washers are needed for mounting the head.


Three small washers have already been threaded onto the fastening screw (31) in the correct sequence. The large washer is attached to the head with an adhesive strip with a red end.

- ⇒ Remove the adhesive tape with the red end.
- ⇒ Loosen the fastening screw (31) on the support arm.
- ⇒ Attach the fastening screw with all three small washers through the adapter of the head.
- ⇒ Make sure that the three small washers (gold / black / gold) and the large washer are in the correct order as shown in the figure.
- ⇒ Insert the adapter with the fastening screw and tighten the fastening screw until the head no longer tilts downward.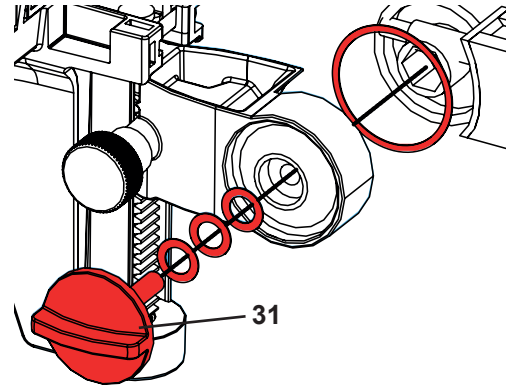

Positioning the monitor

- ⇒ Position the monitor so that it is directly in the field of view. The distance to the observer should be between 50 and 70 cm.
- ⇒ Align the monitor (tilt, rotate using stand) so that the viewing direction is perpendicular to the monitor in all planes.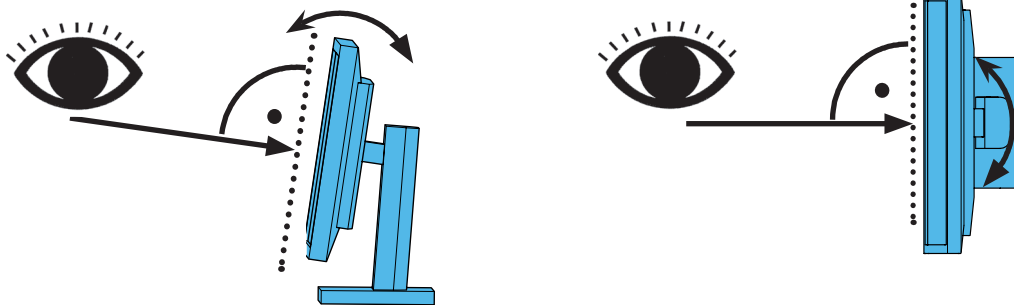

Adjusting the clamping force

In order to achieve a stable position of the head, the clamping forces of the three joints (31, 32, 33) can be adjusted individually.

- Swiveling the head right / left
 - Screw the clamping screw (34) with the included tool (9).
- Height adjustment
 - Screw the clamping screw (35) with the included tool (9).


Electrical connection

! Several plugs and related connector sockets are defined with a color coding. When connecting the components, make sure that the color coding is correct! Improperly connected components can become damaged by too high voltage!

Connections on the head

The USB flash drive and the cables are connected on the rear of the head unit.


- ⇒ Monitor cable ==> HDMI connection (21).
 - Plug the HDMI cable (36) into the retaining clip and then plug it straight into the socket.
- ⇒ Foot switch cable ==> connection (23).
- ⇒ Mouse cable ==> connection (24).
- ⇒ USB memory stick ==> connection (22).
- ⇒ Network extension cable ==> in connection (25).
- ⇒ Power supply of the head (integrated into the support arm) ==> connection (20).
- ⇒ Renfert CONNECT stick ==> Renfert CONNECT interface (26).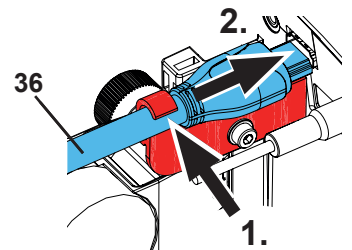

EASY view 3D head power supply

The EASY view 3D power supply (4) has a cable with an angled jack plug (37).

- ⇒ Plug the jack into the socket on the cable of the power supply, which protrudes at the base of the support arm.
 - Ensure that the color coding is correct between the connection cable and socket!**

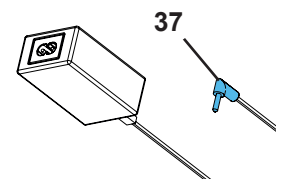! Never plug the jack plug into the connection for the foot switch (23)! This will destroy the head.

- ⇒ Insert the power cord into the power supply unit and into the building wall socket.

Connecting the monitor

Only for item no. 2400 3000.

Item no. 2400.4000 does not include a monitor!

If you use EASY view+ with your own monitor, follow its safety instructions and instructions for use.


This monitor must feature full 1920 x 1080 HD resolution.

Details can be found in the instructions for use in the internet (see chapter "Introduction").

See images at the end of this manual.

Take the following steps when using the monitor supplied by Renfert:

- ⇒ Monitor cable ==> DVI input of the monitor (41).
- ⇒ Secure plug with integrated screws.
- ⇒ Power supply ==> DC 12 V (40).

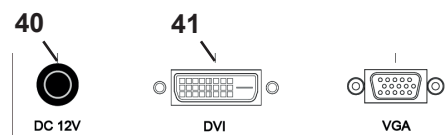Ensure that the color coding is correct between the connection cable and socket!

- ⇒ Insert the power cord into the power supply unit and into the building wall socket.

Renfert CONNECT stick

You can connect the EASY+ 3D to the Renfert app using the Renfert CONNECT stick and, for example, download images on your mobile device.

- ⇒ Insert the Renfert CONNECT stick into the Renfert CONNECT interface (26).

Make sure that the Renfert CONNECT stick is inserted in the correct port. If it is inserted in a different port, it will not work and may be damaged.

3D calibration

Due to vibrations during transport, the factory setting of the 3D calibration may be misaligned.

This results in a blurred 3D image.

In this case, please check the 3D calibration and reset it if necessary.

You can find a description of how to do this on the Internet at www.renfert.com/p918.

Enter the following item number: 2400 3000.

Switch to the "Download" area and select "EASY view + 2400X000 | Calibration".

Operation

Switching on

- ⇒ Switch on the monitor by pressing the power button (8).
- ⇒ Switch on the head using the power switch (7).
 - ◆ The lighting goes on.
 - ◆ **After about 20 seconds**, an image appears on the monitor.
 - ◆ The version number of the EASY view+ software is displayed on the bottom left of the screen.

The cone of light on the table top describes an area whose center is captured by the optics.

Stand-by

If there is no movement below the head for more than 10 minutes and the mouse has not been moved for more than 10 minutes, the lighting and the monitor output are switched off (monitor power button (8) flashes).

The stand-by mode is ended when the mouse is moved or movement takes place in the detection range of the optics.

Adjusting the monitor

In most cases, the monitor can be used with its basic settings.

Details on the monitor can be found in the instructions for use in the internet.

Renfert APP

If EASY view+ 3D is equipped with a Renfert CONNECT stick, it can be connected to the Renfert app.

Depending on your mobile device, you can download the Renfert app using the App Store or Play Store.

For further instructions on how to connect the Renfert CONNECT stick to the app and integrate it into the WiFi, please refer to the user guide of the Renfert app.

Set up

There are two options for focusing the image:

- Coarse adjustment by moving the head with the support arm.
- Fine adjustment through height adjustment (6).

The following procedure is recommended:

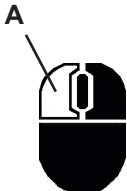
- ⇒ Align the head parallel to the table.
- ⇒ Place the object on the desk pad within the cone of light.
- ⇒ Set the head to the middle of the height adjustment.
- ⇒ Set the head on the support arm so that the object can be seen on the monitor and if necessary adjust the height adjustment.

This allows you to quickly correct the focus for objects with different heights.

System settings

The current settings of the system are displayed on the monitor at the bottom left.

As soon as the mouse is moved, the monitor displays the available settings and with which element of the mouse the setting can be changed.

| 2D / 3D representation | Magnification (4 x, 15 x, 20 x) | Object mode (ceramic, metal, standard, neutral) |
|---|--|---|
|  |  |  |
|  |  |  |
| ⇒ Left mouse button (A) ◆ Switch between 2D and 3D display | ⇒ Mouse wheel (B) rotation forward / backward ◆ Switch of magnification between 4 x, 15 x and 20 x ⇒ Press / rotate the mouse wheel (B) ◆ Adjust the brightness of an object mode (see instructions for use on the internet). | ⇒ Right mouse button (C) ◆ Switch object modes |

Screen shot / still image

Using the foot switch, the monitor image can be frozen (still image) or also stored on the USB memory stick (screen shot).

Still image

- ⇒ Press the foot switch for more than 1 sec.
 - ◆ The still image is displayed, meaning the monitor image is frozen.
- While the still image is displayed, the following actions can be performed:
- ⇒ Briefly press the foot switch.
 - ◆ The screen contents are saved to the USB stick as a screen shot.
- ⇒ Press the foot switch for more than 1 sec.
 - ◆ Switch back to the live image.

Saving a screen shot

Screen shots are only saved as 2D images.

- ⇒ Briefly press the foot switch.
 - ◆ The screen contents are saved to the USB stick as a screen shot.
 - ◆ The screen shot icon appears on the monitor.
 - ◆ The file name is displayed on the monitor during the save operation.
 - ◆ No other actions are possible during the save operation (approx. 5 sec.).

The file name has the following format:

- IMGxxx.jpg
- xxx = sequential counter 000 to 999

If the USB memory stick is then connected to a PC, the file names can be changed as usual, for example, to assign them to a customer.

Accessing the USB stick via a network

If the EASY view+ 3D head is connected to a local network via the network port (25,) and the network extension cable (already inserted in the cable channel), the images (screen shots) of the USB stick in the head can be accessed over the network. Details can be found in the instructions for use in the internet.

Working with 3D

When working in 3D mode, the 3D goggles must be worn.

The 4x magnification does not allow for a 3D illustration.

Split screen

The split screen feature allows an additional image to be displayed on the monitor.

With the split screen function, inserted images are only displayed in 2D.

- ◆ Open split screen main menu:
- ⇒ press the right mouse button until the split screen main menu appears.

Operation and navigation in the split screen menus:

- ⇒ Roll mouse wheel
 - ◆ Select menu item / sub-menu
- ⇒ Click left mouse button
 - ◆ Select menu item / sub-menu
- ⇒ Click right mouse button
 - ◆ returns to the previous menu level
- ⇒ Click and hold the right mouse button
 - ◆ Close the split screen menu.
- ⇒ Exit
 - ◆ Close the split screen menu.

As long as a split screen menu is open, the system settings (2D / 3D, 4x/ 15x / 20x zoom, object mode) cannot be changed.

For details to the split screen function, please refer to the user instruction in the internet.

Monitor

Details on the monitor can be found in the instructions for use in the internet. Renfert recommends not changing the factory settings of the monitor as the basic settings for the EASY view+ 3D.

Cleaning / Maintenance

The device does not contain any internal parts that require maintenance.

Wipe the housing, including the lighting, only with a slightly moistened cloth. Do not use disinfectant or cleaning agents or compressed air.
Clean monitor only with a dry cloth. Because of the natural product used, the desk pad may build an oily coat. It is possible to wipe it off with a grease dissolving cleaning agent such as a commercially available dishwashing liquid.

CAUTION!
Risk of getting burned on hot lenses of the lighting!
Clean only when the unit is switched off and has cooled down.

Removing the cable from the cable duct / guiding it into the cable duct

See images at the end of this manual.

Spare parts

You can find components subject to wear and the spare parts on the spare part list in the internet at www.renfert.com/p918.

Enter the following item number: 2400 3000.

The components excluded from the warranty (such as consumables or parts subject to wear and tear) are marked on the spare part list.

Serial number and date of manufacturing are shown on the type plate of the unit.

Troubleshooting

Details can be found in the instructions for use in the internet.

Disposal Instructions

Within the European Union units with this symbol should not be disposed of in unsorted domestic waste.

For more information regarding proper disposal please apply at your local authorities.

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declare under our sole responsibility, that the product(s)

| Name | Part No. |
|------------------------------|-----------|
| EASY view+ | 2400 4000 |
| EASY view+ 3D | 2400 3000 |
| EASY view 3D Converter 3D/2D | 2400 0501 |
| EASY view 3D HDMI-Splitter | 2400 0503 |

is (are) in conformity with the relevant regulatory requirements by compliance with the UK designated standards.

UK legislation

Electromagnetic Compatibility Regulations 2016

Electrical Equipment (Safety) Regulations 2016

The Restriction of the Use of Certain Hazardous Substances in Electrical and Electronic Equipment Regulations 2012

Designated standards

EN 61326-1:2013


EN 61000-3-2:2014

EN 61000-3-3:2013

EN 61010-1:2010/A1:2019/AC:2019-04

EN 62368-1:2014/AC:2015

EN IEC 63000:2018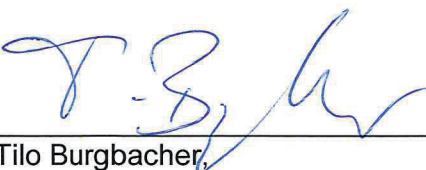

Tilo Burgbacher
Engineering Director







Hilzingen, 02.11.2022

The Technical Documentation has been retained by Renfert GmbH







making work easy

Kabel herausnehmen • Removing the cable • Retirer le câble • Togliere il cavo • Retirar el cable • Вынуть кабель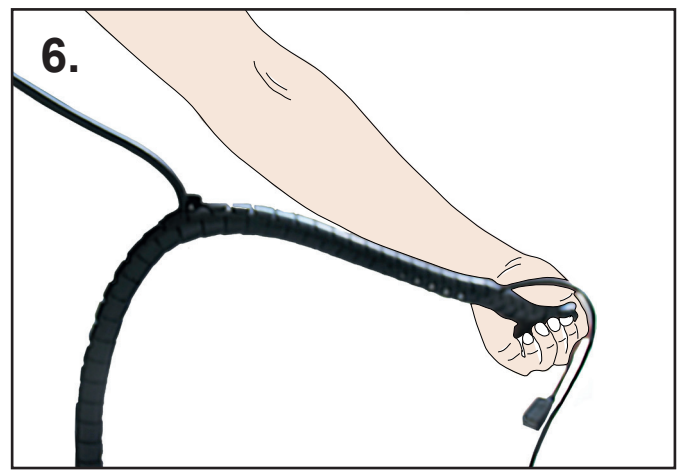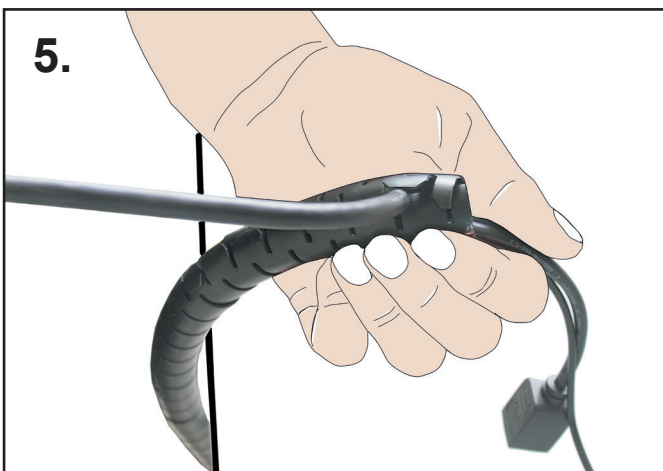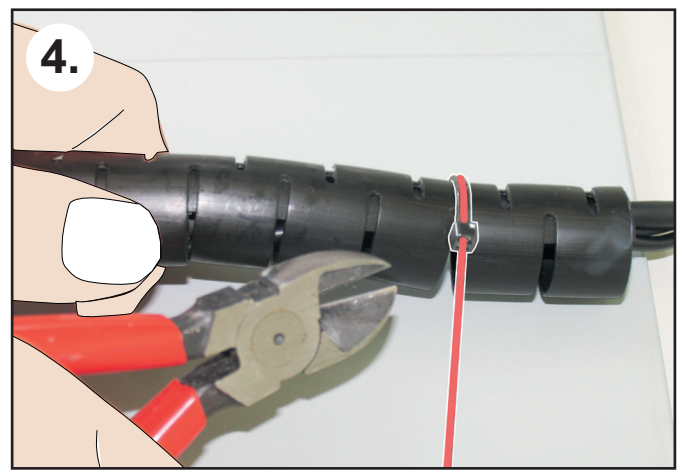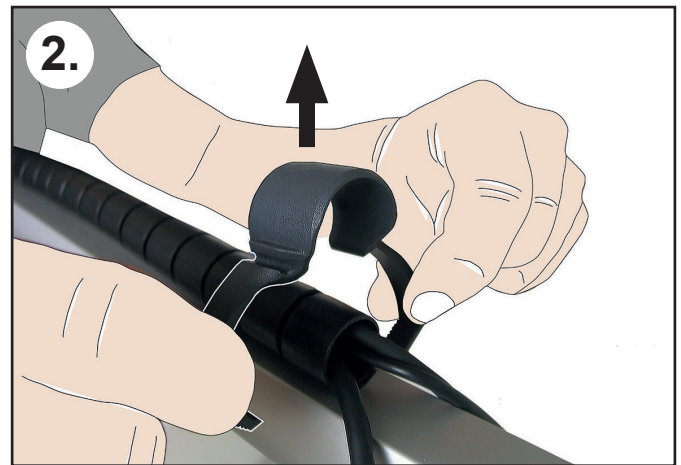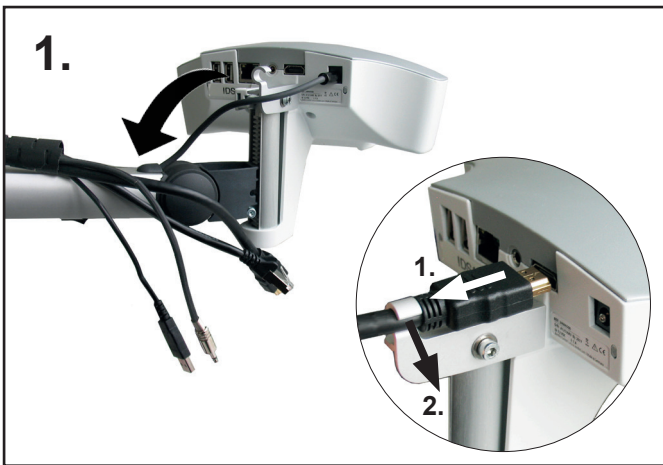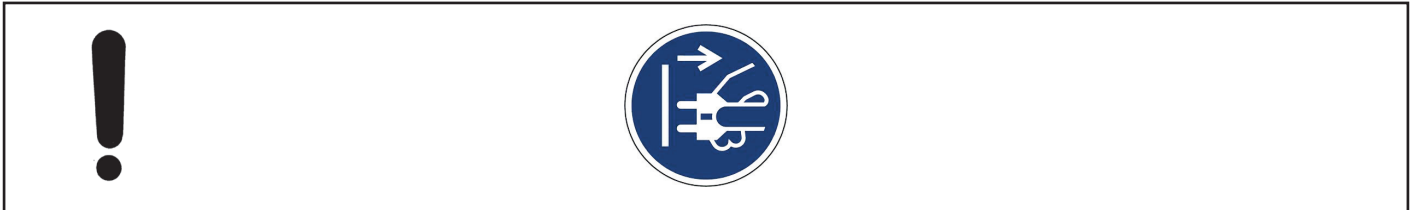

Kabel einziehen • Laying the cable • Insérer le câble • Inserire il cavo •
Instalar el cable • Вставить кабель

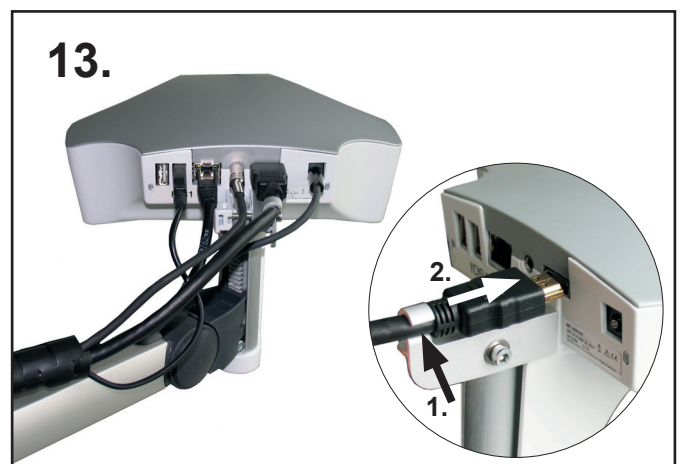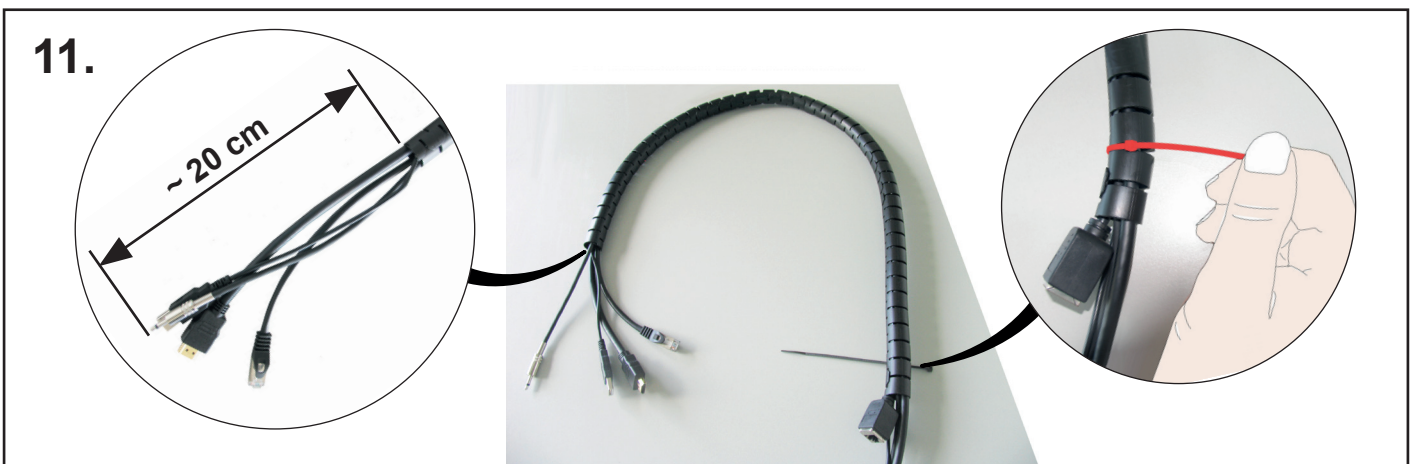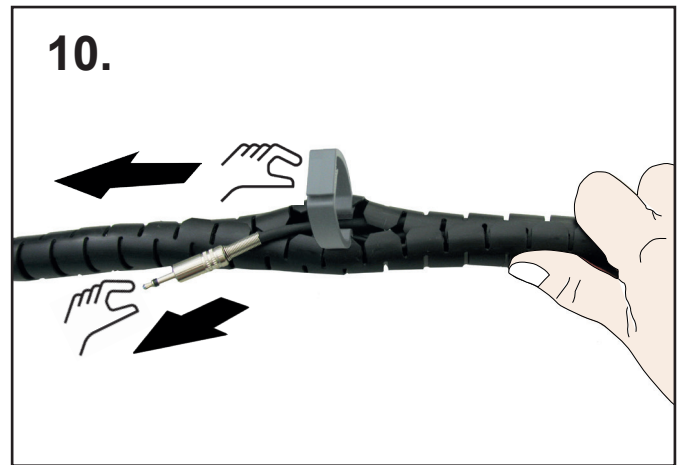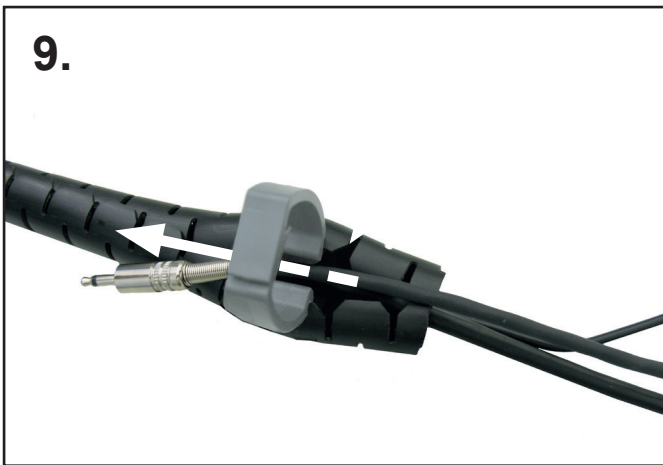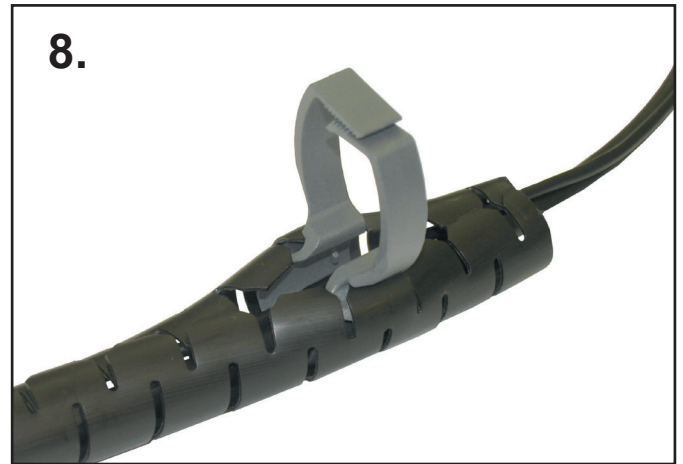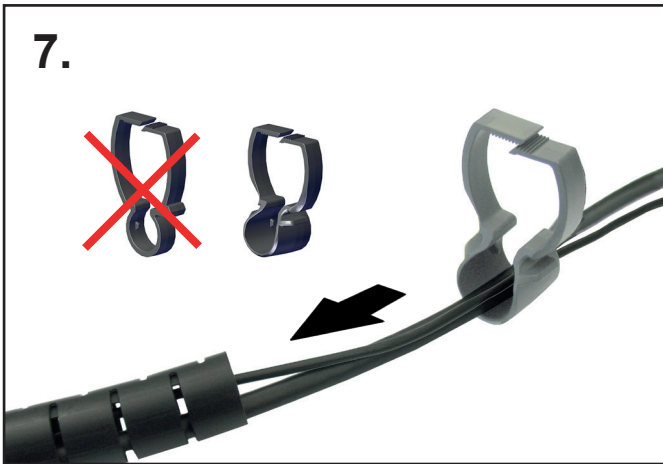Technical data

Head (incl. support arm)

| | |
|--|-------------------------------|
| Nominal voltage: | 100 / 120 / 230 V |
| Permissible mains voltage: | 100 - 240 V |
| Mains frequency: | 50 / 60 Hz |
| Power consumption: | 13.5 W |
| Video resolution: | FullHD (1920 x 1080) |
| Focus distance: | 430 mm [17 inch] |
| Magnification: | 4 x / 15 x / 20 x |
| Approx. illuminance: | 40,000 lux |
| Color temperature: | 5,000 kelvin |
| USB memory stick capacity: | 4 GB |
| Number of storable images, approx.: | 600 |
| Max. video sequence length, approx.: | 15 minutes. |
| Max. distance between mounting point of support arm and screen center: | 615 mm [24.2 inch] |
| Action radius of support arm: | 850 mm [33.5 inch] |
| Max. table top thickness with universal clamp: | 18 - 74 mm [0.71 - 2.92 inch] |
| Max. table top thickness with screw mounting: | max. 26 mm [1.02 inch] |
| Weight (incl. support arm): | 2.1 kg [4.63 lbs] |
| Weight (Head): | 0.7 kg [1.54 lbs] |

Monitor

Only for the monitor included in the scope of delivery for item no. 2400.3000.

| | |
|---|--|
| Nominal voltage: | 100 / 120 / 230 V |
| Permissible mains voltage: | 100 - 240 V |
| Mains frequency: | 50 / 60 Hz |
| Power consumption: | 25 W |
| Screen resolution: | 1920 x 1080 pixels |
| Dimensions with stand (Width x Height x Depth): | 511 x 377 x 170 mm [20.12 x 14.84 x 6.69 Inch] |
| Diagonal screen size: | 546.1 mm [21.5 Inch] |
| Weight (incl. power supply): | 3.65 kg [8.0 lbs] |

**Hochaktuell und ausführlich auf ...
Up to date and in detail at ...
Actualisé et détaillé sous ...
Aggiornato e dettagliato su ...
La máxima actualidad y detalle en ...
Актуально и подробно на ...**

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Renfert GmbH • Untere Gießwiesen 2 • 78247 Hilzingen/Germany