making work easy

- DE
- EN
- FR
- IT
- ES
- PT
- TR
- RU

- ZH
- JA
- KO
- AR

SILENT compact

Quick Start Guide

Made in Germany

21-6816 13022023

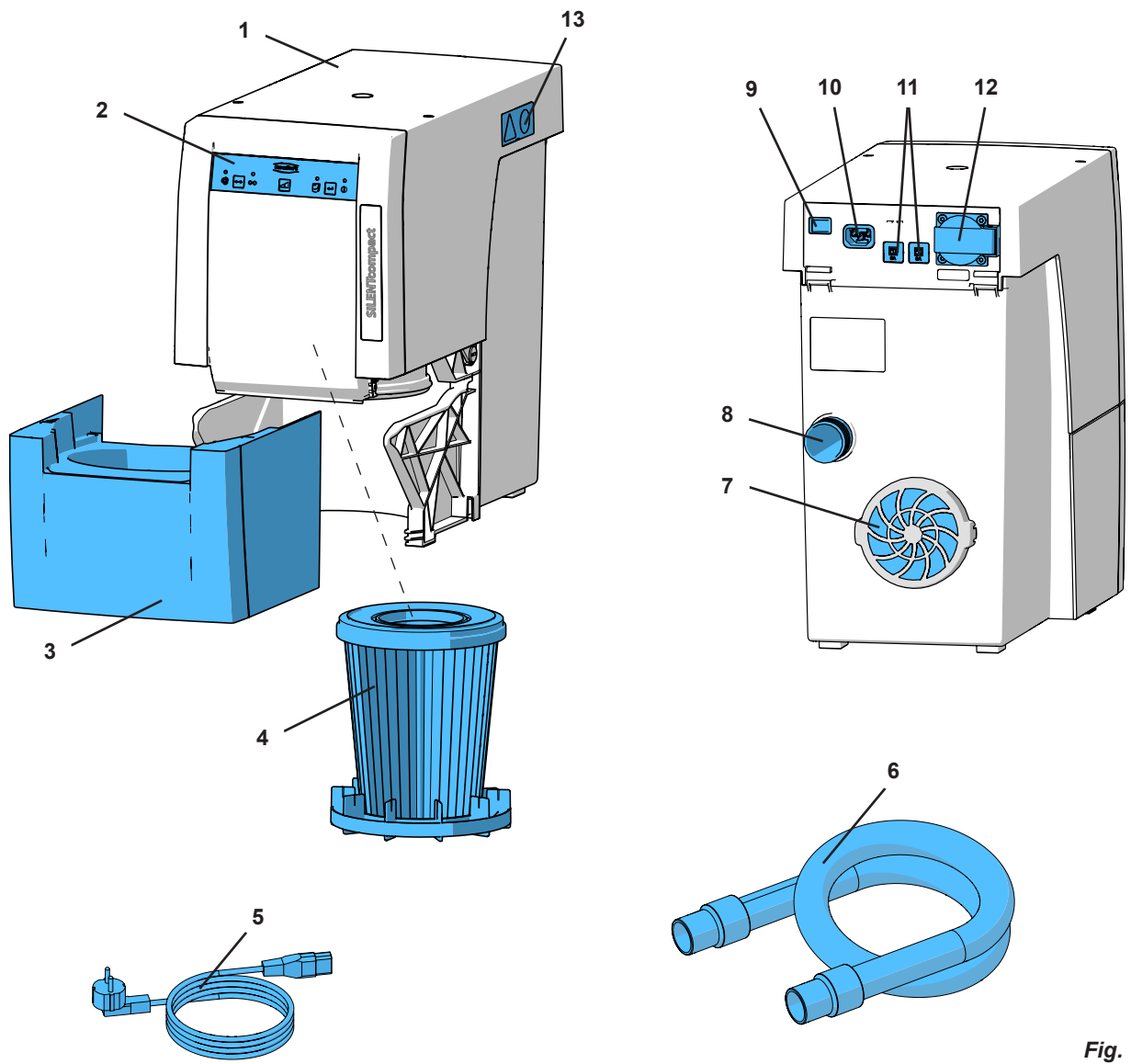
Fig. 1

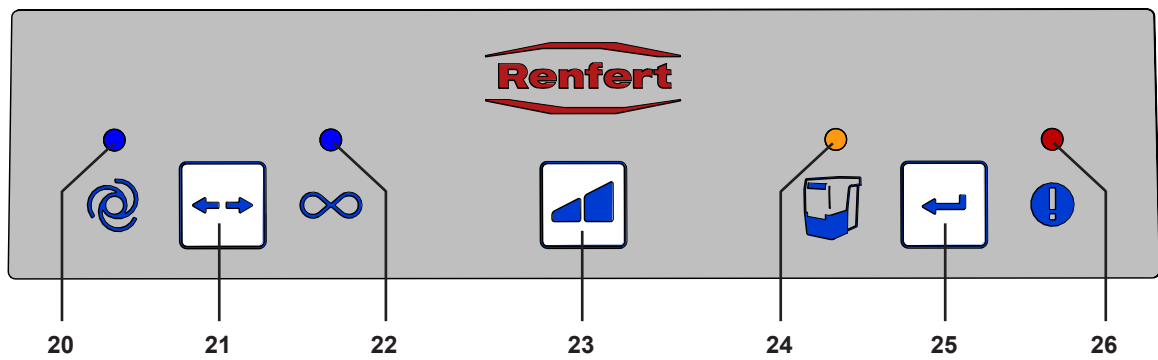
Fig. 2

Einleitung

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Introduction

This Quick Start Guide contains instructions on the operation of the product and relevant safety information.

For detailed information about the device, see the instructions for use which you can download from our website under www.renfert.com/p915.

Enter the following item number: 29340000

Symbols

In the instructions for use and on the unit itself you will find these symbols with the following meanings:

Danger

This indicates a direct risk of injury. Consult accompanying documents!

Electrical current

This indicates a risk of hazard due to an electrical current.

Attention

Disregarding this warning may result in damage to equipment.

Note

This provides the operator with useful information to improve and ease use.

The device complies with the requirements of the applicable EU directives.

This product complies with the relevant UK legislation.

See UKCA Declaration of Conformity in the Internet under www.renfert.com.

The device is subject to the EU directive 2002/96/EG (WEEE directive).

There is a FAQ video available on this topic in our video gallery under www.renfert.com/p49.

Other symbols are explained as they occur.

Safety

Intended Use

This device is designed to extract dry, non-explosive dust.

The unit is intended solely for use in a commercial dental laboratory and dental practice.

The intended use also includes compliance with the instructions specified by the manufacturer concerning operation, servicing and maintenance.

The extraction unit can be used in combination with a SILENT flow sensor within the scope of measures to protect against exposure to dust *) as defined by the German ordinance on hazardous substances ("GefStoffV") / respective employers' liability insurance association.

For this purpose, the information on low-dust disposal of the extracted material (chapter "Empty Dust Drawer") and on low-dust replacement of the fine filter (chapter "Change Fine Filter") in particular must be observed.

In doing so, both the instructions for use of the extraction unit as well as those of the SILENT flow sensor must be observed.

*) Note for customers in Germany: A validated and recognized dust protection measure in accordance with testing principle GS-IFA-M20 (issued by the German Institute for Work and Health IFA) shall only be deemed to have been provided when all components, i.e. the flow rate monitoring unit such as the SILENT flow sensor, the extraction unit, and the detection unit, are validated and recognized.

In the case of multi-workbench extraction units and the use of Y-junctions or extractor switches, every suction channel / every suction point must be fitted with a flow rate monitoring unit.

Improper Use

The equipment is not suitable for use with dental CAM units! For information concerning SILENT extraction units for CAM machines, please refer to www.renfert.com.

Fire-promoting, easily flammable, red-hot, burning or explosive materials must not be suctioned into the device. It is not permitted to suction liquids. This device is not intended for private, household use.

Any use other than specified in these instructions is deemed improper and constitutes a misuse of the device. The manufacturer shall not be liable for damages caused by improper use.

Only spare parts and accessories supplied or authorized by Renfert GmbH may be used with this product. If other spare parts or accessories are used, this could have a detrimental effect on the safety of the device, increase the risk of serious injury and lead to damage to the environment or the device itself.

Hazard and Warning Information

General Information

- ▶ If the device is not used in compliance with the supplied instructions, the safety of the device can no longer be guaranteed.
- ▶ The device may only be operated using a mains cable with the country-specific plug system. Any necessary alterations must be carried out by a qualified electrician.
- ▶ The device may only be operated if the information on the identification plate conforms to the specifications of your local mains power supply.
- ▶ The device may only be plugged into outlets which are connected to the protective conductor system.
- ▶ The mains plug must be easily accessible.
- ▶ Disconnect the device from the mains before carrying out work on the electrical parts.
- ▶ Check connection cables (such as power supply cords), tubes and housing (i.e. the key-pad) regularly for damage (i.e. kinks, cracks and porosity) or signs of ageing. Devices with damaged connection cables, tubes or housing parts or other defects must not be operated!
- ▶ Defective devices must be put out of service immediately. Remove the mains plug and ensure the device is not used. Send the device for repair!
- ▶ Only operate the device under supervision.
- ▶ Observe the accident prevention regulations by the trade association!
- ▶ It is the responsibility of the operator that national regulations during operation and regarding a repeated safety inspection of electrical equipment are complied with. For Germany these are the regulation 3 by DGUV (German Statutory Accident Insurance) in relation with VDE 0701-0702 (Association for Electrical, Electronic and Information Technologies).
- ▶ Information on REACH and SVHC is available on our website www.renfert.com, in the Support area.

Specific Information

- ▶ The German ordinance on hazardous substances (“GefStoffV”) or equivalent national regulations must be observed during operation, cleaning, and maintenance.
- ▶ The power socket on the device may only be used for the purpose specified in the instructions for use. The connection of other electronic devices could lead to material damage.
- ▶ Before connecting an electronic device to the power socket, ensure that the electronic device is first switched off.
- ▶ Read the operating instructions of the other appliance and comply with the safety instructions contained in the document.
- ▶ Please observe the national regulations and permitted exposure to dust in a working environment. Please ask the “National Institute for Occupational Safety and Health” or other responsible authority.
- ▶ Please observe the Safety Data Sheets of the materials to be extracted.
- ▶ Always wear protective gear, when extracting hazardous materials.
- ▶ It is necessary to wear suitable personal protective equipment when emptying the dust drawer or cleaning, depending on the type of extracted material.
- ▶ When disposing of the extracted material or used filter, please observe the local specifications and accident prevention regulations!
- ▶ Make sure the dust drawer is fully closed during operation.
- ▶ Do not operate without a suction hose.
- ▶ Do not extract flammable or explosive gasses, fumes or dust.
- ▶ The following applications involve increased risks and are therefore not permitted:
When cleaning milling chambers and SLM printing systems by suction, a sufficiently high concentration and purity (i.e. unmixed with other dental dusts such as gypsum, resins) can result in spontaneous ignition or the milling dust exploding or printing powder because of a potential exothermic reaction (e.g. due to oxidation).
An alternative cleaning method must be selected for the following milling or printing media in particular (such as manual cleaning):
 - Wood
 - Titanium / titanium-aluminum
 - Light metals and light metal alloys (e.g. aluminum, magnesium)
 - Cobalt chrome powder (e.g. for use in SLM systems)If large quantities of light metals such as titanium alloy are processed (e.g. with sandpaper), producing very fine grinding dust, a sufficiently high concentration and purity may result in spontaneous combustion because of a potential exothermic reaction.

- ▶ **Do not extract hot materials.**
- ▶ **Do not extract liquids.**
- ▶ **If the dust extractor is employed to extract hazardous materials, appropriate personal protective gear must be worn and steps must be taken to ensure that the exhaust air is properly ventilated. Please refer to the associated safety data sheets for specific requirements.**
- ▶ **Dispose of extracted material according to local statutory regulations.**

Authorized Persons

Operation and maintenance of the device may only be performed by qualified personnel.

Product Description

Components and Functional Elements

See Fig. 1 (on inside cover)

- | | |
|---|-----------------------------|
| 1 SILENT compact | 8 Suction port |
| 2 Key-pad | 9 On / Off switch |
| 3 Dust drawer | 10 Power supply |
| 4 Fine filter | 11 Device protection switch |
| 5 Mains cable | 12 Unit power socket |
| 6 Suction hose | 13 Service flap |
| 7 Exhaust air filter / exhaust air outlet | |

See Fig. 2 (on inside cover)

- | | |
|--|---------------------------------|
| 20 Display automatic operation | 23 Suction level key |
| 21 Operating mode key, automatic, continuous operation | 24 Display empty suction drawer |
| 22 Display continuous operation | 25 Enter key, save input |
| | 26 Display error message |

Setting Up

Setup

The extraction unit is a free standing appliance which must not be operated in a lying position.

If the device is to be installed in a cabinet, an opening for the exhaust air with the following dimensions must be provided, in addition to the openings for the suction hoses:

- Circular opening: Min. 120 mm diameters.
- Rectangular opening: Min. 170 x 65 mm.

Connection

- ⇒ Insert the suction tube (6) into the suction port (8).
- ⇒ Connect the suction tube to the relevant suction point.
- ⇒ If necessary, shorten the suction tube.

Caution, risk of injury!

When shortening the suction tube, please ensure that the integrated wire is cut as straight as possible.

If the diameter size does not correspond, please use an adapter to prevent a loss in suction performance.

Long suction hoses, tight bends and kinks will considerably reduce the extraction force at the extraction point.

- ⇒ Connect the electronic device to the power socket (12) at the back.

When connecting an electronic device to the extraction unit, please ensure that the maximum permissible capacity for connected devices is not exceeded (Technical data).

- ⇒ Switch the device OFF at the On / Off switch (9).
- ⇒ Connect the power cable (5) to the power supply (10).
- ⇒ Insert the power plug into the building installed wall socket.

Operation

Switching the Unit On

The extractor is switched ON and OFF at the On / Off switch (9).

When the unit is switched on:

- ◆ All 4 displays light up (the display carries out a functional control),
- ◆ The suction unit performs an automatic filter cleaning sequence (for approx. 8 seconds, loud vibrating noise).

Select Operating Mode: automatic / continuous operating mode

• Automatic operating mode (20):
In order to function, the extraction unit is dependent on an electronic device connected to the power socket (12).

• Continuous operating mode (22):
The extraction unit runs continuously.

⇒ Press the operating mode key (21).

- ◆ To change the operating mode.

If the suction to a connected electrical appliance (e.g. a handpiece) does not react as required (e.g. does not activate, even though a connected device is in operation) then the switch-on threshold for automatic operation must be adjusted. This is also possible whilst the suction is on.

Suction Level

⇒ Press the key suction level (23).

- ◆ Change between the two suction levels.

The extraction unit always starts with the last set suction level.

Operation with the SILENT flow sensor

If the extraction unit is operated in combination with a flow rate monitoring unit such as the SILENT flow sensor, the following must be observed:

- ▶ **If the flow rate is less than the defined threshold value for a certain period, this is detected by the SILENT flow sensor, the corresponding indicator begins to flash, and a warning tone is output every 30 seconds.**

If the flow rate is too low at the suction point being monitored, safe operation is no longer guaranteed and the level of exposure to dust can be harmful to health!

In this case, the following measures must be taken:

⇒ Stop working at all suction points of the affected extraction unit. This applies to the individual suction channels of any multi-workbench extraction unit as well as to all suction points operated via a Y-junction or via extractor switches.

⇒ Determine why the flow rate is too low and resolve the issue.

Cause	Measure
Suction level is too low	• Choose a higher suction level
Filter blockage	• Clean the filter.
Cleaning the filter does not result in the expected improvement.	• Replace the fine filter. • For low-dust replacement of the fine filter, observe Chapter "Change fine filter".
Blockage in the suction point (suction mouth)	• Disconnect the suction hose at the suction point and check whether there is a blockage in the suction point; remove where appropriate.
Blockage in the suction line	• Check the suction line at all disconnection points / service openings section by section for blockages.
Leakage in the section of the suction line between the SILENT flow sensor and the extraction unit.	• Check the installation of the SILENT flow sensor in accordance with its instructions.

Determining the cause and measures can be carried out in the order shown.

After every measure, you should switch on the extraction unit again and check whether the minimum flow rate is again exceeded so that correct operation is possible again.

⇒ Work should only resume after the fault has been rectified and the minimum flow rate is exceeded again.

Cleaning / Maintenance

Opening the device, other than for the processes described below, is not permissible!

EN

Cleaning

Use a damp cloth to clean the outside of the unit.

Do not use abrasive or solvent-based cleaning agents.

Empty Dust Drawer

After the set time period has expired a signal to empty the dust drawer will occur (factory settings 50 h). Before the dust drawer is emptied, a filter cleaning function is carried out so that loose dust particles fall into the dust drawer.

The time period has expired:

- ◆ An acoustic signal will occur 3 times.
- ◆ The display dust drawer (24) lights up.

When the suction has stopped, an acoustic signal occurs 3 times and a filter cleaning function is carried out.

The following procedure is recommended for low-dust disposal of the extracted material

- ⇒ Pull the dust drawer out towards the front.
- ⇒ Place the dust drawer in a plastic bag and seal the bag tightly to prevent dust escaping
- ⇒ Turn the dust drawer with the plastic bag so that the extracted material falls out of the dust drawer.
- ⇒ Leave the closed plastic bag to stand for at least 30 minutes so that the dust that has been raised can settle.
- ⇒ Open the plastic bag and carefully remove the dust drawer.
- ⇒ Seal the plastic bag again tightly to prevent dust escaping and ensure disposal.
- ⇒ Wipe the dust drawer with a damp cloth and insert it again until it locks into place.
- ⇒ Suction up any dust that has been released.

So that you do not have to wait for the dust to settle when emptying the dust drawer and can quickly resume working, we recommend purchasing a second dust drawer. Alternatively, you can also always empty the drawer when work has been completed so that the dust does not settle overnight.

⇒ Press the enter key (25) (the counter which determines the time period is re-set).

- ◆ An acoustic signal confirms the input.
- ◆ The display dust drawer (24) light disappears.

If the dust drawer is not emptied, the display for the dust drawer will remain alight (24). When the unit is switched off / on an acoustic signal will occur to remind that the dust drawer needs emptying.

For information on setting the required time period see the quick reference card which is supplied or go to the instructions for use in the Internet.

Change Fine Filter

- ▶ If the suction power is no longer sufficient in spite of a filter cleaning, the fine filter must be changed.
- ▶ In general, the fine filter should be changed every 2 years.

Do not clean the fine filter manually (e.g. with compressed air, brush, water, etc.), as this leads to damage of the filter material!



Do not use compressed air!

Using compressed air to blow off deposits of dust is generally not permitted.

There is a risk of increased exposure to dust when replacing the fine filter. That's why the following measures must be taken:

- ⇒ Keep the plastic bag ready that contains the new filter.
- ⇒ Keep an additional dust extraction device or suction tube from another workbench extraction unit ready.
- ⇒ Wear at least an FFP2 mask for respiratory protection.
- ⇒ Clean the filter 2x:
 - Switch the extraction unit off / on.
 - Filter cleaning is carried out.
 - Repeat the procedure.
 - Switch off the extraction unit.
- ⇒ Disconnect the power plug, disconnect the suction hose.
- ⇒ Place the extraction unit so that it can be turned around.
- ⇒ Pull out the dust drawer.
- ⇒ Place the extraction unit to one side.
- ⇒ Unlock the clamping nut (1), loosen (2), remove, and ensure disposal.
- ⇒ Pull out the fine filter
- ⇒ Place the fine filters immediately in the plastic bag and seal the bag tightly to prevent dust escaping.
- ⇒ Ensure proper disposal.
- ⇒ Suction any fallen dust out of the dust room and clean room.
- ⇒ Turn the extraction unit upside down
- ⇒ Replace the O-ring of the vibration unit
- ⇒ Insert a new fine filter; ensure that it is correctly positioned.
- ⇒ Attach a new clamping nut and tighten as shown.
- ⇒ Turn the extraction unit back to its previous position.
- ⇒ Insert the dust drawer.
- ⇒ Connect the suction hoses, insert the power plug, and place the extraction unit back in position.
- ⇒ Use the additional dust extraction device to suction up dust that has been released.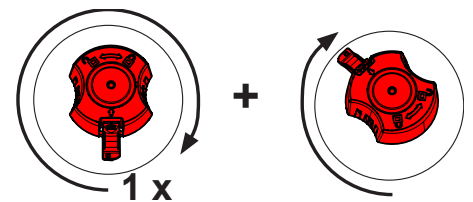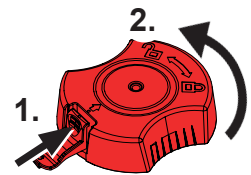

When installing the fine filter please ensure that it is positioned correctly, otherwise leakages may result.

See the assembly instruction at the end of the instruction manual, which is also attached to the new fine filter.

Spare Parts

You can find components subject to wear and the spare parts on the spare part list in the internet at www.renfert.com/p918.

Enter the following item number: 29340000

The components excluded from the warranty (such as consumables or parts subject to wear and tear) are marked on the spare part list.

Disposal Information

Within the European Union, units with this symbol should not therefore be disposed of in unsorted domestic waste.

Please contact your local authorities for more information on proper disposal.

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declare under our sole responsibility, that the product(s)

Name	Part No.
SILENT TS	2921 0050
SILENT TS2	2930 0050
SILENT TC	2935 0000 / 2935 0000 03
SILENT TC2	2936 0000 / 2936 0000 03
SILENT EC2	2937 0000 / 2937 0000 03
SILENT compact	2934 0000 / 2934 0000 03
SILENT compactCAM	2934 2000 / 2934 2000 03 2934 2500 / 2934 2500 03
SILENT powerCAM EC	2939 0000
SILENT powerCAM TC	2938 0000
iVAC silent	36 2934 2000

is (are) in conformity with the relevant regulatory requirements by compliance with the UK designated standards.

UK legislation

Electromagnetic Compatibility Regulations 2016

Supply of Machinery (Safety) Regulations 2008

The Restriction of the Use of Certain Hazardous Substances in Electrical and Electronic Equipment Regulations 2012

Designated standards

EN 61326-1:2013

EN ISO12100:2010

EN 61010-1:2010/A1:2019/AC:2019-04

EN IEC 63000:2018

Tilo Burgbacher,
Engineering Director

Hilzingen, 02.11.2022



The Technical Documentation has been retained by Renfert GmbH


making work easy

90002 1635


Feinfilterset • Fine filter set • Jeu filtre fin • Kit filtro fine
Juego elemento filtrante fino • Комплект фильтров тонкой очистки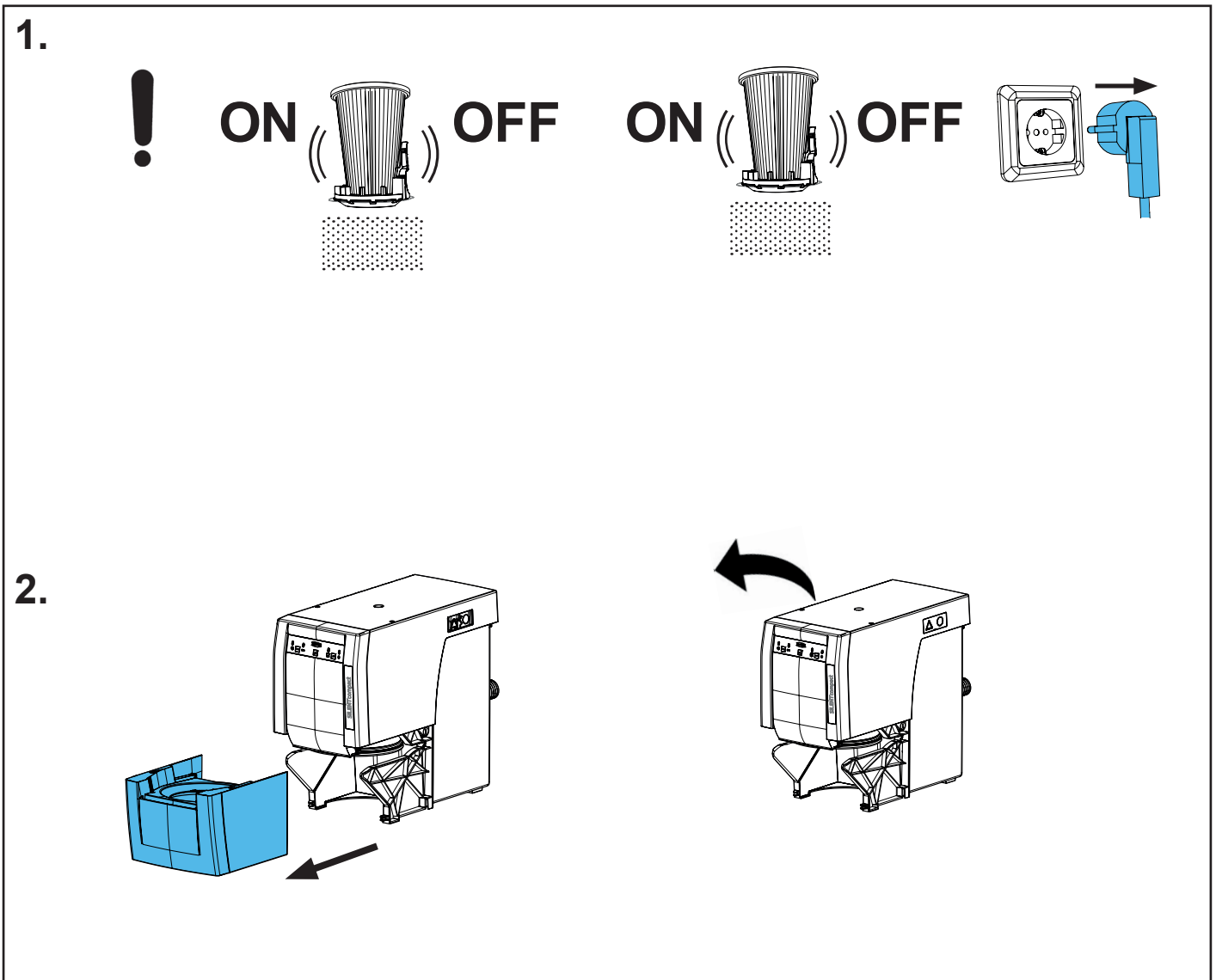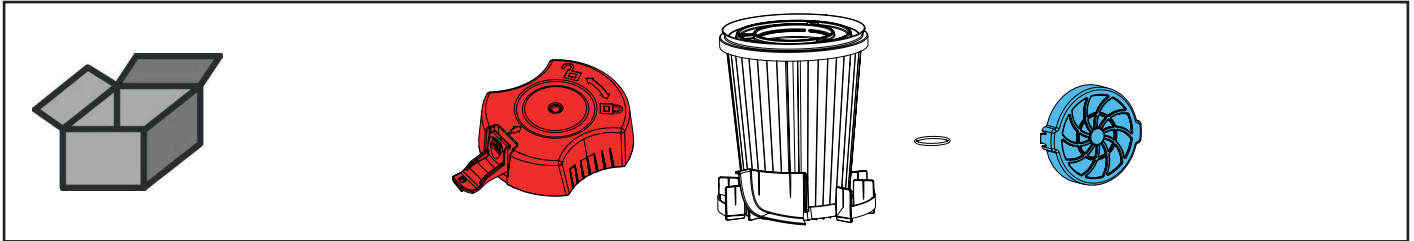


3.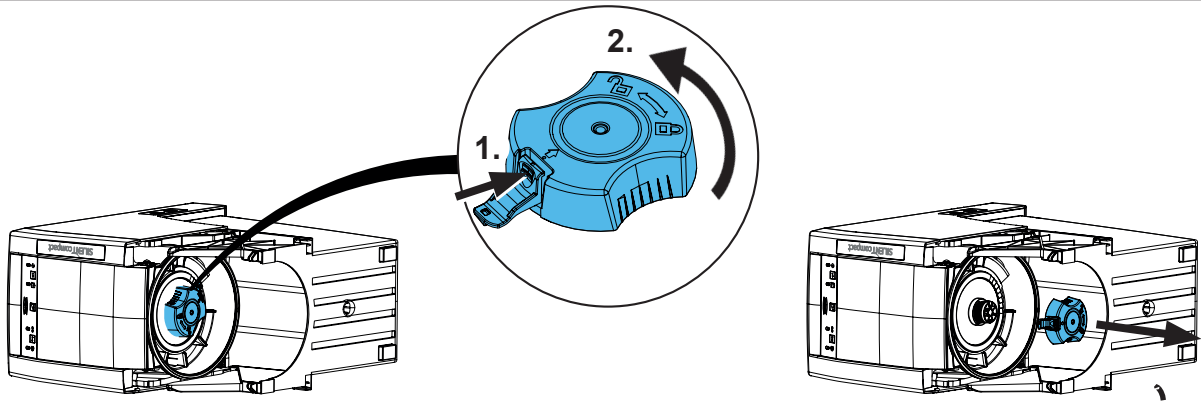


4.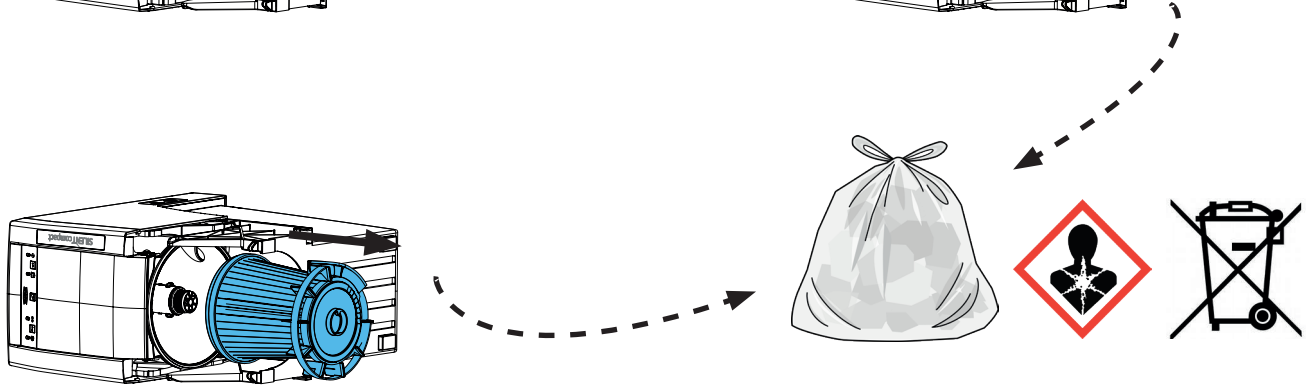

5.
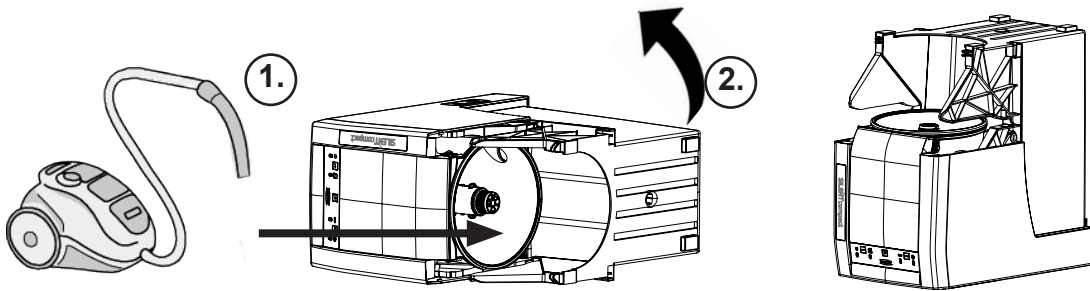



6.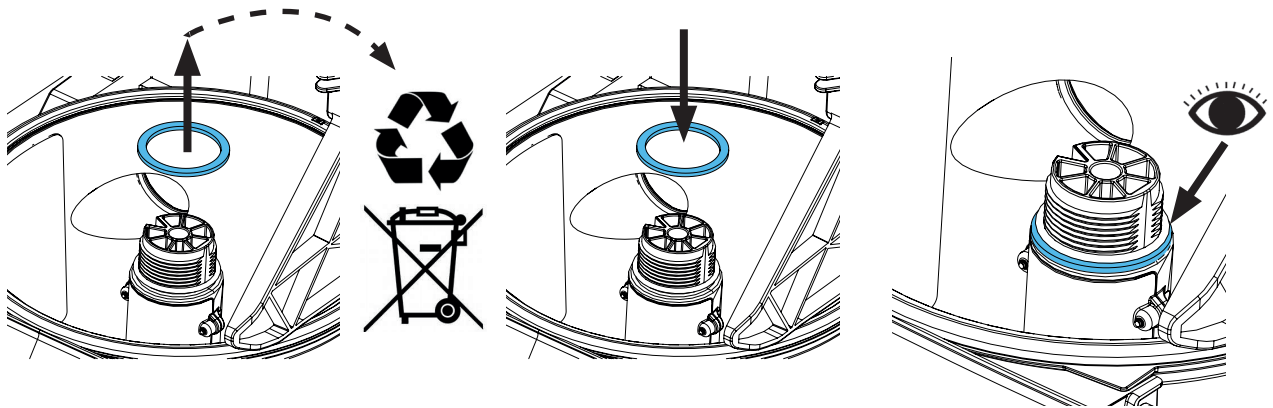





5.

i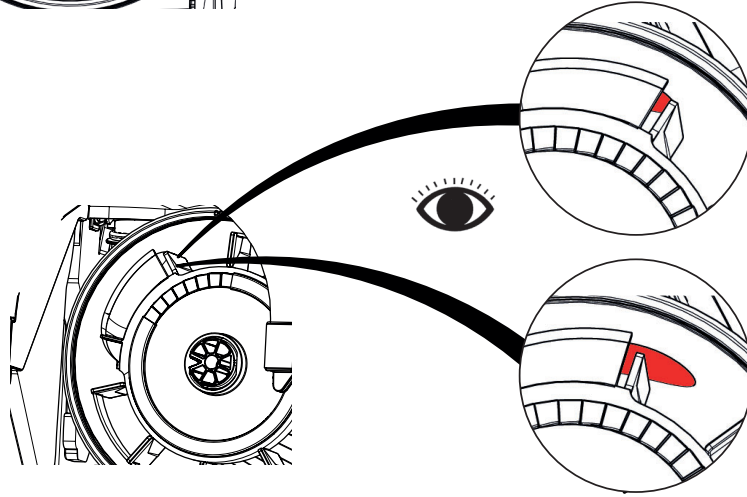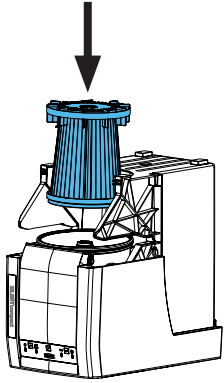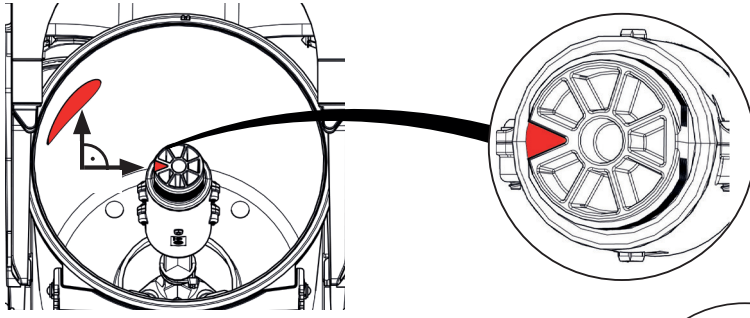


6.


red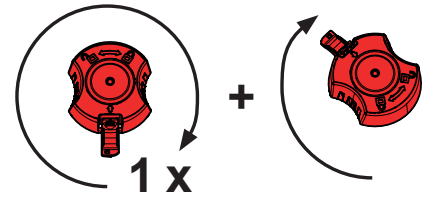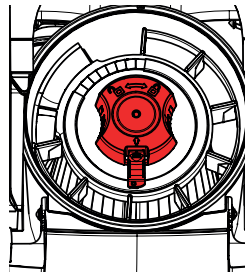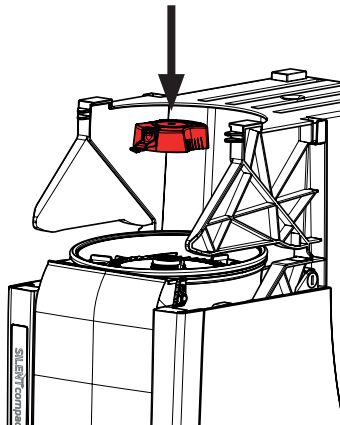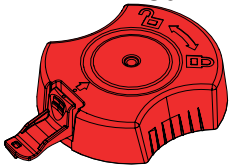





7.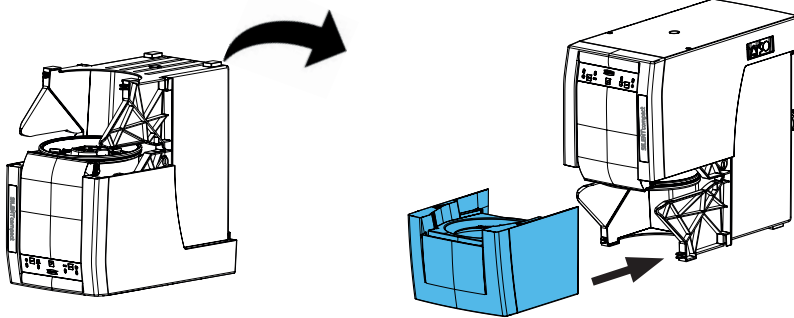


8.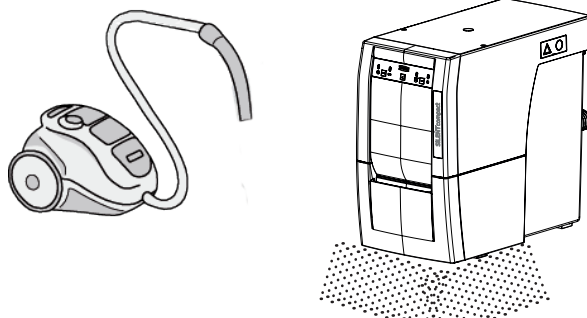




1.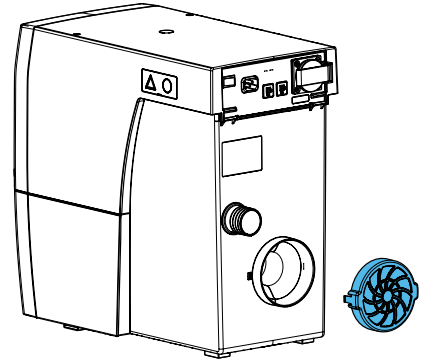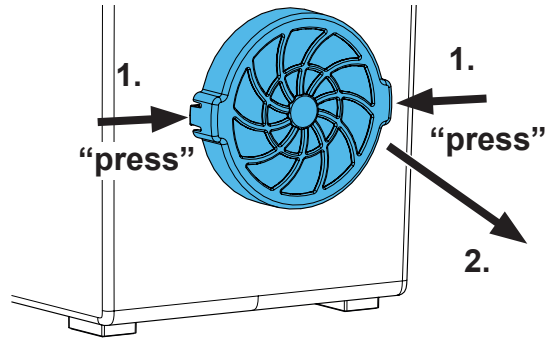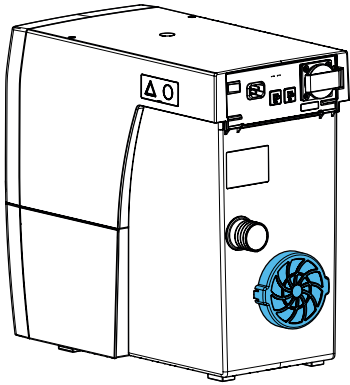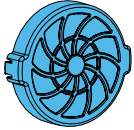

2.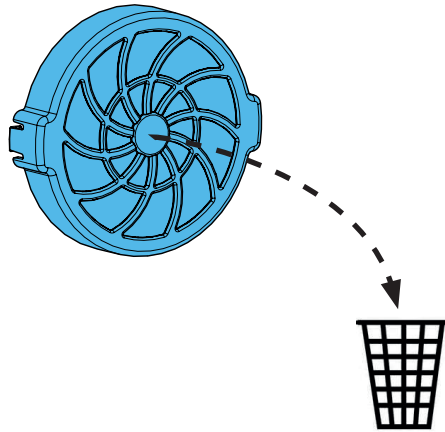

3.

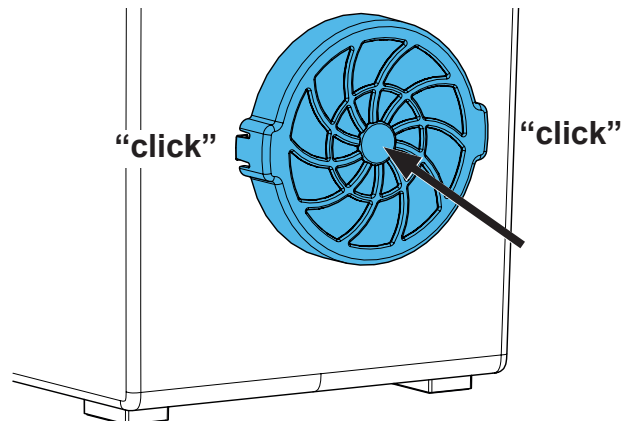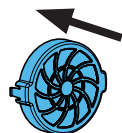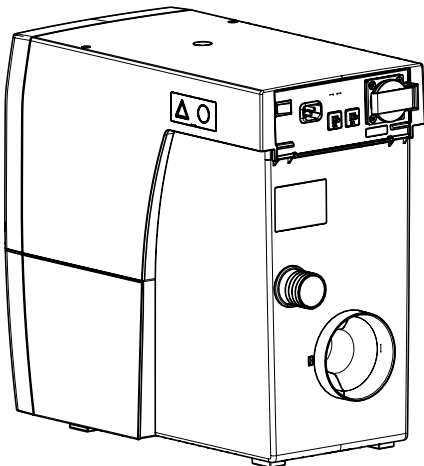Technical Data

	2934 0000	2934 1000	2934 1500
Nominal voltage:	230 V	120 V	100 V
Permissible mains voltage:	220 - 240 V	120 V	100 V
Mains frequency:	50 / 60 Hz	50 / 60 Hz	50 / 60 Hz
Suction turbine power *):	490 W	480 W	480 W
max. connecting power *):	1350 W	480 W	320 W
Total connected power *):	1840 W	960 W	800 W
LpA **) (at max. volume flow):	55 dB(A)		
Flow rate, max. ***):	2500 l/min [1.47 ft ³ /s]		
Negative pressure, max. ****):	219 hPa [3.2 psi]		
Filterquality:	Class M according to EN 60335-2-69		
Weight (empty):	approx 13,2 kg [approx 29.1 lbs]		
Dimensions (width x height x depth):	245 x 440 x 500 mm [9.6 x 17.3 x 19.7 inch]		

- *) Power consumption at nominal voltage
- **) Sound pressure levels according to EN ISO 11202
- ***) Free-blowing turbine at nominal turbine voltage
- ****) At nominal turbine voltage

**Hochaktuell und ausführlich auf ...
Up to date and in detail at ...
Actualisé et détaillé sous ...
Aggiornato e dettagliato su ...
La máxima actualidad y detalle en ...
Актуально и подробно на ...**

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Renfert GmbH • Untere Gießwiesen 2 • 78247 Hilzingen/Germany