making work easy

- de
- en
- fr
- it
- es
- pt
- tr
- ru
- zh
- ko
- ar

SILENT powerCAM TC

Quick Start Guide

Made in Germany

21-2963 12092024

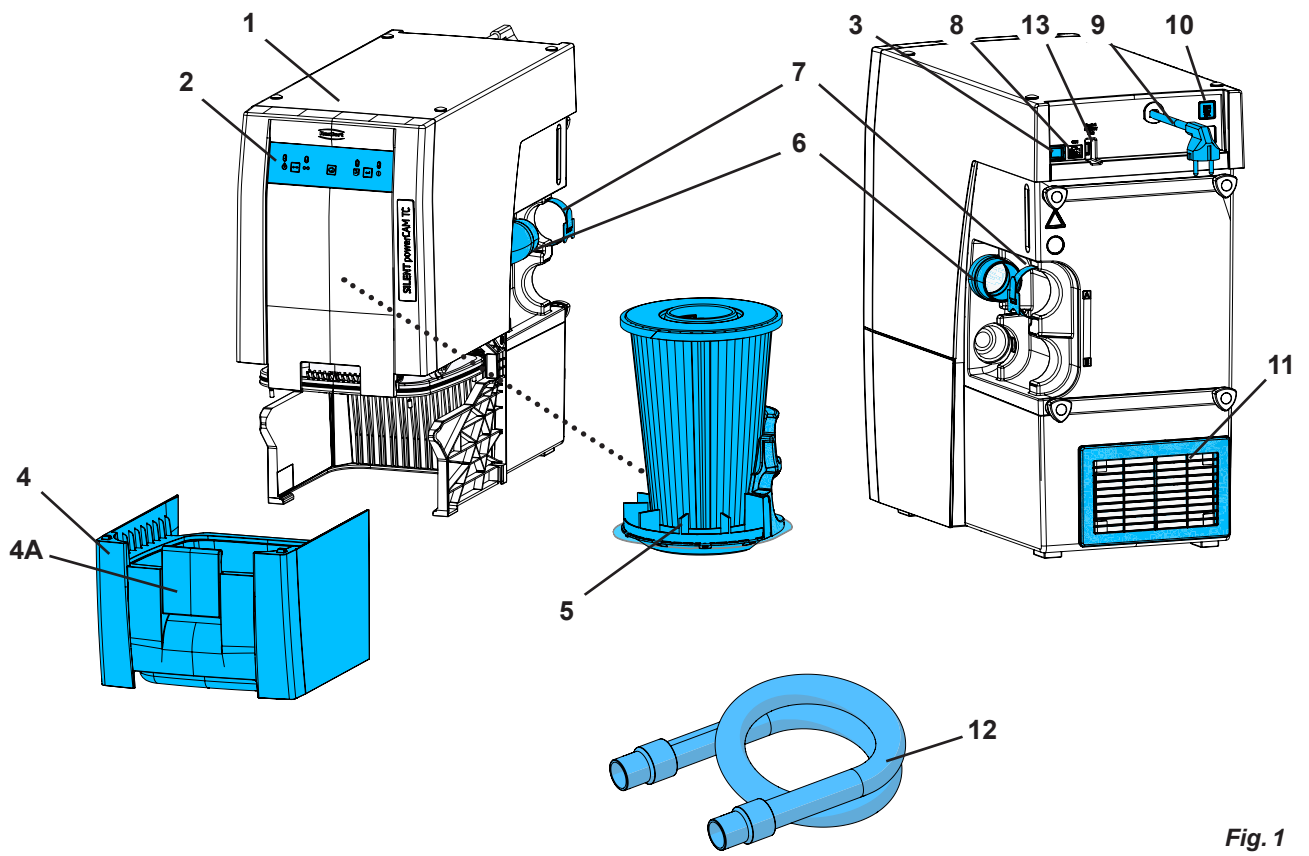
Fig. 1



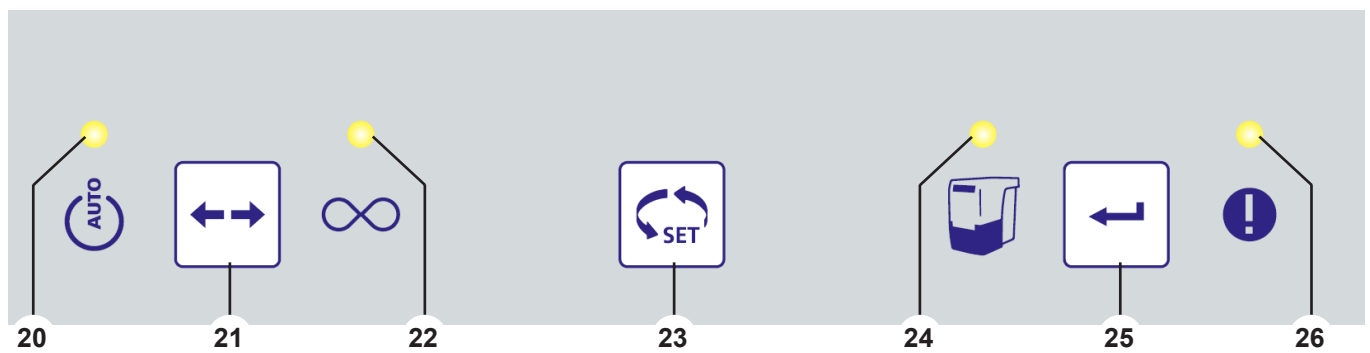
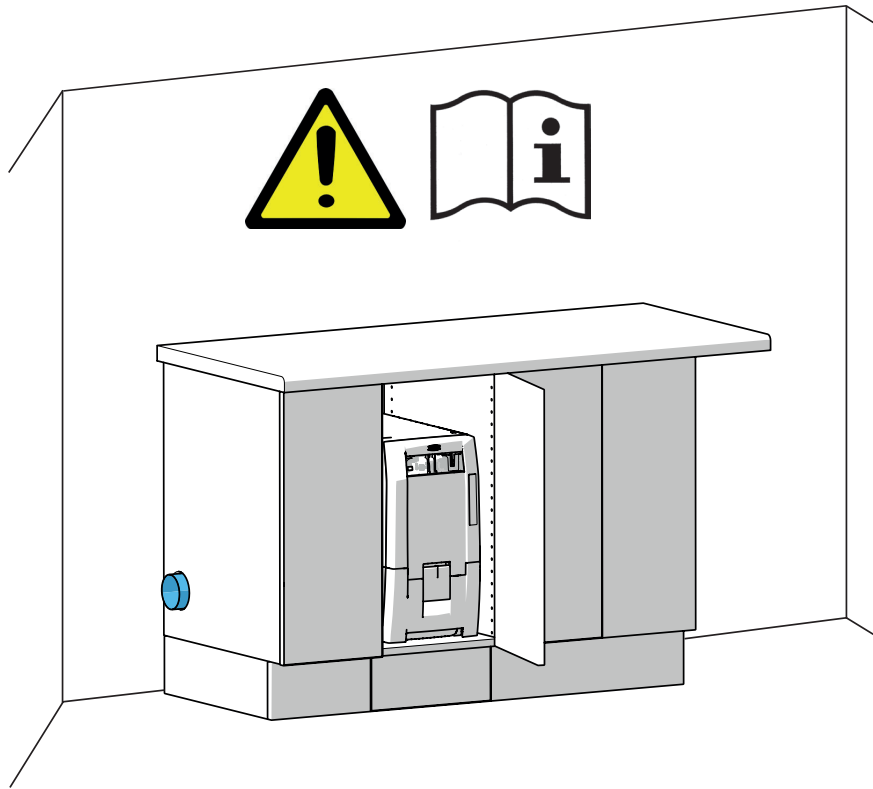
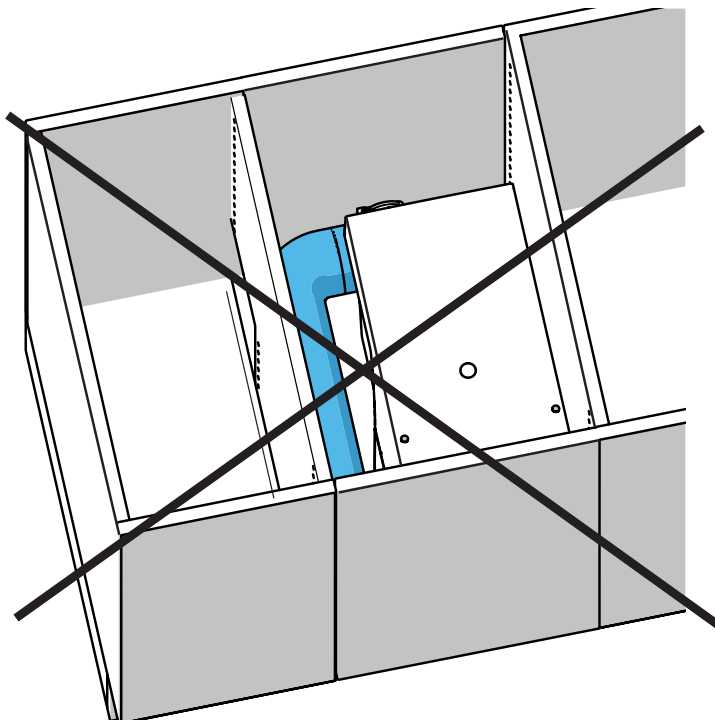
Fig. 2

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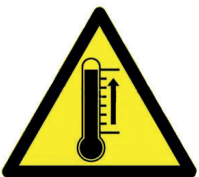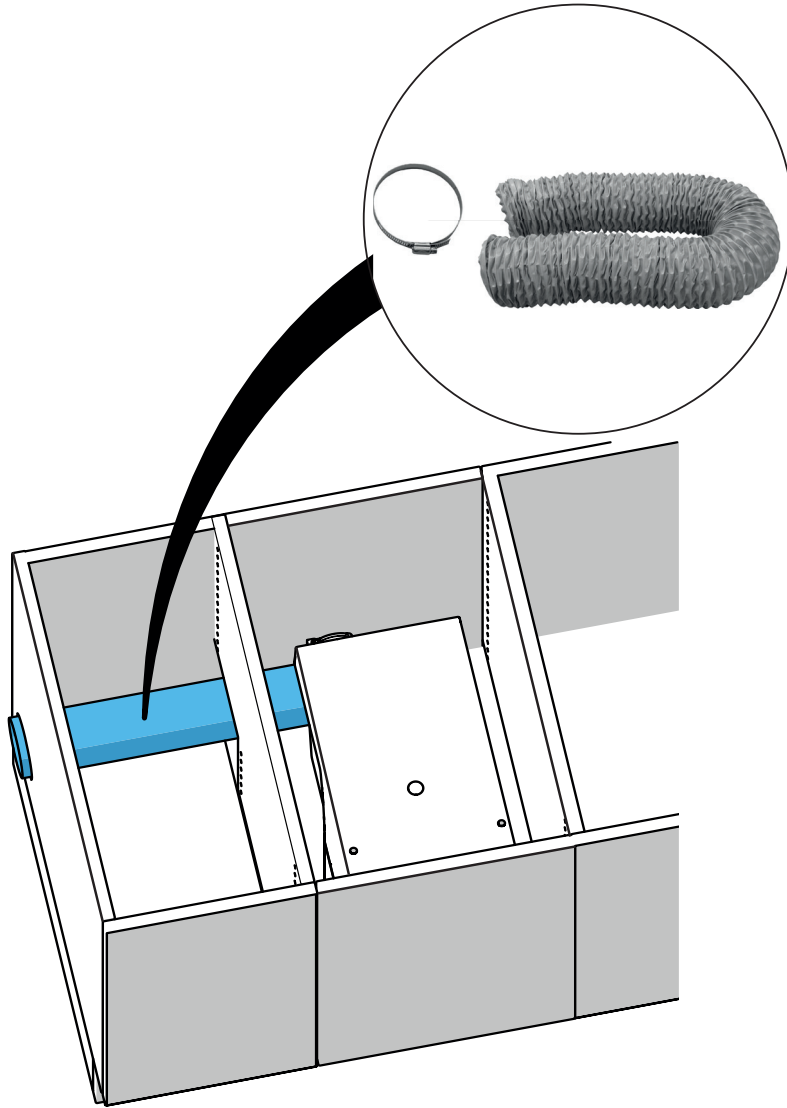Introduction

This Quick Start Guide contains instructions on the operation of the unit and relevant safety information.

Detailed information about the unit is provided in the instructions for use, which you can download from our website at www.renfert.com/p915.

Enter the following item number: 29380000.

Symbols used

Symbols with the following meanings are used in these instructions or on the equipment:

Danger

Immediate danger of injury. Consult accompanying documents!

Electrical voltage

Danger due to electrical voltage.

Caution

Risk of damage to the equipment if the instruction is not followed.

Notice

Provides the operator with useful information that eases use.

The equipment complies with applicable EU directives.

This product complies with the relevant UK legislation.

See UKCA Declaration of Conformity in the Internet under www.renfert.com.

The equipment is subject to the provisions of the EU directive 2002/96/EC (WEEE directive).

Other symbols are explained as they occur.

Safety

Intended use

This extraction unit is intended for use with dental milling machines for the suction of dry dust, generated whilst milling commercially available milling blocks and blanks.

The unit is intended for commercial use only.

The intended use also includes compliance with the instructions specified by the manufacturer concerning operation and maintenance.

Improper use

Flammable, easily combustible, hot, burning, or explosive materials may not be suctioned using the unit. It is not permitted to suction liquids.

This unit is not intended for private, household use.

Any use other than specified in these instructions is deemed improper use.

The manufacturer shall not be liable for resulting damages.

Only spare parts and accessories supplied or authorized by Renfert GmbH may be used with this product. If other spare parts or accessories are used, this could have a detrimental effect on the safety of the equipment, increase the risk of serious injury and lead to damage to the environment or the product itself.

Hazard and warning information

General information

- ▶ **If the unit is not used in accordance with the supplied instructions for use, the intended safety can no longer be guaranteed.**
- ▶ **The unit may only be operated using a power cord equipped with a plug appropriate to the local power supply. Any necessary modifications must be carried out by a qualified electrician.**
- ▶ **Only operate the unit if the information on the nameplate corresponds to the specifications of your local mains power supply.**
After removing the dust drawer, you can find the nameplate inside the unit on the lower left side.
- ▶ **The unit may only be plugged into electrical sockets that are connected to the protective conductor system.**

- ▶ The power plug must be easily accessible.
- ▶ Disconnect the unit from the mains power supply before carrying out work on the electrical parts.
- ▶ Regularly check connecting cables (such as power cords), hoses and housing (for example, the control panel) for damage (for example, kinks, tears, porosity) or deterioration due to aging. Do not operate equipment with damaged connection cables, hoses or housing parts or other defects!
- ▶ Defective equipment must be taken out of service immediately. Disconnect the power plug and secure the equipment against further use. Send the equipment in for repair!
- ▶ Please observe the national accident prevention regulations!
- ▶ It is the responsibility of the operator to ensure that national regulations are complied with during operation and regarding recurring safety inspection of electrical equipment. In Germany, these are DGUV Regulation 3 in conjunction with VDE 0701-0702.
- ▶ Information on REACH and SVHC is available on our website under www.renfert.com, in the Support area.

Specific information

- ▶ For connection to a CAM system, observe the operating instructions for the CAM system as well as the safety instructions that they contain.
- ▶ Observe national regulations and permitted dust exposure levels in a working environment. Refer to your national association for occupational safety and health or the responsible authority.
- ▶ Observe the safety data sheets when suctioning hazardous materials.
- ▶ Wear personal protective equipment when suctioning hazardous materials.
- ▶ Depending on the type of extracted material, wear appropriate personal protective equipment when emptying the dust drawer or cleaning.
- ▶ When disposing of the extracted material or used filters, observe local provisions and accident prevention regulations!
- ▶ Suctioning may only be performed with the dust drawer closed.
- ▶ Do not operate without a suction hose.
- ▶ Do not suction flammable, easily combustible, or explosive gases or fumes.
- ▶ The following applications involve increased risks and are therefore not permitted:
When cleaning milling chambers and SLM printing systems by suction, a sufficiently high concentration and purity (i.e., unmixed with other dental dusts such as plaster, resin) can result in spontaneous combustion or the milling dust or printing powder exploding because of a potential exothermic reaction (e.g., due to oxidation). An alternative cleaning method must be selected, particularly for the following milling or printing media (e.g., manual cleaning):
 - Wood
 - Titanium / titanium-aluminum
 - Light metals and light metal alloys (e.g., aluminum, magnesium)
 - Cobalt-chrome powder (e.g., for use in SLM systems)
- ▶ If large quantities of light metals such as titanium alloy are processed (e.g., with sandpaper), producing very fine grinding dust, a sufficiently high concentration and purity may result in spontaneous combustion because of a potential exothermic reaction.
- ▶ Do not suction hot materials.
- ▶ Do not suction liquids.
- ▶ If the extraction unit is used to suction hazardous materials, appropriate personal protective equipment must be worn, and steps must be taken to ensure that the exhaust air is properly ventilated. Refer to the safety data sheets for the corresponding requirements.
- ▶ Dispose of extracted material according to statutory regulations.

Authorized persons

Operation and maintenance of the equipment may only be performed by trained personnel.

Product description

General description

The equipment is an extraction unit for suctioning dust that builds up in dental CAM systems.

It can be operated manually or controlled using a CAM system.

The extraction unit is equipped with a bi-directional interface for communicating status information to the CAM system and for registering control commands from the CAM system.

Assemblies and functional elements

See Fig. 1 (on inside cover)

- | | |
|----------------------|--|
| 1 SILENT powerCAM TC | 8 CAM interface |
| 2 Control panel | 9 Power cord |
| 3 On / off switch | 10 Protective device switch |
| 4 Dust drawer | 11 Exhaust air filter / exhaust air outlet |
| 4A Clamp fastener | 12 Suction hose and end bushings |
| 5 Fine filter | 13 Renfert CONNECT port |
| 6 Suction port | |
| 7 Velcro® strip | |

See Fig. 2 (on inside cover)

- | | |
|---|-------------------------------|
| 20 Display: CAM operation | 23 Selection key |
| 21 Operating mode key: CAM operation / continuous operation | 24 Display: Empty dust drawer |
| 22 Display: Continuous operation | 25 Enter key, saves input |
| | 26 Error display |

Startup

Setup

The extraction unit is a standing device. Do not operate it on its side.

If the extraction unit is placed in a closed cabinet, the warm exhaust air must be transported out of the cabinet using one of the measures shown in figures A, B, or C at the start of this document.

- A: External air duct (see accessories at www.renfert.com).
- B: Opening in cabinet rear panel, specified dimensions, observe the minimum and maximum distances.
- C: Remove the back panel of the cabinet, distance from the back of the cabinet to the wall min. 50 mm.

Connection

Caution: Risk of injury!

When shortening the suction hose, make sure to cut the integrated wire as straight as possible.

Long suction tubes, tight bends and kinks greatly reduce the suction power at the suction point.

- ⇒ Shorten the suction hose if necessary.
- ⇒ Open the Velcro® strip (7).
- ⇒ Insert the suction hose end bushing (12) into the suction port (6).
- ⇒ Secure the suction hose with the Velcro® strip (7).
- ⇒ Connect the suction hose to the suction point of the CAM system. In doing so, observe the instructions for use of the CAM system.

If the diameter is not suitable, use an adapter (see accessories) to avoid loss of suction.

- ⇒ Insert the interface cable for the CAM system in the CAM interface (8).

To connect the CAM system to the CAM interface (8), see the interface description in the complete instructions for use available online and observe the instructions for the CAM system.

For remote control of the extraction unit, only use the CAM interface provided for this purpose. In the case of CAM systems that only provide a switched mains voltage, it is essential that a type F interface cable is used.

If the extraction unit is very frequently switched on and off using a power supply to the CAM system, this can damage the electronics of the CAM system and of the extraction unit.

Before making the electrical connection, check that the voltage indicated on the nameplate matches the local voltage supply.

- ⇒ Switch off the unit at the on/off switch (3).
- ⇒ Unroll the power cord (9) and insert the power plug into a building electrical socket.

Operation

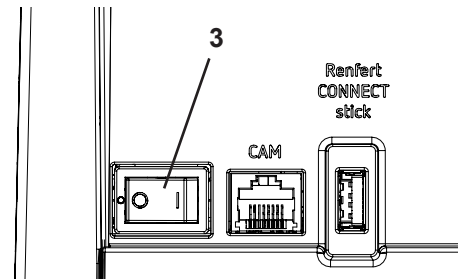
Switching on

The extraction unit is switched on and off via the on / off switch (3).

After activation:

- ◆ The extraction unit performs an automatic filter cleaning sequence (for approx. 8 seconds, loud vibrating noise).

Afterwards, the extraction unit is in the last operating mode set.

Selecting the operating mode: CAM operation / continuous operation

- CAM operation (20):

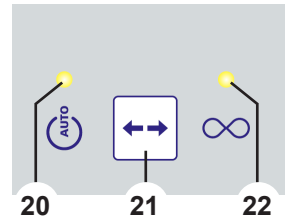
- ◆ The extraction unit responds to the control signals of the CAM system.

- Continuous operation (22):

- ◆ The extraction unit is in continuous operation.

⇒ Press the operating mode key (21).

- ◆ Switching the operating mode.

The extraction unit is suitable for dry dust only!

When connecting to a CAM system with a wet and dry function, ensure that no residual moisture from the CAM system enters the extraction unit.

Always check that the extraction unit is in proper working order before milling:

- ▶ **Error display (26) is off.**
- ▶ **Suction turbine operates correctly without any particularly unusual noise or odor.**

Cleaning / Maintenance

The unit does not contain any internal parts that require maintenance. Opening the equipment other than as specified below is not permitted!

Cleaning

To clean the unit, only wipe the outside with a damp cloth. Do not use any solvent-based or abrasive cleaning agents.

Emptying the dust drawer

After a set time interval is reached (see Chap. 6.2.1), you are prompted to empty the dust drawer:

- ◆ An acoustic signal sounds three times.
- ◆ The dust drawer display (24) lights up.

The next time the extraction unit comes to a stop, an acoustic signal sounds three times after which the filter is cleaned.

Once the filter has been cleaned:

- ⇒ Release the clamp fastener (4A) of the dust drawer.
- ⇒ Pull the dust drawer (4) forward and empty.

During disposal, observe local provisions and accident prevention regulations!

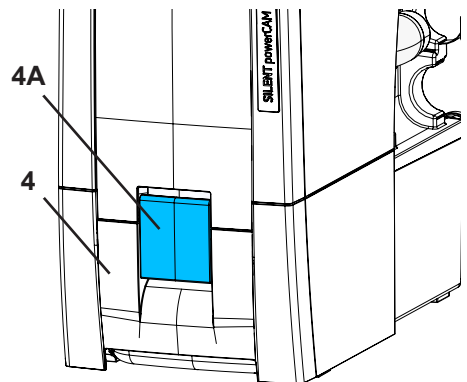
Personal protective equipment must be worn depending on the extracted material.

- ⇒ Insert the dust drawer again and push it all the way in.
- ⇒ Close the clamp fastener (4A).
- ⇒ Press the Enter key (25). The counter for recording the time interval is reset.
 - ◆ An acoustic signal confirms your input.
 - ◆ The dust drawer display (24) switches off.

If the dust drawer is not emptied, the dust drawer display (24) remains lit up. After switching the unit on and off, an acoustic signal sounds three times again to indicate that the dust drawer must be emptied. If heavy extracted material (e.g., zirconia dust), 2 waste bags are recommended (bag-in-bag solution) for safe removal and safe transport.

To set the time interval, consult the instructions for use available online.

Replacing the fine filter

If cleaning the filter does not have a sufficient impact (suction power has not improved), the fine filter must be replaced.

This is detected by the extraction unit and indicated by the error display (26).

! If you are prompted to replace the fine filter, or if the suction power does not significantly improve or only improves temporarily despite cleaning the filter several times, the fine filter must be replaced as soon as possible. Further operation can lead to damage to the unit.

► In general, the fine filter should be changed every 2 years.

When inserting the new fine filter, pay attention to correct seating, as otherwise leaks will occur. For this purpose, see the assembly instructions at the end of the instructions for use, which are also included with the new fine filter.

Spare parts

You can find wear parts and spare parts in the spare parts list available online at www.renfert.com/p918.

Enter the following item number: 29380000.

The components excluded from the warranty (wear parts, consumables) are marked in the spare parts list.

Disposal information

Equipment with this symbol may not be disposed of in unsorted domestic waste within the European Union.

Please contact your local authorities for more information on proper disposal.

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declare under our sole responsibility, that the product(s)

Name	Part No.
SILENT TS	2921 0050
SILENT TS2	2930 0050
SILENT TC	2935 0000 / 2935 0000 03
SILENT TC2	2936 0000 / 2936 0000 03
SILENT EC2	2937 0000 / 2937 0000 03
SILENT compact	2934 0000 / 2934 0000 03
SILENT compactCAM	2934 2000 / 2934 2000 03 2934 2500 / 2934 2500 03
SILENT powerCAM EC	2939 0000
SILENT powerCAM TC	2938 0000
iVAC silent	36 2934 2000

is (are) in conformity with the relevant regulatory requirements by compliance with the UK designated standards.

UK legislation

Electromagnetic Compatibility Regulations 2016

Supply of Machinery (Safety) Regulations 2008

The Restriction of the Use of Certain Hazardous Substances in Electrical and Electronic Equipment Regulations 2012

Designated standards

EN 61326-1:2013

EN ISO12100:2010

EN 61010-1:2010/A1:2019/AC:2019-04

EN IEC 63000:2018

Tilo Burgbacher,
Engineering Director

Hilzingen, 02.11.2022

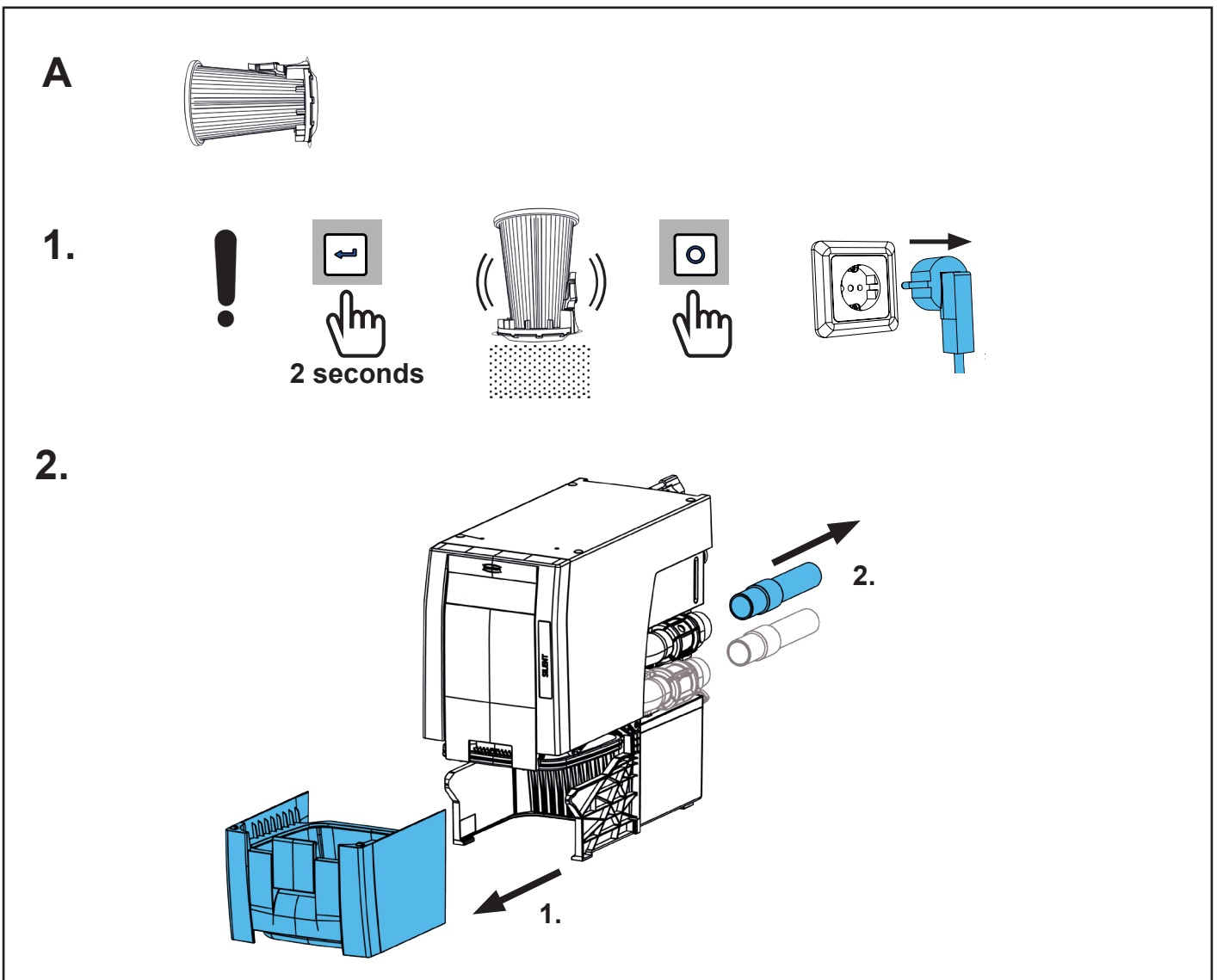
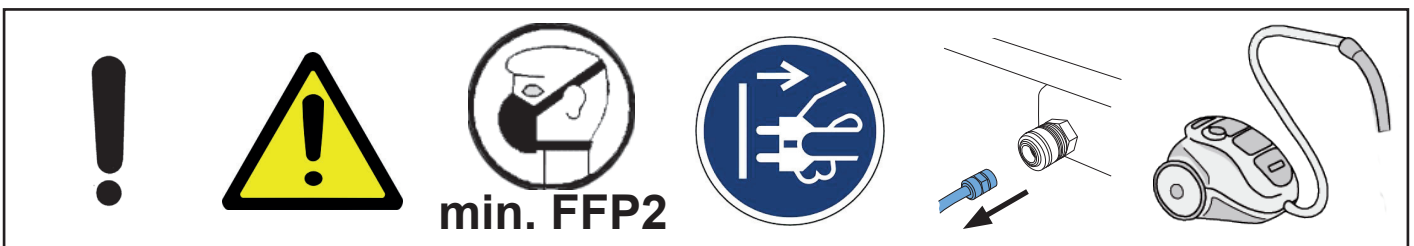
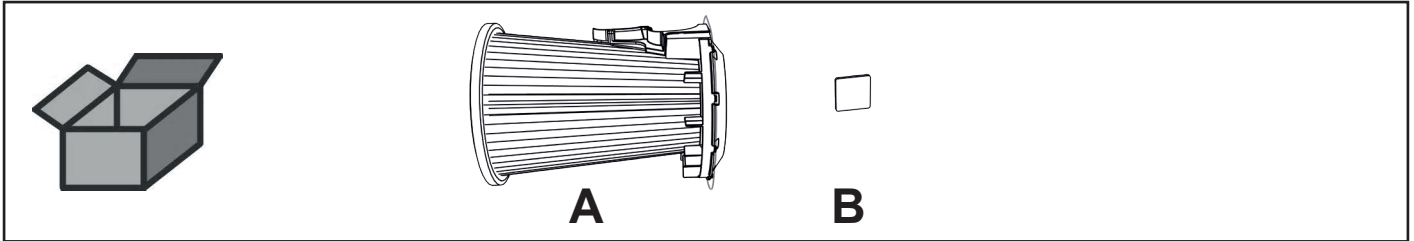
The Technical Documentation has been retained by Renfert GmbH


making work easy

90002 1532



Feinfilter SILENT TC • Fine filter SILENT TC • Filtre fin SILENT TC
Filtro fino SILENT TC • Filtro fino SILENT TC • Фильтр тонкой очистки SILENT TC

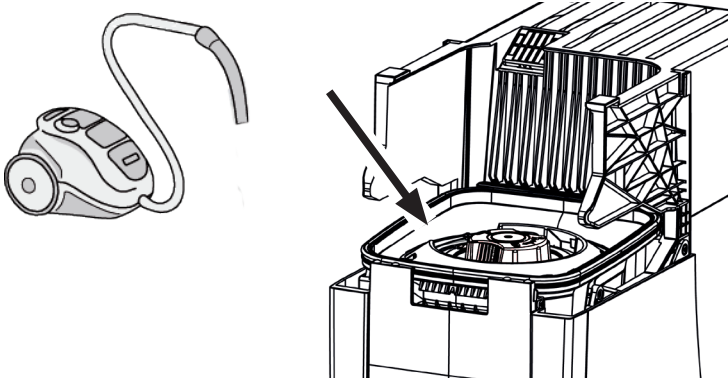
3.

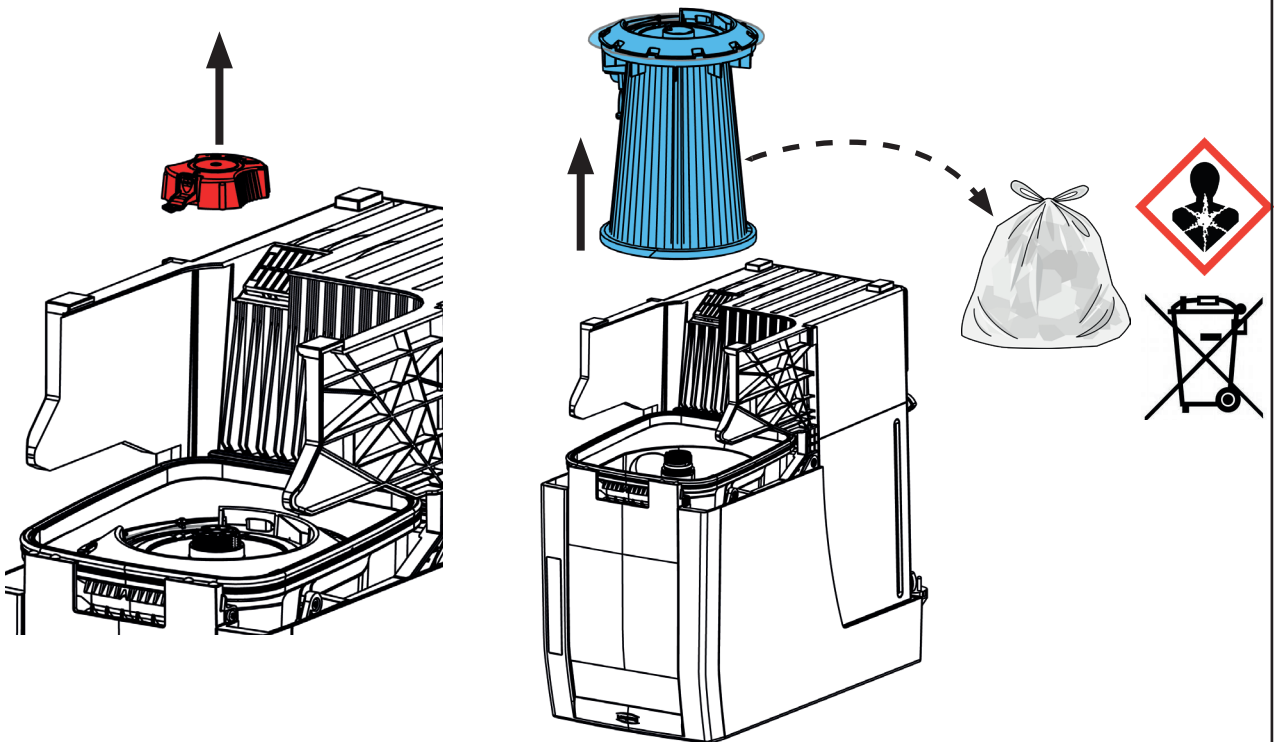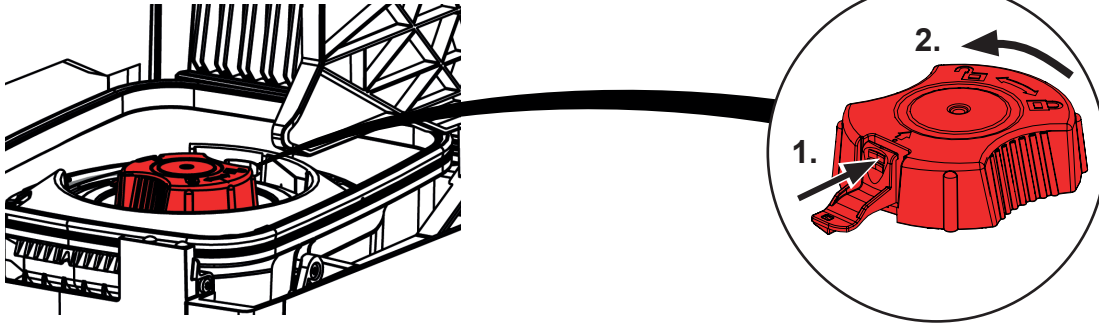
4.

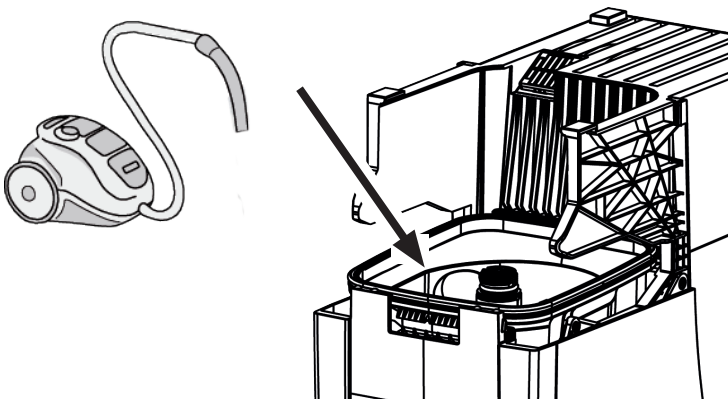
5.

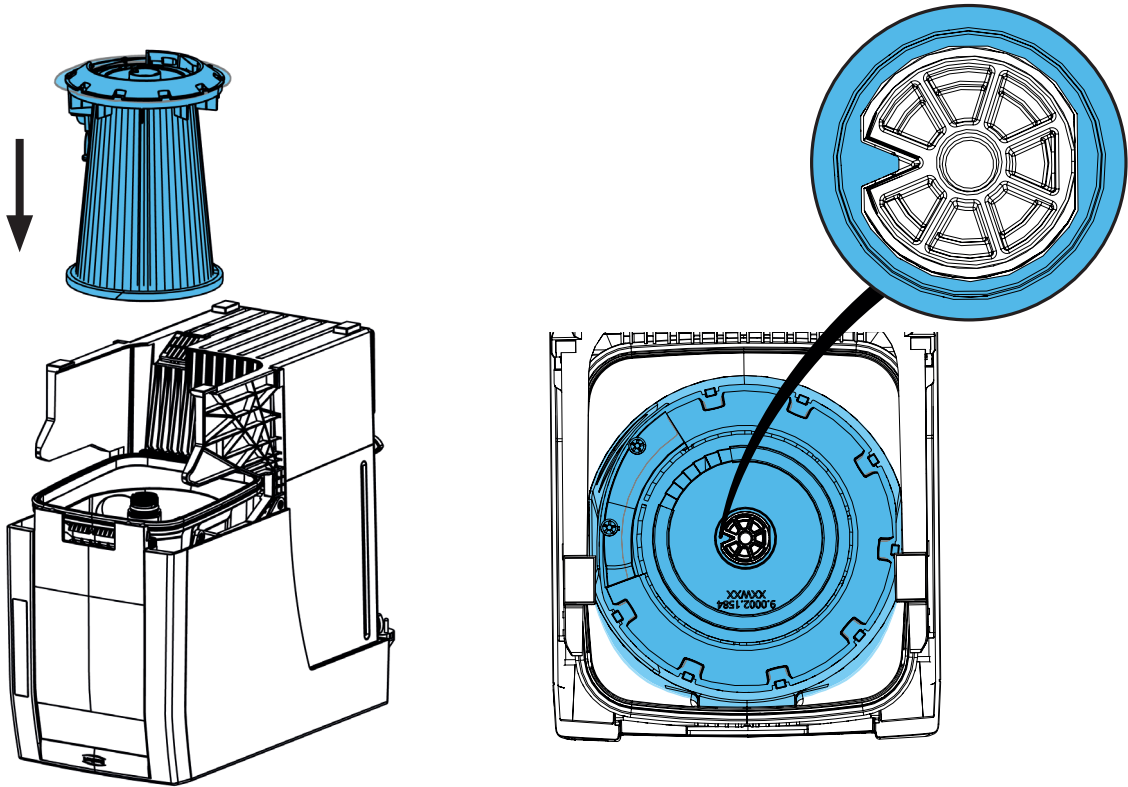6.

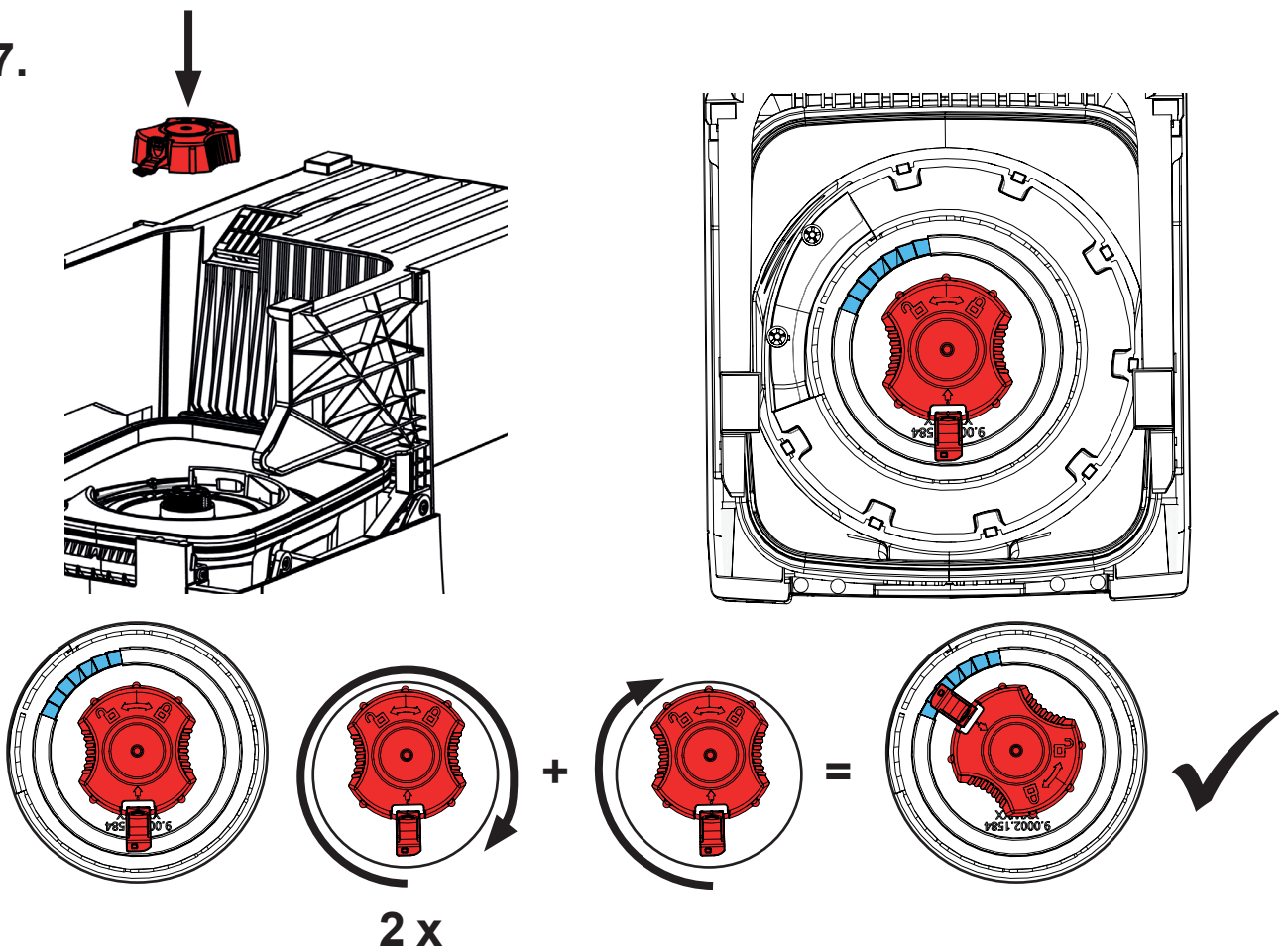
7.

B

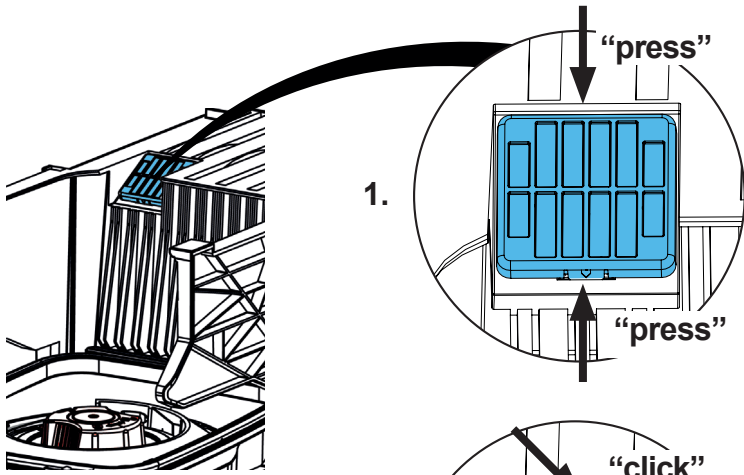8.

1.

"press"

"press"

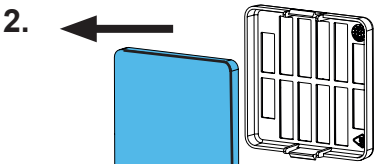
4.

"click"

"click"

2.

3.



closed surface

open surface

i

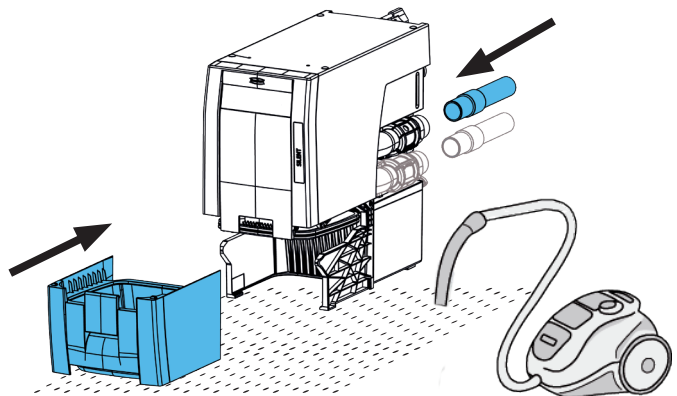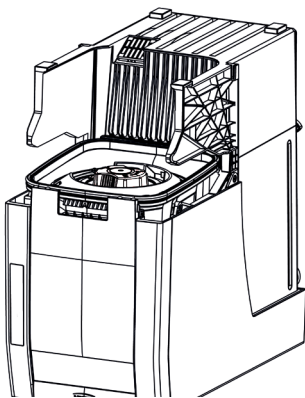9.

Technical data

	SILENT powerCAM TC	
	2938 0000	2938 1000
Nominal voltage:	230 V	120 V
Permissible mains voltage:	220 - 240 V	120 V
Mains frequency:	50 / 60 Hz	50 / 60 Hz
Power consumption *):	850 W	800 W
Mains input fuse:	15 A (T)	
LpA **) (at max. volume flow):	54.3 dB(A)	
Ø suction port: - inner	42 mm [1.65 inches]	
Volume flow, max. ***):	3980 l/min [2.34 ft ³ /s]	
Vacuum pressure, max. ****):	281 hPa [4.1 psi]	
Fine filter: - Filter surface, approx. - Filter quality	0.9 m ² [1390 sq inch] Class M in accordance with EN 60335-2-69	
Fill volume dust drawer, approx.:	7 l [1.85 US gal]	
Weight (empty), approx.:	22.0 kg [48.5 lbs.]	
Dimensions (width x height x depth):	275 x 535 x 540 mm [10.8 x 21.1 x 21.2 inches]	
CAM interface:	RJ45 port	
Input signal: - Low - High	0 - 5 V 15 - 24 V	
Output signal: - Max. pull-up voltage (U) - Max. switching current (I) - Internal series resistance	24 V 5 mA 150 Ohm	

*) Power consumption at nominal voltage

**) Noise level in accordance with EN ISO 11202

***) Free-blowing turbine at nominal turbine voltage

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Renfert GmbH • Untere Gießwiesen 2 • 78247 Hilzingen/Germany