making work easy

- DE
- EN
- FR
- IT
- ES
- PT
- TR
- RU
- CS
- UK
- ZH
- JA
- KO
- AR


Waxprofi

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
Made in Germany

21-9902 15112022


1a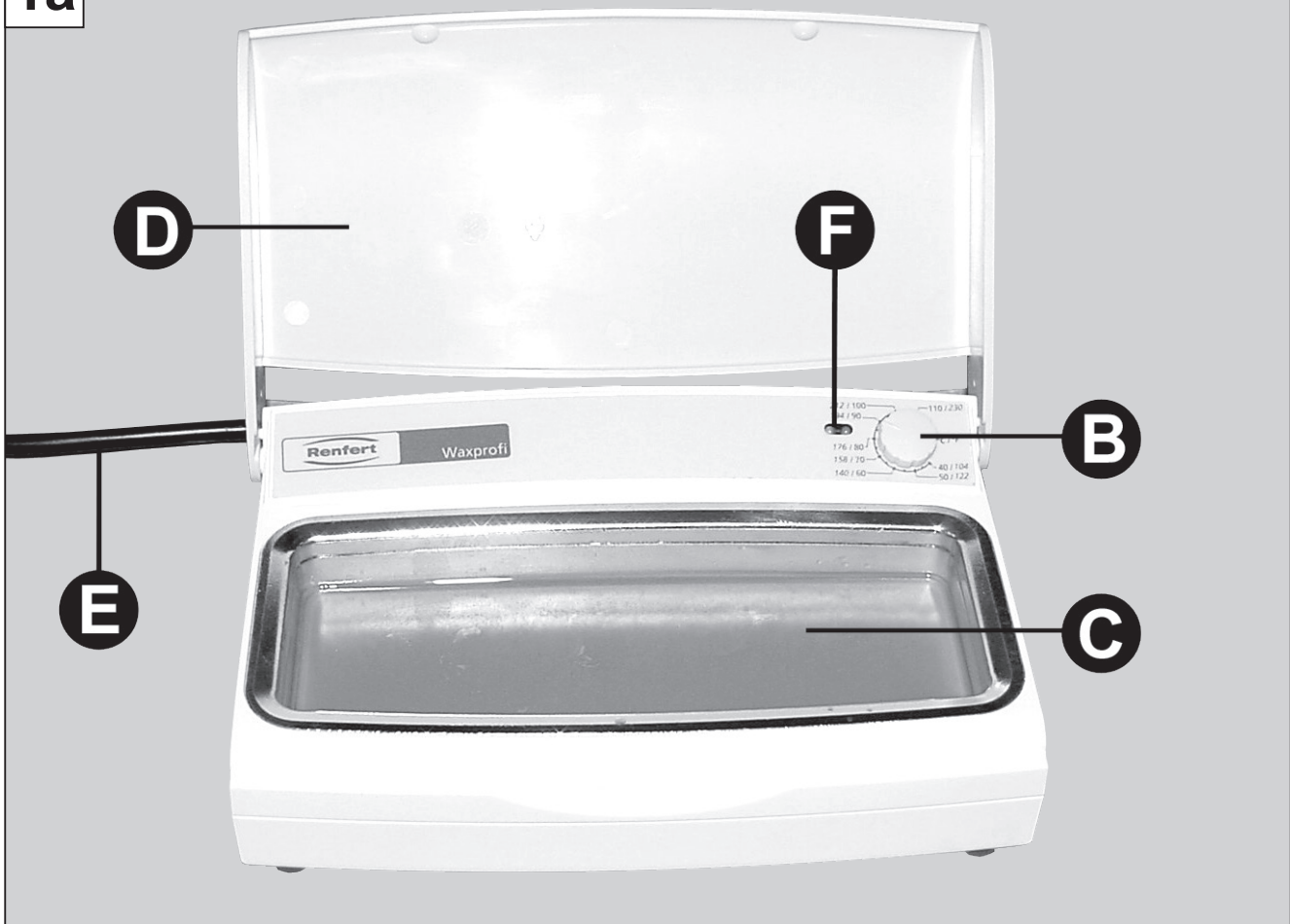


1b

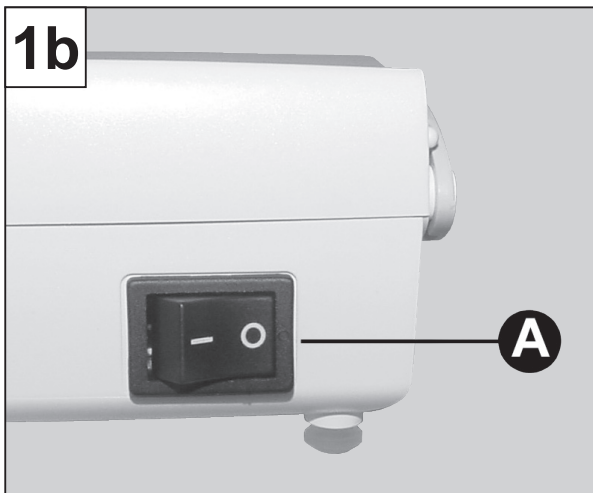
2

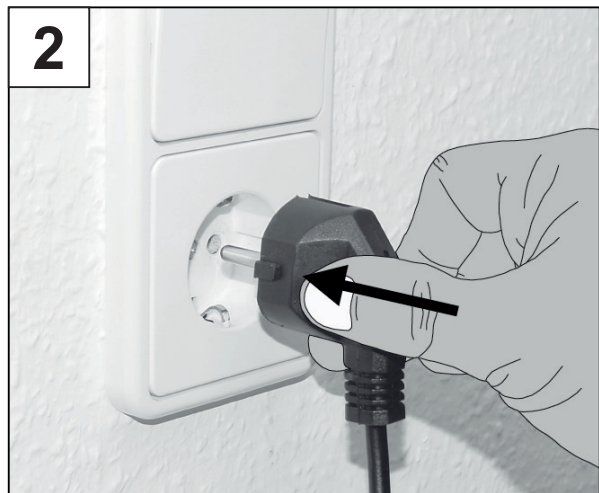3

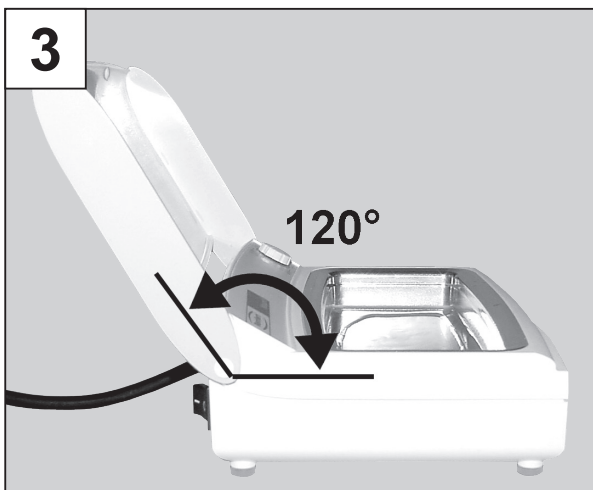
4a

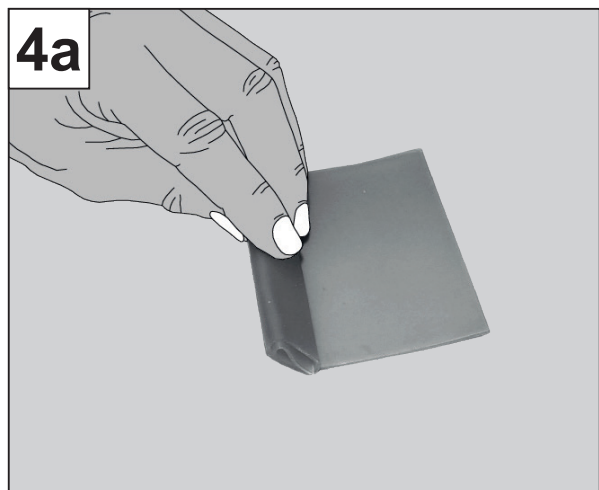4b


5

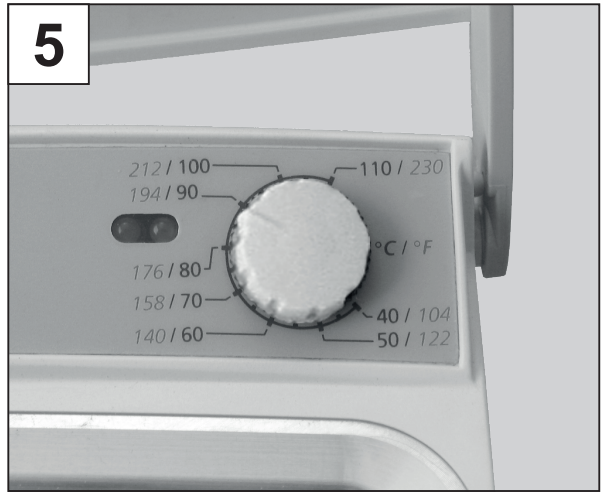6

7

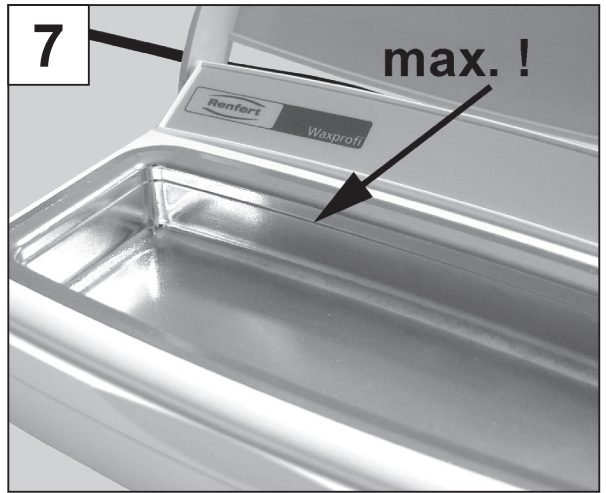1 Einleitung

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1 Introduction

We are pleased with your decision to purchase a *Waxprofi* denture wax heating unit. Please observe the following information in order to ensure long and trouble-free service.

2 Application Area

The Waxprofi wax heating unit is used to heat base waxes to their ideal processing temperature for full and partial dentures.

In the large pot, the wax is held at a liquid, viscous, or spreadable consistency, as required, ready for further processing with the preheated wax knife and Bunsen burner or an electronic waxing-up instrument.

2.1 Ambient Conditions

(in accordance with IEC 60664-1)

The *Waxprofi* may only be operated:

- Indoors.
- Up to an altitude of 2,000 m (6,500 ft) above sea level.
- At an ambient temperature of 5 – 40 °C (41 – 104 °F).
- At a maximum relative humidity of 80 % at 31 °C (87.8 °F), decreasing linearly to 50 % relative humidity at 40 °C (104 °F).
- With a mains power supply where current fluctuations do not exceed 10 % of the nominal value.
- Under Level 2 contamination conditions.
- Under overvoltage category II conditions.

3 Safety Information

3.1 Symbols Used

The following symbols are employed in these instructions and on the unit:

Danger
An imminent danger of injury exists.

Electrical Power
Danger of electrical shock.

Attention
Failure to observe this information can result in damage to the unit.

Note
This is a note intended to provide the operator with useful information to make operation easier.

The device complies with the requirements of the applicable EU directives.

This product complies with the relevant UK legislation.
See UKCA Declaration of Conformity in the Internet under www.renfert.com.

The device is subject to the EU directive 2002/96/EG (WEEE directive).

For indoor use only.

Before opening the unit, disconnect it from the power supply by pulling the wall plug.

3.2 Safety Information

- ▶ The *Waxprofi* is an electrical device with potential hazards. This unit may only be locked and used by authorized specialist personnel after an examination with regard to conformity with applicable national standards.
- ▶ Before starting the unit, compare the voltage information on the nameplate with the specifications of your local power supply.
- ▶ Units with damaged power cords or other defects may not be operated.
- ▶ During operation, always unroll the power cord (overheating hazard in case of a short circuit).
- ▶ Liquid waxes can cause burns. Never move the unit while it is operating!
- ▶ During operation, the wax pot are heated up to its rim. Improper use or handling can result in burns (for up to 30 min. after the unit is switched off).
- ▶ The *Waxprofi* is designed to be used exclusively in conjunction with commercially available dental waxes. Never use the unit to heat other types of materials!
- ▶ Please note: Overfilled pots can overflow!
- ▶ Materials whose flash point is under 175 °C (347 °F) may not be stored in proximity to the *Waxprofi* or come in contact with the *Waxprofi*.
- ▶ Never rinse the unit with water or submerge it. The mains power will result in electric shock!
- ▶ It is the responsibility of the operator that national regulations during operation and regarding a repeated safety inspection of electrical equipment are complied with. For Germany these are the regulation 3 by DGUV (German Statutory Accident Insurance) in relation with VDE 0701-0702 (Association for Electrical, Electronic and Information Technology).
- ▶ Only spare parts and accessories supplied or authorized by Renfert GmbH may be used with this product. If other spare parts or accessories are used, this could have a detrimental effect on the safety of the device, increase the risk of serious injury and lead to damage to the environment or the device itself.
- ▶ Information on REACH and SVHC is available on our website www.renfert.com, in the Support area.

3.3 Liability Exclusion

Renfert GmbH shall be absolved from all claims for damages or warranty if:

- ▶ The product is employed for any purposes other than those cited in the operating instructions.
- ▶ The product is altered in any way other than those alterations described in the operating instructions.
- ▶ The product is repaired by other than an authorized facility or if any but Renfert OEM parts are employed.
- ▶ The product continues to be employed, despite obvious safety faults.
- ▶ The product is subjected to mechanical jarring or is dropped.

4 Operating Elements

(Figure 1a + 1b)

- A) On / Off switch
 - B) Rotary potentiometer for temperature adjustment
 - C) Wax pot
 - D) Lid
 - E) Power cord
 - F) LED indicator
- >> Operating indicator = green;
>> Heating phase = red

5 Commissioning / Operation

1. Connect the *Waxprofi* to the mains power outlet (Figure 2).
2. Open lid (D). The lid can be locked in place at a 120° angle (Figure 3).
3. It may be necessary to fold wax sheets (Figure 4a).
4. Add the waxes to be heated to the pot (Figure 4b).

Fill up the pot only to the circular marking (Figure 7)!

Folding or breaking the denture wax sheets to fit the pot volume is the most practical method. When operating the unit for the first time, fill the wax pot in stages.

5. Switch the *Waxprofi* on at switch (A).

5.1 Setting the Temperature

The desired temperature is adjusted at the rotary potentiometer (B). The scale divisions are identified in both °C (blue) or °F (red) (Figure 5).

The temperature is infinitely adjustable within the range is between 40 - 110 °C (104 - 230 °F).

We recommend the following procedure when commissioning the *Waxprofi* in order to obtain the desired wax consistency:

1. Set the *Waxprofi* temperature to 15 - 20 °C (59 - 68 °F) above the liquefaction point of the base wax.
2. Depending on the desired consistency, reduce the *Waxprofi*'s temperature as follows:

a) Liquid consistency:

Once the wax has completely melted, turn the *Waxprofi*'s temperature down to the liquefaction point of the wax.

b) Viscous or spreadable consistency:

Once the wax has liquefied to the point where it fills the pot to approx. the $\frac{3}{4}$ -full level (Figure 6), reduce the *Waxprofi*'s temperature in stages until the desired consistency is obtained.

5.2 Waxprofi + Bunsen Burner

When processing wax with the *Waxprofi* in conjunction with a Bunsen burner, the wax knife need only be briefly heated in the burner flame since the already heated wax in the *Waxprofi* requires hardly any additional heat in order to be processed.

This saves both time and makes handling the base wax significantly easier.

5.3 Waxprofi + Electric Wax Knife

Match your *Waxprofi* to your electric wax knife. This offers the best prerequisites for quickly and precisely performing the wax tasks for full and partial dentures.

Teeth can set up and gums can be modelled without the delays inherent in the melting time, and without the distraction of the burner flame. The wax is never overheated. This keeps contraction to a minimum, thus saving the time required for subsequent corrective work or allowing you to complete your set up more rapidly. Good results can be obtained with Renfert's *Waxlectric I or II* electric waxing-up instruments. Using the large tip (Art. No. 2155-0105) or the chamfer tip (Art. No. 2155-0106) we recommend setting the *Waxlectric* to a temperature of 130 - 150 °C (266 - 302 °F).

Please note:

If the power supply is interrupted (e.g., as the result of a mains power failure) while the *Waxprofi* is switched on, the unit returns to its previous operating mode once power is restored.

Notes / Tips on Operation:

- The *Waxprofi* can also be used for modelling wax or light wax.
- Refill the pot regularly. Renfert recommends GEO pink modelling wax, No. 647-0000.
- Close the lid to reduce the heating time.
- Do not overheat the wax in the *Waxprofi*.
- Use the inner rim of the pot to strip off excess wax.
- When working with wax in a viscous or spreadable consistency, do not close the lid during short work interruptions, otherwise the trapped heat will completely liquefy the wax.
- When working with the *Waxlectric*, good results can be obtained with tip numbers 2155-0104, -0105, -0106, -0110, and -0111 at a temperature of 130 - 150 °C (266 - 302 °F).
- Some waxes may form a surface skin with reduced temperatures. The skin can be easily dissolved by means of stirring.
- To obtain a viscous or spreadable consistency, heat the wax until a majority has been liquefied (approx. $\frac{3}{4}$ of the pot content), then turn down the *Waxprofi*'s temperature until the desired consistency is achieved.
- A uniform consistency can be obtained more quickly by stirring the pot contents.
- Some waxes may separate, that is, individual colour pigments separate out of the base material. Simply stir the wax to return it to its original, homogenous state.
Exceptions to this are when the *Waxprofi* is used in conjunction with:
 - Solidus 84 (Yeti);
 - "Hardy" modelling wax (Gebdi).

6 Cleaning and Maintenance

The *Waxprofi* is maintenance free.

From time-to-time, you should wipe your unit down with a damp cloth.

Do not use solvents or cleansers.

Never rinse or spray your *Waxprofi* directly with water!

If the unit switched off, you can remove hardened wax residue by carefully scraping it.

Warming the unit's surfaces with a hair drier will make it easier to remove wax residue.

The *Waxprofi* lid can be removed, allowing dirt and other contaminants to be easily cleaned under a steam jet.

7 Spare Parts

You can find components subject to wear and the spare parts on the spare part list in the internet at www.renfert.com/p918.

Enter the following item number: 14400000 or 14401000.

The components excluded from the warranty (such as consumables or parts subject to wear and tear) are marked on the spare part list.

Serial number and date of manufacturing are shown on the type plate of the unit.

8 Warranty

Provided the unit is properly used, Renfert **warrants** the *Waxprofi* for a **period of 3 years**.

The original sales receipt from the dealer must be provided when making warranty claims. Parts which are subject to natural wear and tear (wear parts) and consumables are excluded from the guarantee. These parts are marked in the spare part list.

The warranty is voided in case of improper use; failure to observe the operating, cleaning, maintenance, and connection instructions; in case of independent repairs or repairs by unauthorized personnel; if spare parts from other manufacturers are employed, or; in case of unusual influences or influences not in compliance with the utilization instructions. Warranty service shall not extend the original warranty.

9 Technical data

Mains voltage:	220 - 240 V~, 50/60 Hz 100 - 120 V~, 50/60 Hz
Power consumption:	120 - 140 W (220 - 240 V~) 120 W (100 - 120 V~)
Minimum operating temperature:	40 °C [104 °F]
Maximum operating temperature:	110 °C [230 °F]
Dimensions	178 x 54 x 120 mm (W x H x D): [7.01 x 2.13 x 4.72 inches]
Pot size:	185 ml [6.26 fl oz]
Usable pot volume:	110 ml [3.72 fl oz]
Weight, approx.:	600 g [21.2 oz]

10 Standard Delivery

1 *Waxprofi* denture wax heating unit

1 Operating instructions

11 Accessories

Renfert recommends the following:

Electric wax knife:

Waxlectric I, incl. 1 handpiece and 1 modelling insert:

No. 2156-0000 230 V

No. 2156-1000 120 V

Waxlectric II, incl. 2 handpieces and 2 modelling insert:

No. 2157-0000 230 V

No. 2157-1000 120 V

Waxlectric tips:

No. 2155-0104 1 blade, narrow

No. 2155-0105 1 blade, wide

No. 2155-0106 1 blade, chamfer

No. 2155-0110 1 blade, lance

Bunsen burner:

Bijou 90:

No. 960-0000 Liquid gas

No. 961-0000 Natural gas E

Safety burners:

No. 933-0100 Liquid gas

No. 932-0100 Natural gas E

Safety burners eco:

No. 934-0100 Liquid gas

No. 935-0100 Natural gas E

Wax:

Nr. 625-0500 GEO special, modelling wax, 500 g

Nr. 625-2500 GEO special, modelling wax, 2500 g

For additional waxes, please refer to the Renfert Catalogue or go to www.renfert.com

12 Disposal instructions for countries in the EU

To conserve and protect the environment, prevent environmental pollution and improve the recycling of raw materials, the European Commission adopted a directive that requires the manufacturer to accept the return of electrical and electronic units for proper disposal or recycling.

Within the European Union, units with this symbol should not therefore be disposed of in unsorted domestic waste.

For more information regarding proper disposal please apply at your local authorities.

13 Troubleshooting List

EN

Error	Possible cause	Corrective action
No display / no operation, although the unit is switched on the main switch (A).	<ul style="list-style-type: none">• Power supply interrupted.• Faulty unit fuse.	<ul style="list-style-type: none">• Check the power supply and power cords.• Have fuse replaced by Service Department.
The unit is operating but there is no increase in temperature.	<ul style="list-style-type: none">• Faulty thermal sensor.• Faulty heater.	<ul style="list-style-type: none">• Have fuse replaced by Service Department.• Have heater repaired by Service Department.

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declare under our sole responsibility, that the product(s)

Name	Part No.
Hotty	1460 0000 / 1460 0000 03
Hotty LED	1461 0000 / 1461 0000 03
Vario E	1452 0000 / 1452 0000 03
Waxprofi	1440 0000 / 1440 0000 03
Waxlectric I & II	2156 0000 / 2156 0000 03 2157 0000 / 2157 0000 03
Waxlectric light I & II	2150 0000 / 2150 0000 03 2151 0000 / 2151 0000 03

is (are) in conformity with the relevant regulatory requirements by compliance with the UK designated standards.

UK legislation

Electromagnetic Compatibility Regulations 2016

Electrical Equipment (Safety) Regulations 2016

The Restriction of the Use of Certain Hazardous Substances in Electrical and Electronic Equipment Regulations 2012

Designated standards

EN 61326-1:2013

EN 61010-1:2010/A1:2019/AC:2019-04

EN IEC 63000:2018

Tilo Burgbacher,
Engineering Director

Hilzingen, 02.11.2022

The Technical Documentation has been retained by Renfert GmbH

**Hochaktuell und ausführlich auf ...
Up to date and in detail at ...
Actualisé et détaillé sous ...
Aggiornato e dettagliato su ...
La máxima actualidad y detalle en ...
Актуально и подробно на ...**

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Renfert GmbH, Industriegebiet, 78247 Hilzingen / Germany
Renfert GmbH • Untere Gießwiesen 2 • 78247 Hilzingen/Germany