Remberti

No. 1262 0001

ENGLISH

## 1. Description

You decided to purchase a high quality product from Renfert. Thank you for the confidence shown.

The special features of this product are:

- mineral glass, coated lenses
- precisely defined degree of magnification (1.5 x to 1.75 x)
- distance from working area: 12–35 cm, no blurred areas, very variable
- weight: approx. 38 g, complete with lenses, extremely light
- frame: silver chrome plated

In order to ensure that your Remberti glasses last a long time, please refer to the following instructions.

## 2. Preparation

In order that your preparation glasses fit well, you can either

- a) make the radius of the bow smaller or larger by bending it, or
- b) change the spacing between head and bow by bending it, or
- c) shorten the bows in case of an extremely short distance between ear and nose.

In this case, hold the plastic protections of the bows at the end and take them off whilst turning. Then, shorten the end of the wire by some millimeters and reattach the plastic protections.


This is especially important for those who usually wear spectacles, as they can adapt Remberti glasses to their everyday spectacles. This prevents both pairs of glasses interfering with one another.

## 3. Handling

The latch mechanism can be locked in three positions.


### Position 1:

Working position – magnifies the area being worked on.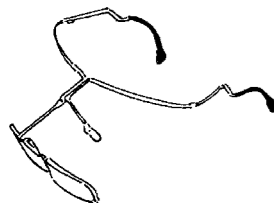


### Position 2:

First latch setting – does not magnify, lenses opened for short-term use.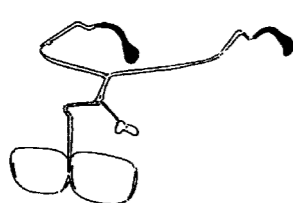

### Position 3:

Lenses opened completely, for longer discussions with the patient or during breaks.

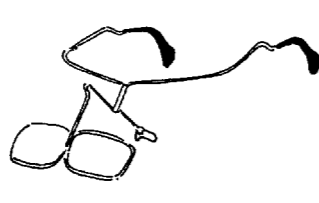## 4. Cleaning / Care

You should clean your Remberti glasses regularly with a lens cleaning cloth. Please **do not use any aggressive cleaning solvent**, as this could wear off the chrome-plating.

## 5. Spare parts

Replacement lenses are available as follows:

- No. 12620100 left, 1 piece  
No. 12620200 right, 1 piece

An optician should replace the lenses in order to ensure that they are aligned correctly.

## 6. Guarantee

Your Remberti glasses are a quality product, meeting the highest standards in material and design. We guarantee each pair supplied to be of the highest standard.

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