making work easy

- de
- en
- fr
- it
- es
- pt
- tr
- ru
  
- zh
- ja
- ko
- ar

# SILENT TC

Quick Start Guide

Made in Germany

21-6832 13022023

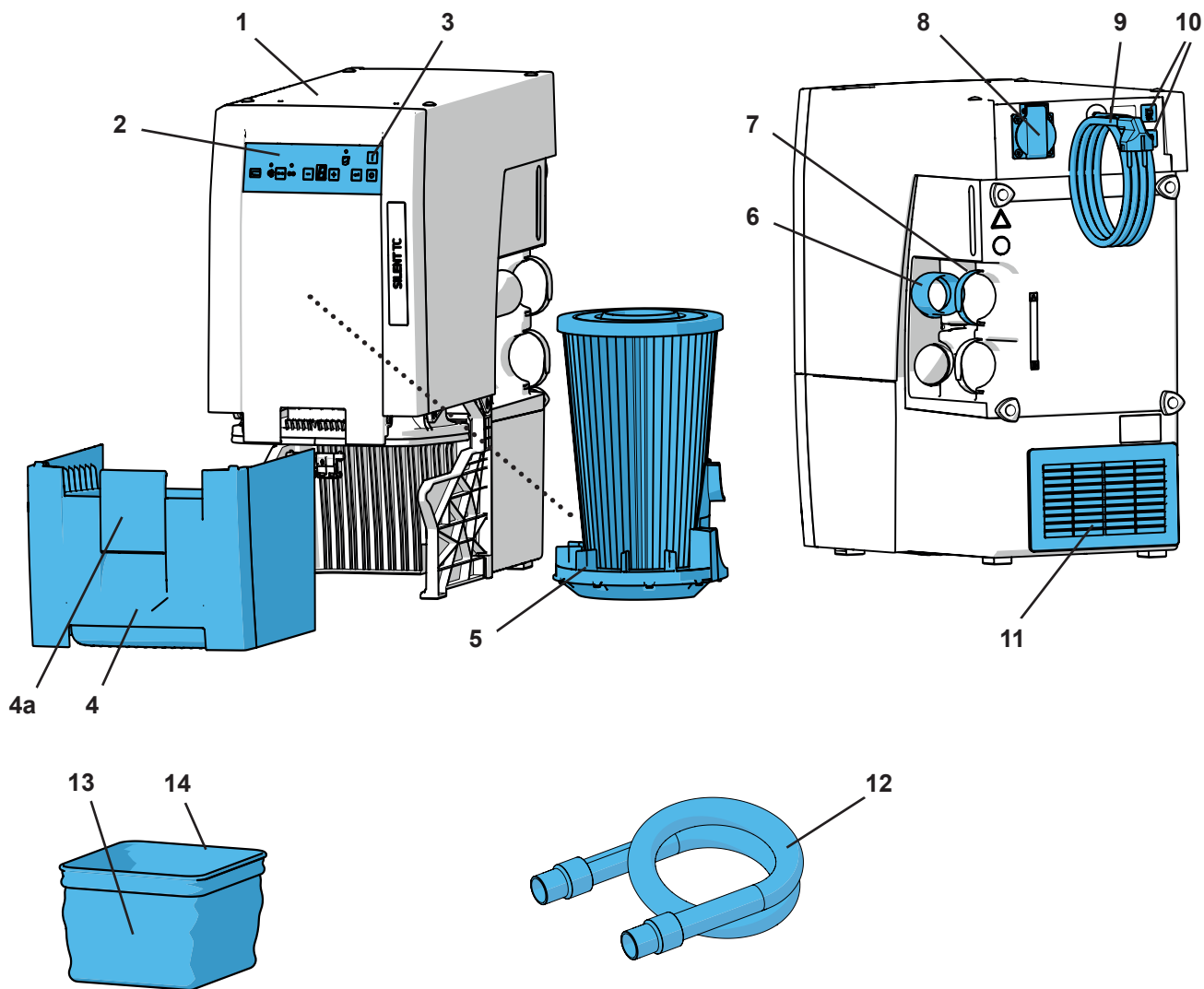
Fig. 1


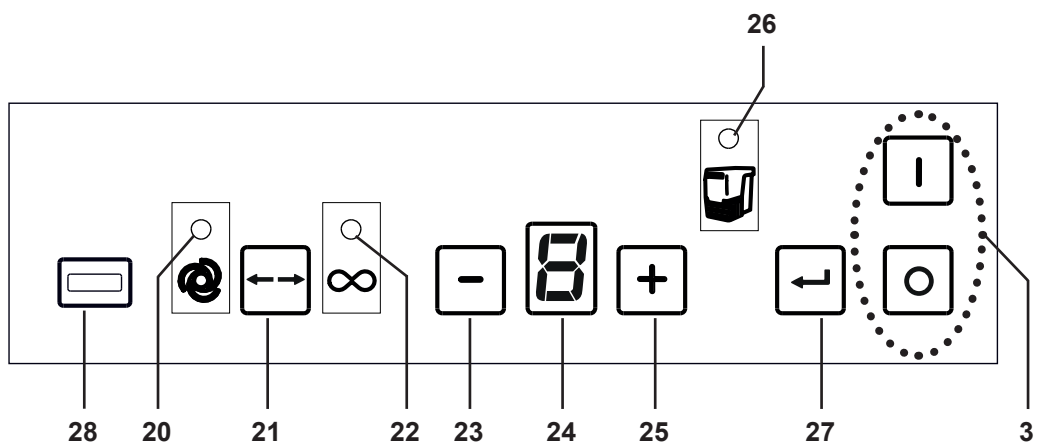
Fig. 2

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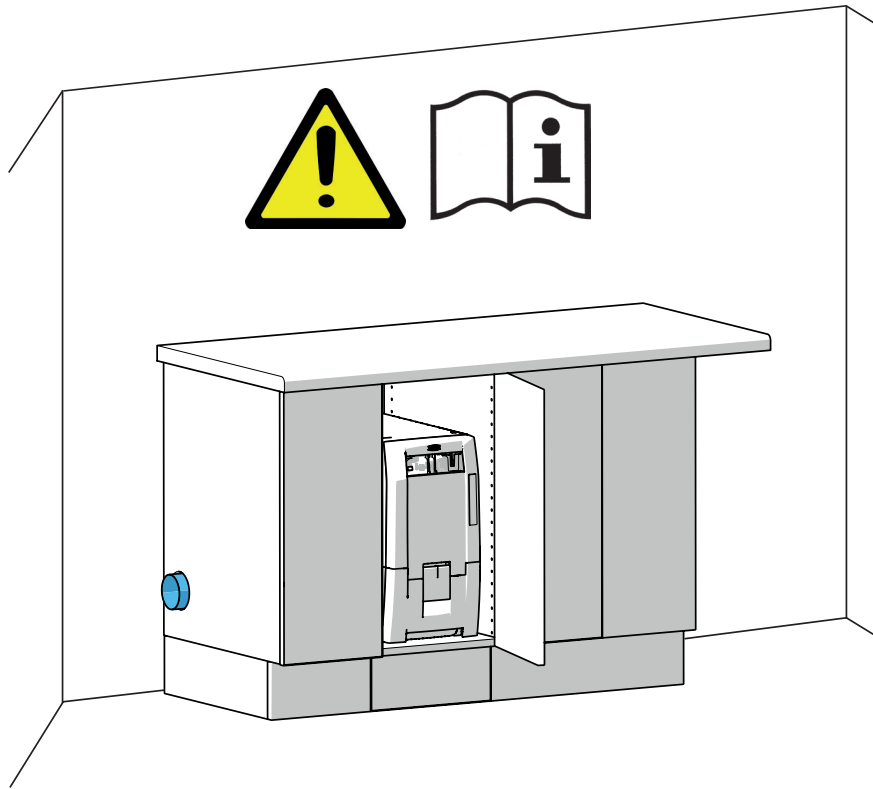# Introduction

This Quick Start Guide contains instructions on the operation of the product and relevant safety information.

For detailed information about the device, see the instructions for use which you can download from our website at [www.renfert.com/p915](http://www.renfert.com/p915).

Enter the following item number: 29350000.

## Symbols used

In the instructions for use and on the unit itself you will find these symbols with the following meanings:

### Danger

**Immediate risk of injury. Consult accompanying documents!**

### Electrical current

**This indicates a risk of injury due to electrical current.**

### Caution

**Risk of damage to the unit if the instruction is not followed.**

### Notice

**A general notice that improves and eases use.**

**The appliance complies with applicable EU directives.**

**This product complies with the relevant UK legislation.**

**See UKCA Declaration of Conformity in the Internet under [www.renfert.com](http://www.renfert.com).**

**Within the EU, this device is subject to the provisions of the directive 2002/96/EC (WEEE directive).**

Other symbols are explained as they occur.

## Safety

### Intended Use

This device is designed to extract dry, non-explosive dust.

The unit is intended solely for use in a commercial dental laboratory and dental practice.

The intended use also includes compliance with the instructions specified by the manufacturer concerning operation, servicing and maintenance.

The extraction unit can be used in combination with a SILENT flow sensor within the scope of measures to protect against exposure to dust \*) as defined by the German ordinance on hazardous substances ("GefStoffV") / respective employers' liability insurance association.

For this purpose, the information on low-dust disposal of the extracted material (chapter "Emptying the dust drawer") and on low-dust replacement of the fine filter (chapter "Replacing the fine filter") in particular must be observed.

**In doing so, both the instructions for use of the extraction unit as well as those of the SILENT flow sensor must be observed.**

\*) Note for customers in Germany: A validated and recognized dust protection measure in accordance with testing principle GS-IFA-M20 (issued by the German Institute for Work and Health IFA) shall only be deemed to have been provided when all components, i.e. the flow rate monitoring unit such as the SILENT flow sensor, the extraction unit, and the detection unit, are validated and recognized.

In the case of multi-workbench extraction units and the use of Y-junctions or extractor switches, every suction channel / every suction point must be fitted with a flow rate monitoring unit.

### Improper Use

**The equipment is not suitable for use with dental CAM units!**

**For information concerning SILENT extraction units for CAM machines, please refer to [www.renfert.com](http://www.renfert.com).**

Fire-promoting, easily flammable, red-hot, burning or explosive materials must not be suctioned into the device. It is not permitted to suction liquids. This device is not intended for private, household use.

Any use other than specified in these instructions is deemed improper and constitutes a misuse of the device. The manufacturer shall not be liable for damages caused by improper use.

Only spare parts and accessories supplied or authorized by Renfert GmbH may be used with this product. If other spare parts or accessories are used, this could have a detrimental effect on the safety of the device, increase the risk of serious injury and lead to damage to the environment or the device itself.

## Hazard and Warning Information

### General Information

- ▶ If the device is not operated according to the enclosed documents, the provided protection is no longer guaranteed.
- ▶ The device may only be operated using a mains cable with the country-specific plug system. Any necessary alterations must be carried out by a qualified electrician.
- ▶ The device may only be operated if the information on the identification plate conforms to the specifications of your local mains power supply.  
After removing the Dust Drawer, you can find the identification plate below, inside the device on the left side.
- ▶ The device may only be plugged into outlets which are connected to the protective conductor system.
- ▶ The mains plug must be easily accessible.
- ▶ Disconnect the device from the mains before carrying out work on the electrical parts.
- ▶ Regularly check connecting cables (such as power cables), hoses and housing (for example, the control panel) for damage (for example, kinks, tears, porosity) or deterioration due to aging. Do not operate units with damaged connection cables, hoses or housing parts or other defects!
- ▶ Defective devices must be put out of service immediately. Disconnect from mains and secure against being switched on again. Send the device in for repair!
- ▶ Only operate the device under supervision.
- ▶ Please observe the national accident prevention regulations!
- ▶ It is the responsibility of the operator that national regulations during operation and regarding a repeated safety inspection of electrical equipment are complied with. For Germany these are the regulation 3 by DGUV (German Statutory Accident Insurance) in relation with VDE 0701-0702 (Association for Electrical, Electronic and Information Technology).
- ▶ Information on REACH and SVHC is available on our website [www.renfert.com](http://www.renfert.com), in the Support area.

### Specific Information

- ▶ The German ordinance on hazardous substances (“GefStoffV”) or equivalent national regulations must be observed during operation, cleaning, and maintenance.
- ▶ The mains socket on the device is only designed for purposes specified in the operating instructions. Connecting other devices may cause material damage.
- ▶ Switch off the electrical device before connecting an electrical appliance to the appliance outlet.
- ▶ Read the operating instructions for the electrical appliance which is to be connected and observe the stipulated safety instructions.
- ▶ Please observe the national regulations and permitted exposure to dust in a working environment. Please ask the “National Institute for Occupational Safety and Health” or other responsible authority.
- ▶ Please observe the Safety Data Sheets of the materials to be extracted.
- ▶ Always wear protective gear, when extracting hazardous materials.
- ▶ It is necessary to wear suitable personal protective equipment when emptying the dust drawer or cleaning, depending on the type of extracted material.
- ▶ When disposing of the extracted material or used filter, please observe the local specifications and accident prevention regulations!
- ▶ Make sure the dust drawer is fully closed during operation.
- ▶ Do not operate without a suction hose.
- ▶ Do not extract flammable or explosive gasses, fumes or dust.
- ▶ The following applications involve increased risks and are therefore not permitted:  
When cleaning milling chambers and SLM printing systems by suction, a sufficiently high concentration and purity (i.e. unmixed with other dental dusts such as gypsum, resins) can result in spontaneous ignition or the milling dust exploding or printing powder because of a potential exothermic reaction (e.g. due to oxidation).  
An alternative cleaning method must be selected for the following milling or printing media in particular (such as manual cleaning):
  - Wood
  - Titanium / titanium-aluminum
  - Light metals and light metal alloys (e.g. aluminum, magnesium)
  - Cobalt chrome powder (e.g. for use in SLM systems)If large quantities of light metals such as titanium alloy are processed (e.g. with sandpaper), producing very fine grinding dust, a sufficiently high concentration and purity may result in spontaneous combustion because of a potential exothermic reaction.
- ▶ Do not extract hot materials.

- ▶ **Do not extract liquids.**
- ▶ **If the dust extractor is employed to extract hazardous materials, appropriate personal protective gear must be worn and steps must be taken to ensure that the exhaust air is properly ventilated. Please refer to the associated safety data sheets for specific requirements.**
- ▶ **Dispose of extracted material according to local statutory regulations.**

## Authorized Persons

Operation and maintenance of the device may only be performed by qualified personnel.

# Product Description

## Component Groups and Functional Elements

See Fig. 1 (on inside cover)

- |                   |  |
|-------------------|--|
| 1 SILENT TC       | 8 Device socket                            |
| 2 Control panel   | 9 Mains cable                              |
| 3 On / off switch | 10 Protective device switch (2 x)          |
| 4 Dust drawer     | 11 Exhaust air filter / Exhaust air outlet |
| 4a Clamp fastener | 12 Suction hose                            |
| 5 Fine filter     | 13 Waste bag (only with 2935 0000)         |
| 6 Suction port    | 14 Retaining ring (only with 2935 0000)    |
| 7 Velcro® strip   |  |

See Fig. 2 (on inside cover)

- |  |   |
|--|---|
| 3 On / off switch  | 24 Display                                  |
| 20 Display Automatic operation                           | 25 [+] -key (increase value)                |
| 21 Start operation key, Automatic / continuous operation | 26 Display empty dust drawer                |
| 22 Display continuous operation                          | 27 Enter key, save input                    |
| 23 [-] key (reduce value)                                | 28 Cover (do not remove, service interface) |

# Initial Operation

## Setting-up

The suction unit is a standing device which must not be operated in a horizontal position.

**!** **If the suction unit is placed in a closed cabinet, the warm exhaust air must be transported out of the cabinet using one of the measures shown in Figs A, B, C at the start of this manual.**

A: External exhaust air duct (see accessories at [www.renfert.com](http://www.renfert.com)).

B: Opening in cabinet rear panel, specified dimensions, observe the minimum and maximum measurements.

C: Remove the back panel of the cabinet, distance from the back of the cabinet to the wall min. 50 mm.

## Connect

**!** **Warning risk of injury!**  
**When shortening the suction tube, please be sure to cut the integrated wire as straight as possible.**

**i** **Long suction tubes, tight bends and kinks greatly reduce the suction power at the suction point.**

⇒ Shorten the suction tube if possible.

⇒ Open the adhesive strap (7).

⇒ Plug the suction tube end piece (12) into the suction socket (6).

⇒ Fix the suction tube in place with the adhesive strap.

⇒ Connect the suction tube to the correct suction point.

**i** **If the diameter is not suitable, use an adapter (see accessories), to avoid loss of suction.**

⇒ Connect the electronic device to the appliance socket (8) on the back.

When connecting electrical appliances to the suction unit, ensure that the maximum permissible power for connected devices is not exceeded (see "Technical data" at the end of this document).

Before making the electrical connection, check that the voltage indicated on the rating plate matches the local voltage supply.

⇒ Switch the device off at the on / off switch (3).

⇒ Unroll the power cord (9) and insert the power plug into the building wall socket.

## Operation

Low-dust disposal of the extracted material is only possible if the waste bag is used.

⇒ Before beginning to work, check that a waste bag has been inserted in the dust drawer.

**Waste bags and the starter kit waste bag are available as accessories.**

### Switching On

The suction unit is switched on and off via the on / off switch (3).

After activation:

- ◆ The suction unit performs an automatic filter cleaning sequence (for approx. 8 seconds, loud vibrating noise).

Afterwards, the suction unit is in the last set mode.

### Select operating mode: Automatic operation / continuous operation

- Automatic mode (20):

The suction is dependent on the electrical device connected to the appliance outlet (8).

- Permanent operation (22):

Suction running continuously.

⇒ Press operating mode key (21).

- ◆ Changing the operating mode.

**If the suction to a connected electrical appliance (e.g. a handpiece) does not react as required (e.g. does not activate, even though a connected device is in operation) then the switch-on threshold for automatic operation must be adjusted.**

### Suction Power

The suction power for the suction unit can be set in 4 levels.

⇒ [-] key (23): Reduce suction performance

⇒ [+] key (25): increase suction performance

### Setting the Parameter

On delivery, there are parameters set, which in most cases, make it possible to work with the suction unit without problems. Only in individual cases, if this is not possible, the parameters can be changed.

The setting of various parameters and the execution of a self-diagnosis are carried out in the programming mode.

**Please refer to the operating instructions on the Internet or Figs. D and E at the beginning of this document to find out how to set the parameters and run self-diagnosis.**

The display (24) shows which parameter are set:

	„d“	Perform self-diagnosis
	„b“	Acoustic signal on / off (decimal point on / off)
	„A“	Time Interval for “Full” Display Dust Drawer
	„c“	Setting the automatic switch-on (calibration)
	„t“	Set after-run (time) for the suction turbine


In order to set the different parameters, the programming mode must be started and the parameter selected.

To do so, press the mode key (21) and enter key (27), see Figs. D and E at the beginning of this document.

Change values with the [-] key and [+] key.

Confirm and save settings with the Enter key (27). Successful saving is acknowledged with an acoustic signal.

If a change is not to be applied, press the mode key (21) again to cancel.

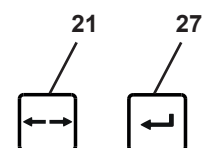### Self Diagnosis (see Fig. D at the beginning of this document)

Start the self diagnosis manually. This is carried out once.

### Acoustic signal (see Fig. D at the beginning of this document)

When on, various inputs are confirmed with an acoustic signal. This beep can be switched on and off.

### Time interval for full display of the dust drawer (see Fig. D at the beginning of this document)

After reaching a set time interval, a request appears to empty the dust drawer.

It is possible to select one of 5 different times.

Time interval / hrs	Value in display (24, Fig. 2)
2	1
5	2
10	3
50	4
100	5

Factory setting

**i** In the case of heavy dust generators (e.g. blasting equipment), the time interval "emptying the dust drawer" should be set to 5 hours, if necessary to 10 hours.

### Automatic switch-on for automatic operation (see Fig. E at the beginning of this document)

The switch-on threshold, which activates and deactivates the suction function when a connected electrical device is operated, can be set.

When setting the switch-on threshold for handpieces, the control unit must be switched on for the first measurement (stand-by) without operating the handpiece

For the second measurement (On), the handpiece should be operated at the speed at which the suction is to be switched on.

**i** For devices with standby mode, these should be switched on approx. 5 minutes before setting to avoid incorrect measurements.

**i** If there is an error with the switch-on function, the display "C" will blink and an acoustic signal is emitted twice. Set the changes again. If necessary, select a higher speed for handpieces. For detailed information on "Troubleshooting", see the instructions for use which you can download from our website at [www.renfert.com/p915](http://www.renfert.com/p915).

### After-run time of the suction turbine (see Fig. E at the beginning of this document)

After-run time = the time between switching off a connected consumer and turning off the suction turbine. There are 6 different times to choose from.

After-run time / sec.	Value in display (24, Fig. 2)
0	1
3	2
5	3
10	4
20	5
30	6

Factory setting

**i** In order to prevent the suction from being switched on / off too often in the case of short interruptions (for example when working with the handpiece), a higher time value can be selected.

## Operation with the SILENT flow sensor

If the extraction unit is operated in combination with a flow rate monitoring unit such as the SILENT flow sensor, the following must be observed:

- ▶ **If the flow rate is less than the defined threshold value for a certain period, this is detected by the SILENT flow sensor, the corresponding indicator begins to flash, and a warning tone is output every 30 seconds.**

**If the flow rate is too low at the suction point being monitored, safe operation is no longer guaranteed and the level of exposure to dust can be harmful to health!**

In this case, the following measures must be taken:

- ⇒ Stop working at all suction points of the affected extraction unit. This applies to the individual suction channels of any multi-workbench extraction unit as well as to all suction points operated via a Y-junction or via extractor switches.
- ⇒ Determine why the flow rate is too low and resolve the issue.

Cause	Measure
Suction level is too low	• Choose a higher suction level
Filter blockage	• Clean the filter.
Cleaning the filter does not result in the expected improvement.	• Replace the fine filter. • For low-dust replacement of the fine filter, observe Chapter 6.3.
Blockage in the suction point (suction mouth)	• Disconnect the suction hose at the suction point and check whether there is a blockage in the suction point; remove where appropriate.
Blockage in the suction line	• Check the suction line at all disconnection points / service openings section by section for blockages.
Leakage in the section of the suction line between the SILENT flow sensor and the extraction unit.	• Check the installation of the SILENT flow sensor in accordance with its instructions.

Determining the cause and measures can be carried out in the order shown.

After every measure, you should switch on the extraction unit again and check whether the minimum flow rate is again exceeded so that correct operation is possible again.

- ⇒ Work should only resume after the fault has been rectified and the minimum flow rate is exceeded again.

## Cleaning / Maintenance

**The device does not contain any internal parts that require maintenance. Opening the device other than described in the instructions is not permitted!**

### Cleaning

To clean the device wipe only with a damp cloth.

Do not use and solvent-based or abrasive cleaning agents.

### Empty Dust Drawer

After reaching a set time interval, a request appears to empty the dust drawer:

- ◆ An acoustic signal sounds 3 x.
- ◆ Display dust drawer (26) lights up.
- ◆ The display (24) shows a container filling up.

Before removing the dust drawer, a filter cleaning function should be performed:

- ⇒ Press Enter key (27) for 2 seconds.
  - ◆ The filter is cleaned (duration approx. 8 sec).
  - ◆ The counter for recording the time interval is reset.


Once the filter has been cleaned:

- ⇒ Release the clamp fastener (4A) of the dust drawer.
- ⇒ Pull the dust drawer (4) out towards the front.
- ⇒ Remove the retaining ring (14, Fig. 1).
- ⇒ Seal the waste bag tightly to prevent dust escaping, remove it, and ensure disposal.

**During disposal, observe local provisions and accident prevention regulations!  
Personal protective equipment must be worn depending on the extracted material.**

- ⇒ Place a new waste bag through the retaining ring and fold the ends over the retaining ring.
  - ⇒ Place the waste bag with the retaining ring in the dust drawer. The retaining ring is held in the dust drawer using magnets. Ensure that the waste bag:
    - Fits against the side walls of the dust drawer;
    - Does not cover the sealing surface.
  - ⇒ Insert the dust drawer again and push it all the way in.
  - ⇒ Close the clamp fastener (4A).
- If the dust drawer has been emptied without prior filter cleaning, the counter must be reset separately to record the time interval.
- ⇒ Press the Enter key (27) (counter for recording the time interval is reset).
    - ◆ Acoustic signal acknowledges the information.
    - ◆ Dust drawer display (26) is deleted.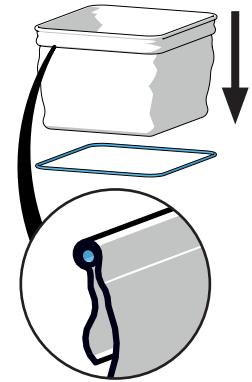

**If the dust drawer is not emptied, the dust drawer (26) display will still light up. After switching the device off and on again, a 3 x acoustic signal is used to indicate the emptying of the dust drawer.**

To set the time interval, see Fig. D at the beginning of this document, or refer to the Instructions for Use on the Internet.

## Change Fine Filter

If the filter cleaning does not show sufficient effect (suction power has not improved), the fine filter must be changed.

This is detected by the suction and indicated by "A" in the display.

**If requested to change the fine filter or the suction performance does not improve substantially or only for a short period, the fine filter must be changed soon. Further operation can lead to damage to the device.**

- ▶ **In general, the fine filter should be changed every 2 years.**

**Do not clean the fine filter manually under any circumstances (e.g., using compressed air, brushes, water, etc.). This damages the filter material.**


**Do not use compressed air!  
Using compressed air to blow off deposits of dust is generally not permitted.**


**There is a risk of increased exposure to dust when replacing the fine filter. That's why the following measures must be taken:**

- ⇒ Keep the plastic bag ready that contains the new filter.
- ⇒ Keep an additional dust extraction device or suction tube from another workbench extraction unit ready.
- ⇒ Wear at least an FFP2 mask for respiratory protection.
- ⇒ Clean the filter 2x
  - Press the Enter key for 2 seconds
  - Filter cleaning is carried out
  - Repeat the procedure
  - Switch off the extraction unit.
- ⇒ Disconnect the power plug
- ⇒ Disconnect the suction hoses.
- ⇒ Place the extraction unit so that it can be turned around.
- ⇒ Pull out the dust drawer.
- ⇒ Turn the extraction unit upside down
- ⇒ Suction the loose dust in the dust room.
- ⇒ Unlock the clamping nut (1), loosen (2), remove, and ensure disposal.
- ⇒ Pull out the fine filter upwards
- ⇒ Place the fine filters immediately in the plastic bag and seal the bag tightly to prevent dust escaping.
- ⇒ Ensure proper disposal.
- ⇒ Suction any fallen dust out of the dust room and clean room.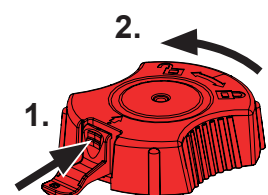

- ⇒ Insert a new fine filter; ensure that it is correctly positioned.
- ⇒ Attach a clamping nut and tighten as shown.
- ⇒ Turn the extraction unit back to its previous position.
- ⇒ Insert the dust drawer.
- ⇒ Connect the suction hoses, insert the power plug, and place the extraction unit back in position.
- ⇒ Use the additional dust extraction device to suction up dust that has been released.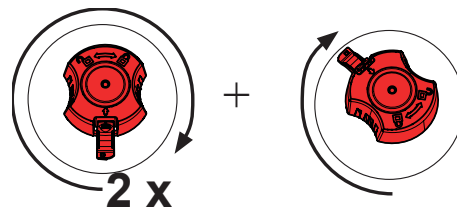


## Spare Parts

You can find components subject to wear and the spare parts on the spare part list in the internet at [www.renfert.com/p918](http://www.renfert.com/p918).

Enter the following item number: 29350000.

The components excluded from the warranty (such as consumables or parts subject to wear and tear) are marked on the spare part list.

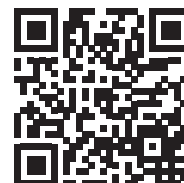## Disposal Information

**Within the European Union, units with this symbol should therefore not be disposed of in unsorted domestic waste.**

Please contact your local authorities for more information on proper disposal.

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declare under our sole responsibility, that the product(s)

Name	Part No.
SILENT TS	2921 0050
SILENT TS2	2930 0050
SILENT TC	2935 0000 / 2935 0000 03
SILENT TC2	2936 0000 / 2936 0000 03
SILENT EC2	2937 0000 / 2937 0000 03
SILENT compact	2934 0000 / 2934 0000 03
SILENT compactCAM	2934 2000 / 2934 2000 03 2934 2500 / 2934 2500 03
SILENT powerCAM EC	2939 0000
SILENT powerCAM TC	2938 0000
iVAC silent	36 2934 2000

is (are) in conformity with the relevant regulatory requirements by compliance with the UK designated standards.

## UK legislation

*Electromagnetic Compatibility Regulations 2016*

*Supply of Machinery (Safety) Regulations 2008*

*The Restriction of the Use of Certain Hazardous Substances in Electrical and Electronic Equipment Regulations 2012*

## Designated standards

EN 61326-1:2013

EN ISO12100:2010

EN 61010-1:2010/A1:2019/AC:2019-04

EN IEC 63000:2018

Tilo Burgbacher,  
Engineering Director

Hilzingen, 02.11.2022

The Technical Documentation has been retained by Renfert GmbH





making work easy

90002 1532



Feinfilter SILENT TC • Fine filter SILENT TC • Filtre fin SILENT TC  
Filtro fino SILENT TC • Filtro fino SILENT TC • Фильтр тонкой очистки SILENT  
TC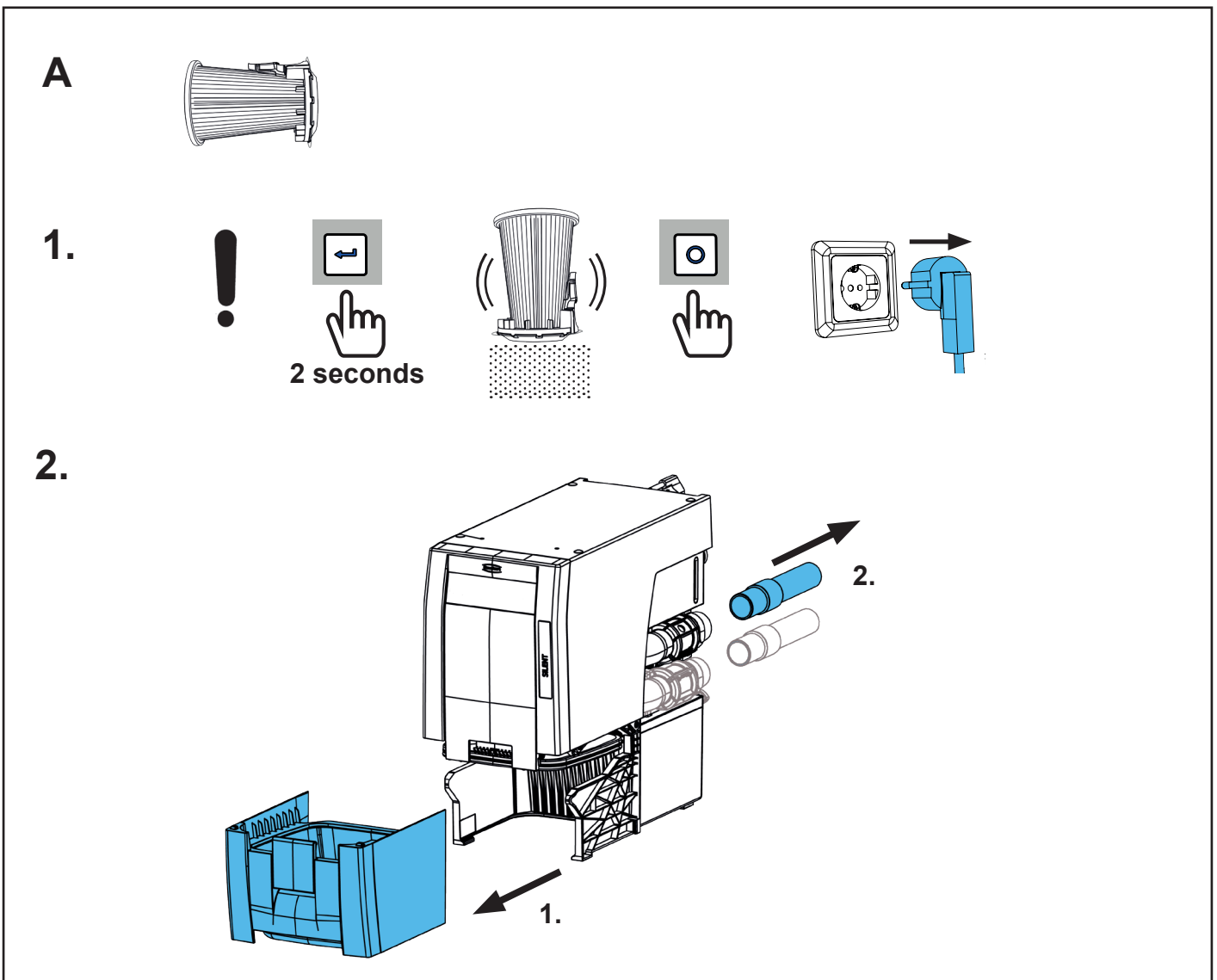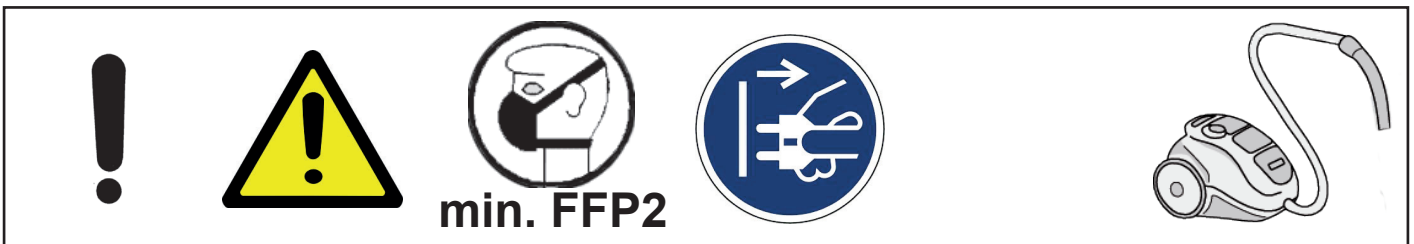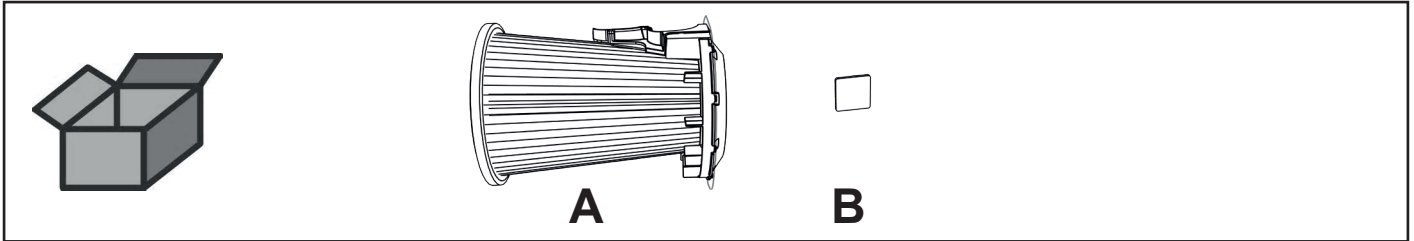


3.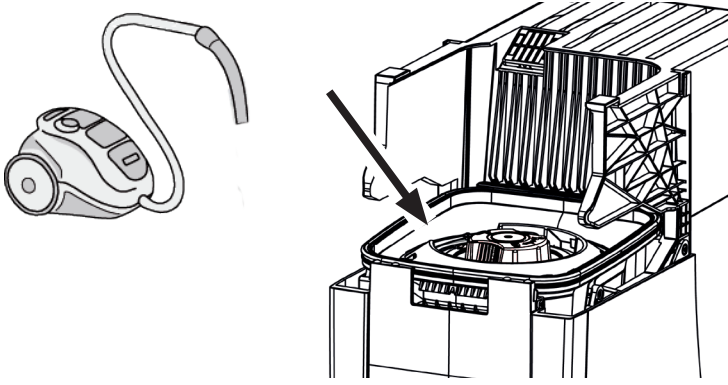


4.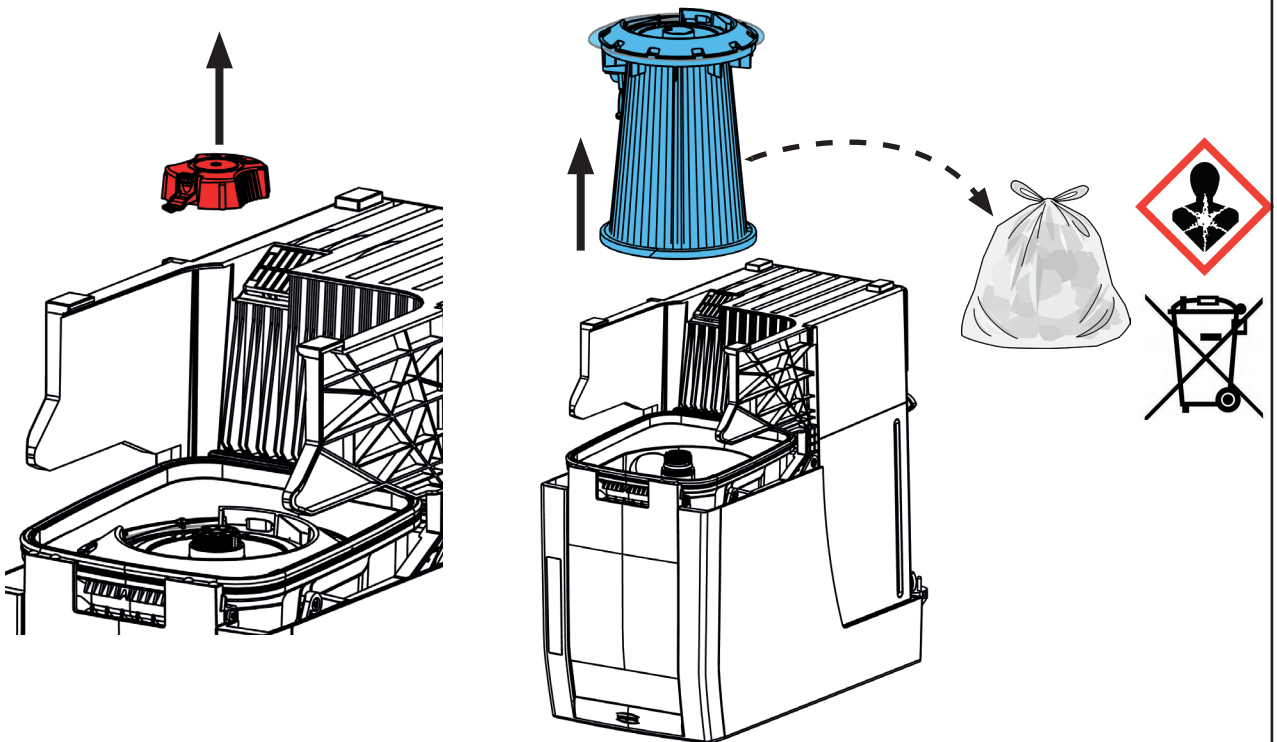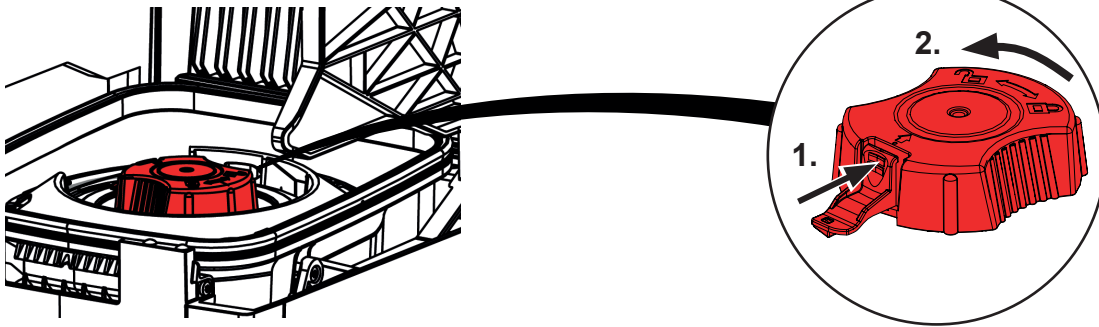


5.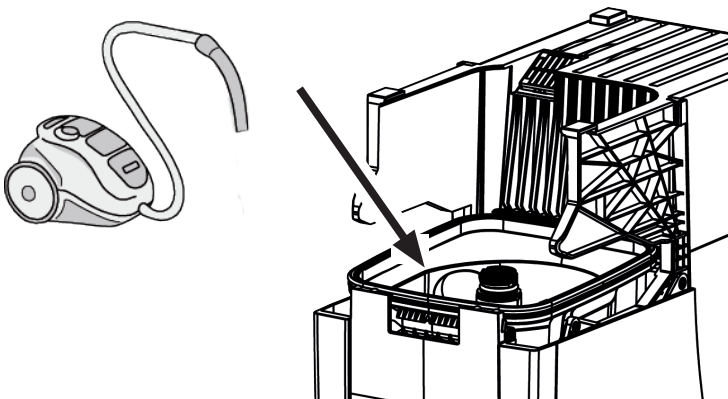

6.

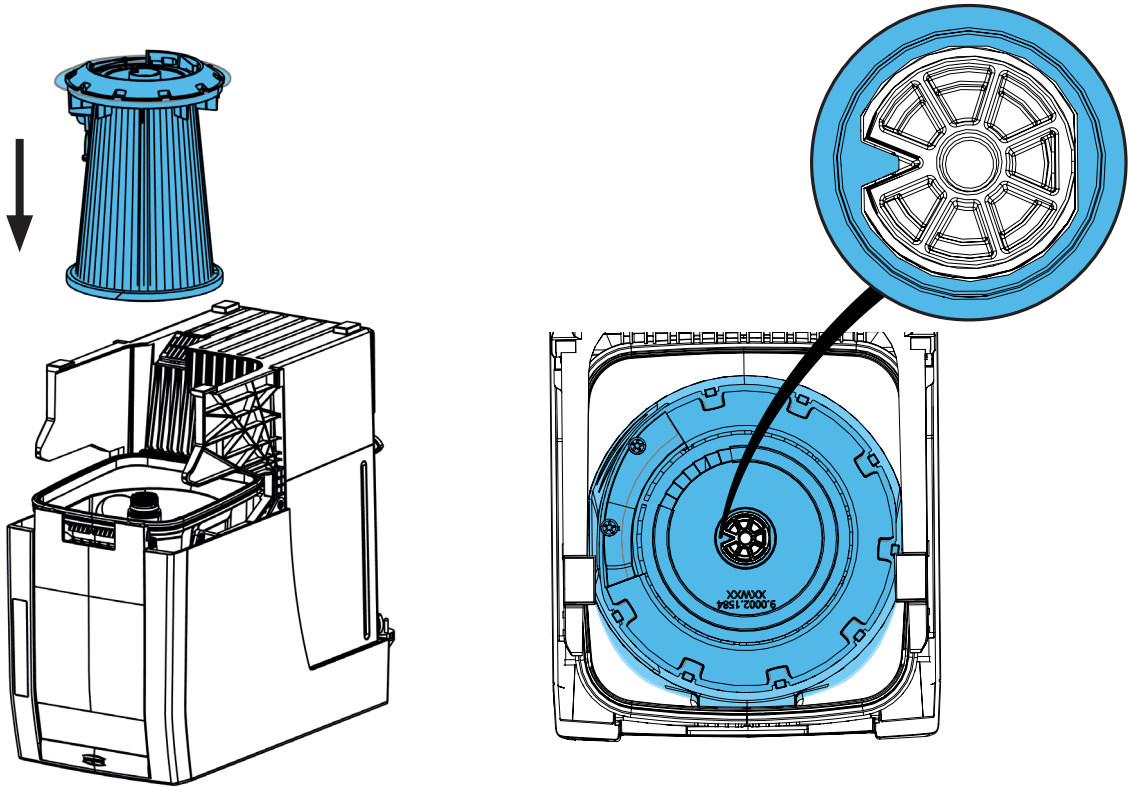
7.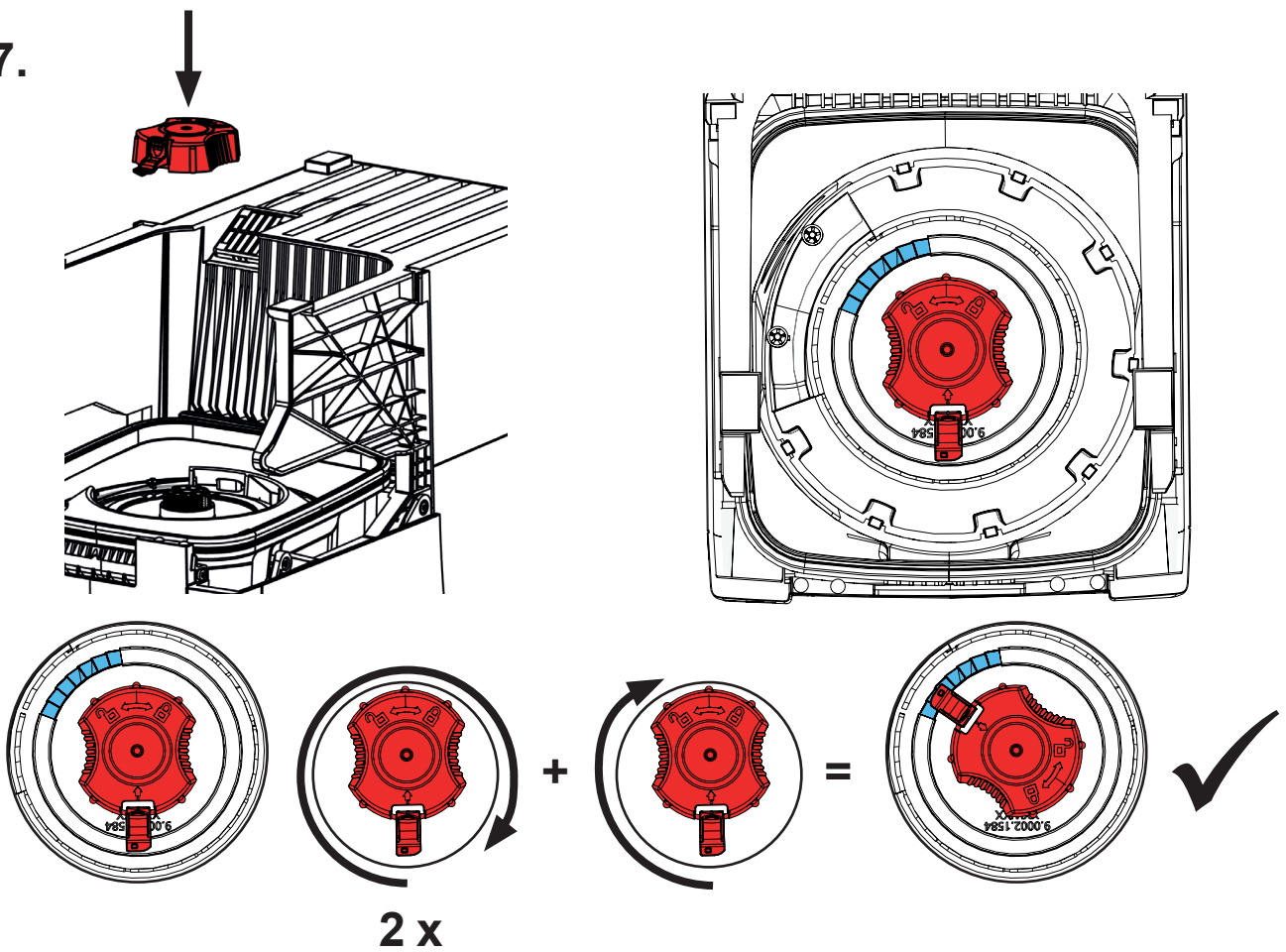


B

8.

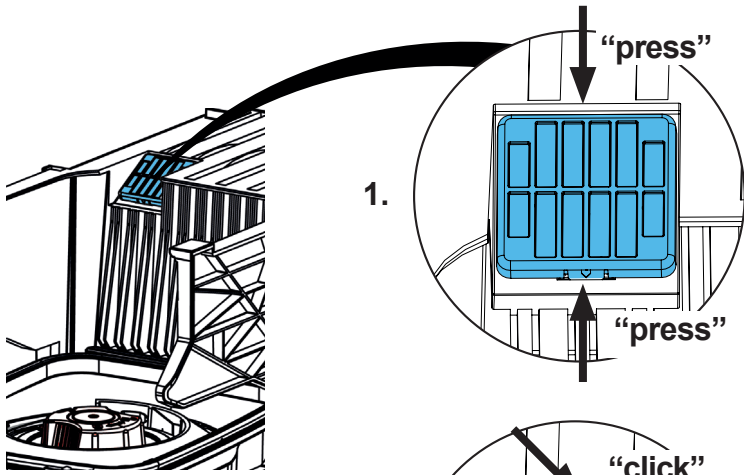
4.
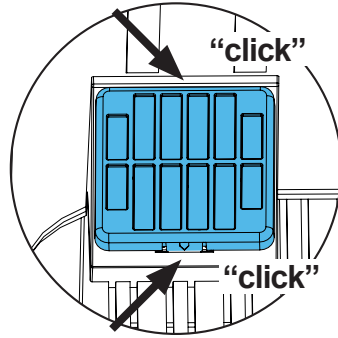


2.

3.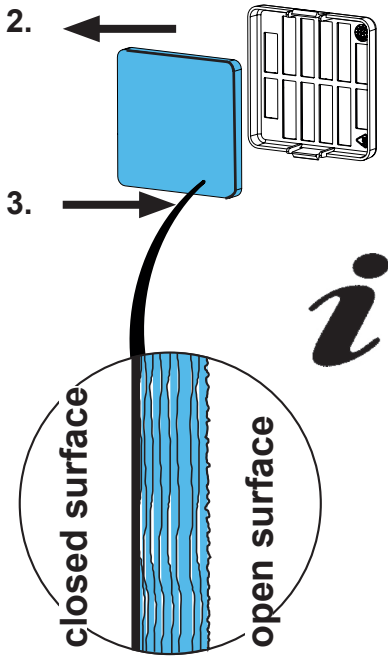

9.

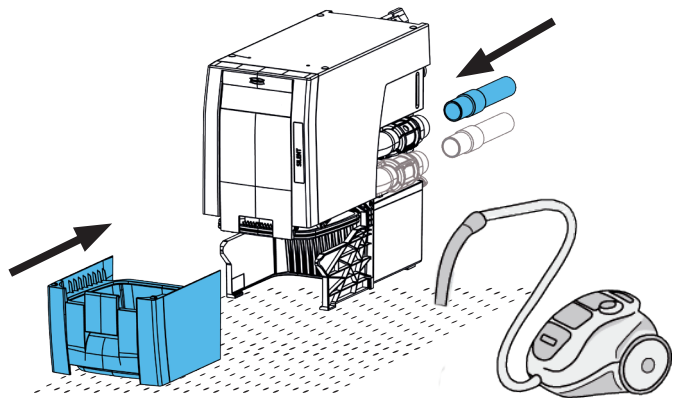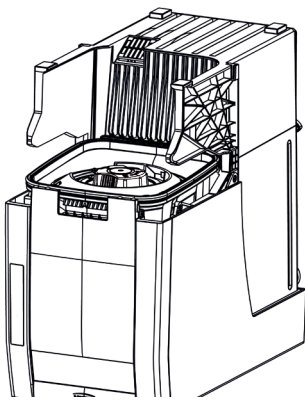# Technical Data

<b>Nominal Voltage:</b>	<b>230 V</b>
Permissible mains voltage:	220 - 240 V
Mains frequency:	50 / 60 Hz
Suction turbine power: *)	850 VA
max. connecting power: *)	2000 W
Total connected power: *)	2850 W
Mains input fuse:	2 x 15 A (T)
LpA **) (at max. volume flow):	54,3 dB(A)
Ø Suction fittings: - internal: - external:	45 mm [1.77 inch] 50 mm [1.97 inch]
max. Volume flow ***):	3980 l/min [2.34 ft <sup>3</sup> /s]
Negative pressure, max. ****):	281 hPa [4.1 psi]
Fine particle filter: - Filter surface area, approx.: - Filterquality:	0.9 m <sup>2</sup> [1390 sq inch] Class M according to EN 60335-2-69
Fill level dust drawer, approx.:	7 l [1.85 US gal]
Dimensions (Width x Height x Depth):	275 x 535 x 540 mm [10.8 x 21.1 x 21.3 inch]
Weight (empty), approx.:	22 kg [48.5 lbs]

- \*) Power consumption at nominal voltage  
 \*\*) Sound pressure levels according to EN ISO 11202  
 \*\*\*) Free-blowing turbine at nominal turbine voltage  
 \*\*\*\*) At nominal turbine voltage

**Hochaktuell und ausführlich auf ...  
Up to date and in detail at ...  
Actualisé et détaillé sous ...  
Aggiornato e dettagliato su ...  
La máxima actualidad y detalle en ...  
Актуально и подробно на ...**

**[www.renfert.com](http://www.renfert.com)**

Renfert GmbH • Untere Gießwiesen 2 • 78247 Hilzingen/Germany  
Tel.: +49 7731 82 08-0 • Fax: +49 7731 82 08-70  
[www.renfert.com](http://www.renfert.com) • [info@renfert.com](mailto:info@renfert.com)

Renfert USA • 3718 Illinois Avenue • St. Charles IL 60174/USA  
Tel.: +1 6307 62 18 03 • Fax: +1 6307 62 97 87  
[www.renfert.com](http://www.renfert.com) • [info@renfertusa.com](mailto:info@renfertusa.com)  
USA: Free call 800 336 7422