making work easy

- de
- en
- fr
- it
- es

ru

zh

ja

ko

MT plus

Quick Start Guide

Made in Germany

21-6810 15112022








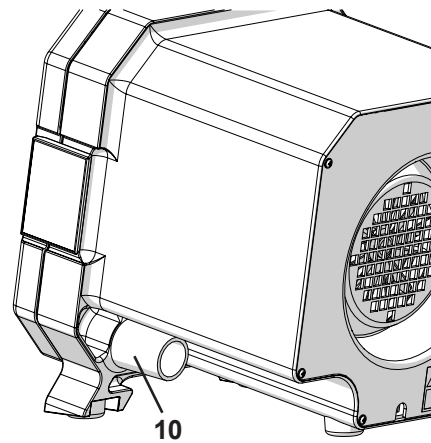
Fig. 1

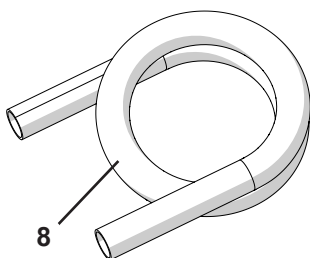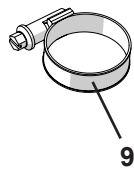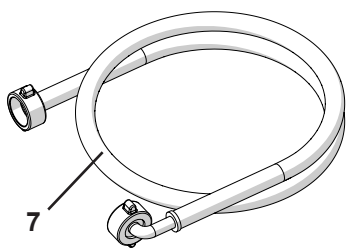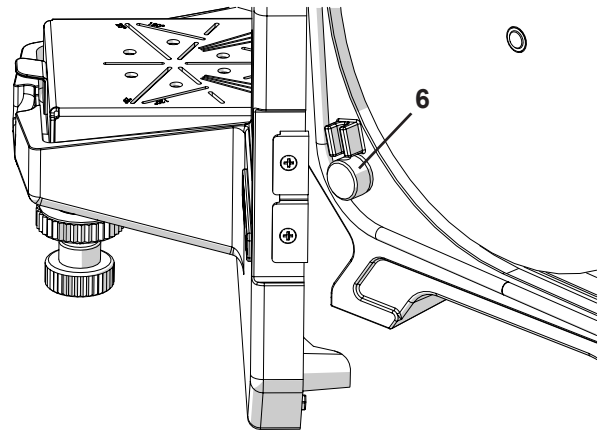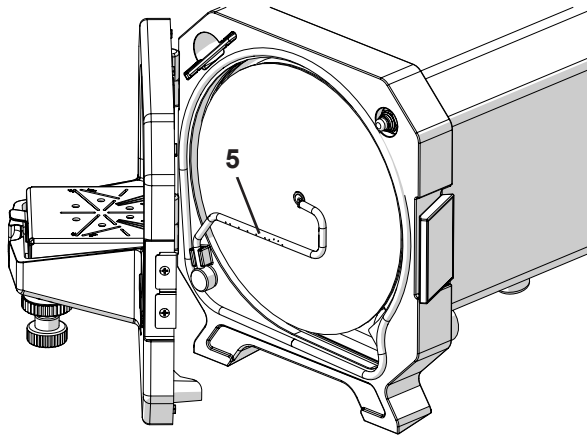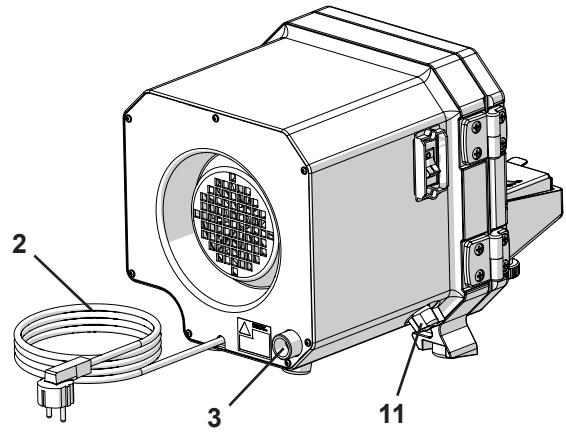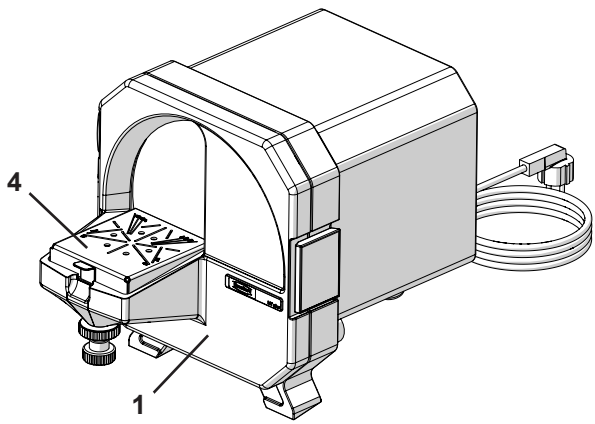Einleitung

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Introduction

This Quick Start Guide contains instructions on the operation of the product and relevant safety information.

Detailed information on the product can be found in the operating instructions which are available for download on our website at www.renfert.com/p915.

Enter the following item number: 18030000 or 18030500.

Symbols

In the instructions for use and on the unit itself you will find these symbols with the following meanings:

Danger

This indicates a direct risk of injury. Consult accompanying documents!

Electrical current

This indicates a risk of hazard due to an electrical current.

Attention

Disregarding this warning may result in damage to equipment.

Note

This provides the operator with useful information to improve and ease use.

The device complies with the requirements of the applicable EU directives.

This product complies with the relevant UK legislation.

See UKCA Declaration of Conformity in the Internet under www.renfert.com.

The device is subject to the EU directive 2002/96/EG (WEEE directive).

Other symbols are explained as they occur.

Safety

Proper Use

The *MTplus* is a model trimmer designed for use in dental technology operations to grind plaster models. It can be employed as either a wet or dry trimmer.

Improper Use

This appliance may not be used for gringing investment material.

Only spare parts and accessories supplied or authorized by Renfert GmbH may be used with this product. If other spare parts or accessories are used, this could have a detrimental effect on the safety of the device, increase the risk of serious injury and lead to damage to the environment or the device itself.

Hazard and Warning Information

General Information

- ▶ If the device is not used in compliance with the supplied instructions, the safety of the device can no longer be guaranteed.
- ▶ Risk of injury!
Risk of injury if the components used are not approved.
Always use original Renfert components.
- ▶ The device may only be operated using a mains cable with the country-specific plug system. Any necessary alterations must be carried out by a qualified electrician.
- ▶ The device may only be operated if the information on the identification plate conforms to the specifications of your local mains power supply.
- ▶ The device may only be plugged into outlets which are connected to the protective conductor system.
- ▶ The mains plug must be easily accessible.
- ▶ Disconnect the device from the mains before carrying out work on the electrical parts.
- ▶ It is the responsibility of the operator that national regulations during operation and regarding a repeated safety inspection of electrical equipment are complied with. For Germany these are the

regulation 3 by DGUV (German Statutory Accident Insurance) in relation with VDE VDE 0701-0702 (Association for Electrical, Electronic and Information Technology).

- ▶ Check connection cables (such as power supply cords), tubes and housing (i.e. the key-pad) regularly for damage (i.e. kinks, cracks and porosity) or signs of ageing. Devices with damaged connection cables, tubes or housing parts or other defects must not be operated!
- ▶ Defective devices must be put out of service immediately. Remove the mains plug and ensure the device is not used. Send the device for repair!
- ▶ Only operate the device under supervision.
- ▶ Please observe the national accident prevention regulations!
- ▶ Information on REACH and SVHC is available on our website www.renfert.com, in the Support area.

Specific Instructions

- ▶ Never reach into the grinding wheel while the unit is connected to the power supply. Inadvertently starting the unit can result in injury!
- ▶ Always keep long hair, loose or wide sleeves or other clothing away from the trimmer. Hair or clothing can be pulled into and wrapped up on the grinding wheel, resulting in injury!
- ▶ The unit is designed solely for grinding plaster models. Cancer causing dusts are released when trimming investment material.
- ▶ Never open the front cover while the motor is running. The motor safety shutoff will not stop the grinding wheel immediately.
- ▶ Only open the front cover after the grinding wheel has come to a complete stop.
- ▶ Always wear protective eyewear when grinding.
- ▶ Do not reach into the grinding wheel while it is turning.
- ▶ Do not work without the model table installed and locked („click“). Small parts or fingers can be drawn into the trimmer, resulting in serious injury.
- ▶ Remove the trimmer table only after the grinding wheel has come to a complete stop and the front cover is opened.
- ▶ The extraction must be selected to match the dust being generated (if necessary, consult the responsible professional organization).
- ▶ Operating the unit can result in a very high noise level >>> always wear ear protection.

Authorized Persons

Operation and maintenance of the device may only be performed by qualified personnel.

Product Description

Component Groups and Functioning Elements

See Fig. 1 (on inside cover)

- | | | | |
|---|------------------|----|--------------------|
| 1 | Front cover | 7 | Water hose |
| 2 | Power cable | 8 | Drainhose |
| 3 | Water connection | 9 | Hose clamp |
| 4 | Trimmer table | 10 | Connection fitting |
| 5 | Spray tube | 11 | Water valve |
| 6 | Blind plug | | |

The other units are covered in the full set of operating instructions available on the internet.

Standard Delivery

- 1 *MTplus*
- 1 Water hose
- 1 Drain hose
- 1 Tool set
- 1 Hardware set
- 1 *Marathon* trimmer disc, only on Nr. 1803-x500
- 1 Quick Start Guide

Installation

Unpacking

⇒ Check to ensure all delivery parts are complete (compare “Standard Delivery”).

Setting Up

⇒ Place the unit on a level and stable base.

⇒ Place appliance with easy access to the mains plug socket.

Wet Trimming

⇒ Connect water hose (7) to water connection (3).

⇒ Connect the water hose to a water supply with a shutoff tap.

⇒ Open the shutoff tap and check for leaks.

⇒ Twist drain hose (8) onto connection fitting (10).

⇒ Fix with a hose clamp (9).

⇒ Place drain water hose with downward slope.

⇒ Unroll the power cable (2) and insert the plug into the socket.

Dry Trimming

⇒ Open the front cover (1).

⇒ Remove the spray tube (5).

⇒ Close the opening with the blind plug (6) from the hardware set.

⇒ Close the front cover (1).

⇒ Push the suction tube onto the connection fitting (10).

⇒ Unroll the power cable (2) and either insert the plug into the automatic switch-on socket of the suction unit or into the mains power socket.

Please observe the maximum connection load for the automatic start and suction. If necessary, connect the suction unit and trimmer to separate power supplies.

Operation

Trimming

⇒ If necessary check and adjust the grinding angle.

⇒ Turn on appliance.

⇒ If necessary, adjust the water level in the water valve (11) or switch the suction on.

⇒ Using both hands, securely set the model on the trimmer table (4) and press it against the disc.

Once the work has been finished:

⇒ Switch off the appliance.

The water supply shutoff tap should be closed when work is interrupted for longer periods and when the appliance is unattended (e.g. over night).

Cleaning / Maintenance

Inside the extractor there are small parts which require maintenance.

Opening the device, other than for the processes described below, is not permissible!

Never clean the device under running water or immerse in water.

⇒ Switch off and unplug the appliance before cleaning or servicing.

Wet Trimming

⇒ Clean the unit using only a damp cloth.

⇒ Remove the front cover (1) and trimmer table (4) and clean with a brush under running water.

Do not remove the grinding table if the front cover is closed otherwise the trimmer disc will be damaged!

Dry Trimming

A raising of plaster dust in aerodynamic dead areas when dry trimming is normal and has no negative influence on the function or grinding power of the MTplus.

⇒ Remove dust using a dry brush or towel or vacuum it clean.

Spare Parts

You can find components subject to wear and the spare parts on the spare part list in the internet at www.renfert.com/p918.

Enter the following item number: 18030000 or 18030500.

The components excluded from the warranty (such as consumables or parts subject to wear and tear) are marked on the spare part list.

Serial number and date of manufacturing are shown on the type plate of the unit.

Disposal Information

Within the European Union, units with this symbol should not therefore be disposed of in unsorted domestic waste.

Please contact your local authorities for more information on proper disposal.

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declare under our sole responsibility, that the product(s)

Name	Part No.
MT plus	1803 0000 1803 0500
MT3	1808 0000 / 1808 0000 03 1808 0500 / 1808 0500 03
MT premium	1807 0000 1807 0500

is (are) in conformity with the relevant regulatory requirements by compliance with the UK designated standards.

UK legislation

Electromagnetic Compatibility Regulations 2016

Supply of Machinery (Safety) Regulations 2008

The Restriction of the Use of Certain Hazardous Substances in Electrical and Electronic Equipment Regulations 2012

Designated standards

EN 61326-1:2013

EN ISO12100:2010

EN 61010-1:2010/A1:2019/AC:2019-04

EN 60825-1: 2014

EN 61000-3-2: 2014

EN 61000-3-3: 2013

EN IEC 63000:2018

Tilo Burgbacher,
Engineering Director

Hilzingen, 02.11.2022

The Technical Documentation has been retained by Renfert GmbH

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Renfert GmbH • Untere Gießwiesen 2 • 78247 Hilzingen/Germany