making work easy

- de
- en
- fr
- it
- es
- pt
- tr
- ru
- pl
- uk
- ja
- ko
- ar

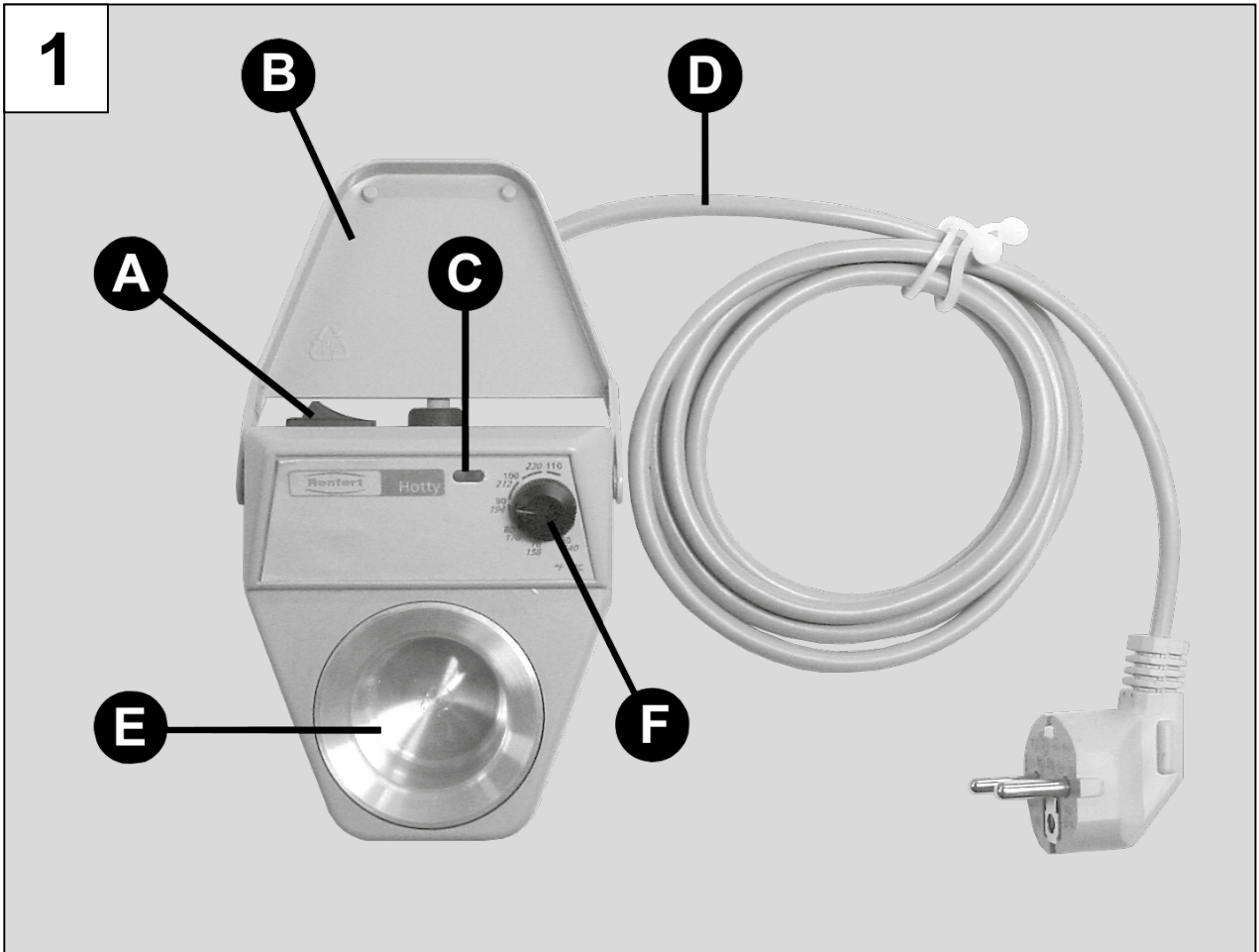
hotty

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Made in Germany

21-9867 15092023

1 Einleitung

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1 Introduction

We are pleased with your decision to purchase a hotty wax heating unit.
Please observe the following information in order to ensure long and trouble-free service.

1.1 Employed Symbols

In these instructions or on the unit itself you will find symbols which have the following meaning:

Danger

This indicates an imminent risk of injury. In these instructions or on the unit itself you will find symbols which have the following meaning.

Electrical hazard

There is a risk of electrical shock.

Attention

Failure to observe the associated information can result in damage to the unit.

Note

The operating instructions contain useful information to make handling easier.

The unit complies with the relevant EC guidelines.

This product complies with the relevant UK legislation.

See UKCA Declaration of Conformity in the Internet under www.renfert.com.

The unit is subject to the EC guidelines 2002/96/EG (WEEE Directive).

► List, particular attention should be paid

- List
- List

⇒ Instructions / required action / input / operational sequence:

You are asked to carry out the specified action in the order indicated.

- ◆ Result of an action / response by the equipment / response by the program:

The equipment or program responds as a result of your actions or when a specific incident occurs.

Other symbols are explained as they occur.

2 Application Area

The hotty is an electronically regulated wax dipping pot. It is ideally suited for thermal processing with all conventional dipping waxes. The following special characteristics make the quality of the hotty apparent to you during operation:

On the one hand, the **precise temperature regulation** (+ / - 2 °C [3.6 °F]) ensures a uniformly high quality copings and, on the other, working together with the heater that reacts optimally to the temperature conditions, prevents an overheating of the wax and resultant damage.

Please note that the warm-up phase is intended to be gentle on the wax. It therefore may take some time before the wax is completely liquefied. The **continuous increase of the operating temperature** prevents the temperature in the wax pot from exceeding the desired setpoint. The quality of the wax is thus maintained.

The hotty offers you **optimal working conditions**. For example, the housing is constructed so that both left- and right-handed individuals can rest their hands on the work while working at the unit. In addition, the dipping pot cover can be swiveled completely out of work area.

3 Safety Information

- Before starting the unit, compare the information on the nameplate with the specifications of your local power supply.
- Units with damaged power cords or other defects may not be operated.
- During operation, always unroll the power cord (D) (overheating hazard in case of a short circuit).
- It is the responsibility of the operator that national regulations during operation and regarding a repeated safety inspection of electrical equipment are complied with. For Germany these are the regulation 3 by DGUV (German Statutory Accident Insurance) in relation with VDE 0701-0702 (Association for Electrical, Electronic and Information Technology).

- ▶ Liquid waxes can cause burns. Do not move the unit when in use!
- ▶ During operation, the wax pot (E) is heated up to its rims. Improper use or handling can result in burns even approx. within 30 minutes after power off.
- ▶ The hotty is designed to be used exclusively in conjunction with commercially available dental waxes. Never use the unit to heat other types of materials!
- ▶ Please note: Overfilled pots can overflow!
- ▶ Materials whose flash point is under 175 °C [347 °F] may not be stored in proximity to the hotty or come in contact with the hotty.
- ▶ Do not rinse or dip into water. Hazardous voltage can cause electric shock!
- ▶ Information on REACH and SVHC is available on our website www.renfert.com, in the Support area.

4 Operating Elements (Fig. 1)

- A) On / Off switch
- B) Lid
- C) LED indicator
 - >> Operation LED = green
 - >> Warm-up phase = red
- D) Power cord
- E) Wax pot
- F) Potentiometer

5 Commissioning / Operation

1. Connect the hotty to the wall outlet (Fig. 2).
2. Swivel the cover (B) back (Fig. 3).
3. Add the dipping wax to the pot (E) (Fig. 4).
4. Switch the hotty on at the main power switch (A).
The operation LED (C - green) then goes on. During the warm-up phase the red LED (C) also remains on.
5. Turn the potentiometer (F) that gives you setpoint between 60 °C [140 °F] and 110 °C [230 °F] in increments of 10 °C [18 °F] until you reach the desired operating temperature (Fig. 5).
6. Once the unit reaches the setpoint temperature, the red LED goes out (C). The red LED would light up shortly when regulating the temperature successively.

5.1 Correct dipping (Fig. 6)

- Support the hand comfortably.
- Do a quick dip to just beyond the preparation border.
- Slowly and evenly remove the die from the wax.
- Just before the tip comes out hold it steady in the pot for a short time so that any excess wax can drip down.

6 Cleaning / Maintenance

The hotty is maintenance free. From time-to-time, you should wipe your unit down with a damp cloth. Never use detergents or solvents!

Never rinse or spray your hotty directly with water.

With the unit switched off, you can remove hardened wax residue by carefully scraping it. Warming the unit's surfaces with a hair drier will make it easier to remove wax residue.

7 Spare Parts

You can find components subject to wear and the spare parts on the spare part list in the internet at www.renfert.com/p918.

Enter the following item number: 14400000 or 14401000.

The components excluded from the warranty (such as consumables or parts subject to wear and tear) are marked on the spare part list.

Serial number and date of manufacturing are shown on the type plate of the unit.

en

8 Warranty

Provided the unit is properly used, Renfert warrants the hotty for a **period of 3 years**. The original sales invoice of the specialist depot is required for a claim under guarantee. Parts subject to natural wear and tear and also consumables are not covered by the guarantee.

The guarantee is rendered void in the case of incorrect use, non-adherence to the operating, cleaning, servicing or connection instructions, repairs carried out by the owner or repairs that are not carried out by a specialist firm, use of replacement parts from another manufacturer or unusual or unapproved uses not specified in the user guidelines.

Successful claims under guarantee do not extend the guarantee period.

9 Technical data

Item number:	1460 0000	1460 1000
Working Voltage:	230 V	120 V
Permissible mains voltage:	220 - 240 V	100 - 120 V
Mains frequency:	50 / 60 Hz	
Maximum power:	25 W (+ / - 10 %)	
Fuse protection: Fuse (not replaceable) / rating:	T 500 mA / 250 V	T 250 mA / 250 V
Minimum operating temperature:	60 °C [140 °F]	
Maximum operating temperature:	110 °C [230 °F]	
Dimensions (W x H x D):	90 x 65 x 98 mm [3.54 x 2.56 x 3.86 inch]	
Weight, approx.:	360 g [12.7 oz]	
Environmental conditions for safe operation:		
Ambient conditions:	Indoors	
Installation height, max.:	2,000 m [6,500 ft] above sea level	
Ambient temperature:	5 - 40 °C [41 - 104 °F]	
Relative humidity:	Max. 80 % at 31 °C [87.8 °F], decreasing linearly to 50 % at 40 °C [104 °F] *)	
Voltage fluctuations in the mains power supply from the nominal value, max.:	10 %	
Contamination level:	2	
Over-voltage category:	II	

*) Between 5 – 30 °C [41 - 86 °F], the unit can be operated at a relative humidity of up to 80 %. At temperatures between 31 - 40 °C [87.8 - 104 °F], the humidity must decrease proportionally in order to ensure operational readiness (e.g., at 35 °C [95 °F] = 65 % humidity; at 40 °C [104 °F] = 50 % humidity). The unit may not be operated at temperatures above 40 °C [104 °F].

10 Standard Delivery

1 hotty wax heating unit

1 Set of operating instructions

11 Accessories

GEO-Dip dipping waxes:

No. 482-3000 GEO-Dip yellow, 200 g

No. 482-3200 GEO-Dip orange, 200 g

No. 482-3300 GEO-Dip green, 200 g

Rewax dipping waxes:

No. 651-0000 Rewax yellow, 210 g

12 Disposal instructions for countries in the EU

To conserve and protect the environment, prevent environmental pollution and improve the recycling of raw materials, the European Commission adopted a directive that requires the manufacturer to accept the return of electrical and electronic units for proper disposal or recycling.

Within the European Union units with this symbol should not therefore be disposed of in unsorted domestic waste:

For more information regarding proper disposal please apply at your local authorities.

13 Troubleshooting Guide

Error	Possible cause	Corrective action
No display / no operation, although the unit is switched on the main switch (A).	<ul style="list-style-type: none">• Power supply interrupted.• Faulty overheat fuse.• Faulty unit fuse.	<ul style="list-style-type: none">• Check the power supply and power cords.• Have fuse replaced by Service Department.• Have fuse replaced by Service Department.
The unit is operating but there is no increase in temperature.	<ul style="list-style-type: none">• Faulty thermal sensor.• Faulty heater.	<ul style="list-style-type: none">• Have probe replaced by Service Department.• Have heater repaired by Service Department.

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declare under our sole responsibility, that the product(s)

Name	Part No.
Hotty	1460 0000 / 1460 0000 03
Hotty LED	1461 0000 / 1461 0000 03
Vario E	1452 0000 / 1452 0000 03
Waxprofi	1440 0000 / 1440 0000 03
Waxelectric I & II	2156 0000 / 2156 0000 03 2157 0000 / 2157 0000 03
Waxelectric light I & II	2150 0000 / 2150 0000 03 2151 0000 / 2151 0000 03

is (are) in conformity with the relevant regulatory requirements by compliance with the UK designated standards.

UK legislation

Electromagnetic Compatibility Regulations 2016

Electrical Equipment (Safety) Regulations 2016

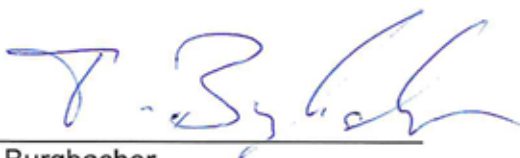
The Restriction of the Use of Certain Hazardous Substances in Electrical and Electronic Equipment Regulations 2012

Designated standards

EN 61326-1:2013

EN 61010-1:2010/A1:2019/AC:2019-04

EN IEC 63000:2018

Tilo Burgbacher,
Engineering Director

Hilzingen, 02.11.2022

The Technical Documentation has been retained by Renfert GmbH

**Hochaktuell und ausführlich auf ...
Up to date and in detail at ...
Actualisé et détaillé sous ...
Aggiornato e dettagliato su ...
La máxima actualidad y detalle en ...
Актуально и подробно на ...**

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Renfert GmbH, Industriegebiet, 78247 Hilzingen / Germany
Renfert GmbH • Untere Gießwiesen 2 • 78247 Hilzingen/Germany