making work easy

- de
- en
- fr
- it
- es
- pt
- ru
- pl
- sv
- hu
- zh
- ja
- ko
- cs
- uk
- fi

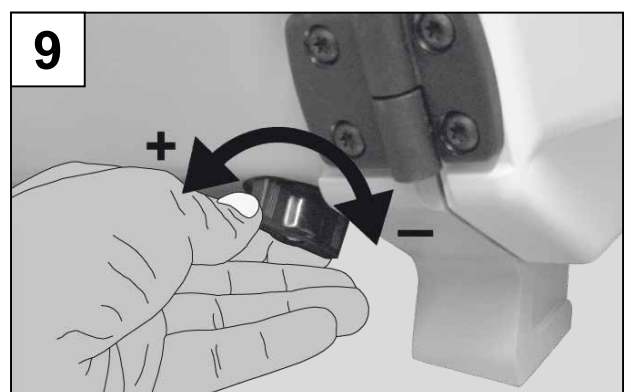
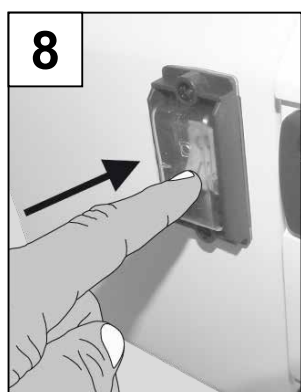
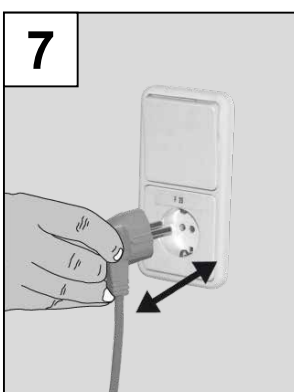
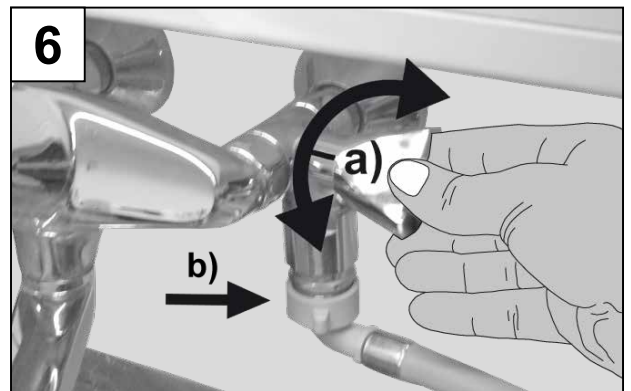
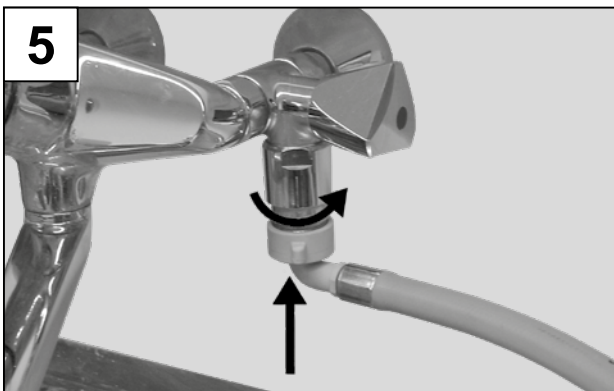
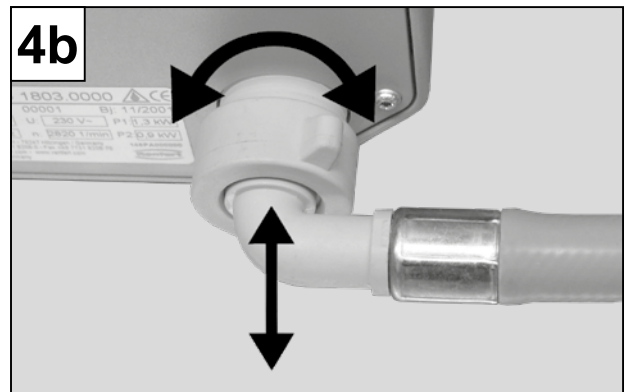
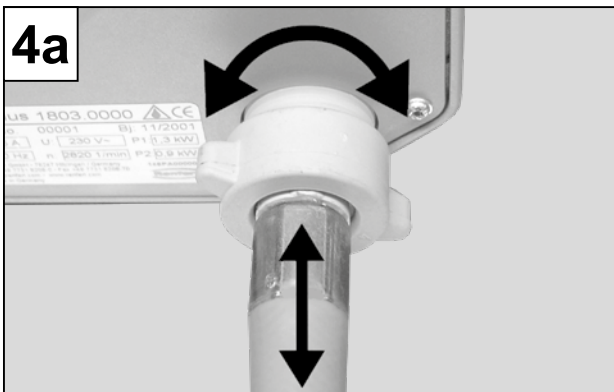
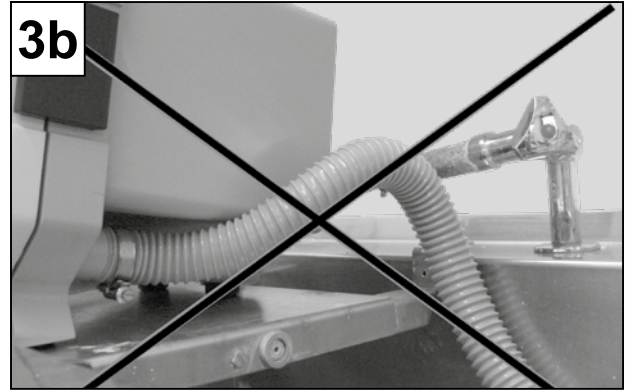
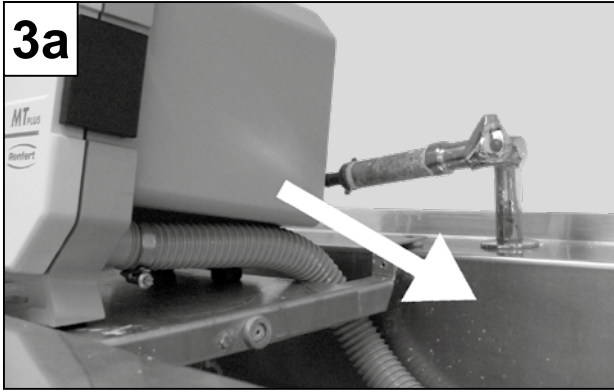
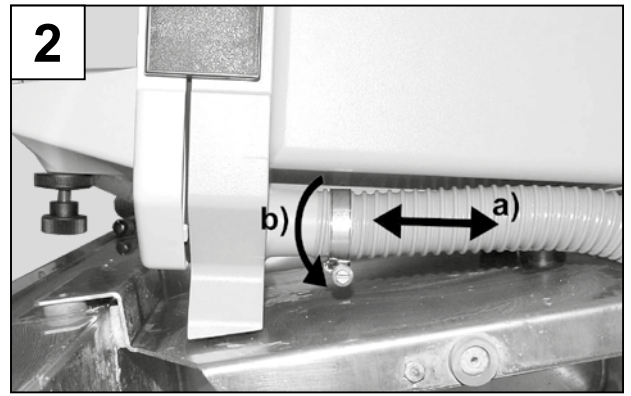
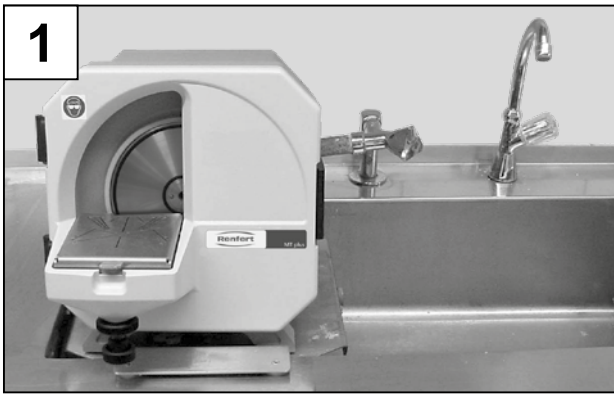
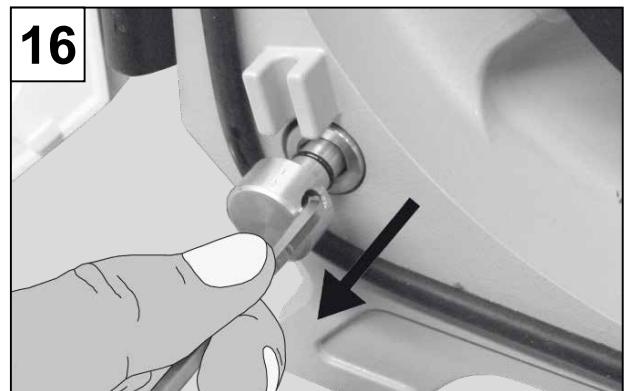
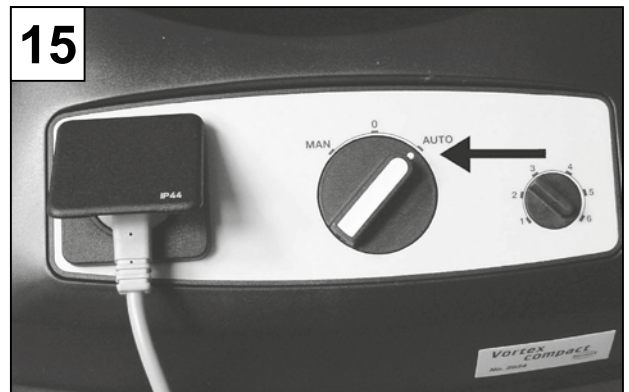
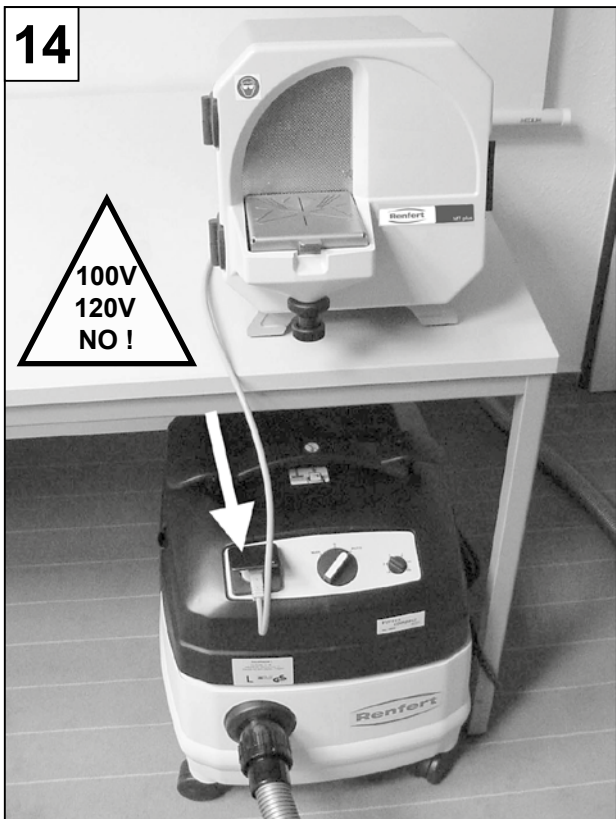
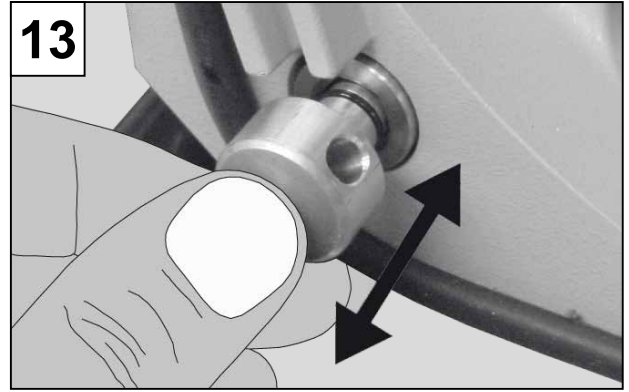
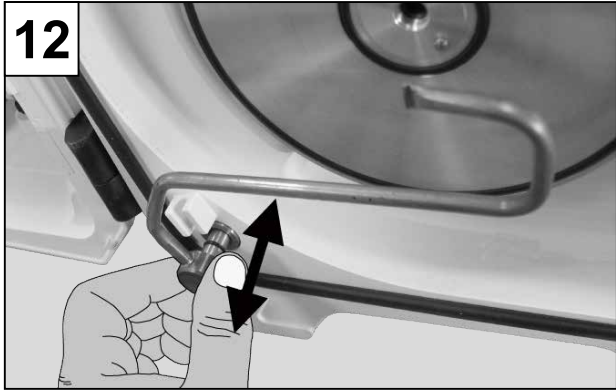
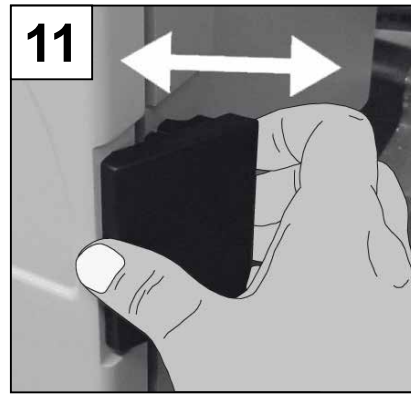
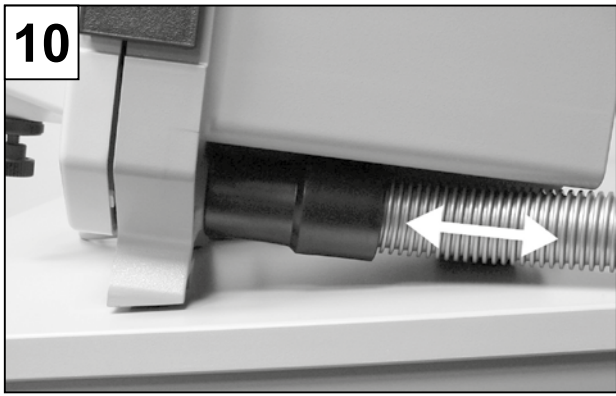
MT plus

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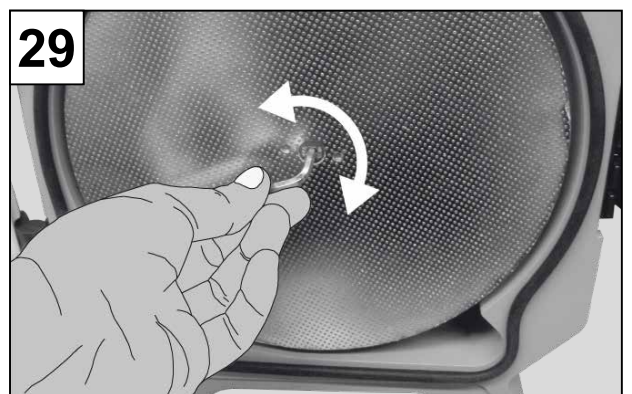
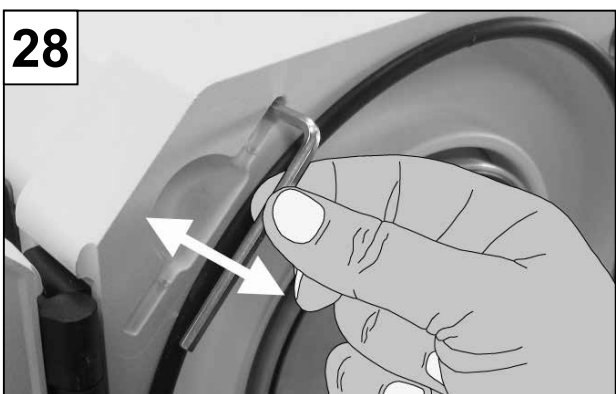
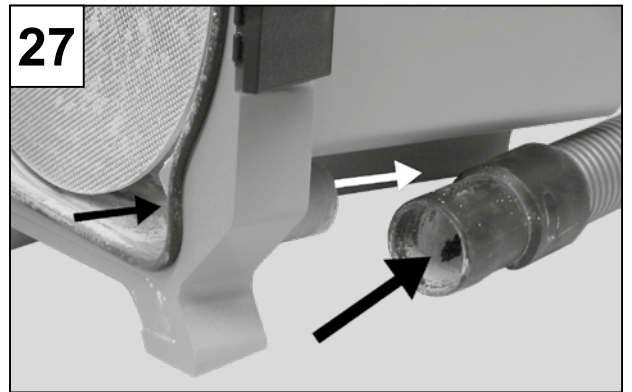
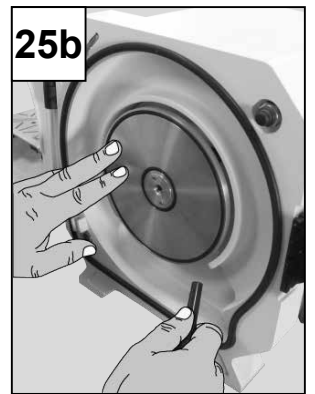
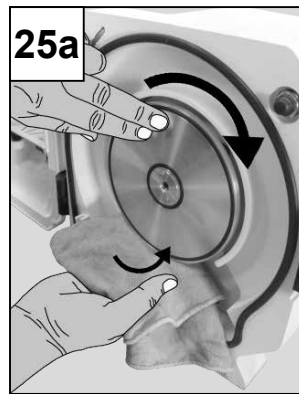
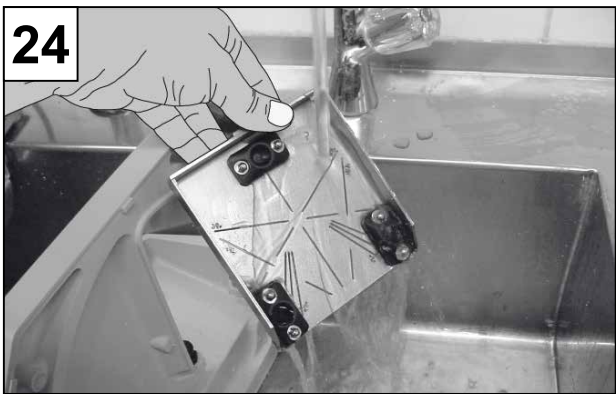
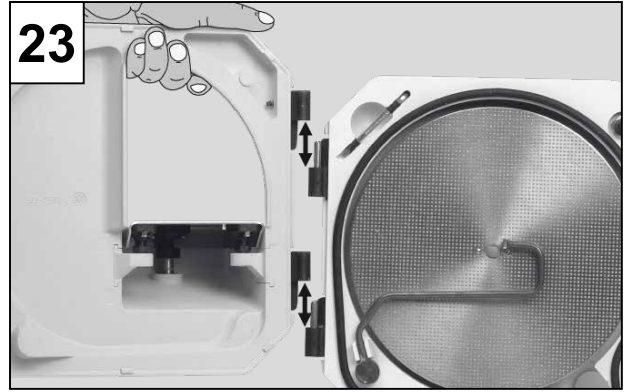
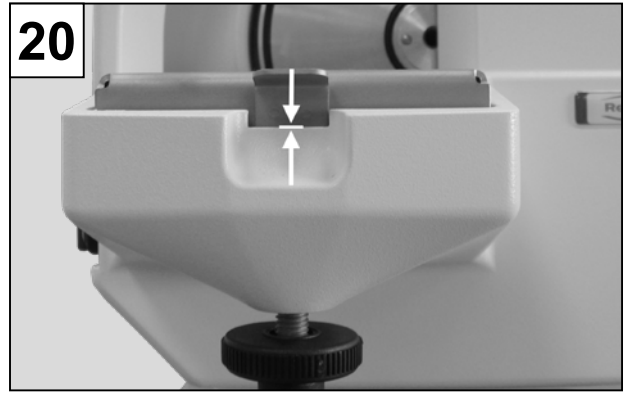
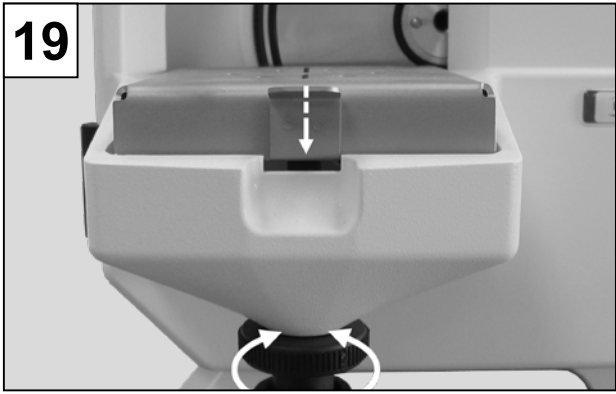
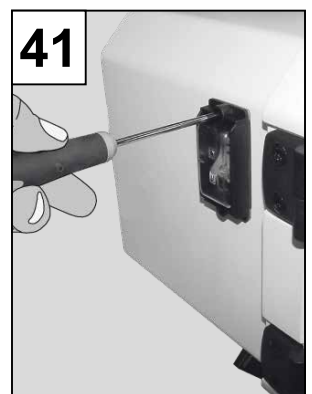
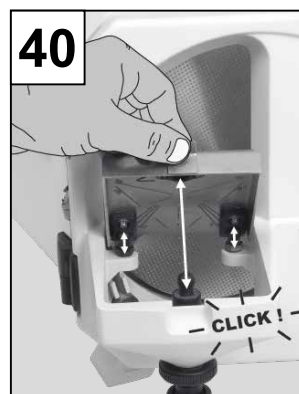
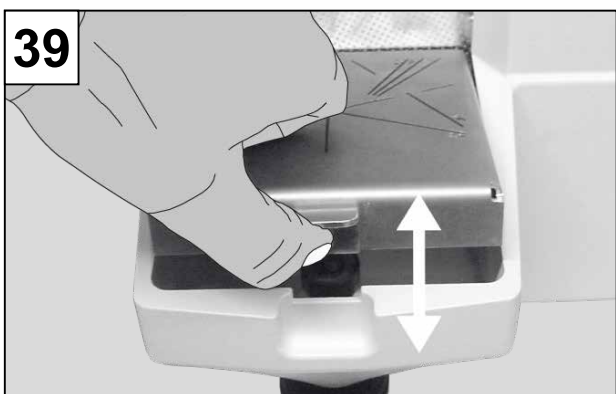
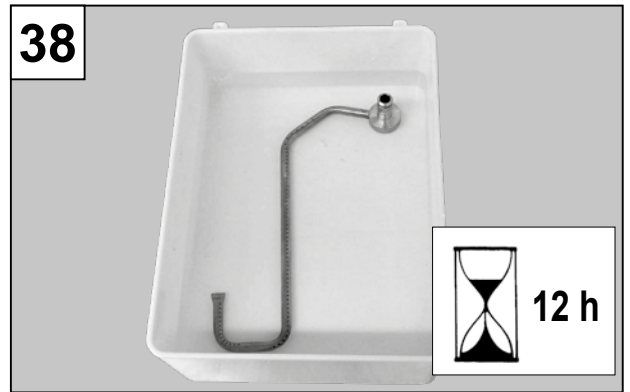
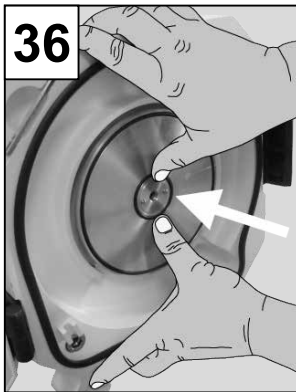
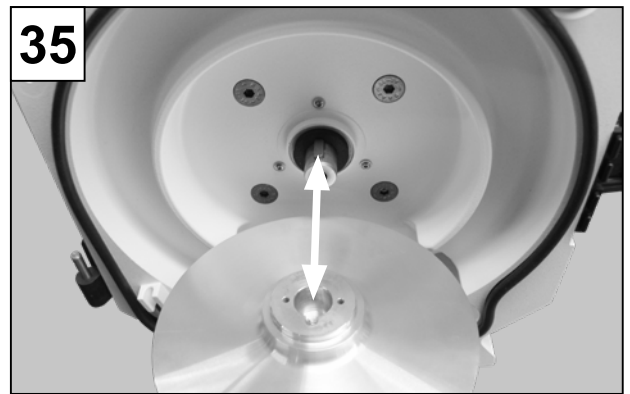
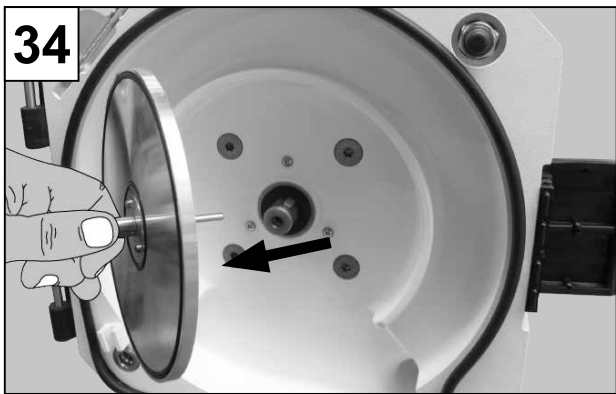
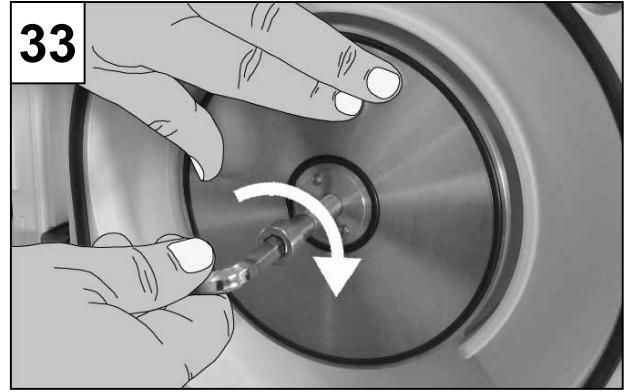
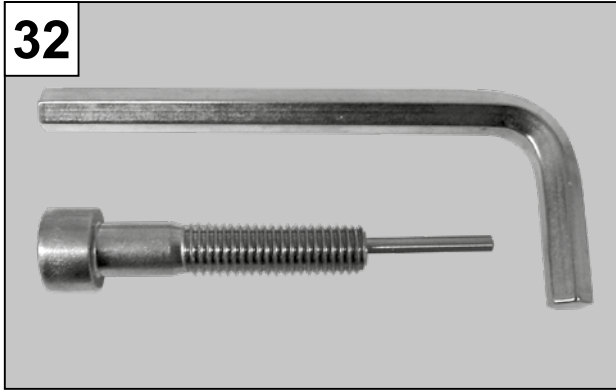
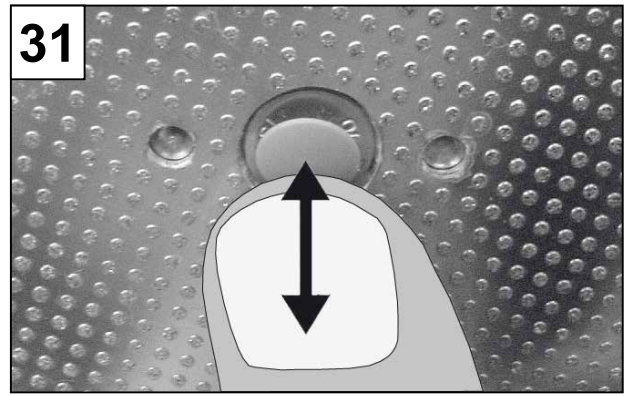
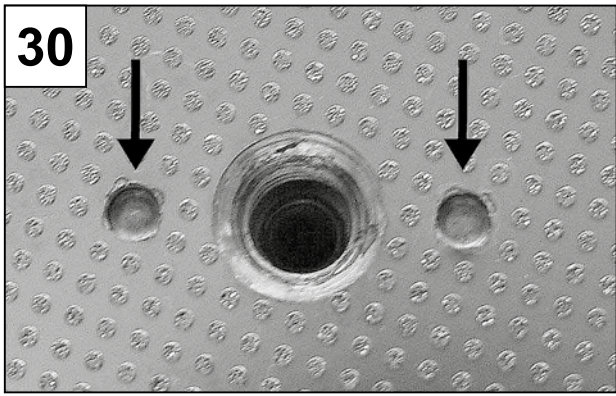
Made in Germany

21-9873 24052024

Inhalt

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Contents

- 1 Introduction 2
- 2 Symbols 2
- 3 Application Area 2
 - 3.1 Ambient Conditions (in accordance with DIN EN 61010-1) 2
- 4 Safety Information 2
 - 4.1 Liability Exclusion 3
- 5 Standard Delivery 3
- 6 Delivery Versions 3
- 7 Accessories 4
- 8 Commissioning / Operation 4
 - 8.1 Mounting the Trimmer Disc 4
 - 8.2 Wet Trimming 4
 - 8.3 Dry Trimming 5
 - 8.4 Conversion from Wet to Dry Operation 5
 - 8.5 Conversion from Dry to Wet Operation 6
 - 8.6 Recommendation for dry or wet trimming 6
 - 8.7 Grinding Angle Adjustment (89° - 99°) 6
 - 8.8 Operation / Trimming 6
- 9 Cleaning / Maintenance 6
 - 9.1 Trimmer Disc Replacement 6
 - 9.2 Flange Replacement / Cleaning 7
 - 9.3 Spray Tube Replacement / Cleaning (Wet Trimmer) 7
 - 9.4 Grinding Table Replacement / Cleaning 7
 - 9.5 Switch Cover Replacement 8
 - 9.6 Spare Parts 8
- 10 Technical Specifications 8
- 11 Troubleshooting Guide 9
- 12 Warranty 10
- 13 Disposal instructions for countries in the EU 10

1 Introduction

We are pleased with your decision to purchase an MTplus. The trimmer sets new standards with regard to function, performance, safety, and design. Please note the following information.

2 Symbols

In the instructions for use and on the unit itself you will find these symbols with the following meanings:

Danger

This indicates a direct risk of injury. Consult accompanying documents!

Electrical current

This indicates a risk of hazard due to an electrical current.

Attention

Disregarding this warning may result in damage to equipment.

Note

This provides the operator with useful information to improve and ease use.

The device complies with the requirements of the applicable EU directives.

This product complies with the relevant UK legislation.

See UKCA Declaration of Conformity in the Internet under www.renfert.com.

The device is subject to the EU directive 2002/96/EG (WEEE directive).

Other symbols are explained as they occur.

3 Application Area

The MTplus is a model trimmer designed for use in dental technology operations to grind plaster models. It can be employed as either a wet or dry trimmer.

3.1 Ambient Conditions (in accordance with DIN EN 61010-1)

The MTplus may only be operated:

- Indoors;
- Up to an altitude of 2,000 m [6,500 ft] above sea level;
- At an ambient temperature of 5 - 40 °C [41 - 104 °F] *);
- At a maximum relative humidity of 80 % at 31 °C [87.8 °F], decreasing linearly to 50 % relative humidity at 40 °C [104 °F] *);
- With a mains power supply where current fluctuations do not exceed 10 % of the nominal value;
- Under level 2 contamination conditions;
- Under overvoltage category II conditions.

*) Between 5 – 30 °C [41 - 86 °F], the unit can be operated at a relative humidity of up to 80 %. At temperatures between 31 - 40 °C [87.8 - 104 °F], the humidity must decrease proportionally in order to ensure operational readiness (e.g., at 35 °C [95 °F] = 65 % humidity; at 40 °C [104 °F] = 50 % humidity). The unit may not be operated at temperatures above 40 °C [104 °F].

4 Safety Information

Please observe all applicable accident prevention regulations together with the following safety notes when trimming:

MTplus is an electrical appliance with a danger potential. This appliance may be connected / used only by authorized and qualified persons after checking its conformity with respect to the national norms.

The unit should only be operated using a mains cable with a country-specific plug system. Any modification required should only be carried out by an electrician.

Never reach into the grinding wheel while the unit is connected to the power supply. Inadvertently starting the unit can result in injury!

Always keep long hair, loose or wide sleeves or other clothing away from the trimmer. Hair or clothing can be pulled into and wrapped up on the grinding wheel, resulting in injury!

-  The unit is designed solely for grinding plaster models. Cancer causing dusts are released when trimming investment material.
-  Never open the front cover while the motor is running. The motor safety shutoff will not stop the grinding wheel immediately.
-  Only open the front cover after the grinding wheel has come to a complete stop.
-  Always wear protective eyewear when grinding.
-  Do not reach into the grinding wheel while it is turning.
-  Do not work without the model table installed and locked („click“). Small parts or fingers can be drawn into the trimmer, resulting in serious injury.
-  Remove the trimmer table only after the grinding wheel has come to a complete stop and the front cover is opened.
-  Operate the equipment only under supervision.
-  It is the responsibility of the operator that national regulations during operation and regarding a repeated safety inspection of electrical equipment are complied with. For Germany these are the regulation 3 by DGUV (German Statutory Accident Insurance) in relation with VDE 0701-0702 (Association for Electrical, Electronic and Information Technology).
-  Information on REACH and SVHC is available on our website www.renfert.com, in the Support area.
- Additional notes for dry trimming:**
-  The extraction must be selected to match the dust being generated (if necessary, consult the responsible professional organization).
-  Operating the unit can result in a very high noise level >>> always wear ear protection.

4.1 Liability Exclusion

Renfert GmbH shall be absolved from all claims for damages or warranty if:

- The product is employed for any purposes other than those cited in the operating instructions.
- The product is altered in any way other than those alterations described in the operating instructions.
- The product is repaired by other than an authorized facility or if any but Renfert OEM parts are employed.
- The product continues to be employed, despite obvious safety faults.

5 Standard Delivery

- 1 MTplus
- 1 Inlet tube
- 1 Drain tube
- 1 Tool set
- 1 Hardware set
- 1 Marathon trimmer disc, only on No. 1803-x500

6 Delivery Versions

- No. 1803-0000 MTplus, 230 V / 50 Hz
- No. 1803-0500 MTplus, 230 V / 50 Hz, incl. Marathon trimmer disc
- No. 1803-4000 MTplus, 120 V / 60 Hz
- No. 1803-4500 MTplus, 120 V / 60 Hz, incl. Marathon trimmer disc
- No. 1803-6000 MTplus 230 - 240 V / 50 Hz, AU/NZ
- No. 1803-6500 MTplus 230 - 240 V / 50 Hz, AU/NZ incl. Marathon trimmer disc

7 Accessories

No. 2011-0000 GO-2011, plaster solvent

Trimming discs - for wet trimming only:

- No. 1803-1000 **Basic kit:**
5 Klettfix silicon carbide trimmer discs (3 x corn 80, 2 x corn 120) + support disc
- No. 1803-1100 **Refill kit:**
5 Klettfix silicon carbide trimmer discs, corn K80
- No. 1803-1200 **Refill kit:**
5 Klettfix silicon carbide trimmer discs, corn K120
- No. 1803-3000 1 Infinity trimmer disc, diamond-coated

Trimming discs - for wet and dry-trimming:

- No. 1803-2000 1 Marathon trimmer disc, partially diamond-coated

Extraction units etc.:

Silent TS:

- No. 2921-0050 Silent TS, 230 V, 50 / 60 Hz
- No. 2921-1050 Silent TS, 100 - 120 V, 50 / 60 Hz
- No. 2921-0002 Dust bag set (5 bags)
- No. 2921-0003 Muffler set, 2 mufflers
- No. 15-0823 Suction hose (by the metre)
- No. 900034305 Hose fitting adapter
- No. 933334240 Suction hose
- No. 900034314 Y-junction

Vortex

- No. 2924-5000 Vortex compact 3L, 230 V
- No. 2924-6000 Vortex compact 3L, 120 V
- No. 2924-5003 5 filter bags
- No. 2924-5005 1 cloth filter
- No. 92923-0001 3 m suction hose with elastic coupling
- No. 92923-0010 1 elastic coupling only, Ø 36 mm
- No. 2-0973 1 90° plastic elbow for drain tube

8 Commissioning / Operation

First, check to make sure that your mains power corresponds with the power requirements given on the trimmer nameplate.

The type of usage wet- (see Chap. 8.1 and 8.2) or dry-trimming (see Chap. 8.1 and 8.3) must be determined prior to use.

8.1 Mounting the Trimmer Disc

Please refer to the trimmer disc instructions (resp. to Chap. 9.1 in this instructions) for the mounting procedure.

Range of application of the discs:

- Klettfix for wet trimming
- Marathon for wet or dry trimming
- Infinity for wet trimming

8.2 Wet Trimming

1. Set the unit up near the water source / drain, with a plaster trap (Fig. 1).
2. Connect the drain hose to the trimmer (Fig. 2a). Turn the drain hose on the connecting branch counter clockwise and fix with a hose clamp (Fig. 2b)!
3. Position the trimmer hose in the drain (Fig. 3a / 3b).
4. Connect the water hose to the trimmer (Fig. 4a / 4b). Observe the necessary water pressure (see Chap. 10 / technical specifications)!
5. Connect the water hose to a water supply with a shutoff tap (Fig. 5).

6. Open the water supply (Fig. 6a) and check for leaks (Fig. 6b).
 7. Plug the power cord into the wall outlet (Fig. 7).
 8. Switch the unit on (Fig. 8).
 9. Adjust the water flowrate (Fig. 9).
- Attention:** the water inlet is closed on delivery.

The MTplus is now ready for use as a wet trimmer.

General information concerning wet trimming:

- An adequate water flow is important, particularly when trimming with diamond-coated discs. Overheating will damage the disc.
- The water supply shutoff tap should be turned off during longer work interruptions or overnight.

8.3 Dry Trimming

- 1a. Connect the trimmer to an external extractor (Fig. 10 illustrates an example using a Renfert extraction hose and coupling).

Attention!

100 V / 120 V units in combination with dust extractors should be separately connected to the mains in order to avoid an overcharging of the extractor and of the electric network (do not use the automatic start-up!). If the electric network is protected by fuse of only 15 A, it is necessary to connect the trimmer and the extractor to separately protected electric circuits.

Recommended dust extractors:

see: Accessories, Extraction units etc.

- 1b. Connect the trimmer to the central extraction (Fig. 10 illustrates an example using a Renfert extraction hose and coupling).

Recommended accessories:

see: Accessories, Extraction units etc.

2. Open the front cover (Fig. 11).
3. Remove the rinse tube (Fig. 12).
4. Close the opening with the blind plug (**completely impress it**) from the hardware set (Fig. 13).
5. Close the front cover (Fig. 11).
- 6a. For extraction **with** automatic start-up:
 - Connect the trimmer power cord to the extractor (Fig. 14).
 - Set the extractor up for automatic operation (Fig. 15).
- 6b. For extraction **without** automatic start-up:
 - Plug the trimmer power cord into the wall outlet (Fig. 7).
 - Switch on the MTplus (Fig. 8), switch the extractor on automatic use.

The MTplus is now ready for use as a dry trimmer.

8.4 Conversion from Wet to Dry Operation

1. Disconnect the unit from the power supply (Fig. 7).
2. Shut off the water supply (Fig. 6a).
3. Remove the water hose (Fig. 4a / 4b).
4. Remove the drain hose (Fig. 2).
5. If necessary, remove the unit from the wet area and **allow to dry completely**.
All remaining moisture must be blown off with compressed air!
6. Remove the trimmer disc (see Chap. 9.1, steps 4 - 7).
7. Clean the area behind the flange first with a wet and then with a dry cloth (Fig. 25a). Moreover blow off with compressed air (Fig. 25b).
8. Once the unit is dry, proceed as described in Section 8.3.

Partially diamond-coated trimmer discs should be used for dry operation.

Recommended trimming disc:

see: Accessories, Trimming discs - for wet and dry trimming.

8.5 Conversion from Dry to Wet Operation

1. Disconnect the unit from the power supply (Fig. 7 or 14).
2. Remove the extractor hose (Fig. 10).
3. Remove the trimmer disc (see Chap. 9.1, steps 4 - 7).
4. Using a brush, thoroughly clean the front cover and the inside of the housing with an extractor and a brush (Fig. 26).
5. Remove the blind plug. Should it be difficult, you can use the Allen key (Fig. 16).
6. Mount the trimmer disc (see instructions for trimmer discs).
7. Install the rinse tube (Fig. 12).
8. Then, proceed as described in Chap. 8.2.

8.6 Recommendation for dry or wet trimming

- Dry trimming is not allowed when the MTplus is equipped for wet trimming.
- Wet trimming is not allowed when the MTplus is equipped for dry trimming.
- Not observing the prior instructions will result in plaster residue mixing with water and hardening to a difficult to remove crust. Such residues particularly on the back of the trimmer disc can lead to imbalance.

8.7 Grinding Angle Adjustment (89° - 99°)

1. Loosen the knurled nut (Fig. 17).
- 2a. Adjust the grinding angle roughly with the adjusting screw (Fig. 18).
If the adjusting screw is turned to the **top**, the angle between model table and trimmer disc is of 89°.
If the adjusting screw is completely turned **out**, the angle between model table and trimmer disc is of 99°.
- 2b. Adjustment of the model table roughly with the adjusting screw (Fig. 18). The lower border of the model table should coincide with the upper border of the model-table support (Fig. 19 + 20).
- 2c. Fine adjustment of the model table at 90° with the adjusting screw (Fig. 18) and set square out of the hardware set (Fig. 21).
3. Retighten the knurled nut (Fig. 17).

8.8 Operation / Trimming

Using both hands, securely set the model on the trimmer table and press it against the disc (Fig. 22).

9 Cleaning / Maintenance

A raising of plaster dust in aerodynamic dead areas when dry trimming is normal and has no negative influence on the function or grinding power of the MTplus.

Never clean the device under running water or immerse in water.

1. Disconnect the unit from the power supply (Fig. 7 or 14).
2. If necessary, shut off the water supply (Fig. 6).
3. Open and unhook the front cover (Fig. 11 + 23).
- 4a. **For wet trimmer:** If necessary, remove the table (Fig. 39 + 40) and clean with a brush under running water (Fig. 24).

Do not remove the grinding table if the front cover is closed otherwise the trimmer disc will be damaged!

- 4b. **For dry trimmer:** Clean using a vacuum and brush (Fig. 26). Clean the drain tube and tube connection (Fig. 27).
5. Reinstall and close the front cover (Fig. 23 + 11).

9.1 Trimmer Disc Replacement

1. Switch the unit off (Fig. 8).
2. Disconnect the unit from the power supply (Fig. 7 or 14).
3. If necessary, shut off the water supply (Fig. 6a).
4. Open the front cover (Fig. 11).
5. If necessary, remove the rinse tube (Fig. 12).

6. Remove the SW4 Allen key from its holder (Fig. 28).
7. Remove the thread cover, loosen the centring screw, and remove the disc (Fig. 31 + 29).
8. If necessary, clean the flange (refer to Chap. 9.2).
9. Place a new disc on the lock pin (Fig. 30).
10. Retighten the centring screw with the Allen key (Fig. 29).
11. Place the thread cover back on the centring screw (Fig. 31).
12. Return the Allen key to its holder (Fig. 28).
13. If necessary, replace the rinse tube (Fig. 12).
14. Close the front cover (Fig. 11).
15. If necessary, open the water supply (Fig. 6a).
16. Plug the power cord into the wall outlet (Fig. 7 or 14).

9.2 Flange Replacement / Cleaning

1. Remove the disc (refer to Chap. 9.1, steps 1 - 7).
2. Have the pulling screw and Allen key at hand >>> see tool kit enclosed (Fig. 32).
3. Screw the pulling screw into the flange (Fig. 33) >>> **The flange may not be removed by hand. Always use the pulling screw to remove the flange!**
4. Remove the flange (Fig. 34).
5. Remove the pulling screw from the flange.
6. Clean the flange and the inside of the housing >>> **Do not damage the seals!**
Recommended for cleaning: >>> see: Accessories
7. Lightly coat the motor shaft and seal with Vaseline.
8. Replace the flange >>> **Note the engagement groove** (Fig. 35).
9. Replace the flange by hand on the shaft (Fig. 36).
10. Reinstall the grinding disc (refer to Chap. 9.1, steps 9 - 16).

9.3 Spray Tube Replacement / Cleaning (Wet Trimmer)

1. Remove the rinse tube (refer to Chap. 9.1, steps 1 - 5).
2. Remove the gaskets (Fig. 37).
3. Allow the rinse tube to soak in decalcifying agent overnight (Fig. 38).
4. Reinstall the gaskets and coat them lightly with Vaseline (Fig. 37).
5. Replace the rinse tube (Fig. 12).
6. Close the front cover (Fig. 11).
7. If necessary, open the water supply (Fig. 6a).
8. Plug the power cord into the wall outlet (Fig. 7 or 14).

9.4 Grinding Table Replacement / Cleaning

1. Switch the unit off (Fig. 8).
2. Disconnect the unit from the power supply (Fig. 7 or 14).
3. Open the front cover (Fig. 11).

! Do not remove the grinding table if the front cover is closed otherwise the trimmer disc will be damaged!

4. Unlatch and remove the grinding table (Fig. 39 + 40).
5. Remove the front cover (Fig. 23).
6. Clean the grinding table and front cover with a brush under running water (Fig. 24).
7. Replace and close the front cover (Fig. 23 + 11).
8. Reinstall the grinding table (Fig. 40).

i *Make sure the retainers are completely free of all plaster residue! Make sure the work table is properly seated on the front cover!*

9. Plug the power cord into the wall outlet (Fig. 7 or 14).

9.5 Switch Cover Replacement

1. Loosen the fastening screws (Fig. 41).
2. Remove the switch cover and replace it with a new one (Art. No. 90003-5788).
3. Retighten the fastening screws.

The MTplus may not be operated with a defective switch cover!

9.6 Spare Parts

You can find components subject to wear and the spare parts on the spare part list in the internet at www.renfert.com/p918.

The components excluded from the warranty (such as consumables or parts subject to wear and tear) are marked on the spare part list.

Serial number and date of manufacturing are shown on the type plate of the unit.

10 Technical Specifications

Mains voltage:	220 - 230 V, 50 / 60 Hz 230 - 240 V, 50 Hz 120 V, 60 Hz 100 V, 50 / 60 Hz
Power consumption:	220 - 230 V: 1300 W (1,743 hp) 230 - 240 V: 1300 W (1,743 hp) 120 V: 1500 W (2,012 hp) 100 V: 1200 W (1,609 hp)
Water consumption, max.:	9 l/min. [0.32 cfm]
Necessary water pressure: - min.: - max.:	1 bar [14.5 psi] 5 bar [72.5 psi]
Speed:	50 Hz: 2850 1/min 60 Hz: 3400 1/min
Connection piece diameters for external extraction:	36 mm [1.42 inch]
Dimensions (W x H x D):	300 x 320 x 410 mm [11.8 x 12.6 x 16.2 inch]
Ø Trimmer disc dimensions:	234 mm [9.2 inch]
Weight:	13,5 kg [30 lbs]
Sound pressure level accord. to DIN 45635-01-KL3, measured under full load: - Lp(A): - K:	74 dB(A) 4 dB

11 Troubleshooting Guide

Fault	Possible cause	Corrective action
Device fails to start.	<ul style="list-style-type: none"> • Power supply not established. • Mains fuse faulty. • Front cover not properly closed. • Motor overheated. 	<ul style="list-style-type: none"> • Check the mains power supply. • Check the mains fuse. • Close the front cover (Fig. 11). • Allow the motor to cool down, then restart at the main power switch.
Trimmer disc wobbles.	<ul style="list-style-type: none"> • Disc is not lying flush on the flange. • Disc not tightened sufficiently. 	<ul style="list-style-type: none"> • Reinstall the disc according to the directions. Pay attention that the disc is mounted exactly on the lock pin (Fig. 30). • Remove all deposits from the back of the disc and from the stopping face of the flange. • Hand-tighten the disc (Fig. 29).
Excessive noise while the trimmer disc is running.	<ul style="list-style-type: none"> • Disc not adequately tightened. 	<ul style="list-style-type: none"> • Hand-tighten the disc (Fig. 29), or reinstall, as required (Chap. 9.1).
Trimmer table wobbling.	<ul style="list-style-type: none"> • Trimmer table incorrectly installed. 	<ul style="list-style-type: none"> • Reinstall the table according to the directions (Fig. 40).

Wet Trimming:

Fault	Possible cause	Corrective action
Disc buildup.	<ul style="list-style-type: none"> • Water volume incorrectly adjusted at the device. • Water supply not fully open. • Spray tube plugged. • Inlet strainer plugged. • Inlet water pressure too low. 	<ul style="list-style-type: none"> • Increase the water volume at the trimmer (Fig. 9). • Open the water supply (Fig. 6). • Clean / decalcify the spray tube (Chap. 9.3). • Remove the strainer with pliers and clean / decalcify it. • The diameter of the inlet pipe is too small resp. too many users are connected to a small pipe >>> Check if water pressure is at least 1 bar (ask your plumber). If necessary the water pressure can artificially increased by a domestic water work (electric pump and pressure tank). • Ask your plumber.
Water sprays out of the grinding opening.	<ul style="list-style-type: none"> • Too much water. • Water drainage impeded. 	<ul style="list-style-type: none"> • Reduce the water volume at the trimmer (Fig. 9). • Check/properly lay out drain tube >>> slope (Fig. 3a + 3b).
No rinse water.	<ul style="list-style-type: none"> • Water connection not properly established. • Spray tube soiled / calcified. 	<ul style="list-style-type: none"> • Check the water connection (Chap. 8.2 steps 4 - 6). • Clean/decalcify the spray tube (Chap. 9.3).
Water is dripping out between the front door and the housing.	<ul style="list-style-type: none"> • Gypsum powder order plaster grains on the sealing surface. 	<ul style="list-style-type: none"> • Clean the sealing surface and the sealing ring.

Dry Trimming:

Fault	Possible cause	Corrective action
Disc buildup.	<ul style="list-style-type: none">• Models are too wet.	<ul style="list-style-type: none">• Clean the disc with a brush under running water or with plaster solvent (Renfert GO-2011 Art. No. 2011-0000).
Excessive dust being blown out of the grinding opening.	<ul style="list-style-type: none">• Extraction not switched on.• Insufficient suction.• Extractor or dust bag full.• Extractor hose blocked.	<ul style="list-style-type: none">• Switch the extraction on.• Increase the suction.• Replace the dust bag.• Check / clean the hose.

12 Warranty

Provided the unit is properly used, Renfert **warrants** all parts of the MTplus for a **period of 3 years**.

In case of any claims for warranty original dealers' invoice is required.

Components subject to natural wear (e.g., trimmer discs) are excluded from this warranty.

The warranty is voided in case of improper use, failure to observe the operating, cleaning, maintenance, and connection instructions, in case of independent repairs or repairs by unauthorized personnel, if spare parts from other manufacturers are employed, or in case of unusual influences or influences not in compliance with the utilization instructions. Warranty service shall not extend the original warranty.

13 Disposal instructions for countries in the EU

To conserve and protect the environment, prevent environmental pollution and improve the recycling of raw materials, the European Commission adopted a directive that requires the manufacturer to accept the return of electrical and electronic units for proper disposal or recycling.

Within the European Union units with this symbol should not therefore be disposed of in unsorted domestic waste:

For more information regarding proper disposal please apply at your local authorities.

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declare under our sole responsibility, that the product(s)

Name	Part No.
MT plus	1803 0000 1803 0500
MT3	1808 0000 / 1808 0000 03 1808 0500 / 1808 0500 03
MT premium	1807 0000 1807 0500

is (are) in conformity with the relevant regulatory requirements by compliance with the UK designated standards.

UK legislation

Electromagnetic Compatibility Regulations 2016

Supply of Machinery (Safety) Regulations 2008

The Restriction of the Use of Certain Hazardous Substances in Electrical and Electronic Equipment Regulations 2012

Designated standards

EN 61326-1:2013

EN ISO12100:2010

EN 61010-1:2010/A1:2019/AC:2019-04

EN 60825-1: 2014

EN 61000-3-2: 2014

EN 61000-3-3: 2013

EN IEC 63000:2018

Tilo Burgbacher,
Engineering Director

Hilzingen, 02.11.2022

The Technical Documentation has been retained by Renfert GmbH

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Renfert GmbH • Untere Gießwiesen 2 • 78247 Hilzingen/Germany