making work easy

- de
- en
- fr
- it
- es
- pt
- tr
- ru
- zh
- ja
- ko
- ar

AUTO spin

Quick Start Guide

Made in Germany

21-6861 11112024

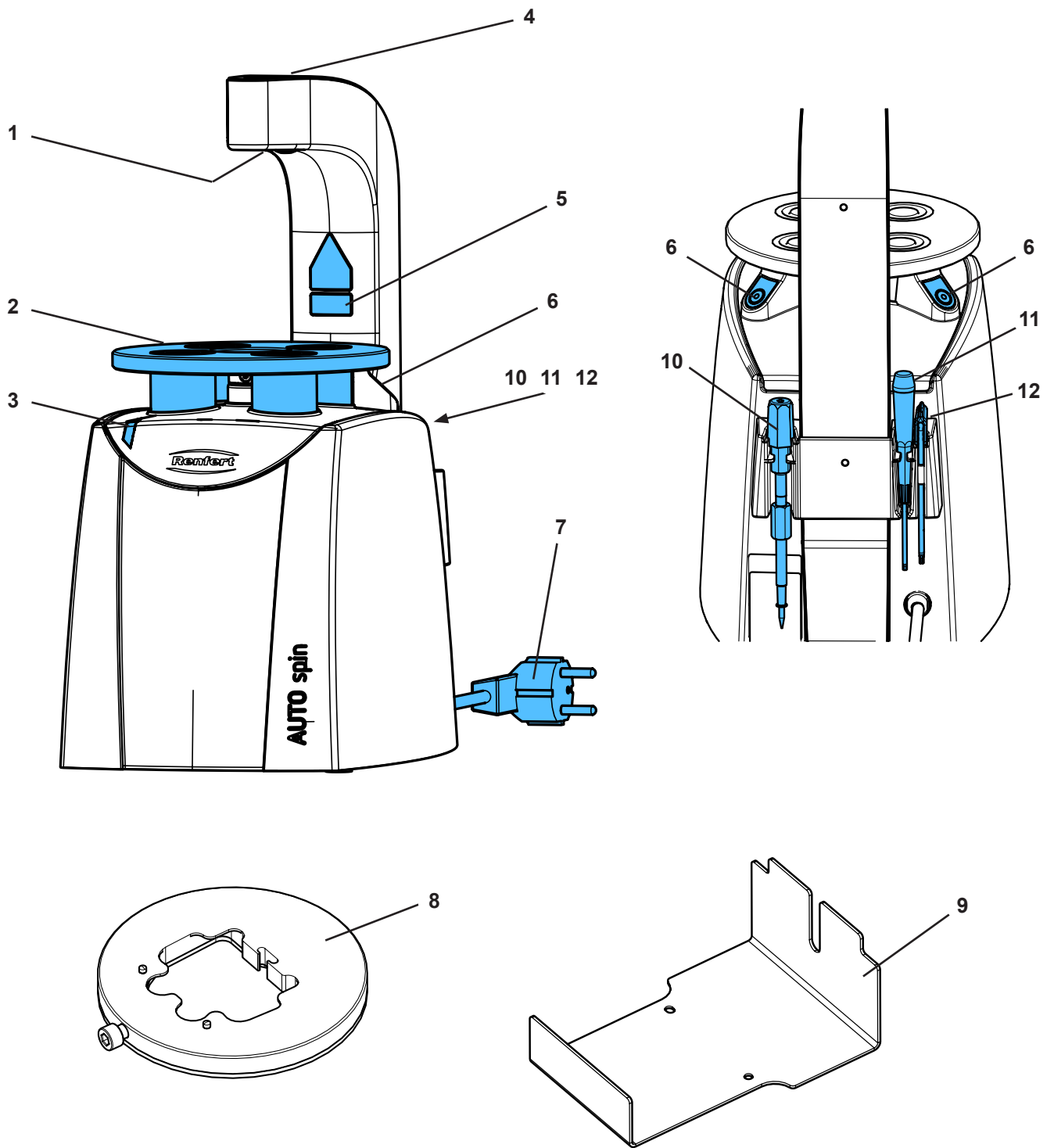
Fig. 1

Einleitung

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Introduction

This Quick Start Guide contains instructions on the operation of the product and relevant safety information.

For detailed information about the device, see the instructions for use which you can download from our website at www.renfert.com/p915.

Enter the following item number: 18600000.

Symbols used

This manual contains symbols also found on the appliance itself with the following meaning:

Danger

Immediate risk of injury. Consult accompanying documents!

Electrical current

This indicates a risk of injury due to electrical current.

Laser

Do not look into the laser beam.

Caution

Risk of damage to the unit if the instruction is not followed.

Notice

A general notice that improves and eases use.

The appliance complies with applicable EU directives.

This product complies with the relevant UK legislation.

See UKCA Declaration of Conformity in the Internet under www.renfert.com.

Within the EU, this device is subject to the provisions of the directive 2002/96/EC (WEEE directive).

Other symbols are explained when displayed.

Safety

Correct use of the unit

The device is a system component for the production of dental section models with a plastic plate as base support.

The device is used to make conical holes in these plastic plates made of POM.

Unapproved Use

Only plastic plates made of POM may be used. The device is not suitable for drilling plastic plates of PMMA (Plexiglas) or other materials.

The device is not suitable to drill in plaster.

Hazards and Warnings

General information

- ▶ Granted protection is not given any longer in case the unit is not operated according to the instruction manual on hand.
- ▶ The unit should only be operated using a mains cable with a country-specific plug system. Any modification required should only be carried out by an electrician.
- ▶ The unit should only be operated if the data on the rating plate corresponds with the data of regional mains voltage.
- ▶ The unit should only be plugged into earthed sockets.
- ▶ The mains socket should be easily accessible.
- ▶ Disconnect the unit from the mains before working on electrical parts.
- ▶ It is the responsibility of the operator that national regulations during operation and regarding a repeated safety inspection of electrical equipment are complied with. For Germany these are the regulation 3 by DGUV (German Statutory Accident Insurance) in relation with VDE 0701-0702 (Association for Electrical & Electronic Technology).

- ▶ Regularly check connecting cables (such as power cables), hoses and housing (for example, the control panel) for damage (for example, kinks, tears, porosity) or deterioration due to aging. Do not operate units with damaged connection cables, hoses or housing parts or other defects!
- ▶ Damaged equipment must be taken out of service immediately. Disconnect from mains and secure against being switched on again. Send the device in for repair!
- ▶ Observe national accident prevention regulations!
- ▶ Information on REACH and SVHC is available on our website www.renfert.com, in the Support area.

Specific information

- ▶ **Caution Laser beam Class 2 laser. Do not look directly into the beam!**
Do not look into the beams reflected by specular surfaces.
- ▶ **Never touch the drill while the unit is connected to the power supply. Inadvertently starting the unit can result in injury!**
- ▶ **Always keep long hair, loose or wide sleeves or other clothing away from the AUTO spin. Hair or clothing can be pulled into and wrapped up on the grinding wheel, resulting in injury!**
- ▶ **Do not touch the rotating drill.**
- ▶ **Operation of the device only with the plate holder in place and plastic plate inserted.**
Risk of injury from rotating drill.
- ▶ **Do not reach under the drilling table during operation, risk of crushing.**
- ▶ **Do not grasp between the drill bit and the drill table during operation, risk of crushing. Risk of injury from rotating drill.**

Approved personnel

The appliance may only be operated and serviced by trained personnel.

Unit must not be used by persons with restricted physical, sensory or mental capabilities, or those lacking the relevant operating experience.

Product description

Assemblies and functional elements

See Fig. 1 (on inside cover)

- | | |
|---------------------------------------|---|
| 1 Target laser | 8 Plate holder |
| 2 Drilling table with holding magnets | 9 Impression holder |
| 3 Operating indicator | 10 Level controller |
| 4 Opening for level controller | 11 Adjusting tool for drilling depth
(screwdriver Torx T5) |
| 5 Laser warning | 12 Tool for drill fixing (angled Torx T10) |
| 6 Operating key, left/right | |
| 7 Power cord with mains plug | |

Standard Delivery

- | | |
|--|---|
| 1 AUTO spin | 1 Tool for drill fixing (angled Torx T10) |
| 1 Drill | 1 Laser sticker set |
| 1 Plate holder | 1 Quick Start Guide |
| 1 Impression holder | 1 AUTO spin system illustrated manual |
| 1 Level controller | |
| 1 Adjusting tool for drilling depth
(screwdriver Torx T5) | |

Commissioning

Unpacking

⇒ Check the delivery for completeness (compare with the “Standard Delivery”).

Setup

⇒ Place the unit on a flat, stable surface.

Do not lift or carry the device by the laser arm.

Replacing the "Laser warning"

⇒ Remove the laser warning in your language from the sticker set and cover over the existing warning label (5).

Electrical connection

⇒ Unroll the power cord and insert the power plug into the building wall socket.

Moving to the start position

If the drill motor is not in its starting position, this is identified and displayed after plugging in the mains plug:

In this case:

- ◆ The operating indicator (3) flashes.

⇒ Press and hold both operating keys (6).

- ◆ The drill motor slowly moves to the starting position.

- ◆ As soon as the start position is reached, the motor stops and the operating indicator lights up permanently.

Operation

Power on / off

The device is switched on and off by pressing one of the operating keys (6).

⇒ To switch on the device, briefly press an operating key.

⇒ To switch off the device, press an operating key for approx. 2 sec.

Auto Off

The device also includes an automatic shutdown feature. If the device is not used for 3 minutes, it turns off automatically.

Level controller

The level controller (10) enables safe planning of the finished model situation.

- No collisions of the pin with the impression.
- No uneven or too high dental arches.

The stroke of the gauge tip in the level controller corresponds to a pin length of 14 mm.

For further handling, please refer to the enclosed AUTO spin system illustrated manual.

Drilling

⇒ Carry out the work steps for creating the model prior to drilling according to the enclosed AUTO spin system illustrated manual.

⇒ Place the prepared unit (plate and impression holder with impression and plastic plate) on the drilling table.

⇒ Align so that the target laser points on the position of the desired pin hole in the impression.

⇒ Press both operating keys (6) simultaneously to initiate the drilling process.

- ◆ The holding magnets secure the plate holder in the desired position.

- ◆ The drill motor starts and drills a hole from below into the plastic plate.

- ◆ The drill motor returns to its start position and stops

- ◆ The holding magnets release the plate holder.

Caution

Danger of drill breakage.

- ▶ **When aligning the target laser, always make sure that the plate holder or metal disc of the plastic plate do not get drilled.**
- ▶ **If the plastic plate is to be used several times, ensure that there is a sufficient distance to already existing drill holes.**


Regularly remove plastic chips that get stuck to the drill.

Remove loose plastic chips from the drill table.


Adjusting the drilling depth

- ⇒ Switch off the device and disconnect it from the power supply.
- ⇒ Remove drill (see Chapter “Changing the drill”).
- ⇒ Insert the drilling depth adjusting tool (11) into the drill chuck from above and adjust the stop screw on the inside:
- ⇒ turn left
 - ◆ Drill hole diameter gets larger.
 - ◆ Pin sits deeper in plastic plate.
- ⇒ turn right
 - ◆ Drill hole diameter get smaller.
 - ◆ Pin sits more shallow in plastic plate.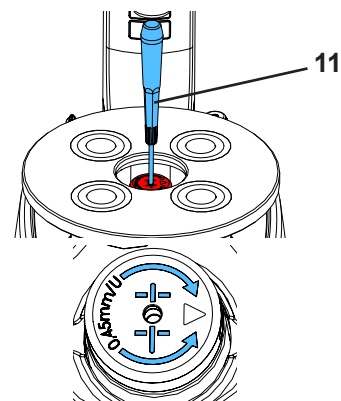

i *One entire revolution of the stop screw changes the drilling depth by 0.45 mm.*

i *A practicable setting is achieved when the inserted pin is flush with the plastic plate when the desired press-in pressure is applied.*

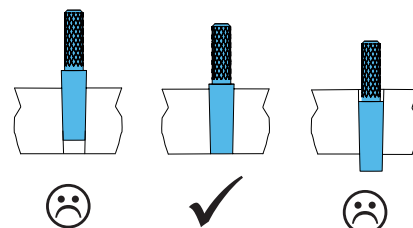Cleaning / Maintenance

The device does not contain any internal parts that require maintenance.

Cleaning


- ⇒ Only wipe the housing with a damp cloth.
- ⇒ Regularly remove loose plastic chips.


Do not use solvents or abrasive cleaners.

Changing the drill

- ⇒ Switch off the device and disconnect it from the power supply.
- ⇒ Turn the drill mount with your fingers until the mark on the drill chuck points to the side.
- ⇒ Use the L-key (12) to loosen the fastening screw until the drill moves (about 1/2 turn).
- ⇒ Remove drill upwards.
- ⇒ Insert new drill.
- ⇒ Re-tighten fastening screw.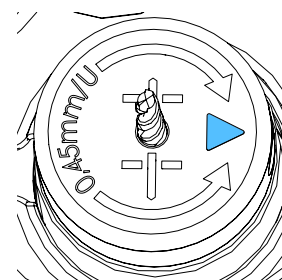

Screw in the screw as far as it will go and then tighten about 1/8 turn.

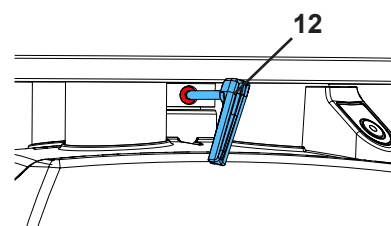Spare parts

You can find components subject to wear and the spare parts on the spare part list in the internet at www.renfert.com/p918. Enter the following item number: 18600000.

The components excluded from the warranty (such as consumables or parts subject to wear and tear) are marked on the spare part list.

Serial number and date of manufacturing are shown on the type plate of the unit.

Disposal Instructions

Within the European Union units with this symbol should not be disposed of in unsorted domestic waste.

For more information regarding proper disposal please apply at your local authorities.

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declare under our sole responsibility, that the product(s)

Name	Part No.
Top spin	1840 0000 / 1840 0000 03
AUTO spin	1860 0000 / 1860 0000 03

is (are) in conformity with the relevant regulatory requirements by compliance with the UK designated standards.

UK legislation

Electromagnetic Compatibility Regulations 2016

Supply of Machinery (Safety) Regulations 2008

The Restriction of the Use of Certain Hazardous Substances in Electrical and Electronic Equipment Regulations 2012

Designated standards

EN 61326-1:2013

EN ISO12100:2010

EN 61010-1:2010/A1:2019/AC:2019-04

EN 60825-1:2014

EN IEC 63000:2018

Tilo Burgbacher,
Engineering Director

Hilzingen, 02.11.2022

The Technical Documentation has been retained by Renfert GmbH

Technical Data

Nominal voltage	230 V	120 V	100 V
Permissible mains voltage:	220 - 240V	120 V	100 V
Mains frequency:	50 / 60 Hz		
Power consumption *): - Between drilling: - During drilling, approx.:	1.5 W 115 W		
Laser: - Laser class: - Wave length: - Power output P0, max.:	2 630 - 680 nm < 1 mW		
LpA *):	< 70 dB (A)		
Dimensions (width x height x depth):	180 x 368 x 275 mm [7.1 x 14,5 x 10.8 inch]		
Permissible drill shaft diameter:	3.00 mm + 0 / - 0.006 mm		
Total drill length (min. - max.):	38 - 40 mm [1.5 - 1.57 inch]		
Weight:	7.6 kg [16.8 lbs]		

*) Sound pressure level in accord. with DIN EN ISO 11202

**Hochaktuell und ausführlich auf ...
Up to date and in detail at ...
Actualisé et détaillé sous ...
Aggiornato e dettagliato su ...
La máxima actualidad y detalle en ...
Актуально и подробно на ...**

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Renfert GmbH • Untere Gießwiesen 2 • 78247 Hilzingen/Germany