Seriennummer und Herstellungsdatum befinden sich auf dem Geräte-Typenschild.  
 Serial number and date of manufacturing are shown on the type plate of the unit.  
 Le numéro de série et la date de fabrication se trouvent sur la plaque signalétique de l'appareil.  
 Il numero di serie e la data di costruzione si trovano sulla targhetta dell'apparecchio.  
 El número de serie y la fecha de fabricación están indicados en la placa identificadora del aparato.  
 Seriennummer en produktiedatum staan op het apparaat-type plaatje.

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
No. 1802-0000

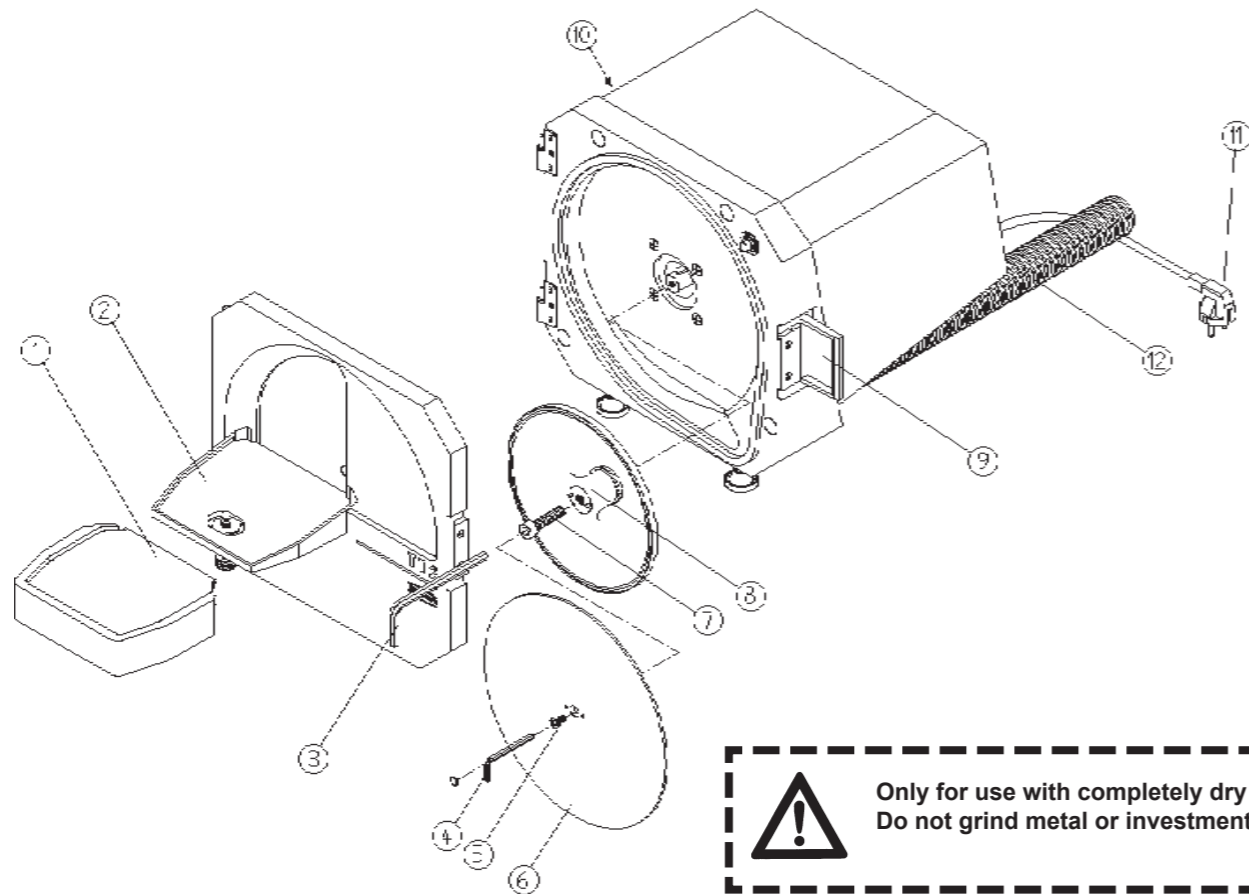ENGLISH

Only for use with completely dry plaster.  
Do not grind metal or investment material!

We are pleased that you decided to buy a TT2 Trimmer, which will set new functional, performance, safety and design standards. To ensure that you enjoy using this product for many years, please follow these instructions carefully.

## 1. Description / Applications

The TT2 is a dry trimmer unit for grinding plaster models in dental laboratories.

### DSS Disc-Stop-System:

The trimmer unit is fitted with a maintenance-free motor brake. The trimmer disc will stop rotating within 5 seconds on switching off the unit or opening the front cover. A hum will be heard from the motor compartment which is normal.

## 2. Installation / Operation

Place the trimmer on a level surface and connect it to an ex-tractor unit that is suitable for capturing plaster dust (e.g. Renfert *Vortex compact EC 2L*, No. 2928-0000 with cloth filter, No. 92924-0005).

For connection to a central extraction unit we recommend the suction hose from our range of accessories (No. 92923-0001, 3 metres) or the muffle with nozzle side Ø 36 (No. 92923-0010). Connect the trimmer, complete with suction hose, to the central extraction unit with the help of a connector piece supplied with your central unit. **For installation of the trimming disc please see point 6.1.**

When placing the trimmer table (1) onto the front cover (2) make sure the table is fixed securely (see Fig. 2). Check that the trimmer disc and the trimmer table are secure.

Ensure that your mains voltage coincides with that quoted on the product identity plate. Once this check is completed insert the plug (11) into the socket. Switch the power on via the switch (10) on the housing.

The TT2 is now ready for use.

## 3. Areas of Caution

Observe the regulations for prevention of accidents of the Employer's Liability Insurance Association and the following safety precautions when trimming:


- Do not touch the disc when trimmer is live because there is danger of injury as soon as the machine runs.
- Do not work at the trimmer with long, untied hair or loose clothes. By winding them into the running machine, there is danger of injury.
- The unit is intended for trimming plaster models only.
- Do not open the front cover whilst the motor is running. The motor switches off automatically when the front cover is opened, however, the disc continues rotating for approx. 5 sec.
- Wear your safety goggles when trimming.
- Open the front cover only after the complete standstill of the grinding disc.
- Never reach into a rotating disc.
- Never work without the model tray. There is a risk of injury if small parts or fingers enter the space between the disc and the casing.

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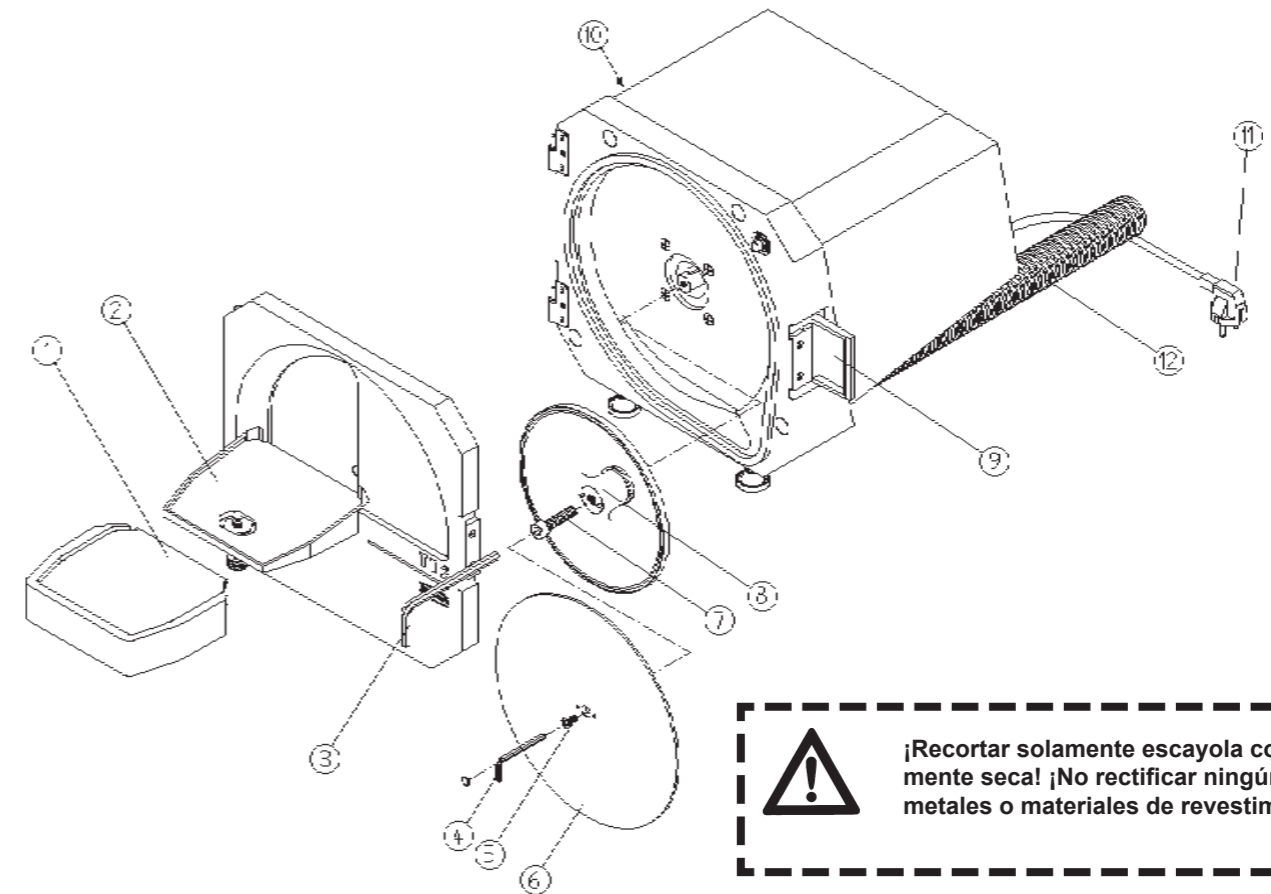- The unit should only be used in conjunction with a dust extractor unit.
- The extractor unit must be suited to capture the relevant dust produced (if in doubt check with your trade association).
- When in use a noise level of 80 dB (A) can occur --> wear ear defenders.

### 4. Operation / Trimming

Hold the model firmly on the trimmer table (1) with both hands and gently press it onto the rotating trimmer disc. The model should, depending on plaster used, have been set for at least one to three hours. The life span of a trimmer disc depends on several factors. Do not grind metal. Muffle rings, pins and retention rings etc., can also damage the trimmer disc. **Do not grind investment material!** After trimming switch off both trimmer and extractor units.

### 5. Troubleshooting Guide

Fault	Cause	Remedy
The unit does not switch on.	<ul style="list-style-type: none"> <li>• The electricity supply is not properly connected.</li> <li>• The electrical safety device is defect.</li> <li>• The front lid is not closed properly.</li> <li>• The motor is overheated.</li> </ul>	<ul style="list-style-type: none"> <li>• Check the electricity supply.</li> <li>• Check the safety device.</li> <li>• Close the front lid.</li> <li>• Let the motor cool down. Switch on the power switch again.</li> </ul>
The disc clogs.	<ul style="list-style-type: none"> <li>• The model is too damp.</li> <li>• The plaster contains too high a percentage of acrylic.</li> </ul>	<ul style="list-style-type: none"> <li>• Clean the disc with a brush under running water or with plaster solvent.</li> </ul>
Too much dust is produced.	<ul style="list-style-type: none"> <li>• The extractor unit is not switched on.</li> <li>• The suction power is too low.</li> <li>• The extractor unit or dust bag is full.</li> <li>• Problem with suction hose.</li> </ul>	<ul style="list-style-type: none"> <li>• Switch on the extractor unit.</li> <li>• Raise the suction power.</li> <li>• Change the dust bag or empty the extractor unit.</li> <li>• Check the suction hose or clear it.</li> </ul>
The disc is wobbling.	<ul style="list-style-type: none"> <li>• Disc is not lying flat against the joint.</li> <li>• Disc has not been tightened sufficiently.</li> </ul>	<ul style="list-style-type: none"> <li>• Refit the disc according to the instructions.</li> <li>• Tighten the screw by using a screw-driver.</li> </ul>
Model tray wobbles.	<ul style="list-style-type: none"> <li>• Model tray is not mounted properly.</li> </ul>	<ul style="list-style-type: none"> <li>• Refit the model tray according to the instructions.</li> </ul>


### 6. Cleaning / Maintenance

A clear extractor vent is important for the smooth functioning of the trimmer. For this reason the unit should be cleaned at least once a week. To do this switch off the unit and unplug it (11). Once the trimmer disc has come to a complete stand-still you can open the lock (9). Now swing the front cover (2) all the way to the left or un hinge it. Clean the front cover using the extractor and a brush. Also use the brush to clean the inner chamber and extractor parts. After cleaning the front cover replace it and close it. Ensure that the spacer unit is present in the hinge bolt.

#### 6.1 Changing the trimmer disc:

Switch off the trimmer and extractor units, unplug (11) and open the front cover. Using the Allen key SW4 (4) loosen the central screw in the middle of the trimmer disc (6) by turning it anti-clockwise. Remove the screw. Clean the support flange if necessary.

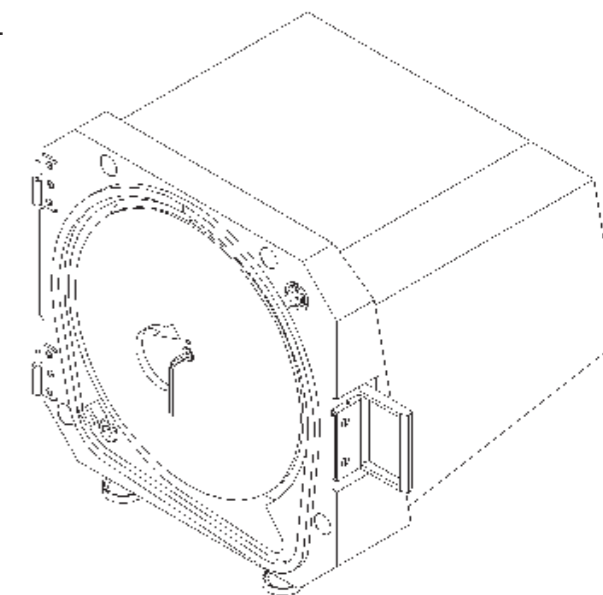Fig. 1

When fitting the new trimmer disc always make sure that the locking pins fit comfortably in the stabiliser holes of the trimmer disc. Tighten the screw (5) with the Allen key by turning it clockwise. After tightening the screw check the rotation of the disc by turning it manually. If the disc touches then it should be removed, rotated by 180°, and then replaced. After changing the disc close the front cover. It can now be plugged in.


### 6.2 Changing / Cleaning the support flange:

Remove the trimmer disc as per the instructions above. Next, turn the screw M10 (7) clock-wise into the support flange (8) using the Allen key SW8 (3) and squeeze the support flange (8) off the motor shaft. Now you can remove the support flange and clean the working area and the support flange. **Do not clean with water!** Remove the screw from the support flange thread again. After rubbing a little Vaseline into the motor shaft and the black rubber washer on the support flange push the support flange back onto the motor shaft. Take note of the position of the take-up on the motor shaft and the position of the take-up nut on the support flange. These must match. Replace the trimmer disc as per the instructions in 6.1.

### 6.3 Changing / Cleaning the trimmer table:

The trimmer table (1) is easily cleaned under running cold water. It can be unhinged from the front plate by pulling it upwards off the adjuster screw and then pulling it sideways. When replacing it the trimmer table is pushed diagonally from above into the trimmer opening. The trimmer table will only lock into the adjuster screw hole (2.3) when the lip (2.1) has snapped on over the edge. In order to ensure a secure hold the trimmer table sides (2.2) must be pressed into the front cover recesses. Always make sure that the trimmer table is seated correctly! To adjust the trimmer table (only for KFO 90°) first unscrew the knurled nut (2.4) clock-wise and re-set the trimmer table via the adjuster screw. After adjusting turn the knurled nut anti-clock-wise to counter the threaded socket. 98° and 90° trimmer tables are available. To make the mounting of the trimmer table easier they can be sprayed in advance with some silicon oil or with washing liquid. **Always make sure that the trimmer table is seated correctly on the front cover!**

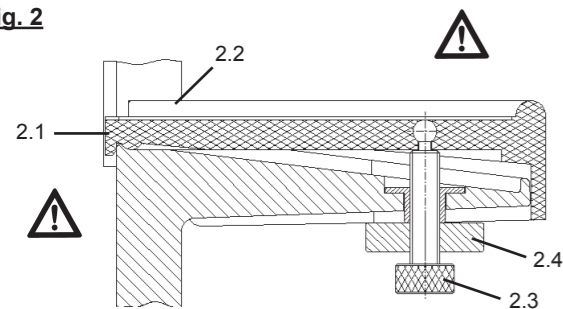Fig. 2

### 7. Spare Parts

You will find the order numbers for wear-and-tear/spare parts in the attached list.

### 8. Guarantee

When used properly Renfert guarantees all parts of the TT2 for 3 years. Parts given to natural wear and tear are excluded from this guarantee (e.g. trimmer discs). The guarantee is void for damage caused by misuse, disregard of the instructions for use, cleaning, maintenance and connection, for self-repaired units or repair by untrained personnel, for use of spare parts from other manufacturers and on influences outside the normal practice or unauthorised in the instruction manual. Services under guarantee do not extend the guarantee.

### 9. Technical Specifications

Tension: 230 V, 50 Hz  
 Power input: 1,630 W  
 Speed: 2,860 r.p.m.  
 Dimensions: 334 x 320 x 456 mm (WxHxD)  
 Weight: 26 kg  
 Level of noise pressure according to DIN 45635-01-KL3 measured when machine was charged totally:  
 Lp(A) = 71 dB(A) K = 8 dB

### 10. Parts supplied

- 1 TT2
- 1 extractor hose
- 1 tool set
- 1 trimmer table KFO 90° (adjustable)
- 1 trimmer table 98° (not adjustable)
- 1 small parts set
- 1 instruction manual with spare part list

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