making work easy

- FAQ - Deutsch
- FAQ - English
- FAQ - Français
- FAQ - Italiano
- FAQ - Español
- FAQ - Português
- FAQ - Türkçe
- FAQ - Русский
- FAQ - Polski
- FAQ - Українська
- FAQ - Suomi
- FAQ - 中文
- FAQ - 日本語
- FAQ - 한국어
- FAQ - اللغة العربية

WAXLECTRIC light

FAQ - frequently asked questions

Made in Germany

20-6130 18042016

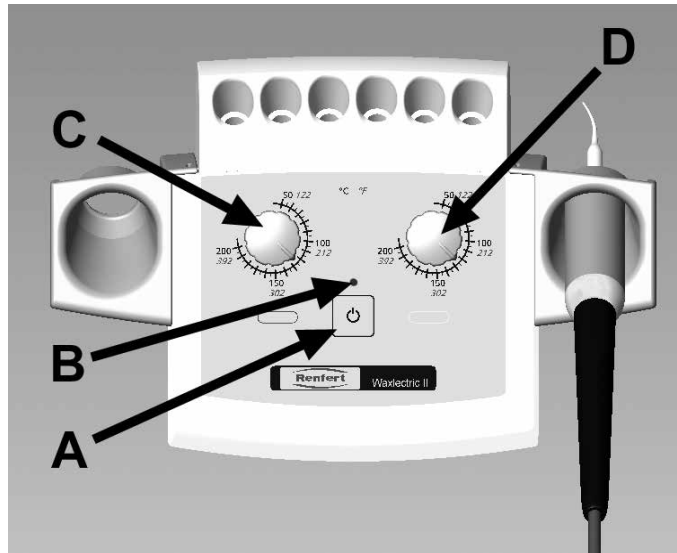
Fig 1 / Fig 2

FAQ - Deutsch

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FAQ - English

For information on operating elements, see [Fig. 1](#) and [Fig. 2](#) on page 2 of this document.

For chapter indications, refer to the complete instruction manual on the internet.

Error	Possible cause	Corrective action
<p>LED B fails to go on.</p>	<ul style="list-style-type: none"> • Control unit not switched on. • Power supply interrupted. • Plug not fully inserted in the control unit. • Faulty control unit. • Faulty transformer. • Short circuit in the tip. 	<ul style="list-style-type: none"> • Switch the control unit on at switch “A”. • Check the power supply and reconnect, as required. • Push the plug in until it unmistakably locks in place. • Have unit replaced by Service Department. • Replace with spare (refer to the spare parts list). • Clean the tip and handpiece (use compressed air). • Replace the tip. • Faulty heater handle --> Replace with spare (refer to the spare parts list).
<p>Unit operating, but no temperature at the tip.</p>	<ul style="list-style-type: none"> • Tip not properly screwed in. • Tip / handpiece contact contaminated. • Faulty tip. • Handpiece cable not properly plugged into the control unit. • Plug on handpiece cable shows signs of corrosion. 	<ul style="list-style-type: none"> • Hand tighten tip. • Clean the contact (use compressed air). • Replace the O-ring (refer to the spare parts list). • Replace with a new tip (refer to Sec. 13 / “Accessories”). • Fully insert the handpiece cable into the socket until the cable locks in place. • Reinsert the plug, clean the contacts, if required.
<p>The tip fails to reach the desired temperature.</p>	<ul style="list-style-type: none"> • Incorrect power supply to the tip because the OEM mains power supply unit is not being used. 	<ul style="list-style-type: none"> • Use the OEM mains power supply unit.

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