making work easy

DE

EN

FR

IT

ES

PT

TR

RU

JA

KO

AR

lay:art oasis

Made in Germany

21-2531 30082019

DE

EN

FR

IT

ES

PT

TR

RU

JA

KO

AR

making work easy

lay:art oasis

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Made in Germany

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EN

lay:art oasis

TRANSLATION OF THE ORIGINAL INSTRUCTIONS FOR USE

Made in Germany

21-2531 30082019

Congratulations, you have chosen the Renfert “lay:art” high-quality, advanced ceramic mixing tray system.

1 Description

The lay:art system consists of a high-grade plastic housing with a lid and high-tech ceramic tray with independent, capillary humidification provided by a high-quality moistening pad.

1.1 Components and functional elements

- 1 Base tray
- 2 Lid
- 3 Ceramic mixing tray
- 4 Moistening pad
- 5 Writable foil
- 6 4 x housing foot
- 7 4 x rest plug
- 8 Non-permanent marker

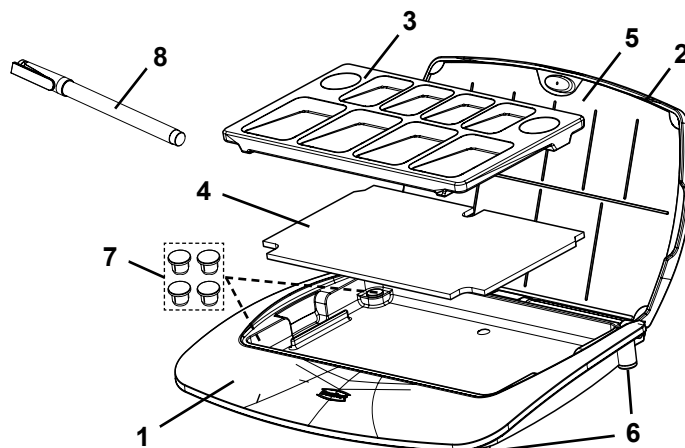
Fig. 1

2 Description of the housing function

You receive the small lay:art housing with your new ceramic mixing tray. The lid can be easily removed from the housing tray at an angle of 90°.

Apply light counter-pressure to the base tray (1) and remove lid (2) upwards.

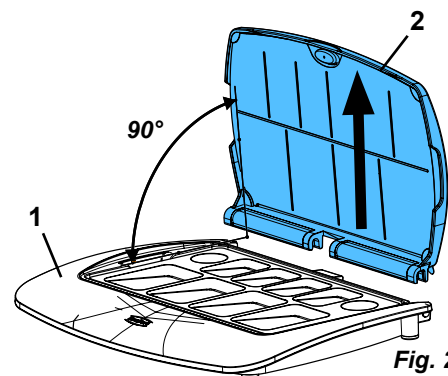
Fig. 2

At an angle of approx. 115° there is a lid locking point, which keeps the lid stable in the opened position.

Caution: Exceeding this point can damage the hinge.

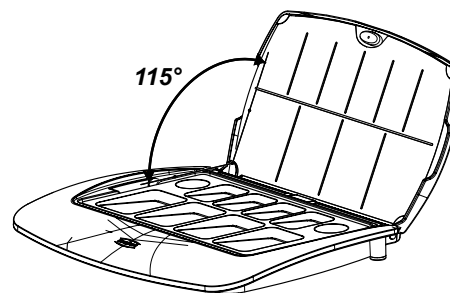
Fig. 3

3 Labelling foil

The writeable foil (5) can be labelled using the non-permanent marker (8) supplied. The foil can be removed from the lid for labelling or replacing. The matt side of the foil is writable. To re-insert, clamp the foil behind the 4 mounting points on the lid. Bend the retaining tabs towards the matted surface (a) before re-inserting.

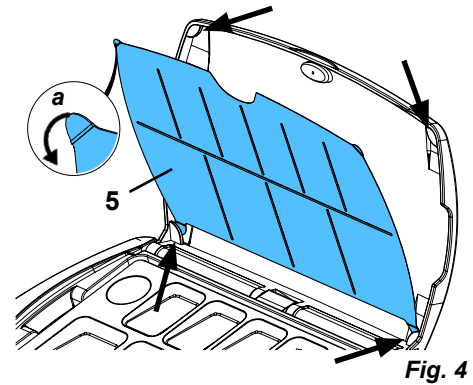
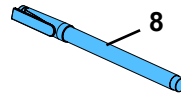
Fig. 4

4 Moistening pad

The following should be noted before first use to guarantee optimum functioning of the moistening pad as a moistening reservoir: Before first use, the moistening pad (4) should be washed out using lukewarm rinsing water (dishwasher detergent) for 45 seconds and then rinsed out under running clear water.

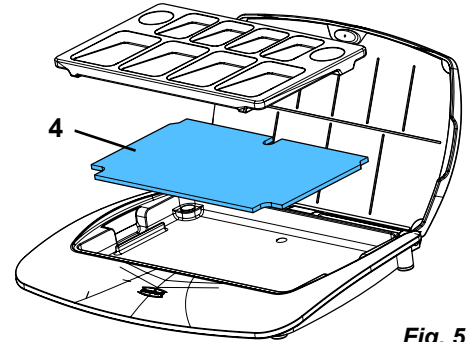
Fig. 5

5 Preparing for use

- After unpacking the housing and ceramic mixing tray, blow clean briefly with compressed air.
- Wash out the moistening pad before first use (see Point 4.).
- Before use, wet the clean moistening pad with distilled water and place the completely soaked moistening pad in the housing base tray.
- Place the ceramic mixing tray on the moistening pad.
- Carefully fill the tray with distilled water until it is visible in the capillary hole openings in the wells.

Tip: Wipe the tray over the pad using a finger before using for quicker moistening.

- The ceramic mixing tray is now ready for use.

6 Use

6.1 Mixing with mixing liquid

Place the required amount of ceramic powder in the ceramic mixing tray. Add the respective quantity of mixing liquid immediately to the ceramic until the required degree of saturation is reached. The ceramic consistency set in this way is maintained over several days due to the capillary effect of the ceramic mixing tray.

6.2 Mixing without mixing liquid

Place the required amount of ceramic powder in the ceramic mixing tray. The capillary effect of the tray moistens the ceramic powder until the optimum degree of saturation is achieved. After mixing thoroughly for a short time, the ceramic is ready to use in the same consistency for several days.

Note that you may have to deviate from the ceramic material manufacturer's instructions with this ceramic mixing technique.

Information: As the water level sinks over time, it is occasionally necessary to refill the distilled water in the housing base tray. The glazed, water-impermeable wells are used for holding modelling fluids, shade mixtures, glazing materials etc.

Never mix the ceramic materials on the ceramic mixing tray using a metal spatula, alternatively use, e.g. our agate spatula (Art. No. 1157-0000). There is the risk of discoloring the surface of the mixing tray and ceramic due to metal abrasion!

7 Care instructions

The mixing tray should be cleaned regularly to maintain consistent quality. Clean it carefully using a steam cleaner or brush it off with distilled water. It is best to clean the housing using commercially available dishwasher detergent and rinse it thoroughly using running water. You can also blow dry the mixing tray using compressed air and blow clean the capillary tubes.

Do not clean using sharp, abrasive or solvent-based cleaners. Use only soft brushes or cleaning cloths!

When using Tricolor stains evaporated staining fluid may dissolve the plastic lid. Do not clean the lid with liquids containing monomer or solvents!

After a long period of use, the capillary effect of the capillary tubes reduces slightly due to contamination in the water and dust formation from the environment. Color residues from the ceramic materials can also form on the surface. Your ceramic mixing tray will be as new again by firing in the preheating furnace:

- Remove the ceramic mixing tray from the housing and completely remove all ceramic residues from the tray. Allow the tray to dry for approx. 3 - 4 hours.
- Place the dry ceramic mixing tray in the cold preheating furnace; placing firing pad, e.g. Fibertray (Art. No. 1919-0000) below the tray ensures safe support.
- The ceramic mixing tray should not be loaded during firing. For this reason, always only fire one ceramic mixing tray.
- Heat the preheating furnace using a maximum heat-rate of 7 - 9 °C/min. to 120 °C and hold for 60 min. Then increase to a final temperature of 800 °C using a low heat-rate of 6 - 8 °C/min.
- Switch off the preheating furnace after reaching the set temperature and cool with the door closed. Remove the ceramic tray from the furnace after it has completely cooled. When the ceramic mixing tray is, at the maximum, warm to the touch, remove it from the preheating furnace using only your hands. Do not use any metal tweezers or furnace tongs!
- Rinse the completely cooled ceramic mixing tray using distilled water to remove any firing residue. Wet clean the bottom of the ceramic mixing tray using 600 grit wet-and-dry sandpaper and follow the sequence at Point 4 (Preparing for use).

8 Replacement parts

10460100 Writable foil S (3 foils)
10455010 Moistening pad (3 pads)

910459991 lay:art housing, small
900030024 lay:art low rest plug (4 plugs)
900030038 Housing feet (4 feet)

9 REACH

Information on REACH and SVHC is available on our website www.renfert.com, in the Support area.

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Renfert GmbH • Untere Gießwiesen 2 • 78247 Hilzingen/Germany