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en | Descaling solution. Danger. Contains: Formic acid. Causes severe skin burns and eye damage. Harmful if inhaled. Do not breathe mist/vapours/spray. Wear protective gloves/protective clothing/eye protection/face protection. IF ON SKIN (or hair): Take off immediately all contaminated clothing. Rinse skin with water or shower. IF IN EYES: Rinse cautiously with water for several minutes. Remove contact lenses, if present and easy to do. Continue rinsing. Immediately call a POISON CENTER. Corrosive to the respiratory tract.

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en | Descaling solution

Danger. Contains: Formic acid. Causes severe skin burns and eye damage. Harmful if inhaled. Do not breathe mist/vapours/spray. Wear protective gloves/protective clothing/eye protection/face protection. IF ON SKIN (or hair): Take off immediately all contaminated clothing. Rinse skin with water or shower. IF IN EYES: Rinse cautiously with water for several minutes. Remove contact lenses, if present and easy to do. Continue rinsing. Immediately call a POISON CENTER. Corrosive to the respiratory tract.

Product description: POWER steamer descaler for descaling the Renfert POWER steamer 1 and 2 is a concentrated descaling solution that is both effective and gentle on material.

Important hazard and safety warnings: The safety data sheet must also be observed. Remove spillages without delay in order to prevent damage to material. Details on how to prepare the device are provided in the POWER steamer instruction manual: www.renfert.com/downloads.

Use / dosage: Rinse the pressure tank through the service opening with water and brush clean. **Fill with 1 liter of water through the inlet. Then fill with 1 liter of Renfert descaling solution.**

Use only diluted! The inlet must remain open during the entire descaling process! A chemical reaction may occur causing foam to develop and descaling solution to spill out – risk of chemical burns!

After approx. 15 minutes, slowly fill the device with a further 0.8 liters (approx.) of water. Allow the descaling solution to soak in.

Recommended exposure time:

Descaling level 1 2–4 hours
Descaling level 2 8–14 hours

Note: If the resulting descaling is not sufficient, repeat the procedure.

After the exposure time has come to an end, carefully open the service opening and drain the descaling solution. Rinse the pressure tank several times with water and brush clean. Close the service opening again.

Disposal: If heavily diluted, the descaling solution that has been used may be disposed of with wastewater.

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