

making work easy

- FAQ - Deutsch
- FAQ - English
- FAQ - Français
- FAQ - Italiano
- FAQ - Español
- FAQ - Português
- FAQ - Türkçe
- FAQ - Русский
- FAQ - Polski
- FAQ - Čeština
- FAQ - 中文
- FAQ - 日本語
- FAQ - 한국어
- FAQ - اللغة العربية
- FAQ - Українська
- FAQ - Română

SILENT powerCAM EC

FAQ - frequently asked questions

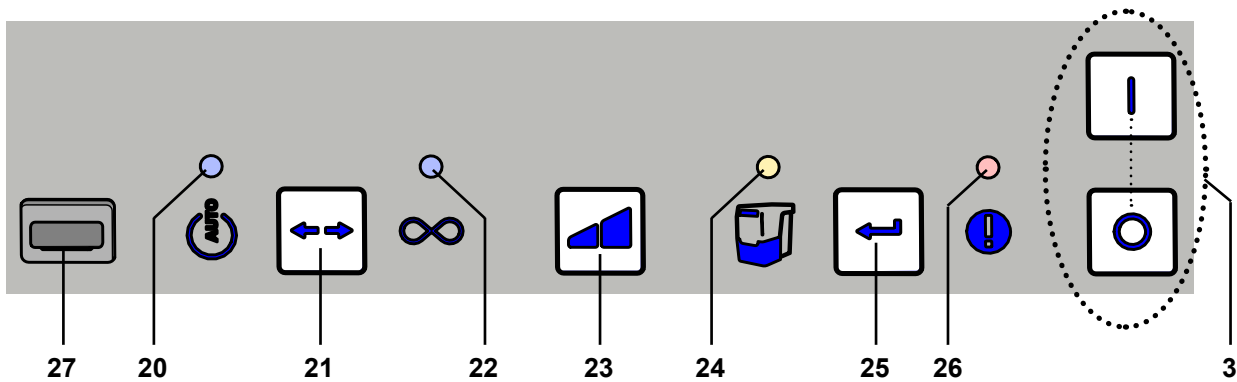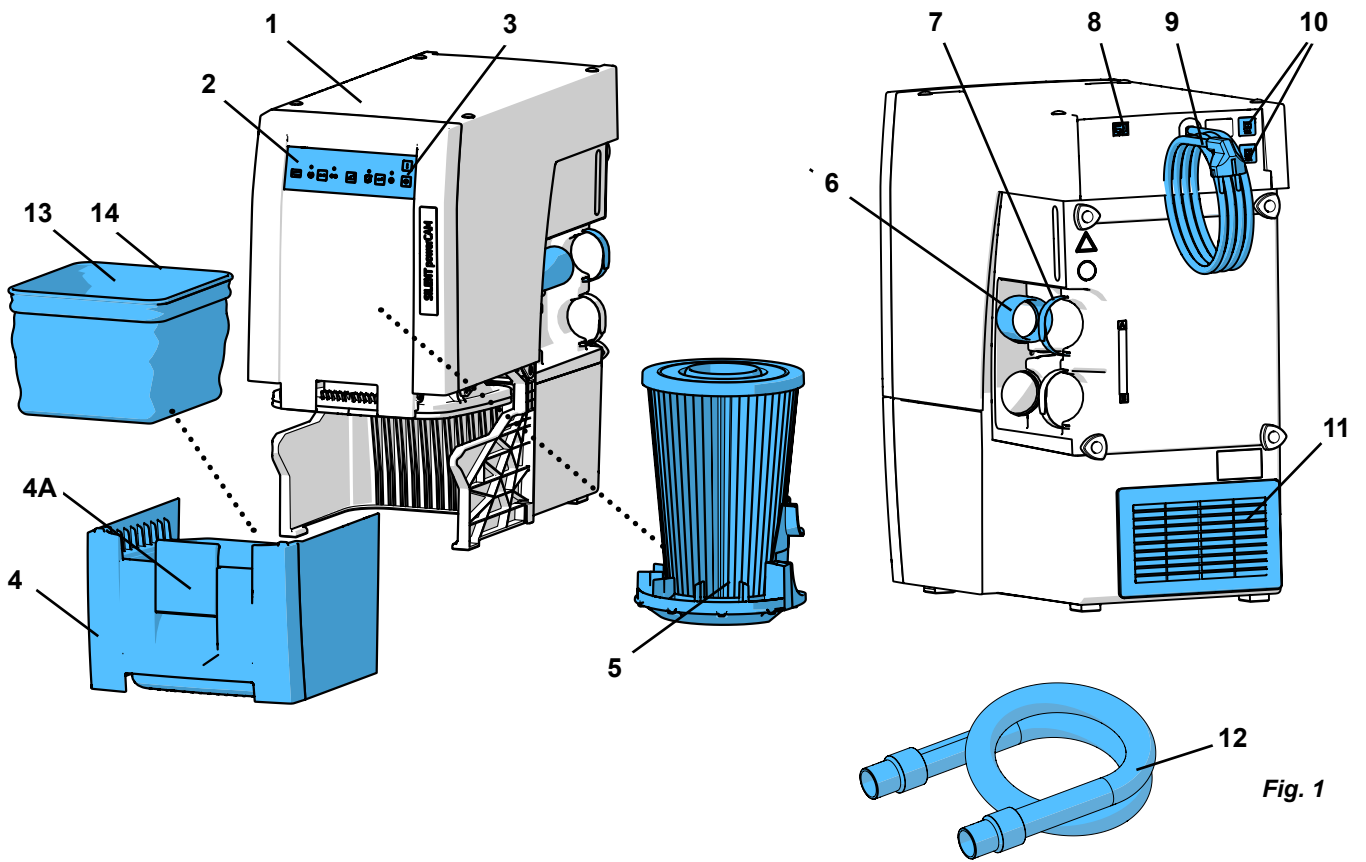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FAQ - English

For information on operating elements, see [Fig. 1](#) and [Fig. 2](#) on page 2 of this document.

For chapter indications, refer to the complete instruction manual on the internet.

Trouble	Possible Cause	Corrective Action
An audible signal sounds, the extraction unit is switched off and filter cleaning takes place.	<ul style="list-style-type: none"> The value has fallen below the internal limit for the flow velocity. Filter cleaning takes place after 8 hours of operation (turbine running time) without being switched off in between times. 	<ul style="list-style-type: none"> Carry on working after the end of the cleaning process. Switch equipment off at the end of each working day using the on / off switch (3, Fig. 1).
After it has been switched on, the dust drawer display (24, Fig. 2) lights up and an audible signal is sounded three times.	<ul style="list-style-type: none"> The time period for emptying the dust drawer has elapsed and the dust drawer has not yet been emptied. The emptying of the dust drawer has not yet been confirmed. 	<ul style="list-style-type: none"> Empty dust drawer and confirm this by pressing the enter key (25, Fig. 2). Confirm that the dust drawer has been emptied by pressing the enter key (25, Fig. 2).
The error message (26, Fig. 2) blinks.	<ul style="list-style-type: none"> The electronic device has become too hot. 	<ul style="list-style-type: none"> Switch the device off and allow it to cool down. Ensure sufficient cooling, e.g. with: <ul style="list-style-type: none"> - Chap. 4.2 Setup. - External ventilation (see chap. 4.6).
All 4 displays flash.	<ul style="list-style-type: none"> Error in the electronics. Suction turbine does not operate. Leakage, because the dust drawer has not been inserted correctly. 	<ul style="list-style-type: none"> Contact Renfert / Service. After changing the suction turbine, check the plug contact of the suction turbine. Check the dust drawer is in the correct position.
The error message lights up (26, Fig. 2) and an acoustic signal occurs 2 x again.	<ul style="list-style-type: none"> The fine filter is so full, that the filter cleaning function alone cannot solve the problem. 	<ul style="list-style-type: none"> Change the fine filter (see accessories and chap. 6.3).
The suction performance is insufficient.	<ul style="list-style-type: none"> Selected extraction force too low. There is a blockage or leak in the suction tube. Dust drawer is not airtight. The fine filter is full. 	<ul style="list-style-type: none"> Select higher suction level. Check suction tube. Please observe the points in chapter 4.4. Check the dust drawer is in the correct position (see chap. 6.2). Switch the unit off and back on in order to initiate the filter cleaning function. If the filter cleaning does not improve the situation, change the fine filter (see chap. 6.3).
Dust drawer is over full.	<ul style="list-style-type: none"> The time period for "empty dust drawer" has been set too high. 	<ul style="list-style-type: none"> Adjust to a lower time period (see chap. 5.5.1).
The signal to empty the dust drawer shows even though the dust drawer is not yet full.	<ul style="list-style-type: none"> The time period for "empty dust drawer" has been set too low. 	<ul style="list-style-type: none"> Adjust to a higher time period (see chap. 5.5.1).

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