making work easy

- de
- en
- fr
- it
- es
- pt
- tr
- ru
- pl
- zh
- ja
- ko
- ar
- da
- cs
- uk

# BASIC eco

Made in Germany

21-6722 18072023

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making work easy

## BASIC eco

TRANSLATION OF THE ORIGINAL INSTRUCTIONS FOR USE

Made in Germany

21-6722 18072023

# Contents

en

<b>1</b>	<b>Introduction</b>	<b>3</b>
1.1	Symbols used	3
<b>2</b>	<b>Safety</b>	<b>3</b>
2.1	Intended use	3
2.2	Meaning of these instructions	3
2.3	Requirements that the operator must meet	3
2.4	Safe condition of the unit	4
2.5	Risks as a result of modifications and repairs	4
2.6	Risk as a result of electric shock and fire	4
2.7	Risk of injury to the eyes or skin	4
2.8	Risk to health as a result of dust	4
<b>3</b>	<b>Product description</b>	<b>5</b>
3.1	General description	5
3.2	Components and functional elements	5
3.3	Standard delivery	6
3.4	Delivery format	6
3.5	Accessories	6
<b>4</b>	<b>Installation</b>	<b>7</b>
4.1	Unpacking	7
4.2	Setting up	7
4.3	Connect foot switch	7
4.3.1	2 Tank version:	7
4.3.2	1 Tank version:	7
4.4	Compressed air connection	7
4.5	Electrical connection	8
4.6	Connect to extraction unit	8
4.7	Filling the sandblasting tanks	8
4.7.1	Authorised sandblasting material	8
<b>5</b>	<b>Operation</b>	<b>9</b>
5.1	Blasting notes	9
5.2	Select blasting pressure	9
<b>6</b>	<b>Cleaning / Maintenance</b>	<b>9</b>
6.1	Sandblasting tank	9
6.2	Cleaning the housing	10
6.3	Cleaning the Interior	10
6.4	Condensation	10
6.5	Blasting nozzle replacement	10
6.6	Cleaning the dosing nozzle	10
6.7	Lamp cover	11
6.8	Spare parts	11
6.9	Shipping Notes	11
<b>7</b>	<b>Trouble shooting</b>	<b>11</b>
<b>8</b>	<b>Technical data</b>	<b>12</b>
<b>9</b>	<b>Warranty</b>	<b>13</b>
9.1	Liability exclusion	13
<b>10</b>	<b>Obligations regarding information</b>	<b>13</b>
10.1	Information on disposal for countries within the EC	13

# 1 Introduction

## 1.1 Symbols used

In these instructions or on the unit itself you will find symbols which have the following meaning:

### **Danger**

This indicates an imminent risk of injury. Please observe the accompanying documents!

### **Electrical hazard**

There is a risk of electrical shock.

### **Attention**

Failure to observe the associated information can result in damage to the unit.

### **Note**

The operating instructions contain useful information to make handling easier.

### **Attention!**

Wear eye protection.

The appliance complies with applicable EU directives.

This product complies with the relevant UK legislation.

See UKCA Declaration of Conformity in the Internet under [www.renfert.com](http://www.renfert.com).

The device is subject to the EU directive 2002/96/EG (WEEE directive).

► List, particular attention should be paid

• List

- List according to importance.

⇒ Instructions / appropriate action / input / operational sequence:

You will be asked to carry out the action in a specified order.

◆ Result of an action / reaction of the device / reaction of the program:

The unit or program reacts as a result of your actions or when a specific incident occurs.

Other symbols are explained as they occur.

## 2 Safety

### 2.1 *Intended use*

The Basic sandblaster is intended for commercial use in dental laboratories in accordance with the instructions for use. The unit is intended for use in the following tasks:

- Removal of residual investment material and of oxides on cast components
- Surface treatment
- Devestment of press ceramics
- Sandblasting of ceramic occlusal surfaces

### 2.2 *Meaning of these instructions*

These instructions for use act as a guide for safe usage of the unit.

If the instructions for use are not observed, accidents may lead to life-threatening electric shock, injury, or damage.

⇒ Only use the unit in accordance with these instructions for use.

⇒ Keep the instructions for use ready with the unit.

⇒ Provide the instructions for use to all subsequent users of the unit.

### 2.3 *Requirements that the operator must meet*

⇒ The product may only be used by persons aged 14 or over who are familiar with the procedures and rules in a dental laboratory.

---

## **2.4 Safe condition of the unit**

---

**Faulty or broken components, or those that are not sealed as required, may cause injuries.**

- ⇒ Check the power cord, housing, and other components such as connection cables, tubes, and the key pad for damage, for example such as kinks, cracks, porosity, and signs of aging.
- ⇒ Defective equipment must be taken out of service immediately. Disconnect the power plug. Send the unit in for repair or dispose of it safely. Until this happens, secure the unit to prevent it from being switched on again so that it is not used inadvertently.
- ⇒ Do not expose the unit to mechanical impact. Do not drop it.

---

## **2.5 Risks as a result of modifications and repairs**

---

**Incorrect modifications and repairs to the unit may cause accidents, fire, and electric shock.**

- ⇒ Only have the unit repaired by qualified specialist dealers or return it.

---

## **2.6 Risk as a result of electric shock and fire**

---

**Risk of electric shock and of fire as a result of excessive operating voltage or if there is no protective conductor.**

- ⇒ Ensure that the power plug is easily accessible.
- ⇒ Only operate the unit using a power cord equipped with a plug appropriate to the local power supply. Only allow qualified electricians to modify power cords.
- ⇒ Only start the unit if the information on the nameplate matches the power supply.
- ⇒ Only connect the unit to electrical sockets that are connected to the protective conductor system.

---

## **2.7 Risk of injury to the eyes or skin**

---

**Injury may occur to the eyes or skin as a result of high pressure or of components fracturing that are subject to high pressure.**

- ⇒ Always wear appropriate protective goggles when carrying out work on the unit.
- ⇒ Never direct the blasting material towards the eyes or unprotected areas of skin.
- ⇒ Never perform sandblasting with the viewing panel open.

---

## **2.8 Risk to health as a result of dust**

---

**Dust emissions may result in damage to health.**

- ⇒ Operate the unit with a suitable extraction unit. The extraction unit must be adapted to the dust that occurs.

## 3 Product description

### 3.1 General description

The sandblasting unit is available in a 1 tank and 2 tank version. The 1 tank version can be upgraded with a second tank (see accessories) to a 2 tank version.

### 3.2 Components and functional elements

- |    |  |    |  |
|----|--|----|--|
| 1  | Hand sleeves                                       | 14 | Air connection (push-in connection)  |
| 2  | Base grid  | 15 | Sandblasting tank  |
| 3  | Selector switch (only on the 2 tanks version)      | 16 | Mixing chamber   |
| 4  | ON / OFF switch (lighting)                         | 17 | Filter cartridge (push-in connection)  |
| 5  | Tank lid   | 18 | Suction tube   |
| 6  | Pressure regulator                                 | 19 | Foot switch connection (B)<br>(push-in connection)                                 |
| 7  | Pressure gauge                                     | 20 | Water trap valve   |
| 8  | Protection screen                                  | 21 | Water trap   |
| 9  | Hand piece   | 22 | Foot switch connection (B)<br>(push-in connection)<br>(only on the 1 tank version) |
| 10 | Foot switch  | 23 | Dosing nipple tool   |
| 11 | Compressed air tube                                | 24 | Connection set   |
| 12 | Power cord   |    |  |
| 13 | Foot switch connection (A)<br>(push-in connection) |    |  |

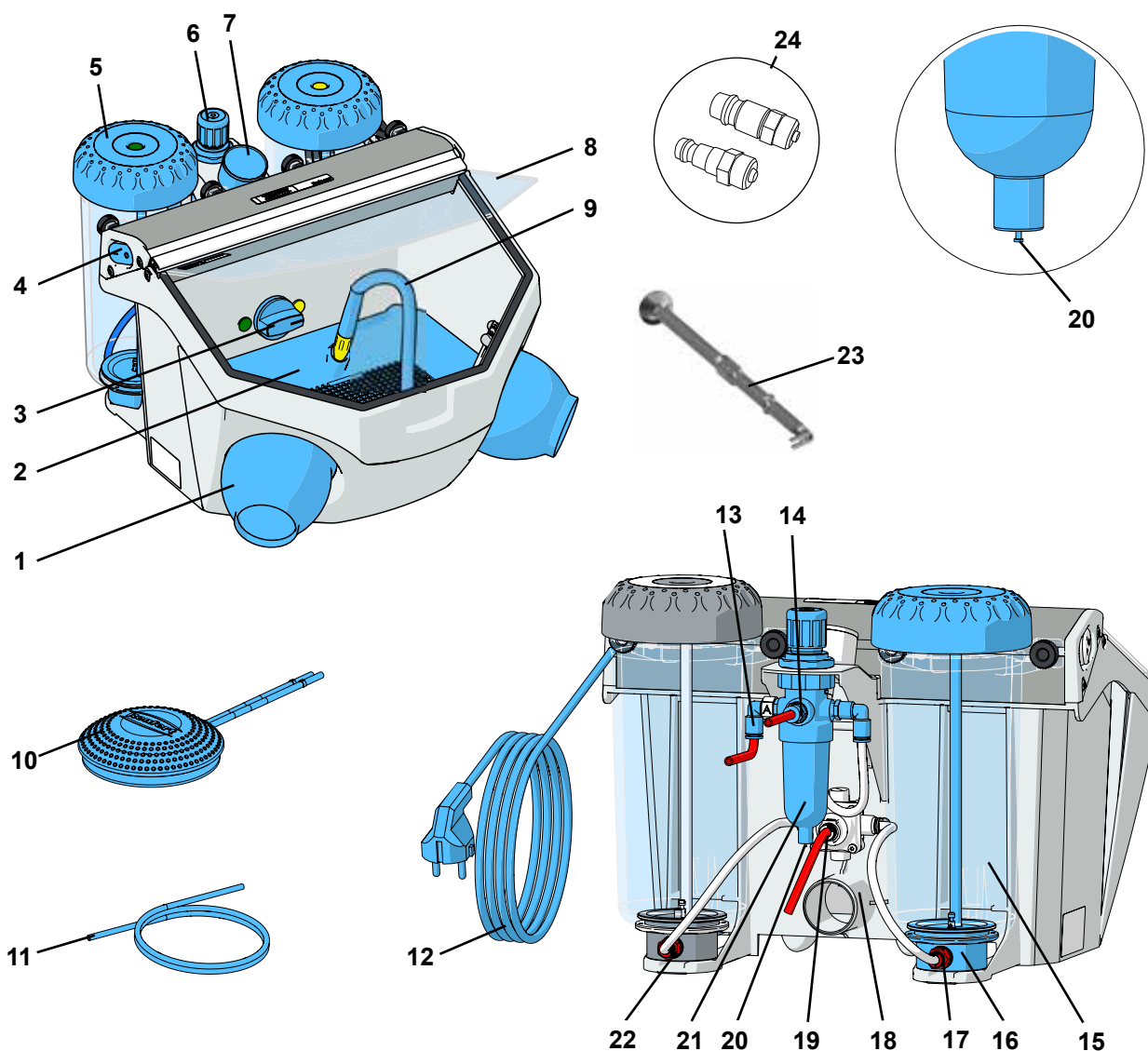
Fig. 1

### 3.3 Standard delivery

- 1 Sandblasting unit *Basic eco*
- 1 Foot switch
- 1 Connection set
- 1 Dosing nipple tool
- 1 Supplement dosing nipple tool
- 1 Quick Start Guide

### 3.4 Delivery format

- 1 tank version: 2949 1XXX, 2949 3XXX, 2949 5050
- 2 tank version: 2949 2XXX, 2949 4XXX, 2949 5025

### 3.5 Accessories

#### Additional tank

- 2947-0050 Additional tank 25 - 70 µm
- 2947-0250 Additional tank 70 - 250 µm

#### Abrasives

##### Cobra aluminium oxide

- 1594-1105 25 µm [500 mesh], white 5 kg canister
- 1594-1205 50 µm [270 mesh], white 5 kg canister
- 1594-2220 50 µm [270 mesh], white 20 kg bucket
- 1584-1005 90 µm [170 mesh], white 5 kg canister
- 1583-1005 110 µm [150 mesh], white 5 kg canister
- 1583-1020 110 µm [150 mesh], white 20 kg bucket
- 1587-1005 125 µm [115 mesh], pink 5 kg canister
- 1587-1020 125 µm [115 mesh], pink 20 kg bucket
- 1585-1005 250 µm [60 mesh], white 5 kg canister
- 1585-1020 250 µm [60 mesh], white 20 kg bucket

##### Rolloblast glass beads

- 1594-1305 50 µm [400-200 mesh] 5 kg canister
- 1594-2312 50 µm [400-200 mesh] 12.5 kg canister
- 1589-1005 100 µm [170-100 mesh] 5 kg canister

#### Nozzles

##### Round nozzles

- 90002-1203 0,4 mm [0.0158 inch]
- 90003-3213 0,6 mm [0.0236 inch]
- 90002-1204 0,8 mm [0.0315 inch]
- 90003-3211 1,0 mm [0.0394 inch]
- 90003-3214 1,2 mm [0.0473 inch]
- 90002-1205 1,4 mm [0.0552 inch]
- 90002-1206 2,0 mm [0.0788 inch]

##### Slotted nozzles

- 90003-1739 1,5 x 3,5 mm [0.0591 x 0.1379 inch]

## 4 Installation

**Risk of injury!**

Loose tubes may hit objects around them.

Tubes must be connected first to the unit.

Only press the foot switch after all tubes have been connected.

### 4.1 Unpacking

⇒ Take the unit and all the accessories out of the package.

⇒ Check to ensure all delivery parts are complete (compare “Included in Delivery”).

### 4.2 Setting up

⇒ Place the unit on a level and stable base.

### 4.3 Connect foot switch

The following tubes (30, 31) are already connected to the foot switch.

#### 4.3.1 2 Tank version:

⇒ Push the tube marked „A“ (30) as far as it will go into the tube connection point „A“ (13).

⇒ Push the tube marked „B“ (31) as far as it will go into the tube connection point „B“ (19).

When inserting the tubes you will notice a slight resistance which must be overcome.

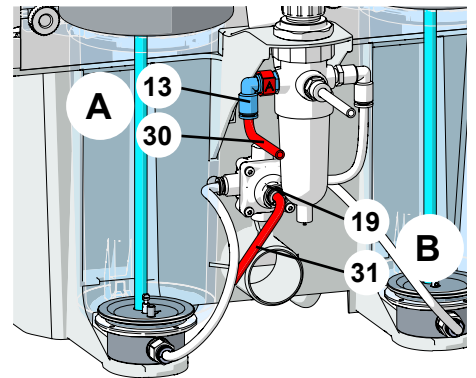
Fig. 2

#### 4.3.2 1 Tank version:

⇒ Push the tube marked „A“ (30) as far as it will go into the tube connection point „A“ (13).

⇒ Push the tube marked „B“ (31) as far as it will go into the tube connection point „B“ (22) (on sandblasting tank).

When inserting the tubes you will notice a slight resistance which must be overcome.

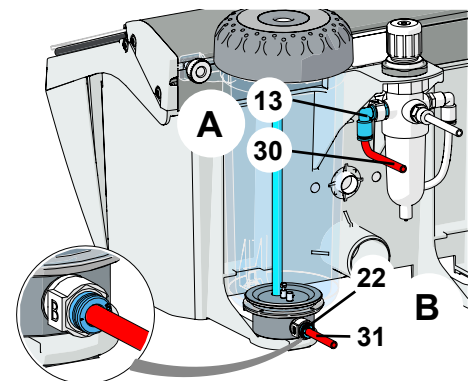
Fig. 3

### 4.4 Compressed air connection

⇒ Push the compressed air tube (11) onto the tube connection point (14) as far as it will go. In doing so there will be a slight resistance to overcome.

⇒ Select the relevant hose connection piece from the enclosed attachment set and fit it onto the end of the tube.

⇒ Attach to the compressed air network.

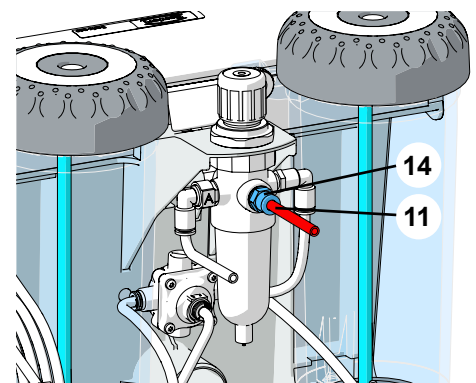
Fig. 4

## 4.5 Electrical connection

Note the instructions in Chap. 2.6 Risk as a result of electric shock and fire!

- ⇒ Unroll the power cable (12, Fig. 1).
- ⇒ Insert the power plug into the building installed wall socket.

## 4.6 Connect to extraction unit

- ⇒ Connect the suction tube from the extraction unit to the suction pipe (18).
- ▶ For suction devices with automatic activation:
  - ⇒ Connect the mains plug from the sandblasting unit into the user socket on the suction device.

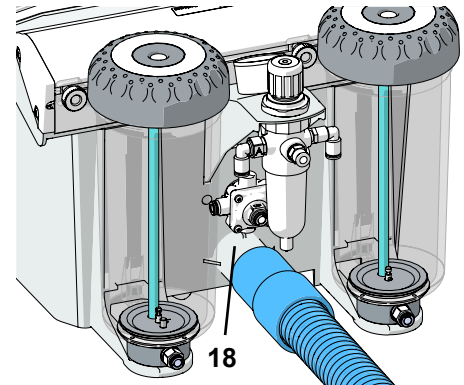
Fig. 5

## 4.7 Filling the sandblasting tanks

Do not write on or stick anything to the sandblasting tank and tank lid.

Do not press the foot switch while filling is in progress!

**Regardless of the pressure display, the sandblasting tanks are only pressurized when the foot switch is pressed.**

- ⇒ Only fill with abrasives up to the max. fill height (thickening of the tank cover underneath the thread).
- ⇒ Clean the thread on the sandblasting tank and tank lid as well as the seal.
- ⇒ Screw the tank lid closed firmly by hand.

**Abrasive residue on the seal causes leakage and premature wear of the seal.**

**Warning: Do not tilt the tank lid when screwing it on.**

**Check that the tank lid is seated securely prior to use.**

**Tank lids that are not closed securely can suddenly blow off.**

**There is a risk of injury as a result of flying parts and sudden emission of abrasive sand.**

- ⇒ Always use only clean, dry abrasive of the appropriate grain size (refer to the accessories).

**Your blasting machine is now ready for use.**

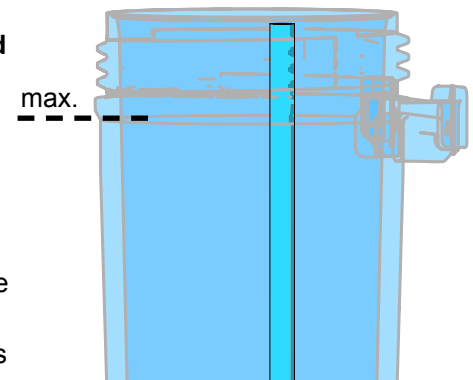
Fig. 6

### 4.7.1 Authorised sandblasting material

- Cobra (Renfert GmbH)
- Rolloblast (Renfert GmbH)
- ROCATEC (3M ESPE)

Sandblasting materials from other manufacturers may be used providing the grain size, grain shape and degree of purity comply with the above mentioned materials.

Renfert GmbH assumes no liability for the function and service life of the devices if other blasting agents are used.

## 5 Operation

- ⇒ Switch the illumination on (4).
- ⇒ Select the blasting tank at the selection switch (3) (only on the 2 tanks version).

**i** **Do not operate the foot switch during tank selection.**

- ⇒ Take the handpiece out of the holder of the relevant sandblasting tank (see color markings).
- ⇒ Activate the foot switch (10, Fig. 1).
  - ◆ The sandblasting material will flow as long as the foot switch is activated.

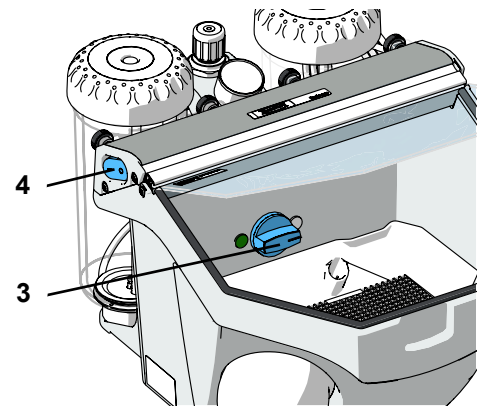
Fig. 7

### 5.1 Blasting notes

Flat blasting with wide nozzles:

90003-1739 1,5 x 3,5 mm [0.059 x 0.138 inch]

When blasting glass ceramics, refer to manufacturer's notes.

### 5.2 Select blasting pressure

- ⇒ Take the handpiece out of the holder of the relevant sandblasting tank (see color markings).
- ⇒ Activate the foot switch.
- ⇒ Set the sandblasting pressure at the pressure regulator (6).
  - ◆ The amount of pressure will be shown on the pressure gauge (7).

**!** **Do not exceed the maximum pressure (see „Technical Data“).**

**i** **In the 2 tank version the amount of pressure selected applies for both blasting tanks.**

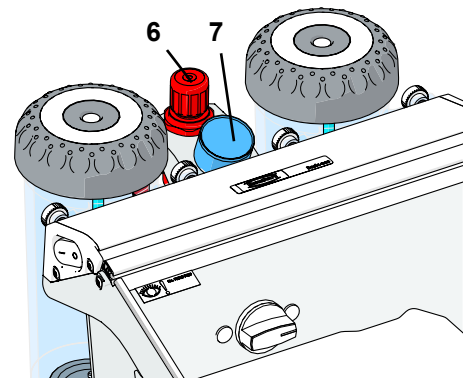
Fig. 8

## 6 Cleaning / Maintenance

**Before cleaning or servicing:**

- ▶ Switch off and unplug the device!
- ▶ Disconnect the appliance from the compressed air supply!

### 6.1 Sandblasting tank

**Check the sandblasting tank or tank lid regularly, at least 1 x a year, by visually inspecting for damage or changes to the material (e.g., cracks ...).**

**If in doubt, replace!**

**The sandblasting tank and tank lid are subject to a natural aging process depending on ambient conditions and must be replaced as standard after every 15 years of operation.**

**Note the year of manufacture on the round stamp engraved on the sandblasting tank and in the tank lid; Our customer service team are happy to advise you.**

Example of an engraved round stamp:

Date of manufacture 01-2003

Fig. 9

**To clean:**

**DO NOT use solvent-based cleaning agents or disinfectants.**

**Solvents and tensides can result in microcracking in the plastic (danger of explosion!).**

- ⇒ Only clean the sandblasting tank and tank lid by rubbing with a dry cloth.

## 6.2 Cleaning the housing

- ⇒ **DO NOT** use solvent-containing cleaning agents or disinfectants.
- ⇒ **DO NOT** use aggressive or abrasive cleaning agents.
- ⇒ Wipe the housing and aluminium frame gently with a slightly damp cloth, do not apply pressure.

## 6.3 Cleaning the Interior

- ▶ **DO NOT** use solvent-containing cleaning agents or disinfectants. (use e.g. soapy water).
- ⇒ Remove bottom grate and the blast protection mat.
- ⇒ Vacuum the blasting chamber.
- ⇒ Check the unit regularly for damage underneath the bottom grate / blast protection mat.
- ⇒ Exchange the damaged bottom grate / blast protection mat.

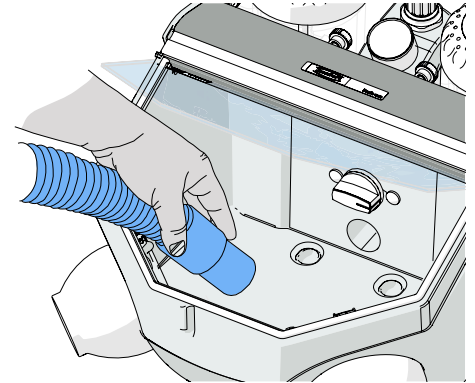
Fig. 10

## 6.4 Condensation

- ⇒ Inspect the water separator (21).
- ⇒ Drain any water off through the valve (20).

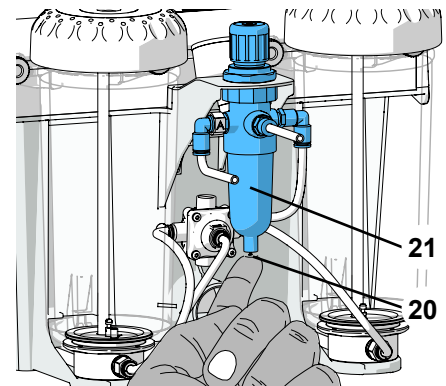
Fig. 11

## 6.5 Blasting nozzle replacement

For installation see accessory supplement.

## 6.6 Cleaning the dosing nozzle

The dosing nozzle from the mixing chamber can become blocked due to contamination and moist blasting sand.

The blasting nozzle is removed for cleaning with the enclosed dosing nipple tool (23, Fig. 1) and replaced again afterwards.

- ⇒ Vacuum clean the blasting tank completely.
- ⇒ Hold the dosing nozzle with the tool and pull upwards (Fig. 12).
- ⇒ Clean the dosing nozzle (e.g. blow with compressed air).
- ⇒ Replace dosing nozzle.

**Before remounting, the dosing nozzle should be completely dry!**

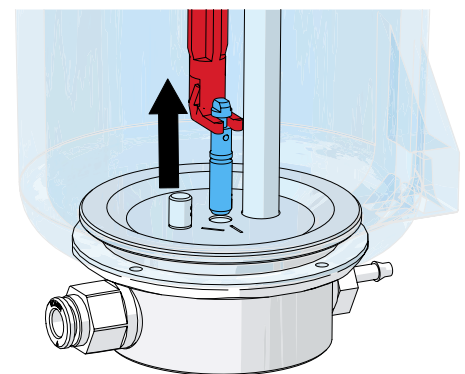
Fig. 12

- ⇒ The dosing nozzle must be adjusted with blasting tanks of 25 - 70 µm (Fig. 13)!

**See also supplement „Cleaning the dosing nozzle“**

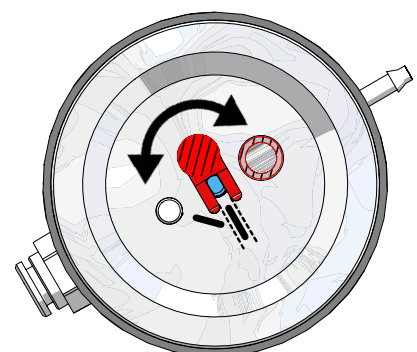
Fig. 13

## 6.7 Lamp cover

It is possible to remove the lamp cover (50) when cleaning (e.g. removing sandblasting residues and dust).

- ⇒ Hold the lamp cover at the back on both sides and pull forwards to remove.
- ⇒ Clean the lamp cover.
- ⇒ Insert the side of the lamp cover marked „TOP“ into the frame and press the back edge up until it locks into place.

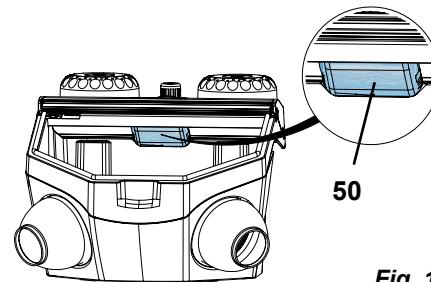
Fig. 14

**!** Do not operate without the lamp cover in place, otherwise the protective shield, which cannot be exchanged, will become matt!

In the course of time, the lamp cover can become matt. This cover can be ordered as a spare part.

## 6.8 Spare parts

Only use accessories and spare parts that have been supplied or approved by Renfert GmbH. Using other accessories or other spare parts can lead to inadvertent accidents and damage.

You can find components subject to wear and the spare parts on the spare part list in the internet at [www.renfert.com/p918](http://www.renfert.com/p918).

The components excluded from the warranty (such as consumables or parts subject to wear and tear) are marked on the spare part list. Serial number and date of manufacturing are shown on the type plate of the unit.

## 6.9 Shipping Notes

- Empty all blasting tanks.
- Order Renfert transport packaging (Tel. +49 7731 8208-383).

The return form can be downloaded from the „Advice“ section on the Renfert website at [www.renfert.com](http://www.renfert.com).

# 7 Trouble shooting

Problem	Possible cause	Corrective action
<b>No air and no abrasive.</b>	<ul style="list-style-type: none"> <li>• Blasting nozzle blocked at the hand-piece.</li> <li>• Foot switch connecting lines kinked.</li> <li>• Tank selection switch in an intermediate position.</li> </ul>	<ul style="list-style-type: none"> <li>• Unscrew the nozzle from the hand-piece and blow it out.</li> <li>• With the nozzle removed, blow out the blasting hose by activating the foot pedal.</li> <li>• Inspect/correct the connection lines.</li> <li>• Turn the switch until you feel it click into place.</li> </ul>
<b>Only air, no or very little abrasive.</b>	<ul style="list-style-type: none"> <li>• The dosage nozzle in the mixing chamber is blocked due to contaminated or moist sand.</li> <li>• Mixing chamber filter combination blocked.</li> <li>• Abrasive contains impurities or is moist.</li> </ul>	<ul style="list-style-type: none"> <li>• Cleaning the dosing nozzle, see chapter 6.6.</li> <li>• Change filter cartridge (17, Fig. 1) (wrench SW14).</li> <li>• Replace the abrasive.</li> </ul>
<b>Pressure control cannot be adjusted.</b>	<ul style="list-style-type: none"> <li>• The pressure control button is locked.</li> </ul>	<ul style="list-style-type: none"> <li>• Pull the pressure control button upwards to unlock.</li> </ul>
<b>Unit blasts continuously.</b>	<ul style="list-style-type: none"> <li>• Foot switch connected incorrectly.</li> <li>• Valve in foot switch is defective.</li> </ul>	<ul style="list-style-type: none"> <li>• Inspect / correct the foot switch connection.</li> <li>• Replace foot switch.</li> </ul>
<b>Lamp is not alight.</b>	<ul style="list-style-type: none"> <li>• Power cord not plugged in.</li> <li>• Illumination defect.</li> </ul>	<ul style="list-style-type: none"> <li>• Plug the power cord into the wall outlet.</li> <li>• Change light using the service or see spare part list.</li> </ul>
<b>Compressed air leaking from the foot switch.</b>	<ul style="list-style-type: none"> <li>• Pneumatic hoses reversed.</li> </ul>	<ul style="list-style-type: none"> <li>• Reverse the connections.</li> </ul>
<b>Foot switch fails to switch on.</b>	<ul style="list-style-type: none"> <li>• No pressure at the compressed air connection.</li> <li>• Pneumatic hoses are kinked.</li> </ul>	<ul style="list-style-type: none"> <li>• Inspect the compressed air lines and/or the compressor.</li> <li>• Reposition the hoses.</li> </ul>
<b>Foot switch hoses cannot be disconnected from the unit.</b>	<ul style="list-style-type: none"> <li>• Push-in connection is not released (Fig. 2 / 3).</li> </ul>	<ul style="list-style-type: none"> <li>• Press the push-in connection ring on the hose side firmly while pulling on the hoses.</li> </ul>

## 8 Technical data

Nominal Voltage	230 V	120 V	100 V
Permissible mains voltage:	220 - 240 V	120 V	100 V
Mains frequency:	50 / 60 Hz		
Power consumption:	9 W		
Maximum connection pressure:	6 - 8 bar [87- 116 psi]		
Working pressure:	1 - 6 bar [14.5 - 87 psi]		
Dimensions (Width x Height x Depth):	350 x 275 x 400 mm [13.8 x 10.8 x 15.7 inch]		
Weight, approx.: 1 tank version: 2 tank version:	4.4 kg [9.7 lbs] 5.5 kg [12.1 lbs]		
Ø Compressed air hose: interior: exterior:	4 mm [157 mil] 6 mm [236 mil]		
Ø Size of pipe union for external extraction unit: interior: exterior:	35 mm [1.38 inch] 40 mm [1.57 inch]		
Blasting chamber volume:	10 l [2.64 US gal]		
Tank capacity, each:	1000 ml [34 fl oz]		
This product contains a light source of energy efficiency class:	G		
<b>Ambient conditions for safe operation</b>			
Ambient conditions:	Indoors		
Installation height, max.:	2,000 m [6,500 ft] above sea level		
Ambient temperature:	5 - 40 °C [41 - 104 °F]		
Relative humidity:	Max. 80 % at 31 °C [87.8 °F], decreasing linearly to 50 % at 40 °C [104 °F] *)		
Voltage fluctuations in the mains power supply from the nominal value, max.:	10 %		
Contamination level:	2		
Over-voltage category:	II		
<b>Ambient conditions for storage and transport</b>			
Ambient temperature:	- 20 – + 60 °C [-4 – + 140 °F]		
Maximum relative humidity:	80 %.		

\*) Between 5 - 30 °C [41 - 86 °F] the device can be operated at a relative humidity of up to 80 %.  
At temperatures between 31 - 40 °C [87.8 - 104 °F] the humidity must decrease proportionally in order to ensure operational readiness (e.g. at 35 °C [95 °F] = 65 % humidity, at 40 °C [104 °F] = 50 % humidity). The device may not be operated at temperatures above 40 °C [104 °F].

## 9 Warranty

Provided the unit is properly used, Renfert warrants all components for **3 years**.

Warranty claims may only be made upon presentation of the original sales receipt from the authorized dealer.

Parts which are subject to natural wear and tear (wear parts) and consumables are excluded from the guarantee. These parts are marked in the spare part list.

The warranty is voided in the case of improper use; failure to observe the operating, cleaning, maintenance and connection instructions; in case of independent repairs or repairs by unauthorized personnel; if spare parts from other manufacturers are employed, or in case of unusual influences or influences not in compliance with the utilization instructions.

Warranty service shall not extend the original warranty.

### 9.1 Liability exclusion

Renfert GmbH shall be absolved from all claims for damages or warranty if:

- ▶ The product is employed for any purposes other than those cited in the operating instructions.
- ▶ The product is altered in any way other than those alterations described in the operating instructions.
- ▶ The product is repaired by other than an authorized facility or if any but Renfert OEM parts are employed.
- ▶ The product continues to be employed, despite obvious safety faults.
- ▶ In the case of mechanical impact or if the unit is dropped.
- ▶ Accessories or consumer products used, which are not supplied or authorised by Renfert GmbH.

## 10 Obligations regarding information

- ▶ Information on REACH and SVHC is available on our website [www.renfert.com](http://www.renfert.com), in the Support area.

### 10.1 Information on disposal for countries within the EC

**The unit must be disposed of by an authorized recycling operation. The selected firm must be informed of all possibly health-hazardous residues in the unit.**

To conserve and protect the environment, prevent environmental pollution and improve the recycling of raw materials, the European Commission adopted a directive that requires the manufacturer to accept the return of electrical and electronic units for proper disposal or recycling.

**Within the European Union units with this symbol should not therefore be disposed of in unsorted domestic waste.**

For more information regarding proper disposal please apply at your local authorities.

**Hochaktuell und ausführlich auf ...  
Up to date and in detail at ...  
Actualisé et détaillé sous ...  
Aggiornato e dettagliato su ...  
La máxima actualidad y detalle en ...  
Актуально и подробно на ...**

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declare under our sole responsibility, that the product(s)

Name	Part No.
Vario basic	2960 0005
Basic quattro IS	2959 0000 / 2959 0000 03
Basic quattro	2958 0000 / 2958 0000 03
Basic master	2948 2000 / 2948 2000 03 2948 2025 / 2948 2025 03 2948 2250 / 2948 2250 03
Basic classic	2947 1050 / 2947 1050 03 2947 1250 / 2947 1250 03 2947 2000 2947 2025 / 2947 2025 03 2947 2250
Basic eco	2949 1050 / 2949 1050 03 2949 1250 / 2949 1250 03 2949 2025 / 2949 2025 03
Basic mobil	2914 3050 2914 3250
Dustex master plus	2626 0105 / 2626 0105 03

is (are) in conformity with the relevant regulatory requirements by compliance with the UK designated standards.

#### UK legislation

*Electromagnetic Compatibility Regulations 2016*

*Electrical Equipment (Safety) Regulations 2016*

*The Restriction of the Use of Certain Hazardous Substances in Electrical and Electronic Equipment Regulations 2012*

#### Designated standards

EN 61326-1:2013

EN 61010-1:2010/A1:2019/AC:2019-04

EN IEC 63000:2018

Tilo Burgbacher,  
Engineering Director

Hilzingen, 02.11.2022

The Technical Documentation has been retained by Renfert GmbH

**Hochaktuell und ausführlich auf ...  
Up to date and in detail at ...  
Actualisé et détaillé sous ...  
Aggiornato e dettagliato su ...  
La máxima actualidad y detalle en ...  
Актуально и подробно на ...**

**[www.renfert.com](http://www.renfert.com)**

Renfert GmbH • Untere Gießwiesen 2 • 78247 Hilzingen/Germany  
Tel.: +49 7731 82 08-0 • Fax: +49 7731 82 08-70  
[www.renfert.com](http://www.renfert.com) • [info@renfert.com](mailto:info@renfert.com)

Renfert USA • 3718 Illinois Avenue • St. Charles IL 60174/USA  
Tel.: +1 6307 62 18 03 • Fax: +1 6307 62 97 87  
[www.renfert.com](http://www.renfert.com) • [info@renfertusa.com](mailto:info@renfertusa.com)  
USA: Free call 800 336 7422

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**Renfert GmbH, Industriegebiet, 78247 Hilzingen / Germany**