making work easy

DE

EN

FR

IT

ES

PT

RU

UK

ZH

JA

KO








Katalysator Nr. 2300 0001 / 2300 3001

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Made in Germany

21-6499 19092019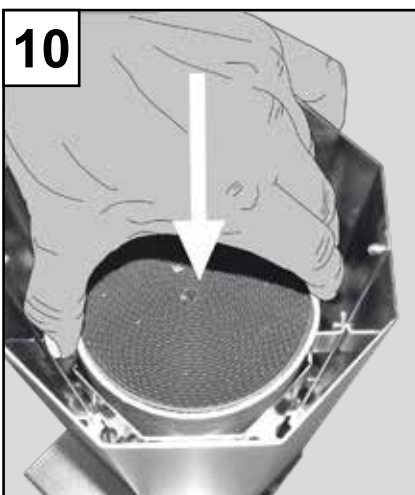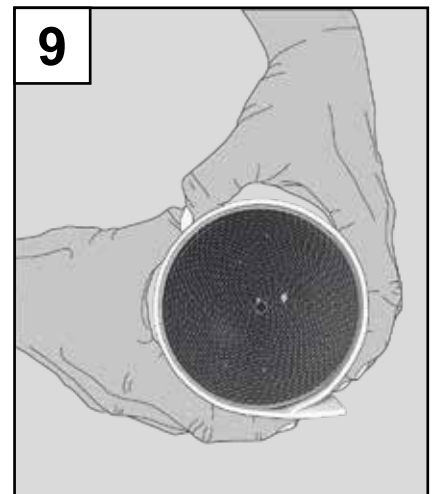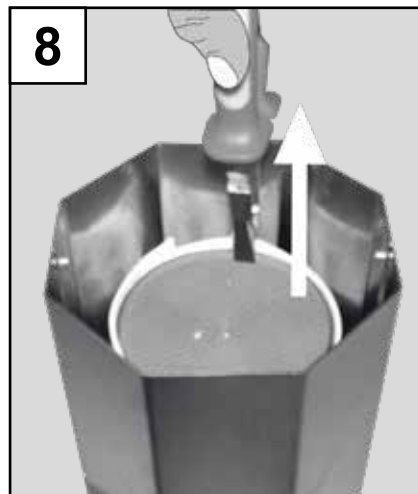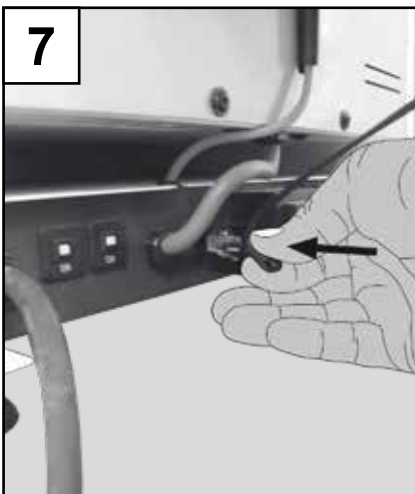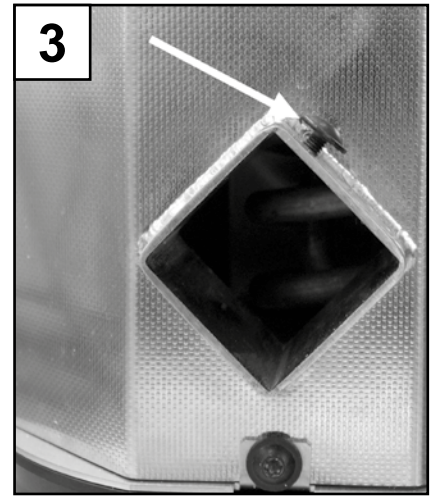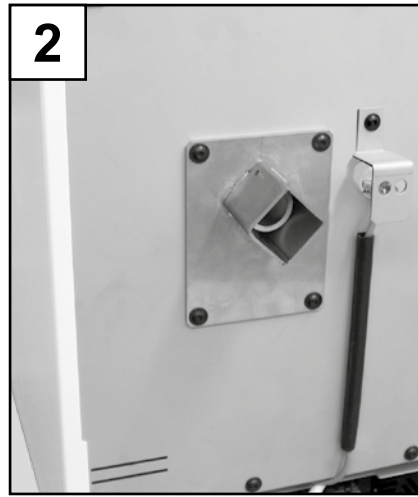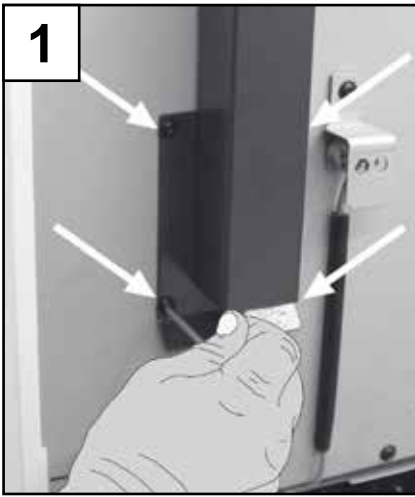

Katalysator

2300-0001 / 2300-3001

DEUTSCH

1. Einleitung

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ENGLISH

1. Introduction

We are pleased with your decision to purchase the *Catalyser*.

Please read the following operating instructions carefully and observe the safety information they contain in order to ensure a long and trouble-free service life.

2. Area of Application

The *catalyst converter* is intended to reduce odours which are the result of heating investments and burning waxes in the *Magma* preheating oven. As a collateral action, this chemical process leads to a reduction of toxic substances.

Depending on the particular materials being processed, hazardous substances may still be generated even when the *catalyst converter* is used.

In such cases, additional measures to remove the flue gases must be implemented. Local ordinances and requirements must be complied with.

2.1 Ambient Conditions (in accord. with DIN EN 61010-1)

The unit may only be used:

- Indoors;
- Up to an altitude of 2,000 m above sea level;
- At an ambient temperature range between 5 - 40 °C [41 - 104 °F *];
- At a maximum relative humidity of 80 % at 31 °C [87.8 °F], dropping in a linear manner to 50 % relative humidity at 40 °C [104 °F *];
- With mains power where the voltage fluctuations do not exceed 10 % of the nominal value;
- Under contamination level 2 conditions;
- Under over-voltage category II conditions.

*) Between 5 – 30 °C [41 – 86 °F], the unit can be operated at a relative humidity of up to 80 %. At temperatures between 31 – 40 °C [87.8 – 104 °F], the humidity must decrease proportionally in order to ensure operational readiness (e.g., at 35 °C [95 °F] = 65 % humidity; at 40 °C [104 °F] = 50 % humidity). The unit may not be operated at temperatures above 40 °C [104 °F].

3. Hazard Information

3.1 Symbology

The following symbols are employed in these instructions and on the unit itself:

Danger

This indicates a direct risk of injury.

Electrical current

This indicates a hazard due to electrical current.

Attention

Failure to observe the associated information can result in damage to the unit.

Note

This provides the operator with useful information to make working with the unit easier.

Only intended for indoor use.

Before opening the unit, disconnect it from the mains power supply by unplugging the power cord from the wall outlet.

Note the operating instructions.

3.2 Hazards

- The *Catalyser* is an electric device and, as such, carries with it a certain inherent potential hazard.
- The unit may only be operated with a power cord equipped with a plug appropriate to the local power supply. Thus plug may only be installed by a qualified electrician.
- The unit may only be taken into service if the information on the nameplate corresponds with the specifications of the local power supply.
- The unit may only be connected to a properly grounded power supply.
- Damaged power cords must be immediately replaced.
- The unit must be disconnected from the power supply prior to beginning any maintenance work on its electrical components.
- Only intended for indoor use. The unit is only designed for dry applications and may not be operated or stored outdoors or under wet conditions.
- Regularly inspect connecting lines and hoses (e.g., the power cord) for damage (e.g., kinks, cracks, porosity) or signs of aging.
- Units exhibiting damaged connecting lines, hoses, or other defects must be taken out of service immediately.
- The unit will become hot during operation. Allow it to cool off before attempting to perform any cleaning or maintenance work.
- Only spare parts and accessories supplied or authorized by Renfert GmbH may be used with this product. If other spare parts or accessories are used, this could have a detrimental effect on the safety of the device, increase the risk of serious injury and lead to damage to the environment or the device itself.
- It is the responsibility of the operator that national regulations during operation and regarding a repeated safety inspection of electrical equipment are complied with. For Germany these are the regulation 3 by DGUV (German Statutory Accident Insurance) in relation with VDE 0701-0702 (Association for Electrical, Electronic and Information Technology) .
- Information on REACH and SVHC is available on our website www.renfer.com, in the Support area.

3.3 Liability Exclusion

Renfert GmbH shall be absolved from all claims for damages or warranty if:

- The product is employed for any purposes other than those cited in the operating instructions;
- The product is altered in any way other than those alterations described in the operating instructions;

- The product is repaired by other than an authorised facility or if any but Renfert OEM parts are employed;
- The product continues to be employed, despite obvious safety faults or damage;
- The product is subjected to mechanical impacts or is dropped.

4. Commissioning

4.1 Mounting the Catalyser

- Loosen the fastening screws on the preheating oven and remove the flue (Figure 1).
- Install the *Catalyser's* mounting plate (Figure 2).
- Before mounting the *Catalyser*, make sure its fastening screw is retracted (Figure 3).
- Push the *Catalyser* completely onto the mounting plate up to the stop (Figure 4).
- Tighten the *Catalyser's* fastening screw (Figure 5).

4.2 Connecting the Catalyser

Before connecting the unit to the wall outlet, make sure the voltage information on the nameplate corresponds to your local power supply!

- Plug the power cord into the wall outlet.
- Use the included cord to connect the *Catalyser* to the oven (Figure 6 / Figure 7).

Make sure none of the cords come into contact with the oven surface.

5. Operation

The Magma preheating oven's controller will automatically detect the presence of a connected Renfert Catalyser.

The *Catalyser* is controlled solely by the preheating oven's internal controller.

There are no operating or control elements on the *Catalyser* itself.

6. Maintenance

Attention!

Unplug the *Catalyser* from the oven prior to beginning any cleaning or maintenance work.

Burn hazard!

The unit will become hot during operation. Allow it to cool off before attempting to perform any cleaning or maintenance work.

6.1 Replacing the Catalyser Cartridge

Replace the Catalyser cartridge when the cleaning effect of the *Catalyser* diminishes.

To do this, proceed as follows:

- Position the oven so that you can easily access the *Catalyser* from above.
- Using a pair of pliers, pull the Catalyser cartridge out from above (Figure 8).
- Remove the insulation.
- Wrap new insulation around the new Catalyser cartridge (Figure 9).
- Insert the insulated Catalyser cartridge into the *Catalyser* (Figure 10).

! Make sure the Catalyser cartridge does not slide out of the insulation during installation.

6.2 Cartridge Disposal

The catalyser cartridge contains residual materials which may represent a health hazard.

All local health ordinances and accident prevention regulations must be observed when disposing of the cartridge.

7. Spare Parts

You can find components subject to wear and the spare parts on the spare part list in the internet at www.renfert.com/p918.

The components excluded from the warranty (such as consumables or parts subject to wear and tear) are marked on the spare part list.

Serial number and date of manufacturing are shown on the type plate of the unit.

8. Warranty

Provided the unit is properly used, Renfert warrants the all components of the *Catalyser* for a period of **3 years**. Warranty claims may only be made upon presentation of the original sales receipt from the authorized dealer.

Parts which are subject to natural wear and tear (wear parts) and consumables are excluded from the guarantee. These parts are marked in the spare part list.

The warranty is voided in case of improper use; failure to observe the operating, cleaning, maintenance, and connection instructions; in case of independent repairs or repairs by unauthorized personnel; if spare parts from other manufacturers are employed, or; in case of unusual influences or influences not in compliance with the utilization instructions.

Warranty service shall not extend the original warranty.

9. Disposing of the Unit

The unit must be disposed of by an authorized recycling operation which must be informed of any residual hazardous material it may contain.

9.1 Disposal Instructions for EU Member Nations

To conserve and protect the environment, prevent environmental pollution and improve the recycling of raw materials, the European Commission adopted a directive that requires the manufacturer to accept the return of electrical and electronic units for proper disposal or recycling.

Within the European Union units with this symbol should not therefore be disposed of in unsorted domestic waste:

For more information regarding proper disposal please apply at your local authorities.

10. Technical Specifications

Mains voltage:	230 V, 50 / 60 Hz
Power consumption:	580 W
Mains fuse:	2 x T5A
Cord length:	1.8 m [70 inches]
Dimensions (W x H x D):	190 x 370 x 170 mm [7.5 x 14.6 x 6.7 inches]
Weight, approx.:	4.8 kg [10.6 lbs]

11. Standard Delivery

- 1 *Catalyser*
- 1 Control cord
- 1 Operating instructions
- 1 Torx wrench

12. Delivery Version

2300-0001	<i>Catalyser</i> , 230V
2300-3001	<i>Catalyser</i> , 230 V, with NEMA6-15P plug

13. CE Conformity

The *catalyst converter* is an accessory to the *Magma* preheating oven – Article No. 2300-xxxx – and may only be operated in conjunction with the oven.

The *catalyst converter* therefore does not require its own, separate CE Conformity Declaration.

Conformity with the associated guidelines has been determined within the context of the conformity examination of the preheating oven.

14. Troubleshooting

EN

Fault	Possible cause	Corrective action
Catalyser fan fails to run.	<ul style="list-style-type: none"> • Mains power cord not plugged in. • Control cord not connected. • Fuse blown. • The oven controller has not switched the <i>Catalyser</i> on. 	<ul style="list-style-type: none"> • Plug the power cord in. • Make sure the control cords is properly seated at both the oven and the <i>Catalyser</i>. • Have the fuse replaced by a qualified electrician. • Refer to the <i>Magma</i> preheating oven's operating instructions.
Catalyser heater fails to heat.	<ul style="list-style-type: none"> • Mains power cord not plugged in. • Control cord not connected. • Fuse blown. • The oven controller has not switched the <i>Catalyser</i> on. 	<ul style="list-style-type: none"> • Plug the power cord in. • Make sure the control cords is properly seated at both the oven and the <i>Catalyser</i>. • Have the fuse replaced by a qualified electrician. • Refer to the <i>Magma</i> preheating oven's operating instructions.
Excessive smoke.	<ul style="list-style-type: none"> • Catalyser cartridge exhausted. 	<ul style="list-style-type: none"> • Replace the Catalyser cartridge (refer to Sec. 6.1).

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Renfert GmbH • Untere Gießwiesen 2 • 78247 Hilzingen/Germany