Renfert-Scanspray extra fine

REF 17312000

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Titandioxid-frei

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ENGLISH | INSTRUCTIONS FOR USE

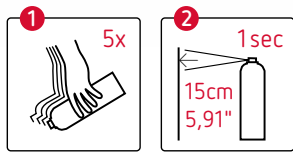
Titanium dioxide-free

Congratulations! On choosing this Renfert-Scanspray extra fine, you have purchased a high quality, innovative product.

Description:

Renfert-Scanspray extra fine is designed for extraoral use, in a laboratory environment, for use on prepared models and impressions prior to scanner exposure. The Renfert-Scanspray extra fine is suitable for all conventional optical scanner systems and produces exact optical image quality.

Thanks to its micro-fine, homogeneous structure, Renfert-Scanspray extra fine can be applied very thinly, enhancing adhesion, especially on polished ceramics/metals and reflecting surfaces.

Application:

Place the precision nozzle onto the spray head prior to initial use.

Shake the can thoroughly for before use.

Hold the can as upright as possible and spray a thin layer onto the surface to be scanned from a distance of approximately 15 cm. Only a very thin, hardly visible layer is necessary. It is advisable to apply the product in small short sprays. This way, only small amounts of powder are distributed, at the same time the nozzle system cleans itself during this procedure. The surface to be sprayed must be clean and dry.

When spraying impression material, it is essential to ensure that the material is completely set first.

Removal:

After scanning, the sprayed surface can be cleaned easily with water and steam cleaning.

Cleaning the nozzle:

The special spray technique means the nozzle will not become blocked, even after frequent use.

If necessary, the nozzle can be cleaned with water or compressed air.

Hazard warnings and safety advice:

Extremely flammable aerosol. Pressurised container: May burst if heated. Causes severe eye irritation. May cause drowsiness and dizziness. Keep away from heat, hot

surfaces, sparks, open flames and other ignition sources. No smoking. Do not spray on an open flame or other ignition source. Do not pierce or burn, even after use. Avoid breathing spray. Wear protective gloves and eye/face protection. If eye irritation persists: Get medical advice/attention.

Protect against sunlight and do not expose to temperatures above 50 °C/122 °F.

Please always observe the information on the safety data sheet.

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Renfert GmbH