help:ex discolor f

Coffee, Tea, Tar Remover | REF 6700 0200

DEUTSCH

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ENGLISH

Danger. Contains: 2-aminoethanol. May be corrosive to metals. Causes severe skin burns and eye damage. Wear protective gloves and eye/face protection. IF ON SKIN (or hair): Take off immediately all contaminated clothing. Rinse skin with water or shower. IF IN EYES: Rinse cautiously with water for several minutes. Remove contact lenses, if present and easy to do. Continue rinsing. Immediately call a POISON CENTER. Absorb spillage to prevent material damage.

Product description:

Ready-to-use special cleaner (alkaline) for the subsequent removal of stubborn discolorations such as tar, coffee, tea stains from removable dental restorations. The cleaning procedure can be carried out as an after-treatment service without the use of the SYMPRO high performance device.

Indication:

Special cleaner for dissolving / removing stubborn discolorations such as tar, coffee, tea stains from removable dental restorations.

Only to be used by trained dental personnel.

Contraindication:

Application in the mouth or on the body is not permitted. The use of the cleaning agent on restorations belonging to patients with a known allergy to individual constituents is contraindicated. The cleaning agent is not intended for multiple use.

**Special hazard warnings:**

Please read the cleaning agent safety data sheet before use.

Download under www.renfert.com

May be corrosive to metals. Causes severe skin burns and eye damage

Wear protective gloves and eye/face protection.

IF ON SKIN (or hair): Take off immediately all contaminated clothing. Rinse skin with water or shower. IF IN EYES: Rinse cautiously with water for several minutes. Remove contact lenses, if present and easy to do, continue rinsing. Immediately call a POISON CENTER/doctor.

The liquids must not come into contact with the following surfaces: PVC, linoleum, natural and artificial stone, marble, stainless steel, wood. In case of contamination and spillage, clean immediately with a copious amount of water in order to avoid material damage.

In case of complaints or events in connection with the medical device, please inform Renfert immediately.

Renfert will investigate the matter and report serious incidents to the authorities in the member states concerned.

**Dosage and application:**

Before use, it is essential to read the comprehensive Instructions for Use of the SYMPRO high-performance cleaning device. Download under www.renfert.com

help:ex discolor f is intended for single-use only, in order to prevent cross contamination.

help:ex discolor f is for application after the cleaning procedure in the SYMPRO high performance cleaning device. The procedure is for application outside the device

- ▶ Place the object into a separate, gray beaker (system component SYMPRO device) and cover completely with help:ex discolor f.
- ▶ Immersion time approx. 5–10 min (outside the device).
- ▶ **Note:** Remove the cleaned objects using only the tweezers.
- ▶ Remove the dissolved residues by brushing with a toothbrush or denture brush and warm water.
- ▶ Carefully rinse the cleaned object with water (at least 5 seconds), examine the restoration visually for damage and disinfect according to the national hygiene guidelines before returning to the patient.
- ▶ **Note:** The cleaning liquid can become contaminated with bacteria from the inserted object. For this reason: Always use fresh cleaning liquid for every new cleaning sequence!
- ▶ Empty the beaker:
 - ▶ Carefully pour the cleaning liquid away
 - ▶ Rinse the beaker with clean water
- ▶ Hygienic preparation of the system components
 - ▶ Please follow the tips in the “Guidelines for hygienic preparation” in the Instructions for Use in the SYMPRO device.

Safety instructions:

The owner and user is responsible for proper disinfection of the cleaning beakers and lids as well as for compliance with national hygiene regulations. In order to avoid unintended effects, do not mix the disinfectant and cleaning liquids together.

Tips on disposal:

Please observe the national regulations for waste disposal when disposing of the used cleaner. This also applies for the introduced contaminants.

For details see SDS Download under www.renfert.com

Storage:

Store in a dry environment and protect against direct sunlight. Observe storage temperature according to product label.

Shelf life:

For the shelf life, please see the product label. Do not use once this date has expired.

Once a bottle has been opened, use within 6 months.

Constituents:

The liquid cleaner contains 2-amino ethanol, potassium hydroxide, glycol ether, < 5% phosphonates, < 5% nonionic tensides and dye.

Disclaimer:

This product has been developed and is intended exclusively for professional use by trained dental professionals. It must be handled in accordance with the information provided in this instruction document. The manufacturer assumes no liability for damages that arise from improper handling or processing.

help:ex discolor f

Coffee, Tea, Tar Remover | REF 6700 0200

ENGLISH

 Unique Device Identifier

 Professional use only

 Observe the Instructions for Use*

 EC conformity

 Medical device

 Manufacturer specifications

 Not intended for reuse

 Batch designation

 Attention: observe accompanying document*

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**Renfert GmbH**