making work easy

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ru

zh

ja

ko

MT3

Quick Start Guide

Made in Germany

21-6809 21112022

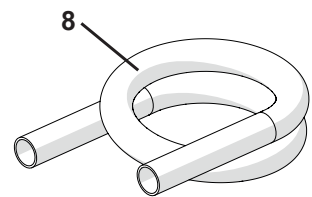
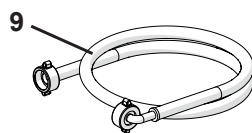
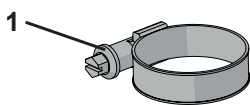
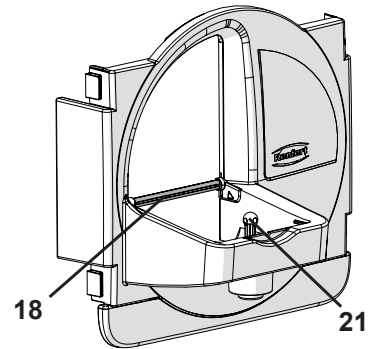
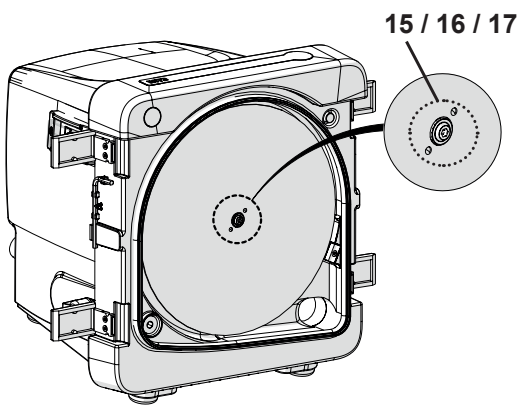
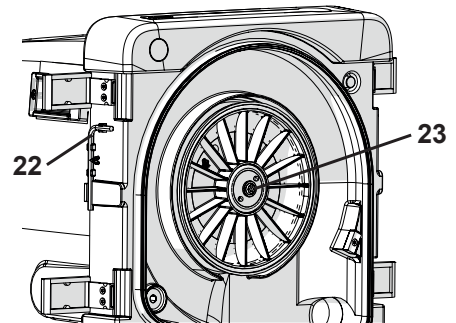
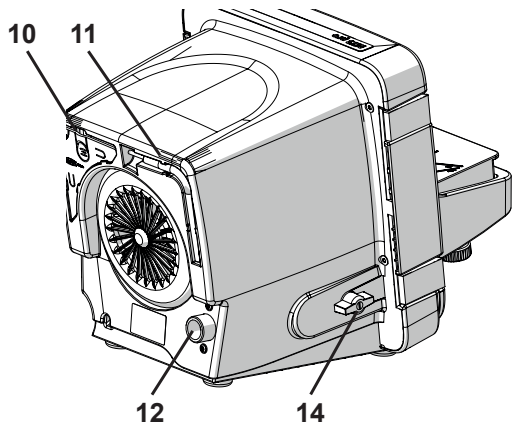
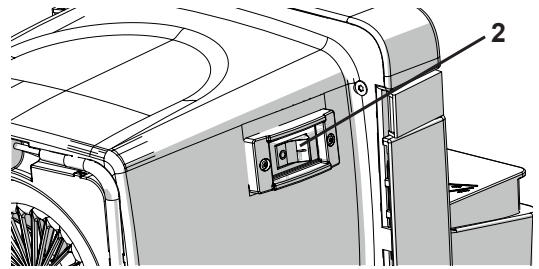
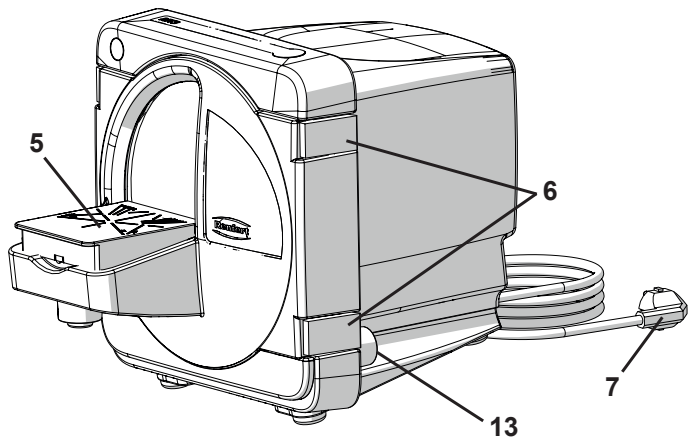
Fig. 1

Einleitung

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Introduction

This Quick Start Guide contains instructions on the operation of the product and relevant safety information.

Detailed information on the product can be found in the operating instructions which are available for download on our website at www.renfert.com.

Symbols

This manual contains symbols also found on the appliance itself with the following meaning:

Danger!
Immediate risk of injury.

High Voltage!
Danger – high voltage.

Caution:
Risk of damage to the unit if the instruction is not followed.

Notice
A general notice that improves and eases use.

Caution:
Wear protective glasses.

IPX2 The appliance is protected against dripping water from 15° angle.

The appliance complies with applicable EU directives.

This product complies with the relevant UK legislation.
See UKCA Declaration of Conformity in the Internet under www.renfert.com.

Within the EU, this device is subject to the provisions of the directive 2002/96/EG (WEEE directive).

Other symbols are explained when displayed.

Safety

Correct use of the unit

This plaster trimmer MT3 is designed solely for wet grinding plaster models.

Unapproved use

Using the appliance for dry grinding is not allowed.

This appliance may not be used for grinding investment material. Cancer causing dusts are released.

Only spare parts and accessories supplied or authorized by Renfert GmbH may be used with this product. If other spare parts or accessories are used, this could have a detrimental effect on the safety of the device, increase the risk of serious injury and lead to damage to the environment or the device itself.

Hazard and Warning Information

General Information

- ▶ If the device is not used in compliance with the supplied instructions, the safety of the device can no longer be guaranteed.
- ▶ **Risk of injury!**
Risk of injury if the components used are not approved.
Always use original Renfert components.
- ▶ The device may only be operated using a mains cable with the country-specific plug system. Any necessary alterations must be carried out by a qualified electrician.
- ▶ The device may only be operated if the information on the identification plate conforms to the specifications of your local mains power supply.
- ▶ The device may only be plugged into outlets which are connected to the protective conductor system.
- ▶ The mains plug must be easily accessible.
- ▶ Disconnect the device from the mains before carrying out work on the electrical parts.
- ▶ It is the responsibility of the operator that national regulations during operation and regarding a repeated safety inspection of electrical equipment are complied with. For Germany these are the regulation 3 by DGUV (German Statutory Accident Insurance) in relation with VDE 0701-0702 (Association for Electrical, Electronic and Information Technology).
- ▶ Check connection cables (such as power supply cords), tubes and housing (i.e. the key-pad) regularly for damage (i.e. kinks, cracks and porosity) or signs of ageing.
Devices with damaged connection cables, tubes or housing parts or other defects must not be operated!
- ▶ Defective devices must be put out of service immediately. Remove the mains plug and ensure the device is not used. Send the device for repair!
- ▶ Only operate the device under supervision.
- ▶ Please observe the national accident prevention regulations!

Specific Instructions

- ▶ Never reach into the grinding wheel while the unit is connected to the power supply.
Inadvertently starting the unit can result in injury!
- ▶ Always keep long hair, loose or wide sleeves or other clothing away from the trimmer. Hair or clothing can be pulled into and wrapped up on the grinding wheel, resulting in injury!
- ▶ The unit is designed solely for grinding plaster models. Cancer causing dusts are released when trimming investment material.
- ▶ Never open the front cover while the motor is running. The motor safety shutoff will not stop the grinding wheel immediately.
- ▶ Only open the front cover after the grinding wheel has come to a complete stop.
- ▶ Always wear protective eyewear when grinding.
- ▶ Do not reach into the grinding wheel while it is turning.
- ▶ Do not work without the model table installed and locked („click“). Small parts or fingers can be drawn into the trimmer, resulting in serious injury.
- ▶ Remove the trimmer table only after the grinding wheel has come to a complete stop and the front cover is opened.
- ▶ The extraction must be selected to match the dust being generated (if necessary, consult the responsible professional organization).
- ▶ Operating the unit can result in a very high noise level >>> always wear ear protection.

Authorized Persons

Operation and maintenance of the device may only be performed by qualified personnel.

Product Description

Component Groups and Functioning Elements

See Fig. 1 (on inside cover)

- | | | | |
|----|------------------|----|-------------------------------|
| 1 | Hose clamp | 13 | Darin water connecting branch |
| 2 | ON / OFF-switch | 14 | Water inlet |
| 5 | Grinding table | 15 | Cover |
| 6 | Door latch (4x) | 16 | Fastening screw |
| 7 | Mains plug | 17 | Lock pins |
| 8 | Drain hose | 18 | Rinse tube |
| 9 | Water hose | 21 | Bracket |
| 11 | Pull-off tool | 22 | Allen key, SW4 |
| 12 | Water connection | 23 | Flange |

Standard Delivery

- 1 Trimmer
- 1 Water hose
- 1 Drain hose
- 1 Allen key, SW4
- 1 Pull-off tool
- 1 Marathon Trimmer Disc
- 1 Quick Start Guide

Installation

Unpacking

⇒ Check to ensure all delivery parts are complete (compare “Standard Delivery”).

Setting Up

- ⇒ Place appliance on solid even working surface.
- ⇒ Place appliance close to a supply water connection and drain.
- ⇒ Place appliance with easy access to the mains plug socket.

Connection

- ⇒ Twist drain hose (8) onto connection fitting (13).
- ⇒ Fix with a hose clamp (1).
- ⇒ Place drain water hose with downward slope.
- ⇒ Connect water hose (9) to water connection (12).
- ⇒ Connect the water hose to a water supply with a shutoff tap.
- ⇒ Open the shutoff tap and check for leaks.
- ⇒ Unroll the power cable and insert the plug (7) into the socket.

Operation

Trimming

⇒ If necessary check and adjust the grinding angle.

⇒ Turn on appliance.

After switching on, wait until the trimmer disc has reached the rated speed (approx 2 - 3 seconds). Start trimming only then.

⇒ Adjust water flow rate at water inlet (14).

⇒ Using both hands, securely set the model on the grinding table and press it against the disc.

Once the work has been finished:

⇒ Switch off the appliance.

The water supply shutoff tap should be closed when work is interrupted for longer periods and when the appliance is unattended (e.g. over night).

Cleaning and Maintenance

The device does not include parts which require maintenance. It is not permitted to open the device!

⇒ Switch off and unplug the appliance before cleaning or servicing.

⇒ Close water supply shutoff tap.

Never clean the device under running water or immerse in water.

Do not steam clean the housing!

⇒ Only wipe with a damp cloth.

⇒ Open door latches (6), remove door and set aside.

⇒ Clean door with a brush or cleaning cloth under running water.

⇒ Wipe the internal space with a brush or cleaning cloth.

⇒ Pull grinding table (5) at leading edge upward from ball head.

⇒ Clean grinding table with a brush under running water.

⇒ Turn rinse tube (18) backwards at curved side and take out.

⇒ Allow the rinse tube to soak in decalcifying agent overnight.

Spare Parts

You can find components subject to wear and the spare parts on the spare part list in the internet at www.renfert.com

The components excluded from the warranty (such as consumables or parts subject to wear and tear) are marked on the spare part list.

Disposal Information

Within the European Union, units with this symbol should not therefore be disposed of in unsorted domestic waste.

Please contact your local authorities for more information on proper disposal.

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declare under our sole responsibility, that the product(s)

Name	Part No.
MT plus	1803 0000 1803 0500
MT3	1808 0000 / 1808 0000 03 1808 0500 / 1808 0500 03
MT premium	1807 0000 1807 0500

is (are) in conformity with the relevant regulatory requirements by compliance with the UK designated standards.

UK legislation

Electromagnetic Compatibility Regulations 2016

Supply of Machinery (Safety) Regulations 2008

The Restriction of the Use of Certain Hazardous Substances in Electrical and Electronic Equipment Regulations 2012

Designated standards

EN 61326-1:2013

EN ISO12100:2010

EN 61010-1:2010/A1:2019/AC:2019-04

EN 60825-1: 2014

EN 61000-3-2: 2014

EN 61000-3-3: 2013

EN IEC 63000:2018

Tilo Burgbacher,
Engineering Director

Hilzingen, 02.11.2022

The Technical Documentation has been retained by Renfert GmbH

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Renfert GmbH • Untere Gießwiesen 2 • 78247 Hilzingen/Germany