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### 1. Fill level in the stainless steel tank

- Fill the ultrasound tank up to the ledge = Recommended fill level including the cleaning agent.
- Never fill to lower than the recommended fill level.
- Do not fill hot liquids into the cold ultrasound tank!

### 2. Stainless steel tank

- Do not place any objects or containers directly onto the base of the tank!
- For use with:
  - Cleaning jar, item no.1850-0006
  - Plastic beaker, item no.1850-0007
 together with:
  - Stainless steel insert lid, item no.1850-0002
- Always allow a min. distance of 15mm between the cleaning jar and the base of the tank!
- Please only use the accessories and equipment stated in the instructions for use (chapter 11).

### 3. Cleaning liquid and cleaning container

- Please only use the permitted cleaning agent in the ultrasound tank.

#### Not to be used as a cleaning agent:

- Flammable liquids (e.g. solvents)

#### Do not put the following directly into the stainless steel tank:

- Cleaning agents containing hydrochloric acid, sulphuric acid, hydrofluoric acid, nitric acid
- Diluted acidic cleaning agent (pH value of < 7)
- Sodium chloride (NaCl) solutions
- Household detergents

The cleaning jar and the plastic beaker are to be used together with the stainless steel insertion lid (see fig. 2) or

- the acid insert tank, item no.1850-0005

### 4. Contaminations

In case of contaminations from

- fluoride deposits (e.g. cement, polishing paste)
- chloride deposits (e.g. disinfection liquid)
- bromide deposits (e.g. solder paste, flux)

do **not** clean directly in the ultrasound tank.

Please use the cleaning jar or acid insertion tank (see fig. 3).

### 5. Emptying

Empty the unit as soon as the liquid is soiled.

*Advice:*

*Empty by tipping the unit over the appliance side at the back right hand side.*

To empty / fill:

- Remove the mains plug
- Ensure that the unit is always dry on the outside.
- Ensure that no abrasive residues (plaster, investment material, polishing residues, metals, etc.) remain in the ultrasound tank, danger of cavitation corrosion!

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\*) Before use, please take time to read through the instructions. These include further and more detailed advice regarding the correct operation and use of this unit.

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