hotty LED

No. 1457-0000 / No. 1457-1000

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ENGLISH

1. Introduction

Thank you for purchasing our *hotty LED* wax dipping unit.

Please observe the following notices, they will help provide you a long and problem free lifetime of your wax dipping unit.

2. Application area

The *hotty LED* is an electronically regulated wax dipping pot. It is ideally suited for melting and retaining any conventional dipping wax.

The *hotty LED* shows its quality in use when it comes to the following points:

The precise temperature control (+/- 2°C [3.6°F]) provides you with a very constant quality of copings. Together with the heating unit, which reacts to the environmental temperature, this excludes the possibility of over heating and consequent damage to the dipping wax.

Please be aware that the initial heating process is very gentle to the dipping wax and it may take some time before the wax is fully melted. Thanks to a slow continual increase of the working temperature, there is no chance that the temperature inside the pot will exceed the set point.

The *hotty LED* offers you optimal working conditions. The housing is designed to let both left and right-handed people rest their hand on the working table whilst dipping. In addition, the lid of the wax-dipping unit can be moved completely out of the working area.

2.1 Environmental conditions

(In accordance with IEC 60664-1)

The *hotty LED* must only be operated:

- in closed rooms
- below an altitude of 2,000 m [6,500 ft] above mean sea level
- under environment temperature between 5 – 40°C [41-104°F]
- at a maximum relative humidity of 80% at 31°C [87,8°F], linear declination to 50% at 40°C [104°F]
- when the mains supply do not fluctuate more than 10% of the Norm.
- at pollution level 2
- with over voltage category II.

3. Safety Information

- **Before initial use, check the voltage recommendations on the name plate (rear side of the unit) and compare with the mains supply.**
- **Units with damaged power cables or other defects are not to be used.**
- **Mains supply cable (G) must be unrolled when in use (danger of over heating in the event of short circuit).**
- **Do not cover the transformer when in use.**
- **The power cables are not to be shortened in any event.**
- **Incorrect use of the socket fittings can cause short-circuiting.**

- Molten wax can cause severe burns to the skin. Do not move the unit once in operation.
- The wax melting pot (E) heats up during use right to the rim. Incorrect use can lead to burns, even 30 min. after the unit has been switched off.
- The *hotty LED* is only intended for use with conventional dental dipping wax. Under no circumstances should it be filled with other materials.
- **Please remember:** overfilled pots can easily overflow!
- Materials with a flash point below 175°C [347°F] are not to be stored in the vicinity of or in contact with the *hotty LED*.
- Do not rinse or dip into water! Mains supply can cause electric shock!

3.1 Liability Exclusion

Renfert GmbH shall be absolved from all claims for damages or warranty if:

- The product is employed for any purposes other than those cited in the operating instructions;
- The product is altered in any way other than those alterations described in the operating instructions;
- The product is repaired by other than an authorised facility or if any but Renfert OEM parts are employed;
- The product continues to be employed, despite obvious safety faults;

- The product is subjected to mechanical jarring or is dropped.

4. Operating Elements (fig. 1)

- A) Lid
- B) On/Off Switch
- C) LED Display
- D) Temperature Setting +/-
- E) Heating pot
- F) Transformer
- G) Power Cable
- H) Connection Cable *hotty LED* / Power Supply

5. Commissioning / Operation

1. Connect *hotty LED* to the transformer using cable supplied (fig. 2).
2. Connect power supply to the mains supply (fig. 3).

Notice: the installation should only be carried out in this order! Make sure the OG-Plug is always fitted securely!

3. Flip the lid (A) open (fig. 4).
4. Fill the pot with dipping wax (E) (fig. 5).
5. Turn on the *hotty LED* by pressing the button (B) (fig.6). The LED-Display now shows the chosen temperature setting (°C or °F), changes then to show the nominal temperature, then changes finally to show the actual temperature.
6. The LED-Display shows a small indicator in the bottom right hand corner (fig. 7). As soon as the nominal temperature has been reached, this indicator will go out.

5.1 Setting the Temperature scale to °C or °F

Please observe that all 230 V units are pre-set to centigrade (°C) and that all 120 V units are pre-set to Fahrenheit (°F).

1. Steps 1-4 (point 5) must be fulfilled before doing this.
2. Press and hold button (B).
3. The LED-Display now shows **C** = degrees Celsius (fig. 8a) or **F** = degrees Fahrenheit (fig. 8b).
4. After 5 seconds, the display changes between **C** and **F** and vice versa. Release the button when the required temperature unit is shown on the display. Following this, you will see the nominal temperature displayed shortly, then the actual temperature.

5.2 Setting the nominal temperature

To set the temperature, press button +/- (D) repeatedly until the required value is displayed (fig. 9). This value is saved automatically – even after the unit is switched off.

Notice: Only use the button (B) when turning the unit on and off. If you turn the mains supply off, the unit will not save any values previously entered. This however, will not affect all other functions.

5.3 Dipping (fig. 10)

- support the hand well.
- dip your object quickly into the wax, past the preparation margin.
- pull the object back out in a slow movement.
- hold shortly before the tip of the die leaves the wax pot, allowing excess wax to drip off.

6. Cleaning and Maintenance

The *hotty LED* is maintenance free. Wipe off the surface from time to time using a damp soft cloth. Never use detergents or solvents!

Never rinse the *hotty LED* with running water!

Cold wax remnants can be removed by carefully scraping them off.

Lightly warming the surface with a hairdryer makes it easier to remove any unwanted wax.

7. Spare Parts

All available spare parts are included in the attached parts list.

8. Guarantee

Under proper use, Renfert warrants the *hotty LED* for a period of 3 years. Conditions for the fulfilment of this warranty include the presentation of original buyers receipt from a recognised dealer. Components subject to natural wear and tear are not included in this warranty.

Warranty becomes void in a case of improper use, failure to observe operating, cleaning, maintenance and connection instructions. Also in the case of independent repairs or repairs carried out by unauthorised personnel, or if unauthorised spare

parts are used and when influences unusual or in non-compliance with the instructions for use apply. Services carried out under warranty do not extend the warranty period.

9. Technical Specifications

Maximum power: 25 W (+/- 10%)

Minimum operating temperature:
60°C [140°F]

Power Supply:
230 V / 50-60 Hz
120 V / 50-60 Hz

Safety Appliances: Fuse (not changeable) rating:
230 V = T 160 mA / 250 V
120 V = T 250 mA / 250 V

Dimensions (w x d x h) :
90 x 98 x 65 mm
[3,55 x 3,86 x 2,56 inch]

Weight (with/without power supply):
approx. 1,7 kg / 200 g

10. Scope of Delivery

1 *hotty LED* Wax dipping unit
1 Power Supply
1 Operating Instructions
1 Spare Parts List

11. Accessories

GEO-Dip Dipping Wax (elastic):

No. 482-3000 *GEO-Dip* yellow, 200 g
No. 482-3200 *GEO-Dip* orange, 200 g
No. 482-3300 *GEO-Dip* green, 200 g

Rewax Dipping Wax (hard, non-elastic):

No. 651-0000 *Rewax* yellow, 210 g
No. 651-0100 *Rewax* green, 210 g

12. Troubleshooting

| Error | Possible Cause | Remedy |
|--|---|--|
| Set values cannot be saved. | <ul style="list-style-type: none"> • The unit was switched on/off by the mains supply. • Faulty memory module. | <ul style="list-style-type: none"> • Turn off unit using the on/off button, not at the mains. • Repair by service. |
| No display / functions although the unit has been turned on using button (B). | <ul style="list-style-type: none"> • Mains supply disconnected. • Thermo-fuse defect. • Main unit fuse defect. • Faulty power supply. • Mains supply too low. • Fuse in the power supply has been activated. The supply has a combined short circuit / thermo-fuse. | <ul style="list-style-type: none"> • Check power supply and cables. • Replacement by service. • Replacement by service. • See spare parts list. • Use correct mains supply. • Disconnect the mains plug and allow the supply to cool down for 5 minutes. Reconnect according to instructions in 5. |
| Unit is working but does not heat up. | <ul style="list-style-type: none"> • Faulty thermal-sensor. • Faulty heating element. | <ul style="list-style-type: none"> • Replacement by service. • Repair by service. |

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