
making work easy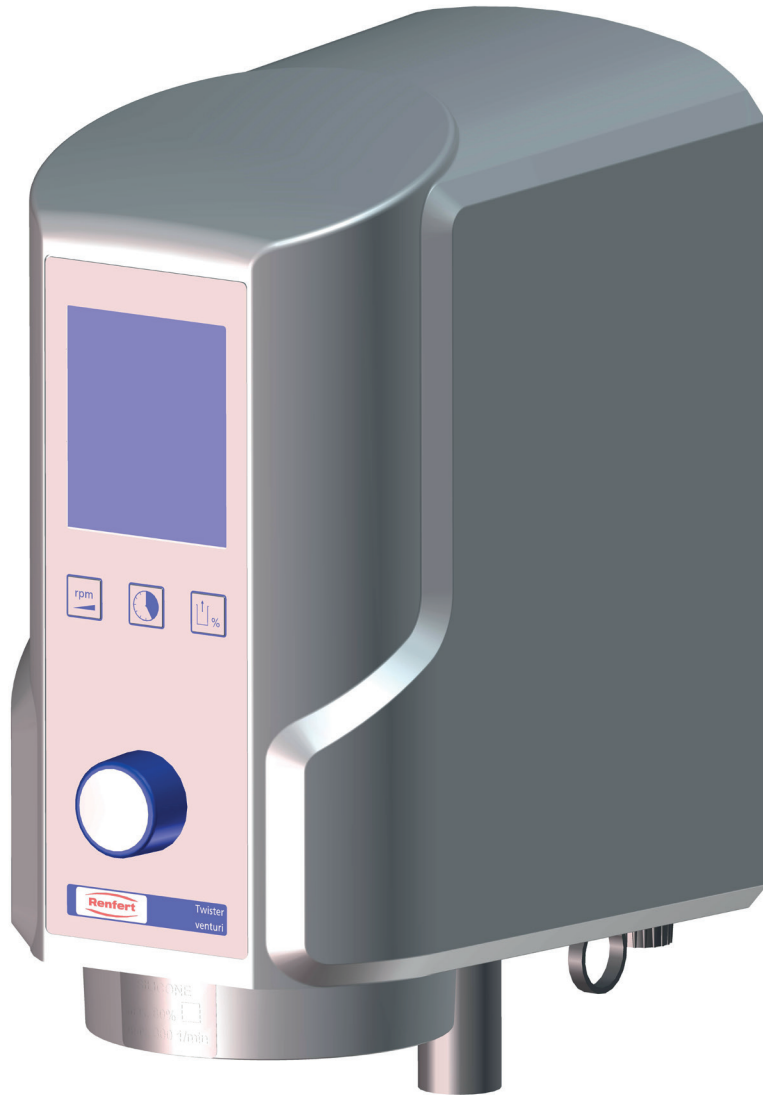

- DE
- EN
- FR
- IT
- ES
- PT
- TR
- RU
- ZH
- JA
- KO
- AR

Twister / Twister venturi

Quick Start Guide

Made in Germany

21-6851 21112022

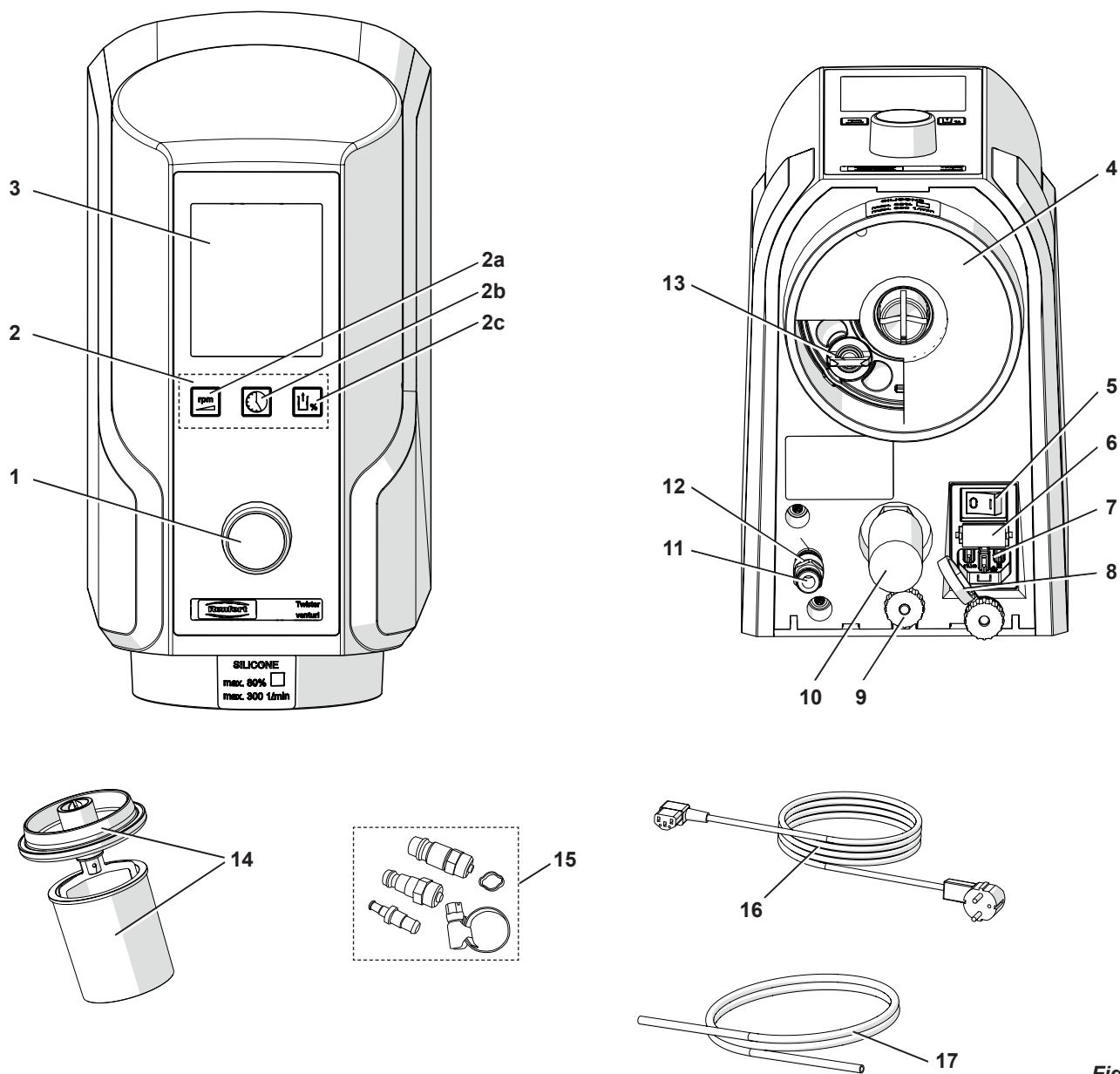
Fig. 1

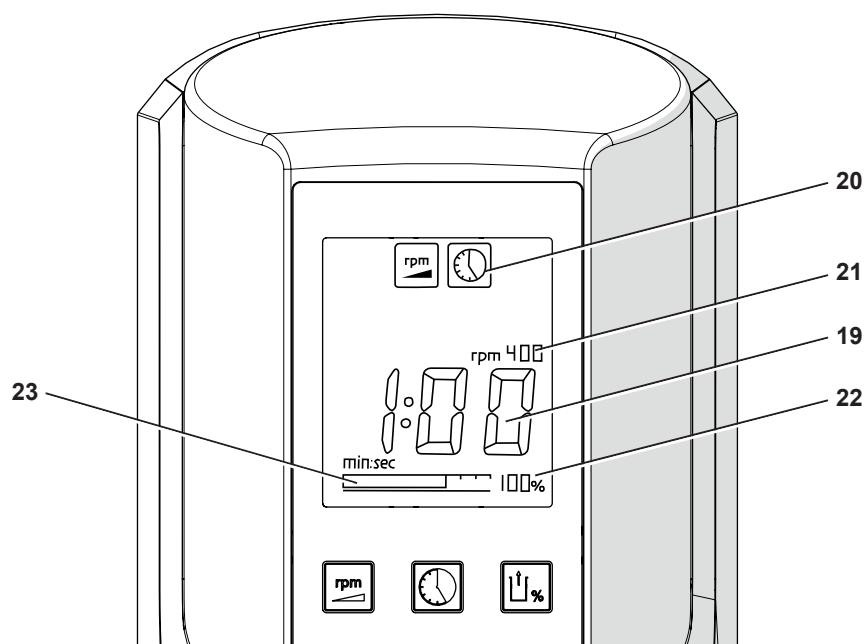
Fig. 2

Einleitung

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Introduction

This Quick Start Guide contains instructions on the operation of the product and relevant safety information. Detailed information on the product can be found in the operating instructions which are available for download on our website at www.renfert.com/p915.

Enter the following item number: 18260000 or 18270000.

Employed Symbols

This manual contains symbols also found on the appliance itself with the following meaning:

Danger!

Immediate risk of injury. Please observe accompanying documentation!

High Voltage!

Danger – high voltage.

Caution

Risk of damage to the appliance if the instruction is not followed.

Notice

A general notice that improves and eases use.

The appliance complies with applicable EU directives.

This product complies with the relevant UK legislation.

See UKCA Declaration of Conformity in the Internet under www.renfert.com.

Within the EU, this device is subject to the provisions of the directive 2002/96/EG (WEEE directive).

Other symbols are explained when displayed.

Safety

Intended use

The Twister / Twister venturi vacuum mixer is designed solely for homogeneous, bubble-free mixing of dental casting and modelling materials such as plasters, investments, and silicones.

Improper use

Only spare parts and accessories supplied or authorized by Renfert GmbH may be used with this product. If other spare parts or accessories are used, this could have a detrimental effect on the safety of the device, increase the risk of serious injury and lead to damage to the environment or the device itself.

Hazards and warnings

General Information

- ▶ **Granted protection is not given any longer in case the appliance is not operated according to the instruction manual on hand.**
- ▶ **The appliance should only be operated using a mains cable with a country-specific plug system. Any modification required should only be carried out by an electrician.**
- ▶ **The appliance should only be operated if the data on the rating plate corresponds with the data of regional mains voltage.**
- ▶ **The appliance should only be plugged into earthed sockets.**
- ▶ **The mains socket should be easily accessible.**
- ▶ **Regularly check connecting cables and hoses (e.g. mains cable) for damage (e.g. kinks, tears, porosity) or deterioration. Do not operate appliances with damaged mains wires, hoses or other defects.**
- ▶ **The appliance should always be operated under supervision.**
- ▶ **Risk of injury!**
Risk of injury if the components used are not approved. Always use original Renfert components.
- ▶ **Switch off and unplug the appliance before carrying out repairs and servicing work on electrical components.**
- ▶ **Switch off and unplug the appliance before cleaning or servicing it.**

- ▶ It is the responsibility of the operator that national regulations during operation and regarding a repeated safety inspection of electrical equipment are complied with. For Germany these are the regulation 3 by DGUV (German Statutory Accident Insurance) in relation with VDE 0701-0702 (Association for Electrical & Electronic Technology).
- ▶ Information on REACH and SVHC is available on our website www.renfert.com, in the Support area.

Specific Information

- ▶ Only to be used indoors. The appliance is only intended for dry operation and should not be used or stored outdoors or in wet conditions.
- ▶ Always read the safety data sheets supplied by the manufacturers when mixing investment materials! Wear the appropriate personal protective equipment as and when required!
- ▶ Never mix flammable or explosive materials.
- ▶ Never connect the paddle without the mixing bowl.
- ▶ Manipulating the automatic bowl coupler and the intake opening can result in damage to the device and personal injury.
- ▶ Switch off the appliance when work is complete.

Authorised persons

The appliance may only be operated and serviced by trained personnel.

Product description

Components and functional elements

See Fig. 1 (on inside cover)

- | | |
|---|---|
| 1 Control knob (set mixing parameters, start, stop, aerate) | 8 Cable safety catch |
| 2 Parameter keys | 9 Knurled nut |
| 2a Speed | 10 Silencer (Twister venturi only) |
| 2b Mixing time | 11 Compressed air connection (Twister venturi only) |
| 2c Vacuum | 12 Inlet filter (Twister venturi only) |
| 3 Display | 13 Sinter filter |
| 4 Sponge filter | 14 Bowl incl. blunger |
| 5 Appliance switch | 15 Pneumatic connection set |
| 6 Fuse | 16 Mains cable |
| 7 Appliance socket | 17 Compressed air hose |

See Fig. 2 (on inside cover)

- | | |
|---|-------------------------------------|
| 19 Large-scale display of parameter (here: mixing time) | 21 Selected speed |
| 20 Parameter symbols; The symbol „vacuum“ only shows when less than 100 % vacuum has been set | 22 Selected vacuum level |
| | 23 Current vacuum level (bar graph) |

Commissioning

Setup

- Operate the device at room temperature 15 - 30 °C [59 - 86 °F].
- Do not place the device under a source of heat.
- Do not expose the device to direct sunlight.
- Do not expose the device to high humidity.

Wall Mounting

Use the drilling template supplied for wall mounting.

Make sure the wall on which you intend to mount the device is capable of bearing its weight!

Please ensure that there are no electric cables or water pipes running through the wall at the place of installation!

Tighten the knurled nut (9) securely.

Benchtop Models

The stand can be used to convert the vacuum mixer to a floor-mounted appliance. The stand is not part of the standard delivery and must be ordered as an optional accessory (see www.renfert.com).

Connection

⇒ Secure the mains cable (16) through the cable safety catch (8).

⇒ Plug mains cable into socket (7) on appliance.

⇒ Plug in at the mains.

Twister venturi only

⇒ Plug the compressed air tube (17) into the tube attachment on the compressed air connection (11) as far as it will go. There will be a slight resistance to overcome when attaching.

⇒ Select the proper coupling from the set (15) provided and install it on the other hose end to connect it to the compressed air system.

⇒ Connect to compressed air line system.

Operation

On

The appliance is switched on and off at the switch (5). The mixing parameters last used will reappear in the display when the appliance is switched on. The factory settings will appear the first time the appliance is switched on.

Standby mode

The appliance will go into standby mode if no button is pressed or it stands idle for more than three minutes. The display will be dimmed when this happens.

Pressing a parameter key, turning the control knob or coupling the mixing bowl ends the standby mode.

Setting Mixing Parameters

⇒ Press parameter key (2a, 2b, 2c).

◆ The symbol of the pressed parameter key (20) is displayed.

◆ The parameter value is displayed in large-scale characters.

⇒ Turn the control knob (1) to change the value.

◆ The modified values are saved when the mixing bowl is coupled to the device.

Mixing Process

Always read the safety data sheets supplied by the manufacturers when mixing investment materials! Wear the appropriate personal protective equipment as and when required!

Observe maximum mark on mixing bowl. Do not fill the mixing bowl over the maximum mark. This limit is valid for powders and liquids measured before mixing. If the mixing bowl is over-filled the suction filter system can become soiled.

⇒ Set mixing parameters as instructed by the manufacturer.

⇒ Mix powder and liquid / duplicating silicone according to the manufacturer's instructions.

⇒ Using a spatula, lightly mix the materials together.

⇒ Install the appropriate paddle.

- ⇒ Couple the mixing bowl (14) to the appliance.
 - ◆ The motor starts briefly for coupling the mixing paddle.
 - ◆ Vacuum generation starts automatically.
- ⇒ Do not let go until the bowl stays in place alone and vacuum level bar (23) has passed the halfway mark!
- ⇒ Start the mixing process: Press the control knob (1).
 - ◆ The appliance beeps when the mixing cycle has finished.
 - ◆ The time which has elapsed since the end of the mixing cycle is shown in the display.
- ⇒ Hold the bowl!
- ⇒ Aerate bowl: Press the control knob (1).
 - ◆ The bowl will be released in a few seconds.
- ⇒ Remove the mixing bowl.

Abort mixing process

- ⇒ Stop the mixing process: Press the control knob once.
 - ◆ The appliance will beep.
 - ◆ Mixing process being aborted.
- ⇒ Hold the bowl!
- ⇒ Aerate bowl: Press the control knob.
 - ◆ The bowl will be released in a few seconds.

i *The parameters can also be changed during the mixing process by pressing the relevant parameter key and turning the control knob.*

i *Changes made to the mixing parameter settings during the mixing process will be applied to that mixing process only and will not be saved.*

Cleaning / Maintenance

Inside the appliance there are no parts which require maintenance.
Opening the device, other than for the processes described below, is not permissible!

Cleaning

Use a damp cloth to clean the outside of the unit.
Do not use solvent-based, aggressive or scouring cleaners.

Do not steam clean the appliance.

Seal Surfaces

Clean mixing paddle / bowl and device / mixing paddle sealing surfaces before each mixing cycle.

Spare Parts

You can find components subject to wear and the spare parts on the spare part list in the internet at www.renfert.com/p918.

Enter the following item number: 18260000 or 18270000.

The components excluded from the warranty (such as consumables or parts subject to wear and tear) are marked on the spare part list.

Disposal instructions

Within the European Union appliances with this symbol should not therefore be disposed of in unsorted domestic waste.

For more information regarding proper disposal please apply at your local authorities.

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declare under our sole responsibility, that the product(s)

Name	Part No.
Twister	1826 0000 / 1826 0000 03
Twister venturi	1827 0000 / 1827 0000 03
Twister evolution	1828 0000 / 1828 0000 03

is (are) in conformity with the relevant regulatory requirements by compliance with the UK designated standards.

UK legislation

Electromagnetic Compatibility Regulations 2016

Supply of Machinery (Safety) Regulations 2008

The Restriction of the Use of Certain Hazardous Substances in Electrical and Electronic Equipment Regulations 2012

Designated standards

EN 61326-1:2013

EN ISO 12100:2010

EN 61010-1:2010/A1:2019/AC:2019-04

EN IEC 61010-2-051:2021/A11:2021

EN IEC 63000:2018

Tilo Burgbacher,
Engineering Director

Hilzingen, 02.11.2022

The Technical Documentation has been retained by Renfert GmbH

Technical Data

	Twister (No. 1826)	Twister venturi (No. 1827)
Mains voltage:	100 - 240 V, 50 / 60 Hz	
Power consumption:	180 VA	
Device fuse:	T4AL, 250 VAC	
Speed:	100 - 450 1/min	
Connection pressure:	--	4,5 - 6 bar [65 - 87 psi]
Air consumption, approx:	--	28 l/min
LpA *) (idling):	< 70 db(A)	
Dimensions (W x H x D):	152 x 285 x 235 mm [6.0 x 11.2 x 9.3 inch]	152 x 320 x 235 mm [6.0 x 12.6 x 9.3 inch]
Weight (without bowl) approx:	5,2 kg [11.5 lbs]	4,0 kg [8.8 lbs]

*) Sound pressure level in accord. with EN ISO 11202

**Hochaktuell und ausführlich auf ...
Up to date and in detail at ...
Actualisé et détaillé sous ...
Aggiornato e dettagliato su ...
La máxima actualidad y detalle en ...
Актуально и подробно на ...**

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Renfert GmbH • Untere Gießwiesen 2 • 78247 Hilzingen/Germany