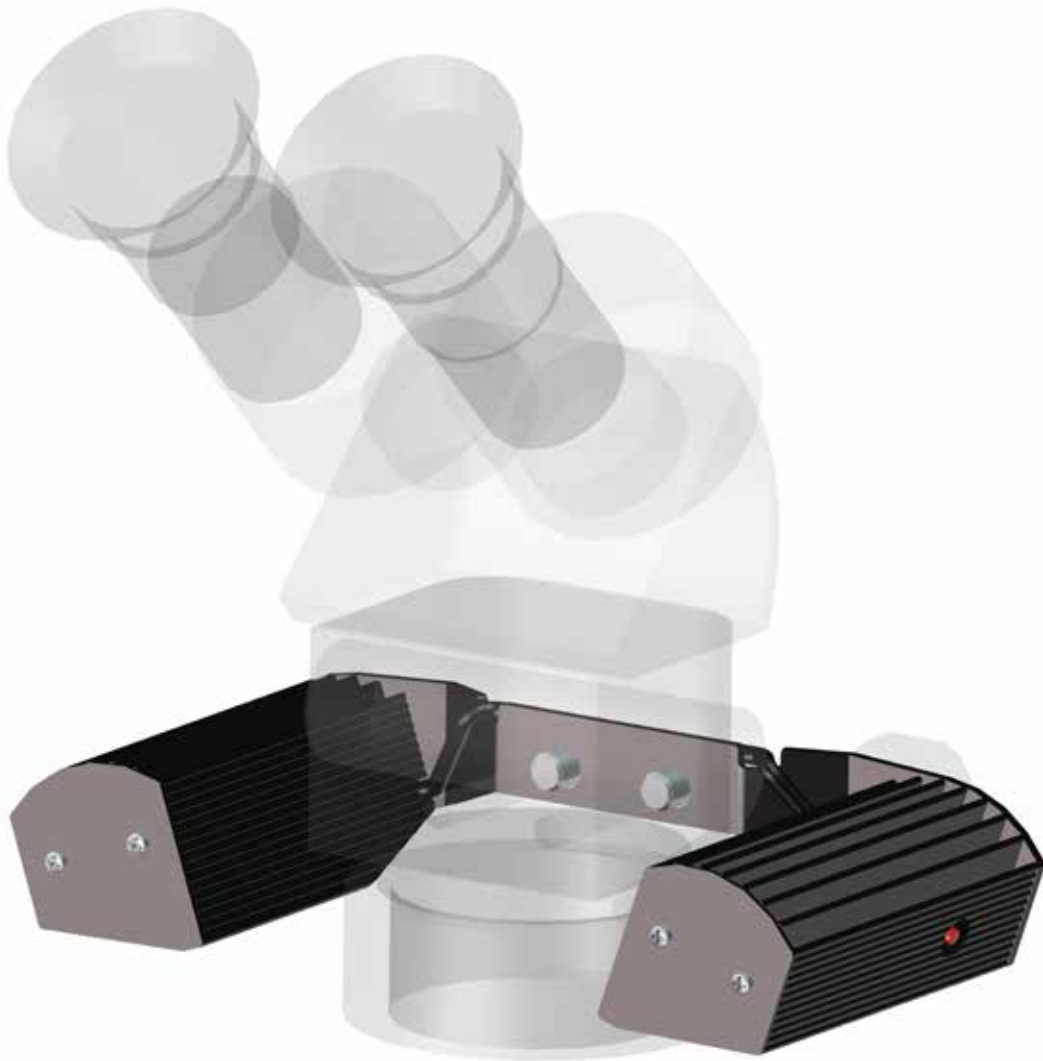
making work easy

DE

EN

FR

IT

ES

PT

TR

RU

PL

ZH

JA

KO

AR

UK

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Made in Germany

21-6687 20112024

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Nr. 2200 5000

DEUTSCH

Inhalt

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1. Introduction

1.1 Symbols

In the instructions for use and on the unit itself you will find these symbols with the following meanings:

Danger

This indicates a direct risk of injury. Consult accompanying documents!

Electrical current

This indicates a risk of hazard due to an electrical current.

Attention

Disregarding this warning may result in damage to equipment.

Note

This provides the operator with useful information to improve and ease use.

The device complies with the requirements of the applicable EU directives.

This product complies with the relevant UK legislation.

See UKCA Declaration of Conformity in the Internet under www.renfert.com.

The device is subject to the EU directive 2002/96/EG (WEEE directive).

► List, particular attention should be paid

- List
- Subsection numbering

⇒ Instructions / appropriate action / input / operational sequence:
You will be asked to carry out the action in a specified order.

- ◆ Result of an action / reaction of the device / reaction of the program:
The unit or program reacts as a result of your actions or when a specific incident occurs.

Other symbols are explained as they occur.

2. Safety

2.1 Intended Use

The product is designed for use on the Renfert Mobiloscope only.
It may only be operated with the mains adapter supplied.

2.2 Improper Use

The product may not be used for general lighting applications.
It may not be built into other units or housings.
The product must not be supplied with power with a different mains adapter.

2.3 Ambient Conditions for Safe Operation

The device may only be operated:

- Indoors,
- Up to an altitude of 2,000 m above sea level,
- At an ambient temperature of between 5 - 35 °C [41 - 95 °F]
- At a maximum relative humidity of 80 % at 31°C [87.8 °F], dropping to a linear of up to 50 % relative humidity at 35 °C [95 °F],
- With mains power where the voltage fluctuations do not exceed 10 % of the nominal value,
- Under contamination level 2 conditions,
- Under over-voltage category II conditions.

2.4 Ambient Conditions for Storage and Transport

For storage and transport the following specifications to ambient conditions apply:

- Ambient temperature -20 – +60 °C [-4 – +140 °F].
- Maximum relative humidity 80 %.

2.5 Hazard and Warning Information

2.5.1 General Information

- ▶ If the device is not used in compliance with the supplied instructions, the safety of the device can no longer be guaranteed.
- ▶ The device may only be operated using a mains cable with the country-specific plug system. Any necessary alterations must be carried out by a qualified electrician.
- ▶ The device may only be operated if the information on the identification plate conforms to the specifications of your local mains power supply.
- ▶ The mains plug must be easily accessible.
- ▶ It is the responsibility of the operator that national regulations during operation and regarding a repeated safety inspection of electrical equipment are complied with. For Germany these are the regulation 3 by DGUV (German Statutory Accident Insurance) in relation with VDE 0701-0702 (Association for Electrical & Electronic Technology).
- ▶ Disconnect the device from the mains before carrying out work on the electrical parts.
- ▶ Check connection cables (such as power supply cords), tubes and housing (i.e. the key-pad) regularly for damage (i.e. kinks, cracks and porosity) or signs of ageing. Devices with damaged connection cables, tubes or housing parts or other defects must not be operated!
- ▶ Defective devices must be put out of service immediately. Remove the mains plug and ensure the device is not used. Send the device for repair!
- ▶ Only operate the device under supervision.
- ▶ The use of unauthorized accessories may be an impediment to the safety of the device. Use spare parts and accessories supplied by Renfert only.
- ▶ Observe the accident prevention regulations by the trade association!
- ▶ Information on REACH and SVHC is available on our website www.renfert.com, in the Support area.

2.5.2 Specific Information

- ▶ Do not look directly into the lamps!
- ▶ Do not cover lights while in operation as this poses a fire hazard!
- ▶ Hot surface. The housing on the LED light fitting becomes hot when in use.

2.6 Authorized Persons

Operation and maintenance of the device may only be performed by qualified personnel.

Any repairs not specifically described in these operating instructions may only be carried out by a qualified electrician.

2.7 Disclaimer

Renfert GmbH shall be absolved from all claims for damages or warranty if:

- ▶ The product is employed for any purposes other than those specified in the operating instructions.
- ▶ The product is altered in any way other than those alterations described in the operating instructions.
- ▶ The product is not repaired by an authorized facility or if non-original Renfert parts are implemented.
- ▶ The product continues to be used despite obvious safety faults or damage.
- ▶ The product is subjected to mechanical impacts or is dropped.

3. Product Description

3.1 General Description

The LED lights can be easily fitted on all Renfert Mobiloscope models. The illumination of the object from both sides and the colour of the lamp ensure a high-contrast, non-glare working surface. External adapter provided for low-voltage power supply.

3.2 Components and Functional Elements

- | | |
|-----------------------|--------------------------|
| 1 LED light fitting | 6 Power supply connector |
| 2 Adapter plate | 7 Mains adapter |
| 3 On / Off button | 8 Mains input |
| 4 Power supply socket | 9 Mains lead |
| 5 Fastening screws | |

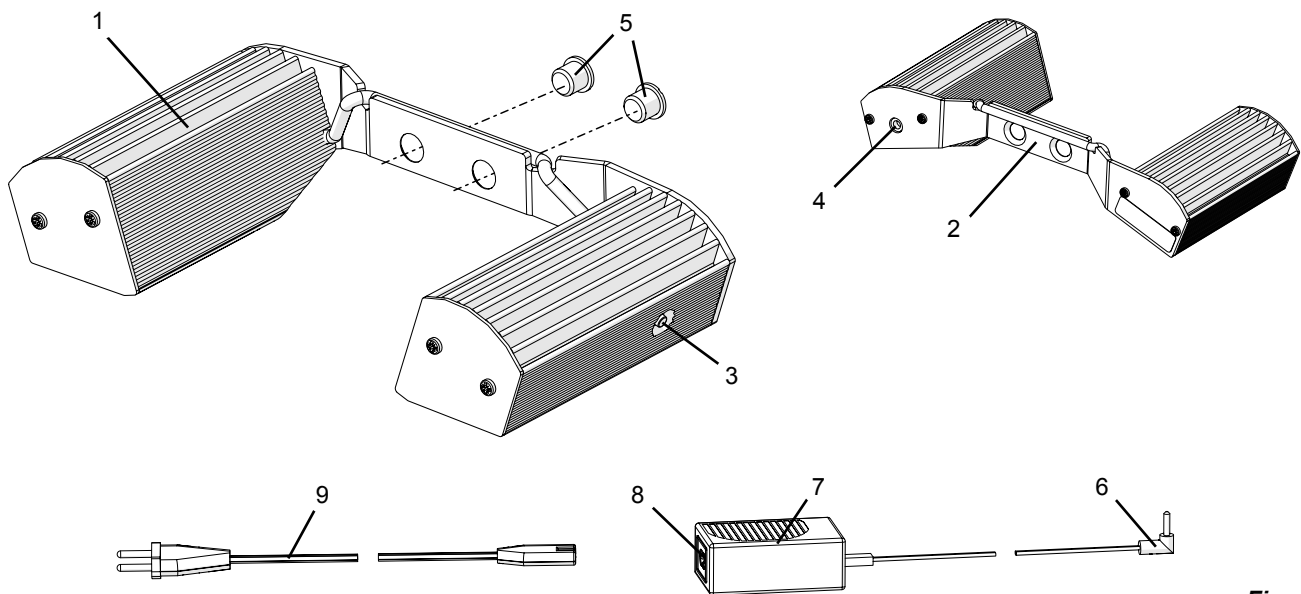
Fig. 1

3.3 Scope of Delivery

- 1 LED light fitting
- 1 Mains adapter
- 1 Mains lead, flat Europlug connector
- 1 Mains lead, NEMA 1-15P plug
- 2 Fastening screws
- 6 Cable ties
- 1 Operating manual

4. Setting Up

4.1 Unpacking

- ⇒ Remove the device and all the accessories from the delivery package.
 - ⇒ Check the delivery for completeness (refer to the "Scope of Delivery" section).
- The product must be fitted on the Mobiloscope before use.

4.2 Fitting the LED lights

- Remove any existing systems before fitting (cf. sec. 4.4).
- ⇒ Remove screws on the back of the Mobiloscope head.
- ⇒ Screw adapter plate with LED light fitting (1) onto Mobiloscope head with the screws provided (5).
- ⇒ Make sure that the connection cable is not caught between the lamps.
- ⇒ Plug the power supply connector (6) into the socket (4).
- ⇒ Fasten cable on the Mobiloscope arm using the ties provided.

! Make sure that the cable does not get caught when the arm is moved and that the plug is not under strain.

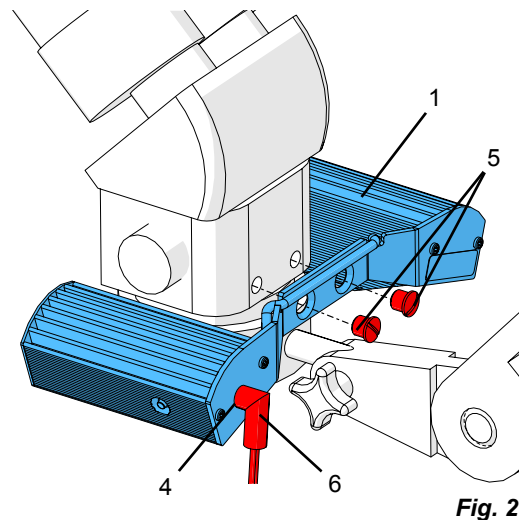
Fig. 2

4.3 Electrical Connection

- ⇒ Plug the mains lead (9, Fig. 1) in the mains input (8, Fig. 1) on the mains adapter (7, Fig. 1).
- ⇒ Plug mains lead in socket.

4.4 Removal of existing Renfert systems

Lamp adapter (article nos. 2200-0501, -1501, -2501):

- ⇒ Pull out connector plug.
- ⇒ Undo fastening screws and remove lamp adapter.
- The cable need not be removed but is not used for the LED light fitting.

Microscope light / cold light source (article nos. 2200-3000, -4000):

- ⇒ Undo knurled screw and pull microscope head up and out of retainer (difficult to remove).
- ⇒ Remove steel ring fastener with cold light source.
- ⇒ Refit microscope head and secure in place with knurled screw.

5. Operation

The On / Off button (3, Fig. 1) is used to switch the LED lights on and off and to change to the dimmer function.

- ⇒ Press On / Off button repeatedly
 - ◆ LED lights on, full strength
 - ◆ LED lights on, dimmed
 - ◆ LED lights off

The housing on the LED light fitting may become hot when in use.

Risk of injury on glass cover on LED lights!

Risk of injury from glass splinters if glass cover is damaged, e.g. from working with partitioning screens.

6. Cleaning / Maintenance

Inside the appliance are no parts which require maintenance.

Opening the device, other than for the processes described below, is not permissible!

The LED light fitting needs no maintenance.

If necessary, wipe the LED light fitting with a slightly damp cloth.

Switch the LED light fitting off and allow to cool before cleaning the housing!

Do not use solvent-based cleaners.

6.1 Spare Parts

You can find components subject to wear and the spare parts on the spare part list in the internet at www.renfert.com/p918.

The components excluded from the warranty (such as consumables or parts subject to wear and tear) are marked on the spare part list.

Serial numbers, manufacture date and device type can be found on the device identification plate.

7. Troubleshooting

Error	Possible Cause	Corrective Action
None of the lamps come on.	<ul style="list-style-type: none"> • Incorrect connection. • Defective mains adapter. • LED light fitting defective. 	<ul style="list-style-type: none"> • Check leads and plug connections. • Order mains adapter from spare parts list. • Send LED light fitting in for repair.
The left lamp does not light up.	<ul style="list-style-type: none"> • Connection cable damaged. 	<ul style="list-style-type: none"> • Send LED light fitting in for repair.
Light not powerful enough.	<ul style="list-style-type: none"> • It has been switched over to dimmer function. • Dirt on glass guards. 	<ul style="list-style-type: none"> • Switch LED lights off and then back on again (cf. also section 5). • Switch off LED light and allow to cool down. Clean glass guards.

8. Technical Data

Nominal voltage:	100 - 240 V
Mains frequency:	50 / 60 Hz
Mains adapter power consumption:	16 W
LED light fitting operating voltage:	12 VDC
LED light fitting current consumption:	1,2 A
Dimensions (Width x Height x Depth):	162 x 34 x 82 mm [6.38 x 1.34 x 3.25 inch]
Weight (excl. mains adapter), approx.:	250 g [0.55 lbs]
This product contains a light source of energy efficiency class:	G

9. Warranty

Provided the unit is properly used, Renfert warrants all components for **3 years**.

Warranty claims may only be made upon presentation of the original sales receipt from the authorized dealer.

The warranty is voided in the case of improper use; failure to observe the operating, cleaning, maintenance and connection instructions; in case of independent repairs or repairs by unauthorized personnel; if spare parts from other manufacturers are employed, or in case of unusual influences or influences not in compliance with the utilization instructions.

Warranty service shall not extend the original warranty.

10. Disposal Information

10.1 Disposing of the Unit

The unit must be disposed of by an authorized recycling operation. The selected firm must be informed of all possible health hazardous residues in the unit.

10.1.1 Disposal Instructions for countries in the EU

To conserve and protect the environment, prevent environmental pollution and improve the recycling of raw materials, the European Commission adopted a directive that requires the manufacturer to accept the return of electrical and electronic units for proper disposal or recycling.

Within the European Union, units with this symbol should not therefore be disposed of in unsorted domestic waste.

Please contact your local authorities for more information on proper disposal.

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declare under our sole responsibility, that the product(s)

Name	Part No.
LED illumination, for Mobiloskop	2200 5000

is (are) in conformity with the relevant regulatory requirements by compliance with the UK designated standards.

UK legislation

Electromagnetic Compatibility Regulations 2016

Electrical Equipment (Safety) Regulations 2016

The Restriction of the Use of Certain Hazardous Substances in Electrical and Electronic Equipment Regulations 2012

Designated standards

EN 62368-1:2014/AC:2015

EN 62031:2008/A2:2015

EN 61347-1:2015


EN 61347-2-13:2014

EN 62493:2010

EN IEC 55015:2019/A11:2020

EN 61547:2009;

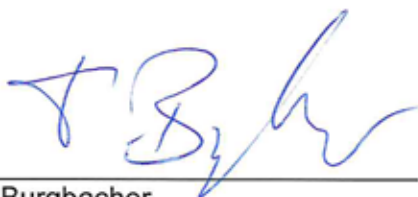EN IEC 63000:2018

Tilo Burgbacher,
Engineering Director

Hilzingen, 02.11.2022

The Technical Documentation has been retained by Renfert GmbH

**Hochaktuell und ausführlich auf ...
Up to date and in detail at ...
Actualisé et détaillé sous ...
Aggiornato e dettagliato su ...
La máxima actualidad y detalle en ...
Актуально и подробно на ...**

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Renfert GmbH • Untere Gießwiesen 2 • 78247 Hilzingen/Germany