Renfert

making work easy

DE

EN

FR

IT

ES

RU

PL

NL

ZH

JA








KO

Alginat-Anmischbecher 500 ml

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Made in Germany

21-6508_20092019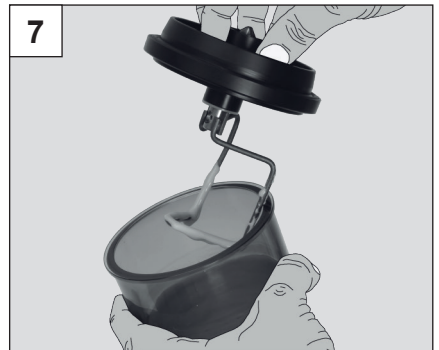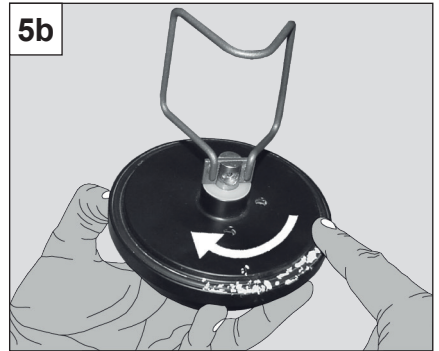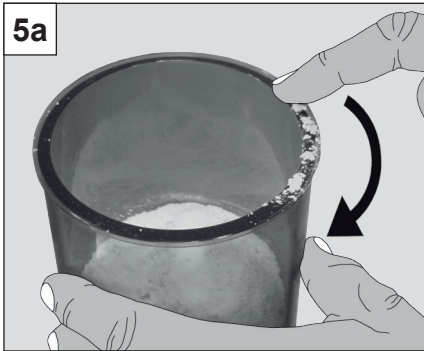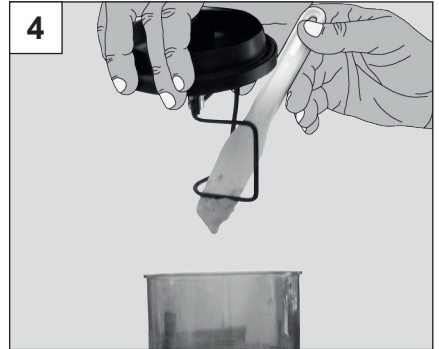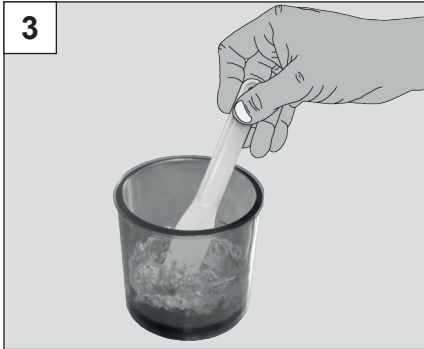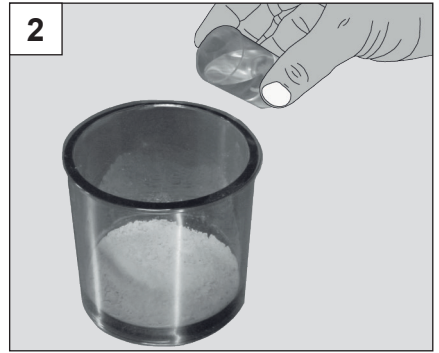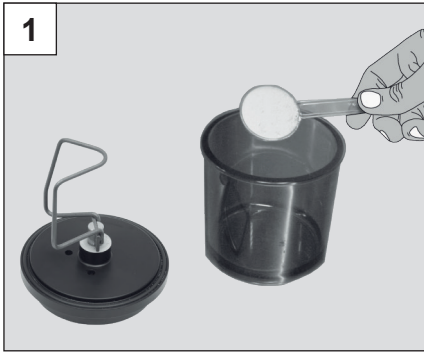

Alginat-Anmischbecher 500 ml

inkl. Rührwerk
Nr. 1823-0500

DE

DEUTSCH

1. Einleitung

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EN

1. Introduction

We are pleased with your decision to purchase an alginate mixing bowl with paddle included.

This paddle guarantees ideal mixing of alginates.

Please, read this instruction manual thoroughly and observe the safety information in order to ensure long and trouble-free operation.

2. Application Area

The alginate paddle is used for bubble-free, homogenous mixing of alginates.

It can be used with all Renfert vacuum-mixers of the *Twister* series(*).

(* Except *Twister pro* and *Twister* produced before March 2004.

3. Safety information

- Only to be used for alginate mixing.
- Please observe the instruction manual of your mixing equipment of the *Twister* series on this subject.

Please observe the instructions of use and safety information of your alginate's manufacturer.

Information on REACH and SVHC is available on our website www.renfert.com, in the Support area.

4. Mixing procedure

For an exact dosage of alginate and water, please use the scoops and vials of your alginate's manufacturer.

1. Using the measuring scoop, put the needed alginate quantity into the bowl (fig. 1).
2. Add the needed quantity of water via the measuring vial (fig. 2).

The bowl and the mixing paddle are both engineered for 1 to 6 scoops of alginate and for the corresponding quantity of water (Do not fill the bowl besides the maximum filling level marked on the bowl).

3. Stir the mixing materials together a little first (fig. 3).
4. Wipe the spatula clean on the lower bar of the mixing paddle (fig. 4).
5. Clean the bowl rim (fig. 5a) as well as the lid margin (fig. 5b)!
6. Insert the mixing paddle (fig. 6).
7. Couple the mixing bowl to the unit. Do not release the bowl until it stays in place by itself.

The further mixing steps should be executed as described in the instruction manual of your mixing unit of the *Twister*-series.

At the end of the mixing procedure take the mixing paddle out of the bowl and wipe the alginate residues on the bowl's rim (fig. 7).

Tip:

Independent tests have shown that the best mixing results can be achieved at a speed of 320-450 rpm and a mixing time of 25-30 seconds.

Please observe also the exact instructions of the alginate's manufacturer..

5. Cleaning

Do not use any detergents containing solvents eg. disinfecting bur cleaners, since they damage the coating on the alginate paddle !

5.1 Alginate mixing paddle / mixing bowl

After mixing, let the alginate residues on the alginate mixing paddle resp. on the alginate mixing bowl shortly set and remove by hand. Afterwards, wipe clean with a humid cloth and rinse under running water.

Do not let the alginate residues harden !

Hardened alginate residues can be removed using commercially available alginate cleaner.

5.2 Sealing

Please, always keep the following sealing surfaces clean, in order to ensure an ideal vacuum build- up and in order to guarantee that the bowl will be securely held in place during mixing:

- a) Sealing ring mixing paddle/ mixing bowl
- b) Sealing ring equipment / mixing paddle

Tip:

Lightly lubricate the sealing rings with vaseline from time to time. This prolongs the sealing' s life and ensures an optimum vacuum performance.

6. Spare Part

No. 90002-1394 Alginate mixing paddle, mounting wrench incl.

7. Standard Delivery

- 1 Alginate mixing bowl 500 ml
- 1 Paddle
- 1 Instruction manual

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Renfert GmbH • Untere Gießwiesen 2 • 78247 Hilzingen/Germany