making work easy

- de
- en
- fr
- it
- es
- pt
- tr
- ru
- zh
- ja
- ko
- ar

TOP spin

Quick Start Guide

Made in Germany

21-6808 09112021

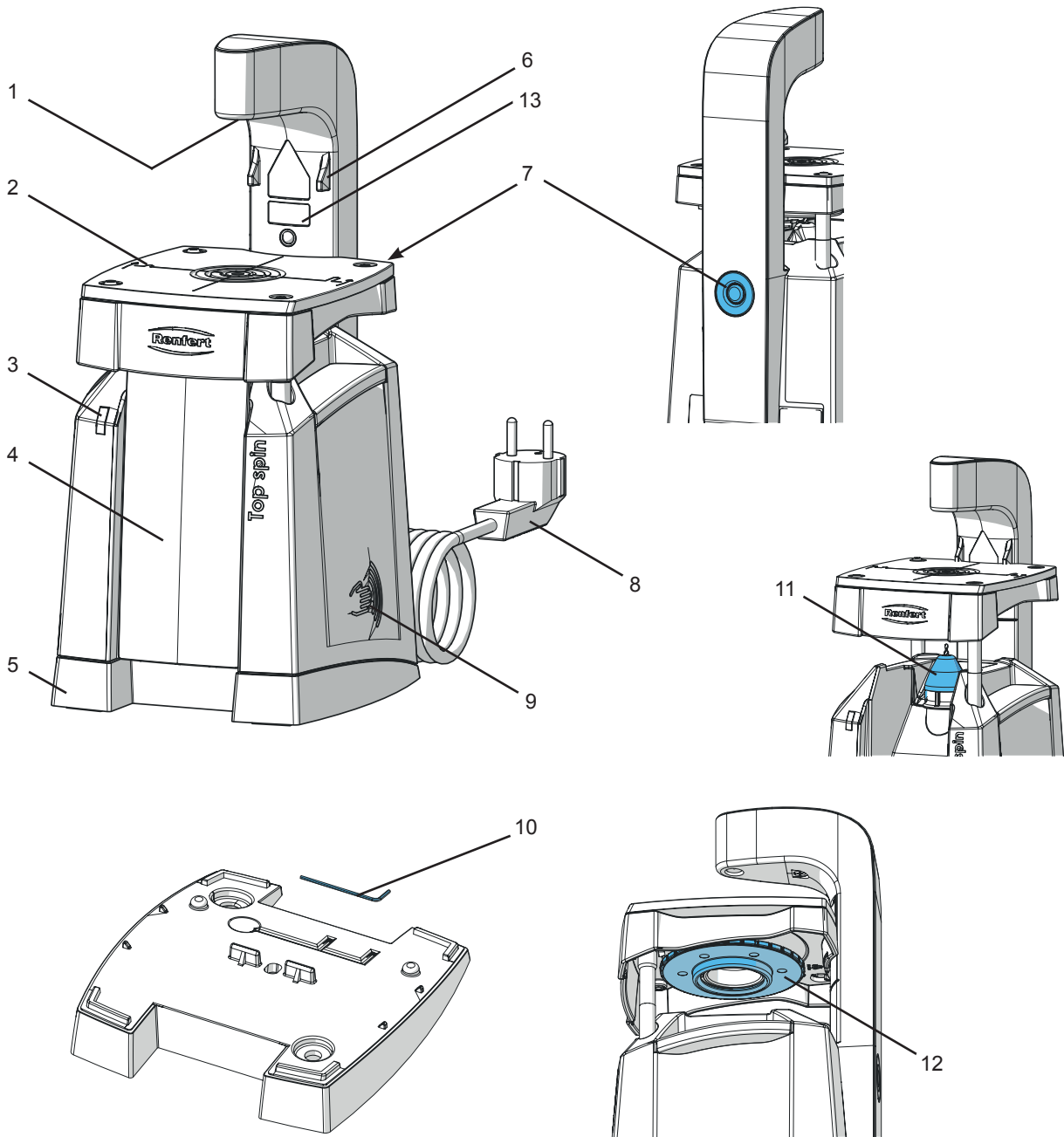
Fig. 1

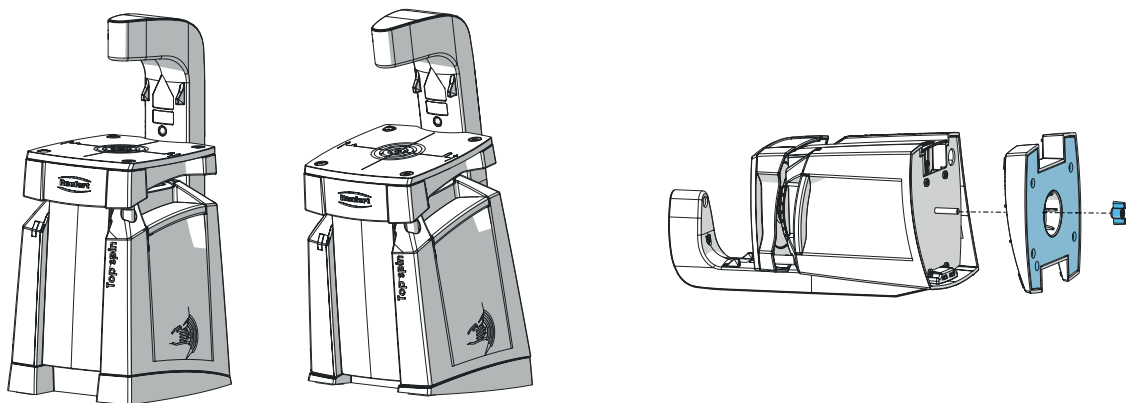
Fig. 2

Einleitung

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Introduction

This Quick Start Guide contains instructions on the operation of the product and relevant safety information.

Detailed information on the product can be found in the operating instructions which are available for download on our website at www.renfert.com.

Symbols

In the instructions for use and on the unit itself you will find these symbols with the following meanings:

Danger

This indicates a direct risk of injury. Consult accompanying documents!

Electrical current

This indicates a risk of hazard due to an electrical current.

Laser

Do not look into the laser beam.

Attention

Disregarding this warning may result in damage to equipment.

Note

This provides the operator with useful information to improve and ease use.

The device complies with the requirements of the applicable EU directives.

This product complies with the relevant UK legislation.

See UKCA Declaration of Conformity in the Internet under www.renfert.com.

The device is subject to the EU directive 2002/96/EG (WEEE directive).

Other symbols are explained as they occur.

Safety

Proper Use

This device is a pinhole drilling unit designed to drill holes for the secure fixation of pins from various different pin systems in dental arches made from plaster or epoxy resin (e.g. on a polyurethane basis).

The device may also be used to drill holes into acrylic base plates. It is exclusively for use in dental laboratories or dental practices.

Improper Use

The device must not be used:

- for drilling into metal
- for milling

The product is not designed for use in the domestic environment.

Any application other than those described in these instructions for use is regarded as improper use. The manufacturer is not liable for damage caused by incorrect or unauthorised use.

Only spare parts and accessories supplied or authorized by Renfert GmbH may be used with this product. If other spare parts or accessories are used, this could have a detrimental effect on the safety of the device, increase the risk of serious injury and lead to damage to the environment or the device itself.

Hazard and Warning Information

General Information

- ▶ If the device is not used in compliance with the supplied instructions, the safety of the device can no longer be guaranteed.
- ▶ The device may only be operated using a mains cable with the country-specific plug system. Any necessary alterations must be carried out by a qualified electrician.
- ▶ The device may only be operated if the information on the identification plate conforms to the specifications of your local mains power supply.
- ▶ The device may only be plugged into outlets which are connected to the protective conductor system.
- ▶ The mains plug must be easily accessible.
- ▶ Disconnect the device from the mains before carrying out work on the electrical parts.
- ▶ Check connection cables (such as power supply cords), tubes and housing (i.e. the key-pad) regularly for damage (i.e. kinks, cracks and porosity) or signs of ageing.
Devices with damaged connection cables, tubes or housing parts or other defects must not be operated!
- ▶ Defective devices must be put out of service immediately. Remove the mains plug and ensure the device is not used. Send the device for repair!
- ▶ Only operate the device under supervision.
- ▶ Please observe the national accident prevention regulations!
- ▶ It is the responsibility of the operator that national regulations during operation and regarding a repeated safety inspection of electrical equipment are complied with. For Germany these are the regulation 3 by DGUV (German Statutory Accident Insurance) in relation with VDE 0701-0702 (Association for Electrical, Electronic and Information Technology).
- ▶ Information on REACH and SVHC is available on our website www.renfert.com, in the Support area.

Specific Instructions

- ▶ **Caution! Laser beam! Laser class 2! Do not look into the laser beam!**
- ▶ Do not use the device for milling. Risk of injury if the drill breaks.
- ▶ Never touch the drill when the unit is connected to the power supply. There is a risk of injury if the unit is accidentally activated!
- ▶ Do not operate the Top spin with long, untied hair or loose / baggy clothing. If anything gets tangled up or drawn in there is a risk of injury.
- ▶ Do not touch the drill when in motion.
- ▶ The unit should only be activated when the model is securely placed in position
- ▶ When drilling, do not hold the model directly over the laser beam - risk of injury.

Authorized Persons

Operation and maintenance of the device may only be performed by qualified personnel.

Product Description

Component Groups and Functioning Elements

See Fig. 1 (on inside cover)

- | | | | |
|---|---------------------|----|------------------------------------|
| 1 | Target laser | 8 | Mains supply cable with plug |
| 2 | Drill table | 9 | Sensor surface (on / off) |
| 3 | Operation display | 10 | Allen key |
| 4 | Dust Drawer | 11 | Dust protection hood |
| 5 | Base | 12 | Depth stop (under the drill table) |
| 6 | Upper stop position | 13 | Warning Laser |
| 7 | Spindle lock | | |

Included in Delivery

- 1 Top spin
- 1 Stepped drill, size: large
- 1 Quick Start Guide
- 1 Allen key
- 1 Replacement threaded pin (see end of the instructions for use)
- 1 Laser sticker set

Installation

Unpacking

⇒ Check to ensure all delivery parts are complete (compare “Included in Delivery”).

Setting Up

- ⇒ Place the unit on a level and stable base.
- ⇒ Decide which working position you require (Fig. 2).

Change laser warning label

⇒ Remove the laser warning label in your language from the sticker set and stick over the present warning label (13, Fig. 1).

Electrical Connection

⇒ Unroll the power cable (8, Fig. 1) and insert the plug into the socket.

Operation

Switching on / off

The unit is switched on and off via the sensor surface (9, Fig. 1).

⇒ To switch the unit on, touch the sensor surface with the palm of your hand for 1 second. Do not apply pressure.

The drilling motor is activated when the drilling table (2, Fig. 1) depressed.

Only activate the drilling motor when there is a drill in place!

Auto Off

If the unit is not used for a duration of 3 minutes, it automatically switches off.

Setting the Drilling Depth

The drilling depth can be set at the adjustable depth stop (12, Fig. 1) underneath the drilling table. If necessary, carry out trial drillings.

The drilling depth changes approximately 0.1 mm for each click on the depth stop.

Drilling

⇒ Place the dental arch onto the drilling table (2, Fig. 1) and align the target laser (1, Fig. 1).

⇒ Hold the dental arch with both hands.

⇒ Press the drilling table down as far as it will go.

- ◆ The motor will be activated automatically and begin drilling from the bottom of the dental arch.

⇒ Guide the drilling table back up.

- ◆ The motor will stop when the drilling table has reached its original position.

Cleaning / Maintenance

Inside the extractor there are small parts which require maintenance.
Opening the device, other than for the processes described below, is not permissible!


Clean the unit using only a damp cloth.

Do not use any scouring cleaning agents or those containing solvents.

Emptying the Dust Drawer

- ⇒ Lift the work table up as far as it will go
- ⇒ Brush the dust residues into the Dust Drawer or remove them by knocking gently (4), and pull the Dust Drawer out by its lower edge.
- ⇒ Empty the Dust Drawer.
- ⇒ First insert the upper part of the Dust Drawer into the housing and then press the lower part in.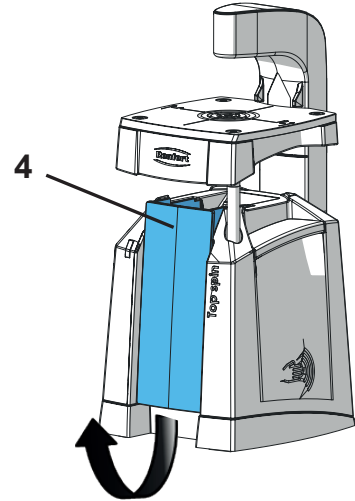

Spare parts

You can find components subject to wear and the spare parts on the spare part list in the internet at www.renfert.com

The components excluded from the warranty (such as consumables or parts subject to wear and tear) are marked on the spare part list.

Disposal Information

Within the European Union, units with this symbol should not therefore be disposed of in unsorted domestic waste.

Please contact your local authorities for more information on proper disposal.

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declare under our sole responsibility, that the product(s)

Name	Part No.
Top spin	1840 0000 / 1840 0000 03
AUTO spin	1860 0000 / 1860 0000 03

is (are) in conformity with the relevant regulatory requirements by compliance with the UK designated standards.

UK legislation

Electromagnetic Compatibility Regulations 2016

Supply of Machinery (Safety) Regulations 2008

The Restriction of the Use of Certain Hazardous Substances in Electrical and Electronic Equipment Regulations 2012

Designated standards

EN 61326-1:2013

EN ISO12100:2010

EN 61010-1:2010/A1:2019/AC:2019-04

EN 60825-1:2014

EN IEC 63000:2018

Tilo Burgbacher,
Engineering Director

Hilzingen, 02.11.2022

The Technical Documentation has been retained by Renfert GmbH

Technical Data

Nominal voltage	230 V
Permissible mains voltage:	100 - 240 V
Mains frequency:	50 / 60 Hz
Power consumption during drilling:	< 12 W
In stand by mode:	< 0.7 W
Laser: - Laser class: - Wave length:	2 630 - 680 nm
Power output P0, max.:	< 1 mW
Acoustic pressure according to DIN EN ISO 11202	LpA < 70 dB (A)
Dimensions (width x height x depth):	153 x 330 x 175 mm [6.0 x 13.0 x 6.9 inch]
Permissible drill shaft diameter:	3.00 mm + 0 / - 0.03 mm
Total drill length (min. - max.):	34 - 46 mm [1.34 - 1.81 inch]
Weight, approx.:	4.0 kg [8.8 lbs]

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Spare threaded pin (for drill stop)
(for its use, please refer to the complete instruction for use in the internet)

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Renfert GmbH • Untere Gießwiesen 2 • 78247 Hilzingen/Germany