making work easy

- de
- en
- fr
- it
- es
- pt
- tr
- ru
- zh
- ja
- ko
- ar

# Twister evolution

Quick Start Guide

Made in Germany

21-6759 28112024

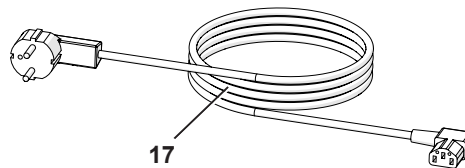
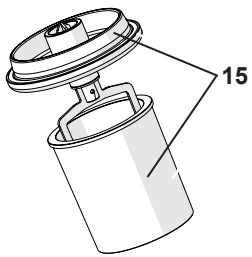
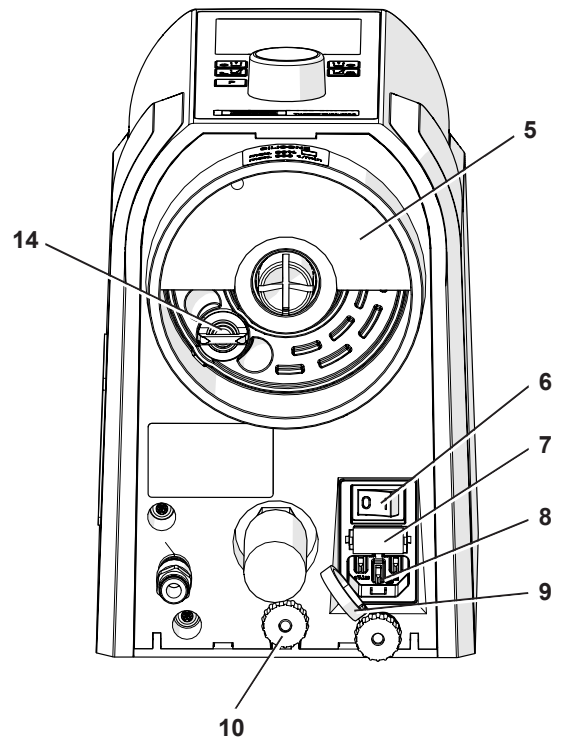
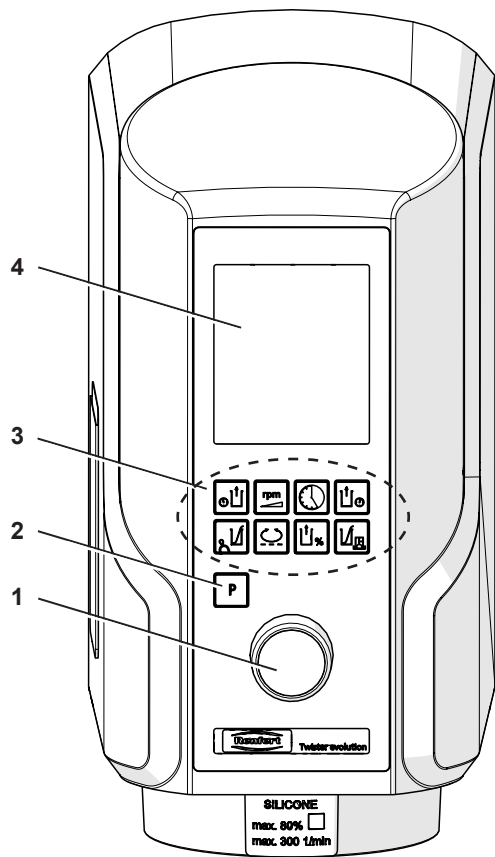
Fig. 1

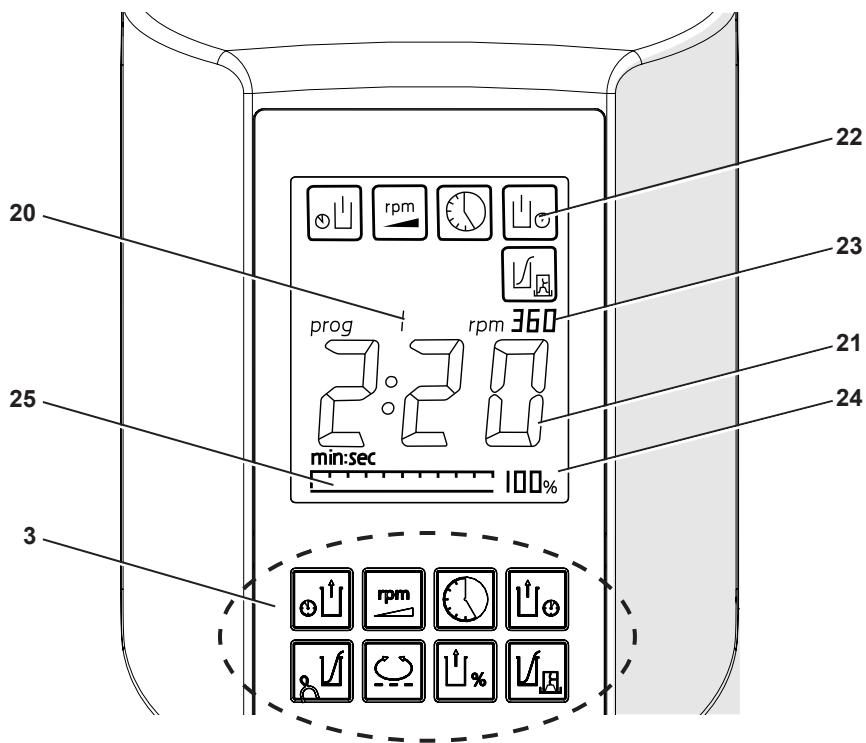
Fig. 2

# Einleitung

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# Introduction

This Quick Start Guide contains instructions on the operation of the product and relevant safety information. Detailed information on the product can be found in the operating instructions which are available for download on our website at [www.renfert.com/p915](http://www.renfert.com/p915).

Enter the following item number: 18280000.

## Employed Symbols

This manual contains symbols also found on the appliance itself with the following meaning:

**Danger!**

**Immediate risk of injury. Please observe accompanying documentation!**

**High Voltage!**

**Danger – high voltage.**

**Caution**

**Risk of damage to the appliance if the instruction is not followed.**

**Notice**

**A general notice that improves and eases use.**

**The appliance complies with applicable EU directives.**

**This product complies with the relevant UK legislation.**

**See UKCA Declaration of Conformity in the Internet under [www.renfert.com](http://www.renfert.com).**

**Within the EU, this device is subject to the provisions of the directive 2002/96/EG (WEEE directive).**

Other symbols are explained when displayed.

## Safety

### Intended use

The Twister evolution vacuum mixer is designed solely for homogeneous, bubble-free mixing of dental casting and modelling materials such as plasters, investments, and silicones.

### Improper use

Only spare parts and accessories supplied or authorized by Renfert GmbH may be used with this product. If other spare parts or accessories are used, this could have a detrimental effect on the safety of the device, increase the risk of serious injury and lead to damage to the environment or the device itself.

### Hazards and warnings

#### General Information

- ▶ **Granted protection is not given any longer in case the appliance is not operated according to the instruction manual on hand.**
- ▶ **The appliance should only be operated using a mains cable with a country-specific plug system. Any modification required should only be carried out by an electrician.**
- ▶ **The appliance should only be operated if the data on the rating plate corresponds with the data of regional mains voltage.**
- ▶ **The appliance should only be plugged into earthed sockets.**
- ▶ **The mains socket should be easily accessible.**
- ▶ **Regularly check connecting cables and hoses (e.g. mains cable) for damage (e.g. kinks, tears, porosity) or deterioration. Do not operate appliances with damaged mains wires, hoses or other defects.**
- ▶ **The appliance should always be operated under supervision.**
- ▶ **Risk of injury!**  
**Risk of injury if the components used are not approved. Always use original Renfert components.**
- ▶ **Switch off and unplug the appliance before carrying out repairs and servicing work on electrical components.**
- ▶ **Switch off and unplug the appliance before cleaning or servicing it.**

- ▶ It is the responsibility of the operator that national regulations during operation and regarding a repeated safety inspection of electrical equipment are complied with. For Germany these are the regulation 3 by DGUV 0701-0702 (German Statutory Accident Insurance) in relation with VDE (Association for Electrical & Electronic Technology).
- ▶ Information on REACH and SVHC is available on our website [www.renfert.com](http://www.renfert.com), in the Support area.

### Specific Information

- ▶ Only to be used indoors. The appliance is only intended for dry operation and should not be used or stored outdoors or in wet conditions.
- ▶ Always read the safety data sheets supplied by the manufacturers when mixing investment materials! Wear the appropriate personal protective equipment as and when required!
- ▶ Never mix flammable or explosive materials.
- ▶ Never connect the paddle without the mixing bowl.
- ▶ Manipulating the automatic bowl coupler and the intake opening can result in damage to the device and personal injury.
- ▶ Switch off the appliance when work is complete.

### Authorised persons

The appliance may only be operated and serviced by trained personnel.

## Product description

### Components and functional elements

See Fig. 1 (on inside cover)

- |   |                       |
|---|-----------------------|
| 1 Control knob (set mixing parameters, start, stop, aerate) | 7 Fuse                |
| 2 Programming key „P“                                       | 8 Appliance socket    |
| 3 Parameter keys  | 9 Cable safety catch  |
| 4 Display   | 10 Knurled nut        |
| 5 Sponge filter   | 14 Sinter filter      |
| 6 Appliance switch  | 15 Bowl incl. blunger |
|   | 17 Mains cable        |

See Fig. 2 (on inside cover)

- |  |                                       |
|--|---------------------------------------|
| 20 Number of program selected                                  | 23 Speed setting in selected program  |
| 21 Large mixing time display                                   | 24 Vacuum setting in selected program |
| 22 Parameter symbol in large-scale display (here: mixing time) | 25 Current vacuum level (bar graph)   |

## Commissioning

### Setup

- Operate the device at room temperature 15 - 30 °C [59 - 86 °F].
- Do not place the device under a source of heat.
- Do not expose the device to direct sunlight.
- Do not expose the device to high humidity.

### Wall Mounting

Use the drilling template supplied for wall mounting.

**Make sure the wall on which you intend to mount the device is capable of bearing its weight!**

**Please ensure that there are no electric cables or water pipes running through the wall at the place of installation!**

**Tighten the knurled nut (10) securely.**

## Benchtop Models

The stand can be used to convert the vacuum mixer to a floor-mounted appliance.

The stand is not part of the standard delivery and must be ordered as an optional accessory (see [www.renfert.com](http://www.renfert.com)).

## Connection

- ⇒ Secure the mains cable (17) through the cable safety catch (9).
- ⇒ Plug mains cable into socket (8) on appliance.
- ⇒ Plug in at the mains.

## Operation

### On

The appliance is switched on and off at the switch (6).

The program and mixing parameter settings last used will appear in the display when the appliance is switched on.

### Standby mode

The appliance will go into standby mode if no button is pressed or it stands idle for more than three minutes. The display will be dimmed when this happens.

Pressing a parameter key, turning the control knob or coupling the mixing bowl ends the standby mode.

## Mixing Process

***A mixing process always takes place according to the values stored in a program. See Chapter "Programming".***

**Always read the safety data sheets supplied by the manufacturers when mixing investment materials! Wear the appropriate personal protective equipment as and when required!**

**Observe maximum mark on mixing bowl. Do not fill the mixing bowl over the maximum mark. This limit is valid for powders and liquids measured before mixing. If the mixing bowl is over-filled the suction filter system can become soiled.**

- ⇒ Select an appropriate mixing bowl size.
- ⇒ Mix powder and liquid as instructed by the manufacturer. Allow plaster to sit for approx. 15 - 20 sec.
- ⇒ Using a spatula, lightly mix the materials together or use the premix function.
- ⇒ Install the appropriate paddle.
  - ◆ Make sure the bowl rim and cover rim are clean!
- ⇒ Press the programming key, „P“ (2).
- ⇒ Select the desired program by turning the control knob (1).
  - ◆ The following will be shown from the selected programme:
    - Program No.
    - Symbols of the used parameter
    - Mixing time
    - Speed
    - Vacuum
- ⇒ Couple the mixing bowl to the appliance.
  - ◆ The motor starts briefly for coupling the mixing paddle.
  - ◆ Vacuum generation starts automatically.
- ⇒ Do not let go until the bowl stays in place alone and vacuum level bar has passed the halfway mark!
  - ◆ Once the minimum vacuum has been achieved (the appliance beeps), the mixing program begins automatically.
    - The remaining mixing time (countdown) is displayed during the mixing process.
    - The appliance beeps once the mixing program has been completed.
    - The time which has elapsed since the end of the mixing cycle is shown in the display.

- ⇒ Aerate bowl.
- ⇒ Hold the bowl!
- ⇒ Press the control knob (1).
  - ◆ The bowl will be released in a few seconds.
- ⇒ Remove the mixing bowl.

### **Abort mixing process**

- ⇒ Stop the mixing process: Press the control knob once.
  - ◆ The appliance will beep.
  - ◆ Mixing process being aborted.
- ⇒ Hold the bowl!
- ⇒ Aerate bowl: Press the control knob.
  - ◆ The bowl will be released in a few seconds.

***i*** ***The parameters can also be changed during the mixing process by pressing the relevant parameter key and turning the control knob.***

***i*** ***Changes made to the mixing parameter settings during the mixing process will be applied to that mixing process only and will not be saved.***

## **Programming**

A mixing process always takes place according to a program and the mixing parameter specified within. The mixing parameter and the specified values can be changed and saved.

### **Programme Selection**

- ⇒ Press the programming key, „P“ (2).
- ⇒ Select the desired program by turning the control knob (1).

### **Setting / Saving Mixing Parameters**

- ⇒ Press the relevant parameter key (3).
  - ◆ Mixing parameter symbol will appear in the display.
  - ◆ Mixing parameter setting is displayed in large-scale characters for approx. 4 sec. or until another key is pressed.
- ⇒ Turn the control knob (1) to change the value.
- ⇒ Repeat the procedure for all parameter to be changed.

### **Saving mixing parameter settings permanently**

- ⇒ Press programme button „P“ (2) until the acoustic signal can be heard (after approx. 2 seconds) which means that all values in the selected programme have been permanently saved.

For detailed information on programming and the influence of the mixing parameter, see the complete instructions in the Internet under [www.renfert.com/p918](http://www.renfert.com/p918).

## Key Symbols / Mixing parameters / Values

Mixing parameters	Symbol	Adjustment range	Factory setting
Program		1 - 100	1
Mixing time		0:00 - 9:55 min:sec	1:00
Speed		100 - 450 1/min	350
Vacuum		70 - 100 *) %	100
Pre-vacuum		0:00 - 1:00 min:sec	0:00
Post-vacuum		0:00 - 1:00 min:sec	0:00
Interval **)		0:00 - 0:30 min:sec	0:00
Premix, plaster		0:00 / 0:15 / 0:20 / 0:25	0:00
Premix, investment		0:00 / 0:15 / 0:20 / 0:25	0:00

\*\*) Direction change:  
The time period after which the current direction of mixing is reversed. This only applies if the mixing time is longer than the interval.

## Cleaning / Maintenance

**Inside the appliance there are no parts which require maintenance. Opening the device, other than for the processes described below, is not permissible!**

### Cleaning

Use a damp cloth to clean the outside of the unit.  
Do not use solvent-based, aggressive or scouring cleaners.

**Do not steam clean the appliance.**

### Seal Surfaces

Clean mixing paddle / bowl and device / mixing paddle sealing surfaces before each mixing cycle.

### Spare Parts

You can find components subject to wear and the spare parts on the spare part list in the internet at [www.renfert.com/p918](http://www.renfert.com/p918).

Enter the following item number: 18280000.

The components excluded from the warranty (such as consumables or parts subject to wear and tear) are marked on the spare part list.

## Disposal instructions

**Within the European Union appliances with this symbol should not therefore be disposed of in unsorted domestic waste.**

For more information regarding proper disposal please apply at your local authorities.

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declare under our sole responsibility, that the product(s)

Name	Part No.
Twister	1826 0000 / 1826 0000 03
Twister venturi	1827 0000 / 1827 0000 03
Twister evolution	1828 0000 / 1828 0000 03

is (are) in conformity with the relevant regulatory requirements by compliance with the UK designated standards.

## UK legislation

*Electromagnetic Compatibility Regulations 2016*

*Supply of Machinery (Safety) Regulations 2008*

*The Restriction of the Use of Certain Hazardous Substances in Electrical and Electronic Equipment Regulations 2012*

## Designated standards

EN 61326-1:2013

EN ISO 12100:2010

EN 61010-1:2010/A1:2019/AC:2019-04

EN IEC 61010-2-051:2021/A11:2021

EN IEC 63000:2018

Tilo Burgbacher,  
Engineering Director

Hilzingen, 02.11.2022

The Technical Documentation has been retained by Renfert GmbH

# Technical Data

	<b>Twister evolution (Nr. 1828)</b>
Mains voltage:	100 - 240 V, 50 / 60 Hz
Power consumption:	180 VA
Device fuse:	T4AL, 250 VAC
Speed:	100 - 450 1/min
LpA *) (idling):	< 70 db(A)
Dimensions (W x H x D):	152 x 285 x 235 mm [6.0 x 11.2 x 9.3 inch]
Weight (without bowl) approx:	5,2 kg [11.5 lbs]

\*) Sound pressure level in accord. with EN ISO 11202

**Hochaktuell und ausführlich auf ...  
Up to date and in detail at ...  
Actualisé et détaillé sous ...  
Aggiornato e dettagliato su ...  
La máxima actualidad y detalle en ...  
Актуально и подробно на ...**

**[www.renfert.com](http://www.renfert.com)**

Renfert GmbH • Untere Gießwiesen 2 • 78247 Hilzingen/Germany  
Tel.: +49 7731 82 08-0 • Fax: +49 7731 82 08-70  
[www.renfert.com](http://www.renfert.com) • [info@renfert.com](mailto:info@renfert.com)

Renfert USA • 3718 Illinois Avenue • St. Charles IL 60174/USA  
Tel.: +1 6307 62 18 03 • Fax: +1 6307 62 97 87  
[www.renfert.com](http://www.renfert.com) • [info@renfertusa.com](mailto:info@renfertusa.com)  
USA: Free call 800 336 7422