making work easy

- de
- en
- fr
- it
- es
- pt
- tr
- ru
- zh
- ja
- ko
- ar

SILENT powerCAM EC

Quick Start Guide

Made in Germany

21-6845 20092024

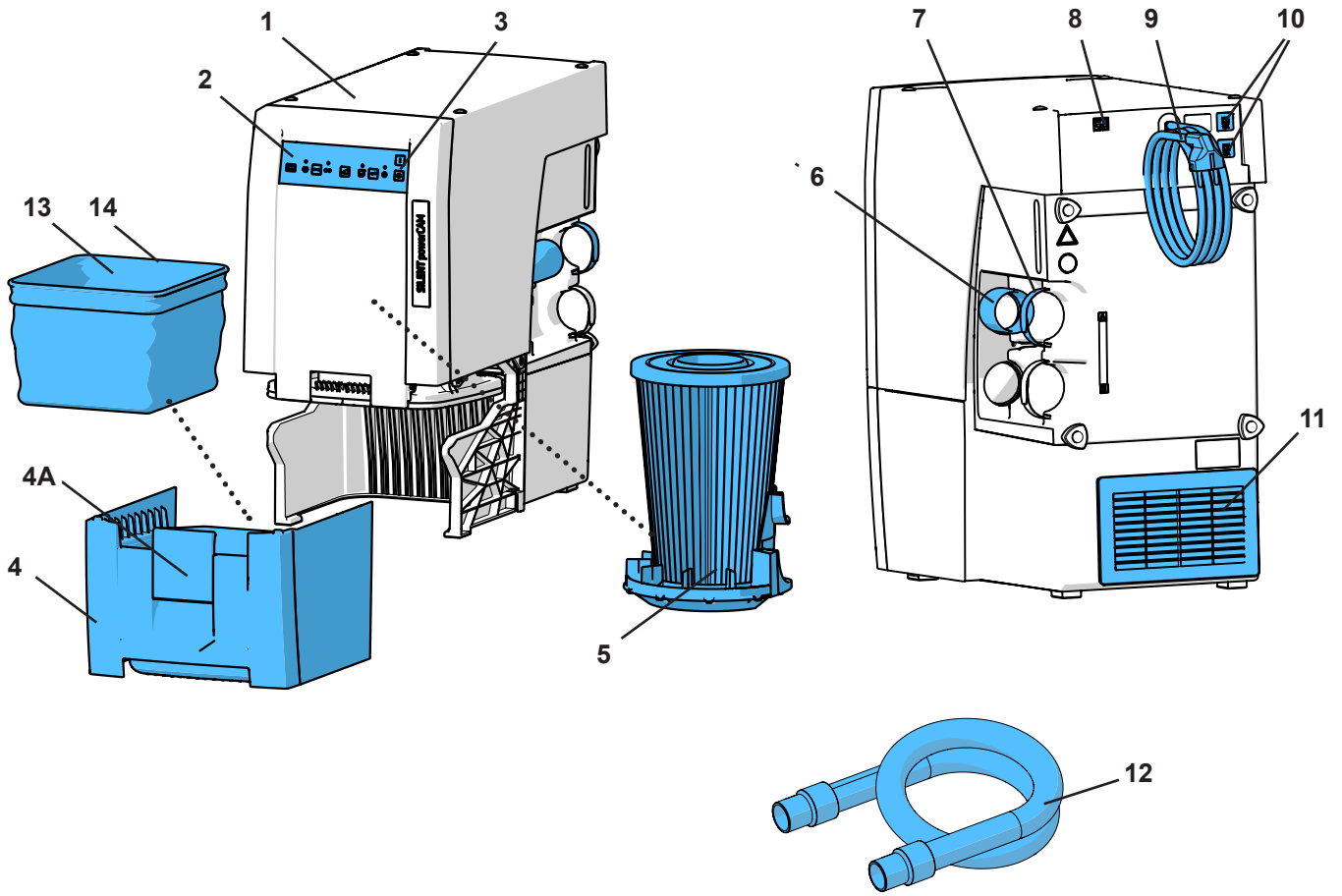
Fig. 1



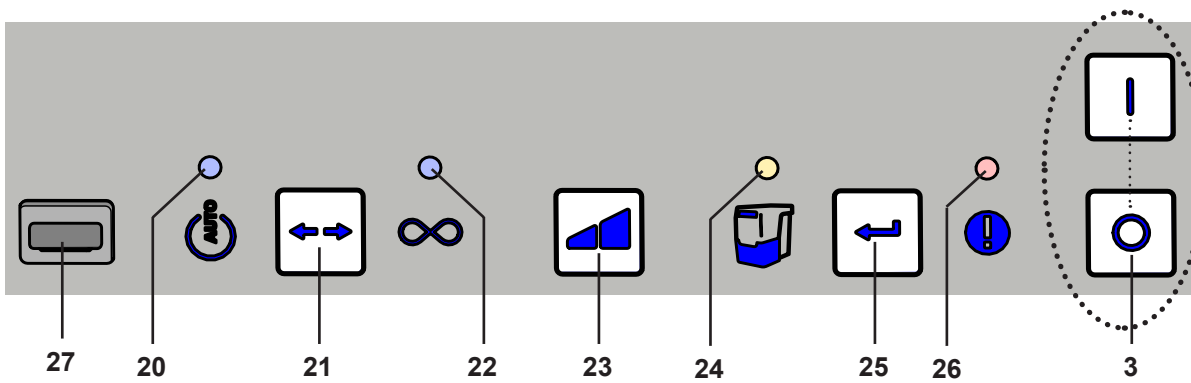
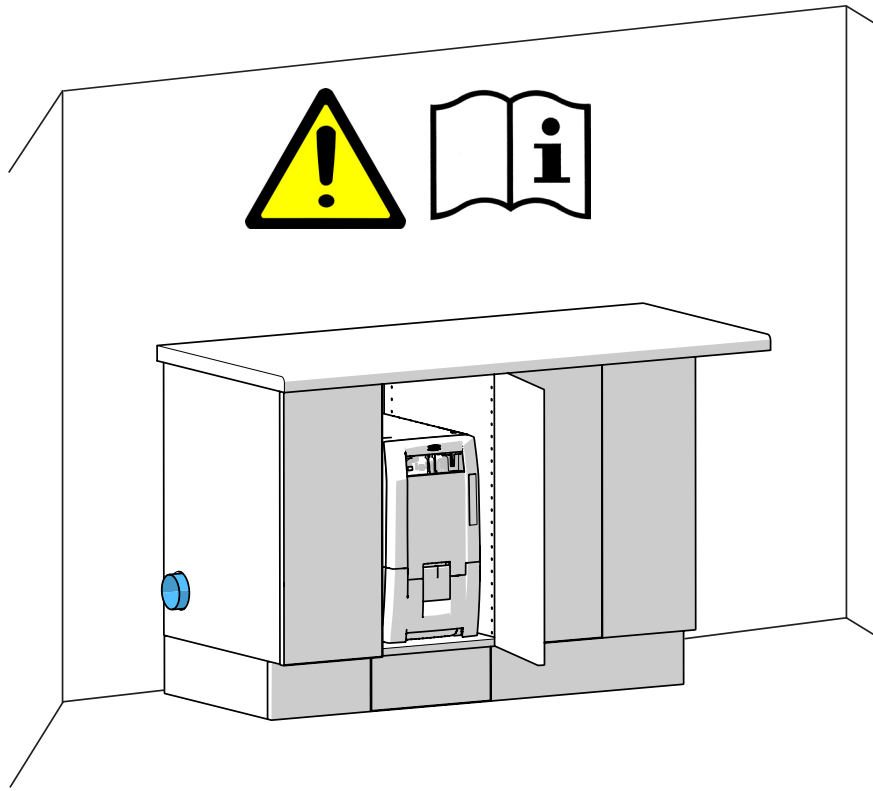
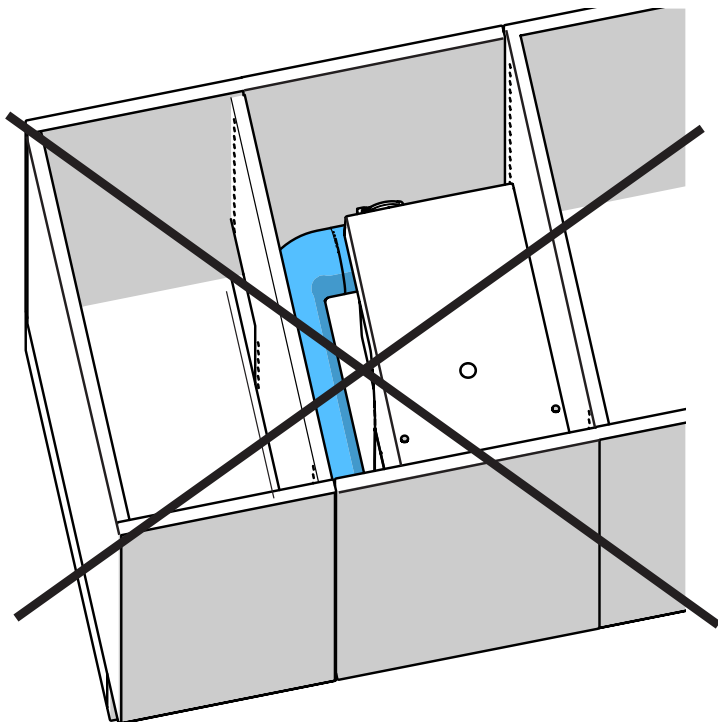
Fig. 2

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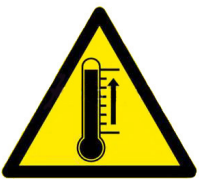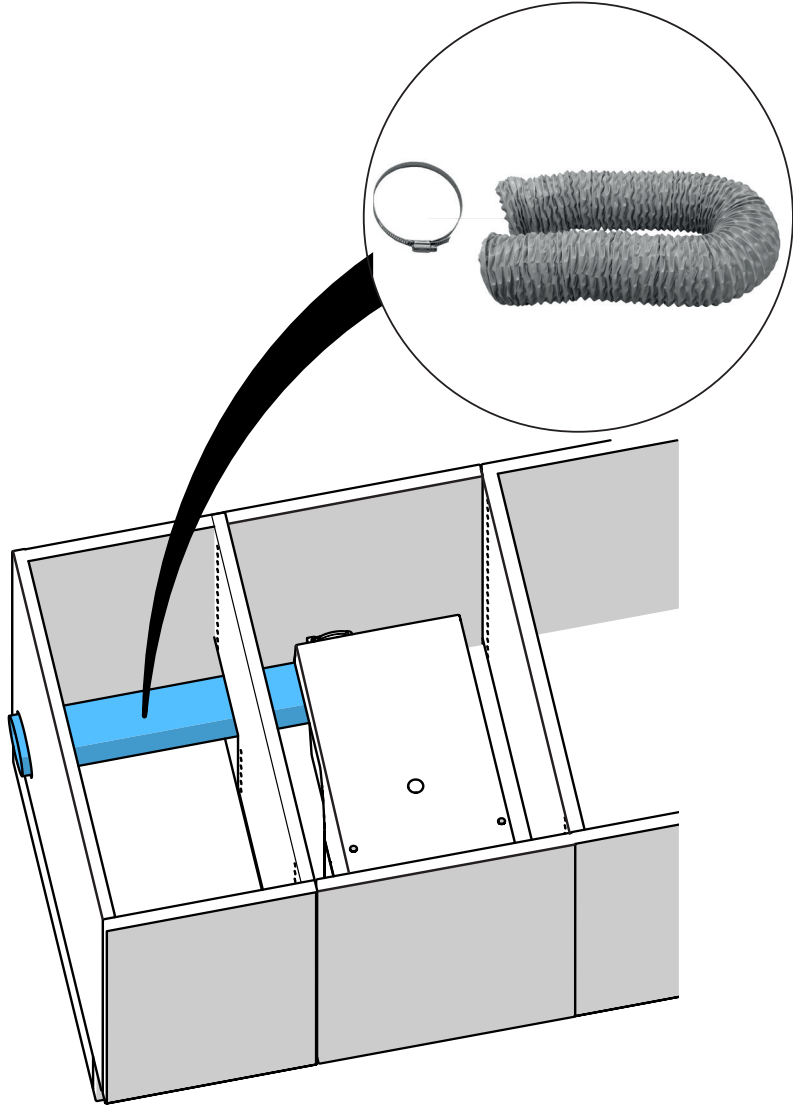Introduction

This Quick Start Guide contains instructions on the operation of the product and relevant safety information.

Detailed information on the product can be found in the operating instructions which are available for download on our website at www.renfert.com/p915.

Enter the following item number: 29390000.

Symbols

In the instructions for use and on the unit itself you will find these symbols with the following meanings:

Danger

This indicates a direct risk of injury. Consult accompanying documents!

Electrical current

This indicates a risk of hazard due to an electrical current.

Attention

Disregarding this warning may result in damage to equipment.

Note

This provides the operator with useful information to improve and ease use.

The device complies with the requirements of the applicable EU directives.

This product complies with the relevant UK legislation.

See UKCA Declaration of Conformity in the Internet under www.renfert.com.

The device is subject to the EU directive 2002/96/EG (WEEE directive).

Other symbols are explained as they occur.

Safety

Intended Use

This extraction unit is intended for use with dental milling machines for the suction of dry dust, generated whilst milling commercially available milling blocks and blanks.

The device is intended exclusively for commercial use.

The intended use also includes compliance with the instructions specified by the manufacturer concerning operation, servicing and maintenance.

Improper Use

Fire-promoting, easily flammable, hot, burning or explosive materials must not be suctioned into the device. It is not permitted to suction liquids.

This device is not intended for private, household use.

Any use other than specified in these instructions is deemed improper and constitutes a misuse of the device.

The manufacturer shall not be liable for damages caused by improper use.

Only spare parts and accessories supplied or authorized by Renfert GmbH may be used with this product. If other spare parts or accessories are used, this could have a detrimental effect on the safety of the device, increase the risk of serious injury and lead to damage to the environment or the device itself.

Hazard and Warning Information

General Information

- ▶ If the device is not used in compliance with the supplied instructions, the safety of the device can no longer be guaranteed.
- ▶ The device may only be operated using a mains cable with the country-specific plug system. Any necessary alterations must be carried out by a qualified electrician.
- ▶ The device may only be operated if the information on the identification plate conforms to the specifications of your local mains power supply. After removing the Dust Drawer, you can find the identification plate below, inside the device on the left side.
- ▶ The device may only be plugged into outlets which are connected to the protective conductor system.

- ▶ The mains plug must be easily accessible.
- ▶ Disconnect the device from the mains before carrying out work on the electrical parts.
- ▶ Check connection cables (such as power supply cords), tubes and housing (i.e. the key-pad) regularly for damage (i.e. kinks, cracks and porosity) or signs of ageing. Devices with damaged connection cables, tubes or housing parts or other defects must not be operated!
- ▶ Defective devices must be put out of service immediately. Remove the mains plug and ensure the device is not used. Send the device for repair!
- ▶ Please observe the national accident prevention regulations!
- ▶ It is the responsibility of the operator that national regulations during operation and regarding a repeated safety inspection of electrical equipment are complied with. For Germany these are the regulation 3 by DGUV (German Statutory Accident Insurance) in relation with VDE 0701-0702 (Association for Electrical, Electronic and Information Technology).
- ▶ Information on REACH and SVHC is available on our website www.renfert.com, in the Support area.

Specific Information

- ▶ When connecting to a CAM system, please observe the CAM device instructions for use and comply with their safety precautions.
- ▶ Please observe the national regulations and permitted exposure to dust in a working environment. Please ask the “National Institute for Occupational Safety and Health” or other responsible authority.
- ▶ Please observe the Safety Data Sheets of the materials to be extracted.
- ▶ Always wear protective gear, when extracting hazardous materials.
- ▶ It is necessary to wear suitable personal protective equipment when emptying the dust drawer or cleaning, depending on the type of extracted material.
- ▶ When disposing of the extracted material or used filter, please observe the local specifications and accident prevention regulations!
- ▶ Make sure the dust drawer is fully closed during operation.
- ▶ Do not operate without a suction hose.
- ▶ Do not extract easily flammable, explosive gasses or fumes.

The following applications involve increased risks and are therefore not permitted:

When cleaning milling chambers and SLM printing systems by suction, a sufficiently high concentration and purity (i.e. unmixed with other dental dusts such as gypsum, resins) can result in spontaneous ignition or the milling dust exploding or printing powder because of a potential exothermic reaction (e.g. due to oxidation).

An alternative cleaning method must be selected for the following milling or printing media in particular (such as manual cleaning):

- Wood
- Titanium / titanium-aluminum
- Light metals and light metal alloys (e.g. aluminum, magnesium)
- Cobalt chrome powder (e.g. for use in SLM systems)

If large quantities of light metals such as titanium alloy are processed (e.g. with sandpaper), producing very fine grinding dust, a sufficiently high concentration and purity may result in spontaneous combustion because of a potential exothermic reaction.

- ▶ Do not extract hot materials.
- ▶ Do not extract liquids.
- ▶ If the extraction unit is used to suction hazardous materials, appropriate personal protective gear must be worn and steps must be taken to ensure that the exhaust air is properly ventilated. Please refer to the associated safety data sheets for specific requirements.
- ▶ Dispose of extracted material according to local statutory regulations.

Authorized Persons

Operation and maintenance of the device may only be performed by qualified personnel.

Product Description

Components Groups and Functional Elements

See Fig. 1 (on inside cover)

- | | | | |
|----|--------------------|----|---|
| 1 | SILENT powerCAM EC | 8 | CAM interface |
| 2 | Key-pad | 9 | Mains cable |
| 3 | On / Off switch | 10 | Device protection switch (2 x) |
| 4 | Dust drawer | 11 | Exhaust air filter / exhaust air outlet |
| 4A | Clamp fastener | 12 | Suction hose with end bushings |
| 5 | Fine filter | 13 | Disposal bag |
| 6 | Suction port | 14 | Holder |
| 7 | Velcro® strip | | |

See Fig. 2 (on inside cover)

- | | | | |
|----|--|----|--|
| 3 | On / Off switch | 24 | Display empty suction drawer |
| 20 | Display CAM-mode | 25 | Enter key, save input |
| 21 | Operating mode key, CAM-mode /
continuous operation | 26 | Display error message |
| 22 | Display continuous operation | 27 | Cover (do not remove, service interface) |
| 23 | Extraction force -key | | |

Setting Up

Setup

The extraction unit is a free standing appliance which must not be operated in a lying position.

If the extraction unit is kept in a closed cupboard, the warm exhaust air must be allowed to escape using one of the methods, shown on the pictures A, B, C at the beginning of this document.

A: External ventilation duct (see accessories at www.renfert.com).

B: Observe the opening in the cabinet, specified measurements, minimum and maximum distances.

C: Remove the back side of the cabinet, the distance from the back side of the cabinet to the wall min. 50 mm.

Connect

Warning risk of injury!

When shortening the suction tube, please be sure to cut the integrated wire as straight as possible.

Long suction tubes, tight bends and kinks greatly reduce the suction power at the suction point.

⇒ If necessary, shorten the suction hose.

⇒ Open the Velcro® strip (7).

⇒ Place the suction hose with the end bushing (12) into the suction port (6).

⇒ Fasten the suction hose with the Velcro® strip.

⇒ Connect the suction hose to the suction port on the CAM device.

If the diameter size does not correspond, please use an adapter (see accessories) to prevent a loss in suction performance.

⇒ Insert the interface cable to the CAM system in the CAM interface (8).

To connect the CAM device to the interface socket on the extraction unit (8), please see the interface specifications in the complete instructions for use in the Internet and check the CAM device instructions.

Only ever use the CAM interface which is provided for remote control of the extraction unit. For CAM systems which only provide a switched mains voltage, it is essential to use “interface cable type F”. If the suction is frequently switched on or off via the CAM power supply, it may cause electronic damage to the CAM system as well as to the SILENT compactCAM.

⇒ Switch the device off at the on / off switch (3).

⇒ Unroll the mains cable (9) and insert the plug into the building's electrical socket.

Operation

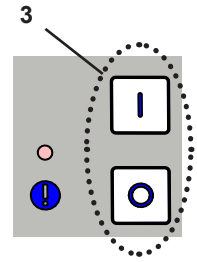
Switching On

The suction unit is switched on and off via the on / off switch (3).

After activation:

- ◆ The suction unit performs an automatic filter cleaning sequence (for approx. 8 seconds, loud vibrating noise).

Then the unit returns to the last set operating mode.

Select Mode: CAM-mode / Continuous Operation

- CAM-mode (20):

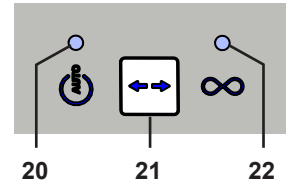
The extraction unit is controlled by the signals provided by the CAM device.

- Continuous operation (22):

The extraction unit runs continuously.

⇒ Press the operating mode key (21).

- ◆ To change the operating mode.

The extraction unit is for use with dry dust only!

When connected to a CAM device with wet and dry function, ensure that no residual moisture from the CAM device enters the extraction unit.

Check that the extraction unit is in full working order for every milling procedure:

- ▶ **Display error (26) is off.**
- ▶ **Suction turbine operates correctly without producing any peculiar noise or odor.**

Extraction force

⇒ Press the extraction force key (23).

- ◆ Switch to another level (only in continuous operation mode).

Setting the Parameter

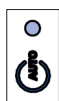
On delivery, there are parameters set, which in most cases, make it possible to work with the suction unit without problems. Only in individual cases, if this is not possible, the parameters can be changed.

The setting of various parameters and the execution of a self-diagnosis are carried out in the programming mode.

Information on how to set the parameters and perform a self-diagnosis can be found in the operating instructions in the Internet or in Figs. D and E at the start of this manual.

The four displays show which parameter is set:

Display CAM-mode (20)

Carry out self-diagnosis

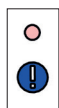
Display continual operation (22)

Audible signal (Buzzer) on / off

Display empty dust drawer (24)

Time interval for full display of the dust drawer


Display error message (26)

Adjust the after-run time of the suction turbine

In order to set the various different parameters, the programming mode must be started and the parameter must be selected.

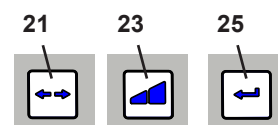- Start programming mode: Press operating mode key (21) for 3 seconds.
- Select the parameter: Press the Operating mode key (21), the corresponding display blinks, the other displays are on.
- Change values: Press the extraction force keys (23).
- Save settings: Press the enter key (25). An audible signal confirms successful saving.
- Exit programming (no changes): Press operating mode key (21) again.

Self-diagnosis (see Fig. D at the start of this manual)

Start the self-diagnosis manually. This is performed once.

Acoustic signal (see Fig. D at the start of this manual)

When switched on, various settings will be confirmed with an acoustic signal. The audible signal can be switched on and off.

Time Interval for “Full” Display Dust Drawer (see Fig. E at the start of this manual)

After reaching a set time interval, a request appears to empty the dust drawer. You can select between 5 times.

Time interval / hrs	Blink signal in the display 
2	1 x
5	2 x
10	3 x
50	4 x
100	5 x

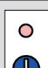
Factory setting

i *With devices which generate large amounts of dust, (e.g. sandblasting units) the time interval “empty dust drawer” should be set to 5 hours, if necessary, to 10 hours.*

Suction turbine after-run time (see Fig. E at the start of this manual)

After-run time = The period of time between the OFF command in the CAM system and moment the suction turbine actually stops functioning.

It is possible to select between 6 different times.

After-run time / Secs.	Blink signal in the display 
0	1 x
3	2 x
5	3 x
10	4 x
20	5 x
30	6 x

Factory setting

i *In order to prevent the suction unit from switching on/off too frequently in the event of short interruptions, it is advisable to select a higher time.*

Cleaning / Maintenance

Inside the appliance there are no parts which require maintenance.
Opening the device, other than for the processes described below, is not permissible!

Cleaning

Use a damp cloth to clean the outside of the unit.
Do not use abrasive or solvent-based cleaning agents.

Empty Dust Drawer

After having reached the previously set time interval a request appears to empty the dust drawer.

- ◆ An audible signal will be emitted 3 x.
- ◆ The display dust drawer (24) lights up.

Before removing the dust drawer, a filter cleaning procedure should be carried out.

⇒ Press the enter key (25) for 2 seconds.

- ◆ The filter cleaning procedure is carried out (duration approx. 8 seconds).

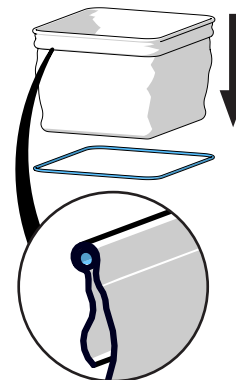
After the filter cleaning has finished:

- ⇒ Unfasten (4A) the dust drawer.
- ⇒ Pull the dust drawer forwards (4).
- ⇒ Remove the holder (14).
- ⇒ Close the disposal bag, remove and dispose of.

Observe the local waste regulation and accident prevention regulations when disposing of the product! Wear the appropriate personal protection equipment according to the type of dust.

- ⇒ Insert a new disposal bag through the holder and fold the ends over the holder ring.
- ⇒ Place the disposal bag with the holder into the dust drawer. The holder ring is fixed in place in the dust drawer with magnets. Ensure that the disposal bag:
 - has contact with the side walls of the dust drawer;
 - does not have contact with the seal surface.
- ⇒ Replace the dust drawer and slide in completely.
- ⇒ Close the fastening (4A).
- ⇒ Press the enter key (25) (the time interval counter is re-set).
 - ◆ An audible signal confirms the drawer has been replaced.
 - ◆ The dust drawer display (24) goes off.

If the dust drawer is not emptied, the dust drawer display (24) remains on. After switching the unit off and on again, a 3 x audible signal indicates that the dust drawer must be emptied.

In case of heavy materials (for ex. zirconium dust) we recommend to use 2 disposal bags for a safe removal and a safe transport (bag in bag solution).

To set the time interval, see Fig. E at the start of this manual, or find information in the instructions for use in the Internet.

Change fine filter

If the filter cleaning does is not sufficient (suction performance does not improve), then the fine filter needs to be changed.

This is detected by the suction unit and indicated by the error display (26).

If a change of the fine filter is requested or the extraction force does not substantially improve or only improves for a short time in spite of repeated cleaning, the fine filter must be changed straight away.

Further operation may cause damage to the unit.

Carry out the automatic filter cleaning process before changing the fine filter.

- ▶ **In general, the fine filter should be changed every 2 years.**

When the new fine filter is inserted, check that it is fitted correctly as otherwise leaks may occur. See the assembly instruction at the end of the instruction manual, which is also attached to the new fine filter.

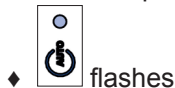
Self-diagnosis

The self-diagnosis helps the control system to check the function of the turbine and electronic parts.

If no fault is detected, the suction is ready for operation after the self-diagnosis has finished. If an error is detected, this is indicated with a signal.

The self-diagnosis is started manually and is carried out once.

⇒ Press the operating mode key (21) 3 seconds.



◆ flashes

⇒ Press the enter key (25).

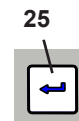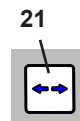◆ Blinks: self-diagnosis is carried out.

No error detected:

◆ Suction is on standby.

Error detected:

◆ An audible signal can be heard.

◆ All displays are alight.

⇒ For troubleshooting, see the complete instructions in the Internet.

Spare Parts

You can find components subject to wear and the spare parts on the spare part list in the internet at www.renfert.com/p918.

Enter the following item number: 29390000.

The components excluded from the warranty (such as consumables or parts subject to wear and tear) are marked on the spare part list.

Disposal Information

Within the European Union, units with this symbol should not therefore be disposed of in unsorted domestic waste.

Please contact your local authorities for more information on proper disposal.

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declare under our sole responsibility, that the product(s)

Name	Part No.
SILENT TS	2921 0050
SILENT TS2	2930 0050
SILENT TC	2935 0000 / 2935 0000 03
SILENT TC2	2936 0000 / 2936 0000 03
SILENT EC2	2937 0000 / 2937 0000 03
SILENT compact	2934 0000 / 2934 0000 03
SILENT compactCAM	2934 2000 / 2934 2000 03 2934 2500 / 2934 2500 03
SILENT powerCAM EC	2939 0000
SILENT powerCAM TC	2938 0000
iVAC silent	36 2934 2000

is (are) in conformity with the relevant regulatory requirements by compliance with the UK designated standards.

UK legislation

Electromagnetic Compatibility Regulations 2016

Supply of Machinery (Safety) Regulations 2008

The Restriction of the Use of Certain Hazardous Substances in Electrical and Electronic Equipment Regulations 2012

Designated standards

EN 61326-1:2013

EN ISO12100:2010

EN 61010-1:2010/A1:2019/AC:2019-04

EN IEC 63000:2018

Tilo Burgbacher,
Engineering Director

Hilzingen, 02.11.2022

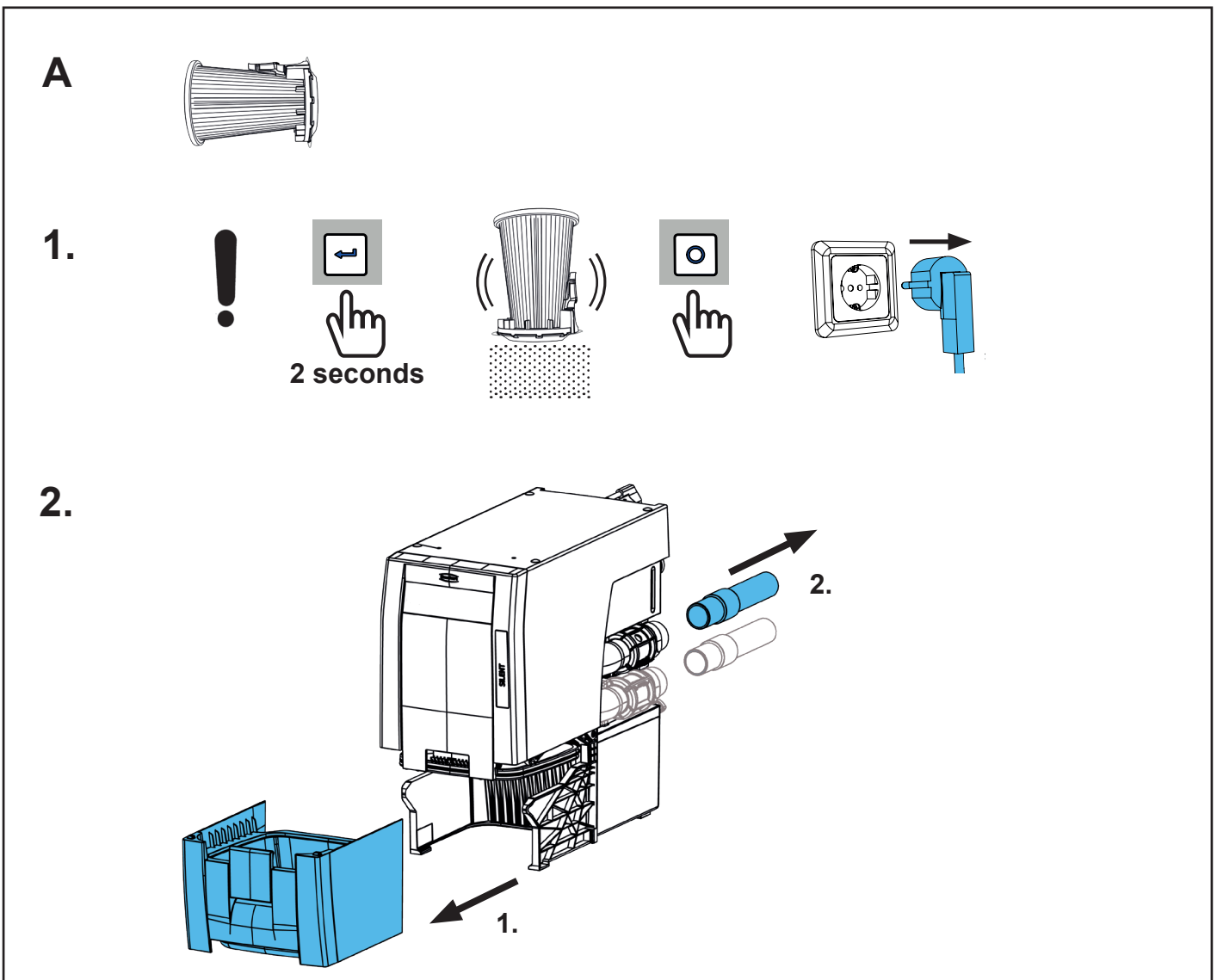
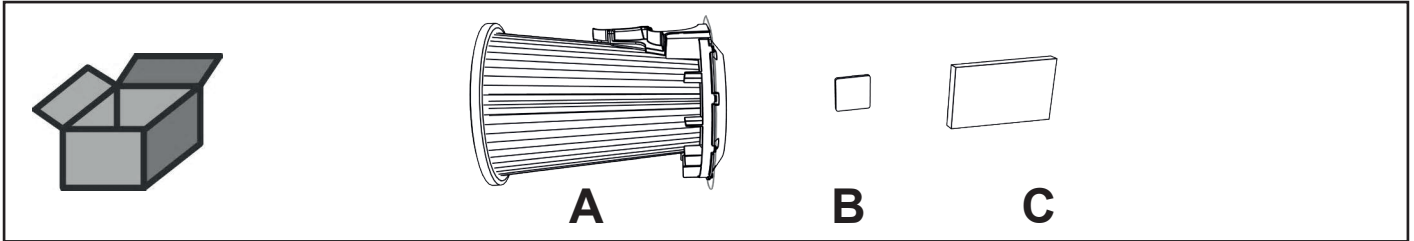
The Technical Documentation has been retained by Renfert GmbH


making work easy

90002 1584



Feinfilter SILENT EC • Fine filter SILENT EC • Filtre fin SILENT EC
Filtro fino SILENT EC • Filtro fino SILENT EC • Фильтр тонкой очистки SILENT
EC

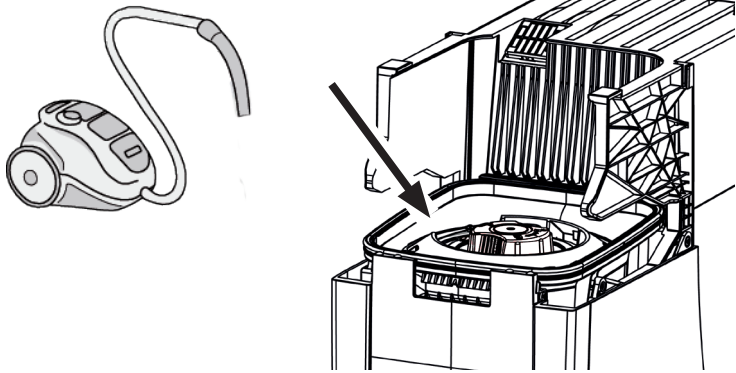
3.

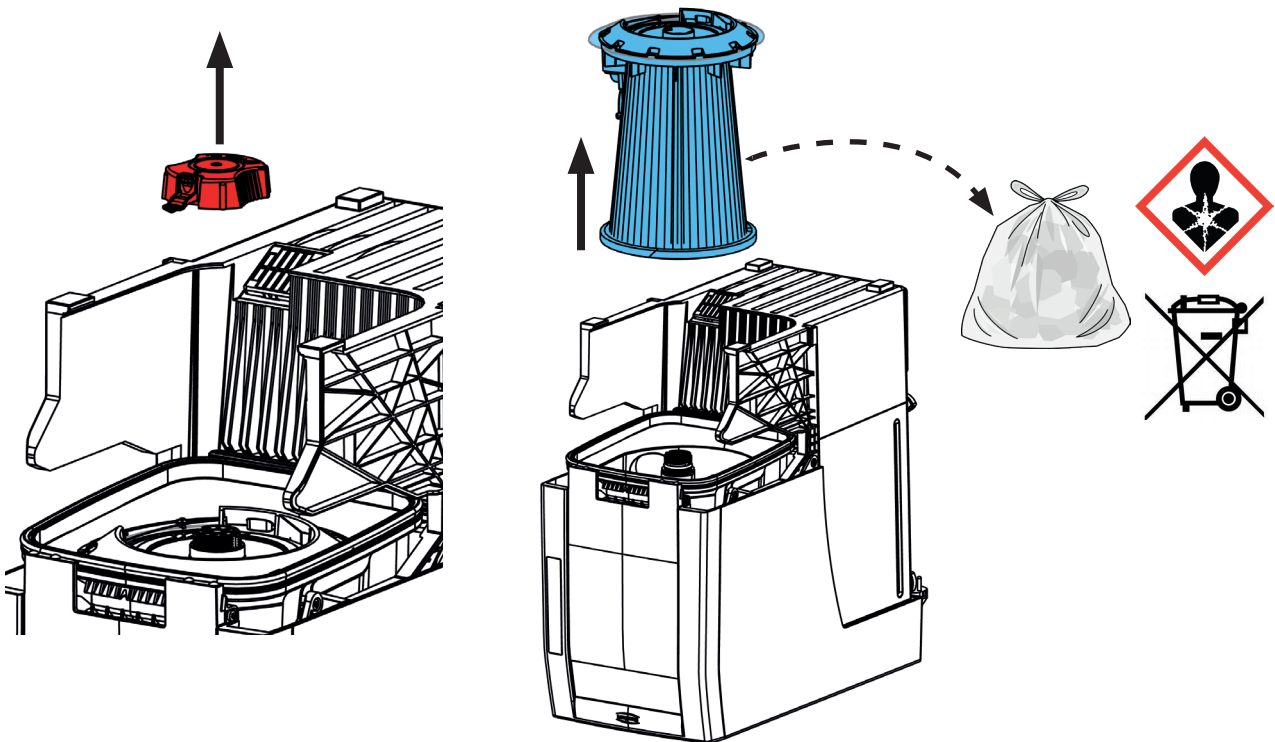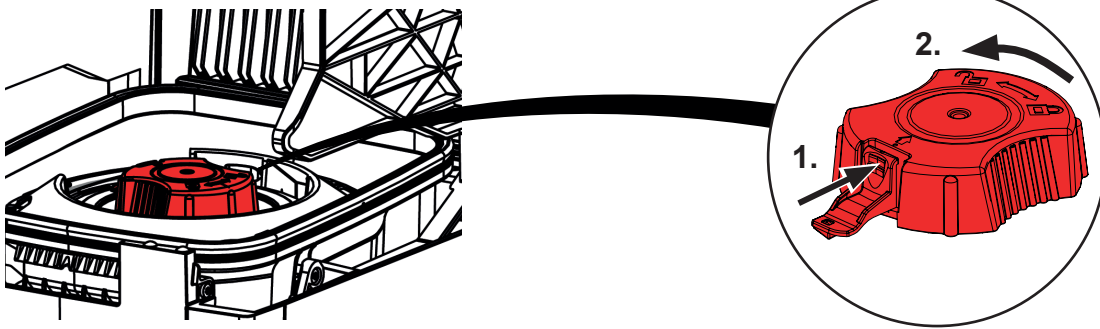
4.

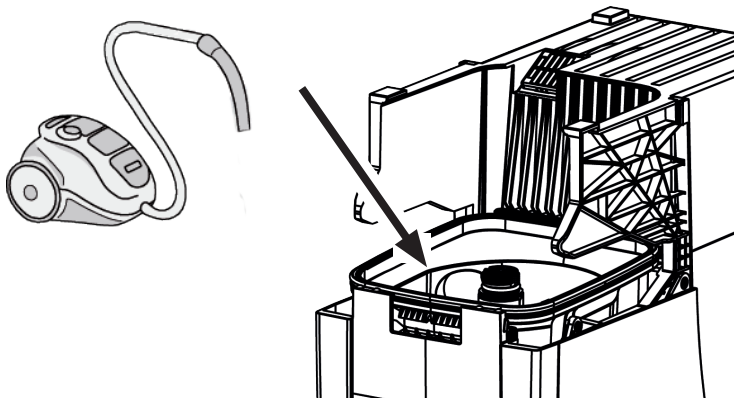
5.

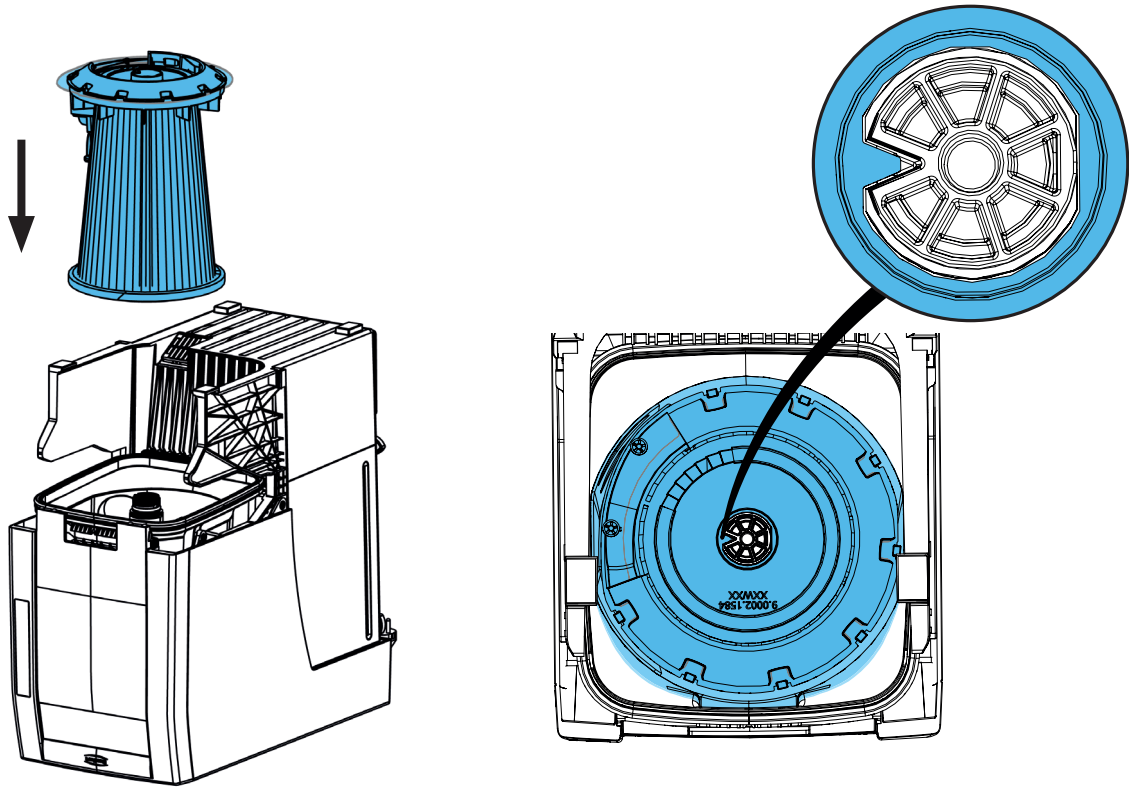6.

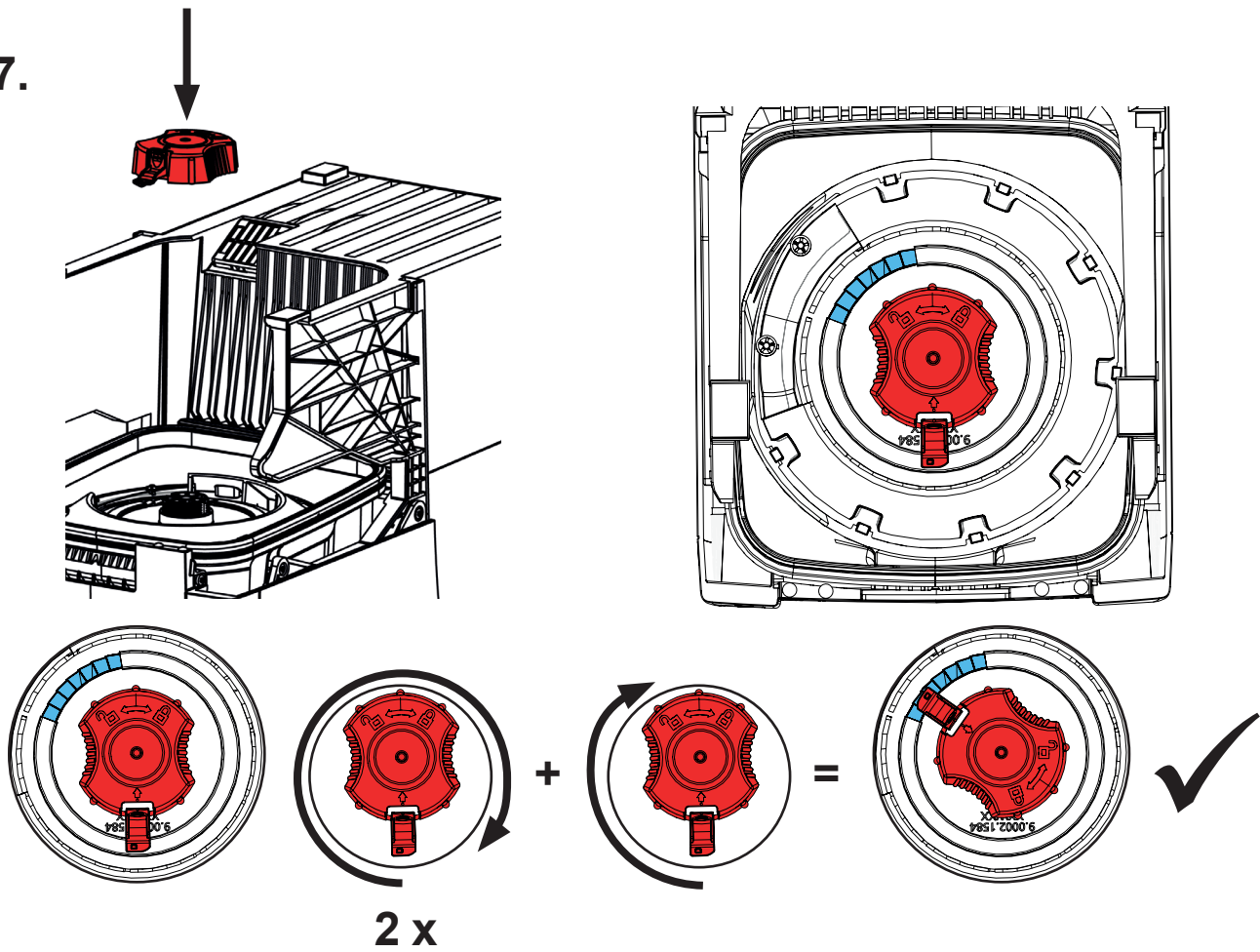7.


B


8.

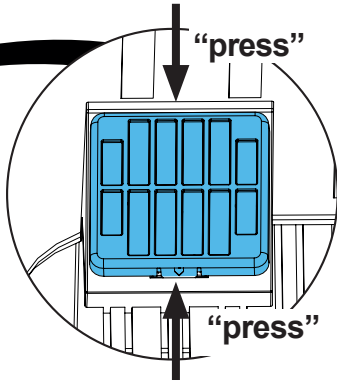
1.

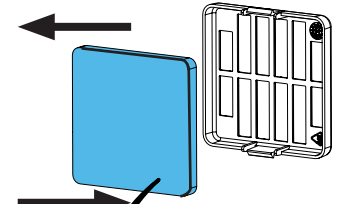2.

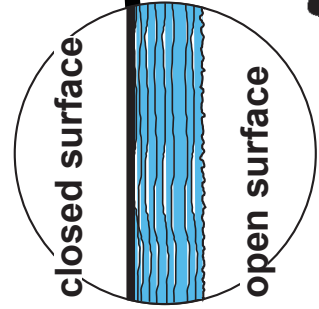
3.



i


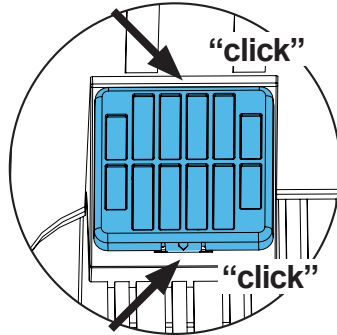
4.

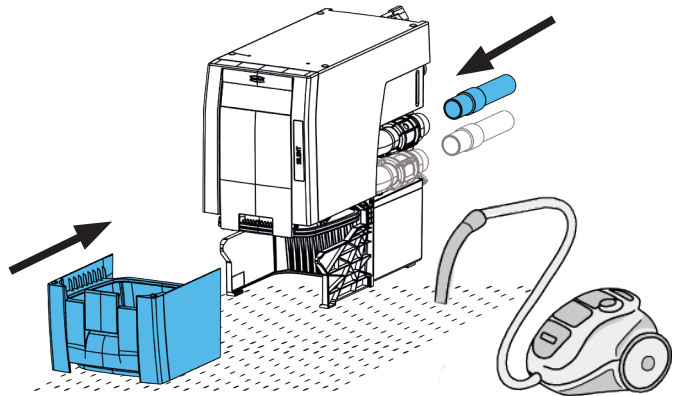
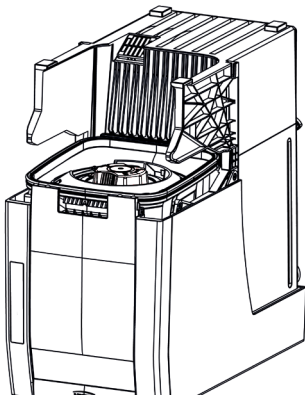
9.


C

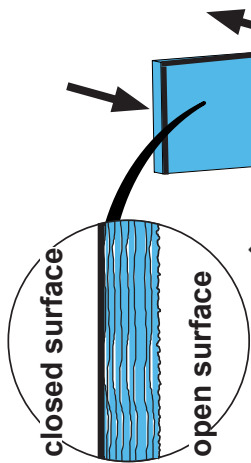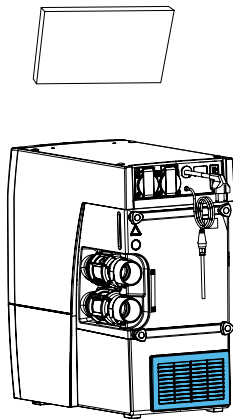
i

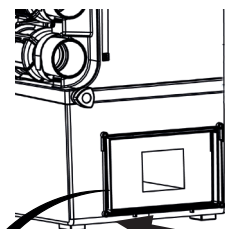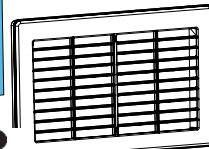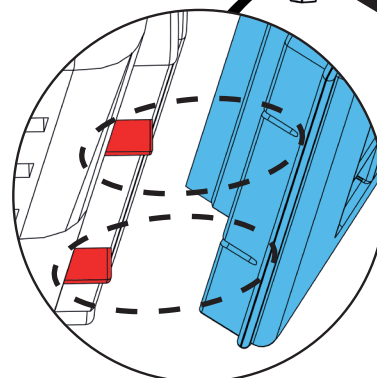1.

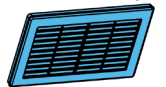2.

Technical Data

	SILENT powerCAM EC		
	2939 0000	2939 1000	2939 2000
Nominal Voltage:	230 V	120 V	100 V
Permissible mains voltage:	220 - 240 V	120 V	100 V
Mains frequency:	50 / 60 Hz	50 / 60 Hz	50 / 60 Hz
Suction turbine power *):	1610 VA	1440 VA	1250 VA
Network input protection:	2 x 15 A (T)		
LpA **) (at max. volume flow):	54,3 dB(A)		
Ø suction fitting: - inside	42 mm [1.65 inch]		
Flow rate, max. ***):	3984 l/min [2.34 ft ³ /s]		
Negative pressure, max. ****):	257 hPa [3.73 psi]	262 hPa [3.80 psi]	255 hPa [3.70 psi]
Fine filter: - Filter surface, approx. - Filter quality	0.9 m ² [1390 sq inch] Class M according to EN 60335-2-69		
Filling volume dust drawer approx.:	7 l [1.85 US gal]		
Weight (empty), approx.:	18.0 kg [39.7 lbs]		
Dimensions (width x height x depth):	270 x 535 x 540 mm [10.6 x 21.1 x 21.2 inch]		
CAM interface:	RJ 45 socket		
Input signal: - Low - High	0 - 5 V 15 - 24 V		
Output signal: - max. pull-up voltage - max. switching current (I) - internal series resistor	24 V 5 mA 150 Ohm		

- *) Power consumption at nominal voltage
 **) Sound pressure levels according to EN ISO 11202
 ***) Free-blowing turbine at nominal turbine voltage
 ****) At nominal turbine voltage

**Hochaktuell und ausführlich auf ...
Up to date and in detail at ...
Actualisé et détaillé sous ...
Aggiornato e dettagliato su ...
La máxima actualidad y detalle en ...
Актуально и подробно на ...**

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Renfert GmbH • Untere Gießwiesen 2 • 78247 Hilzingen/Germany