making work easy

 lay:art

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EN

Instruction Manual

Congratulations, and thank you for choosing the high quality and state-of-the-art porcelain mixing palette system „lay:art“ by Renfert.

1. Description

The system consists of a high quality plastic case with a lid and a glass mixing palette.

1.1 Components and functional elements

- | | |
|------------------|--|
| 1 Base | 6 4 x Rest plugs (tall) |
| 2 Lid | 7 Non-permanent marker |
| 3 Mixing palette | 8 4 x Moistening strips |
| 4 Labelling foil | 9 Tool compartment
(only crystal aqua xl) |
| 5 4 x Case feet | |

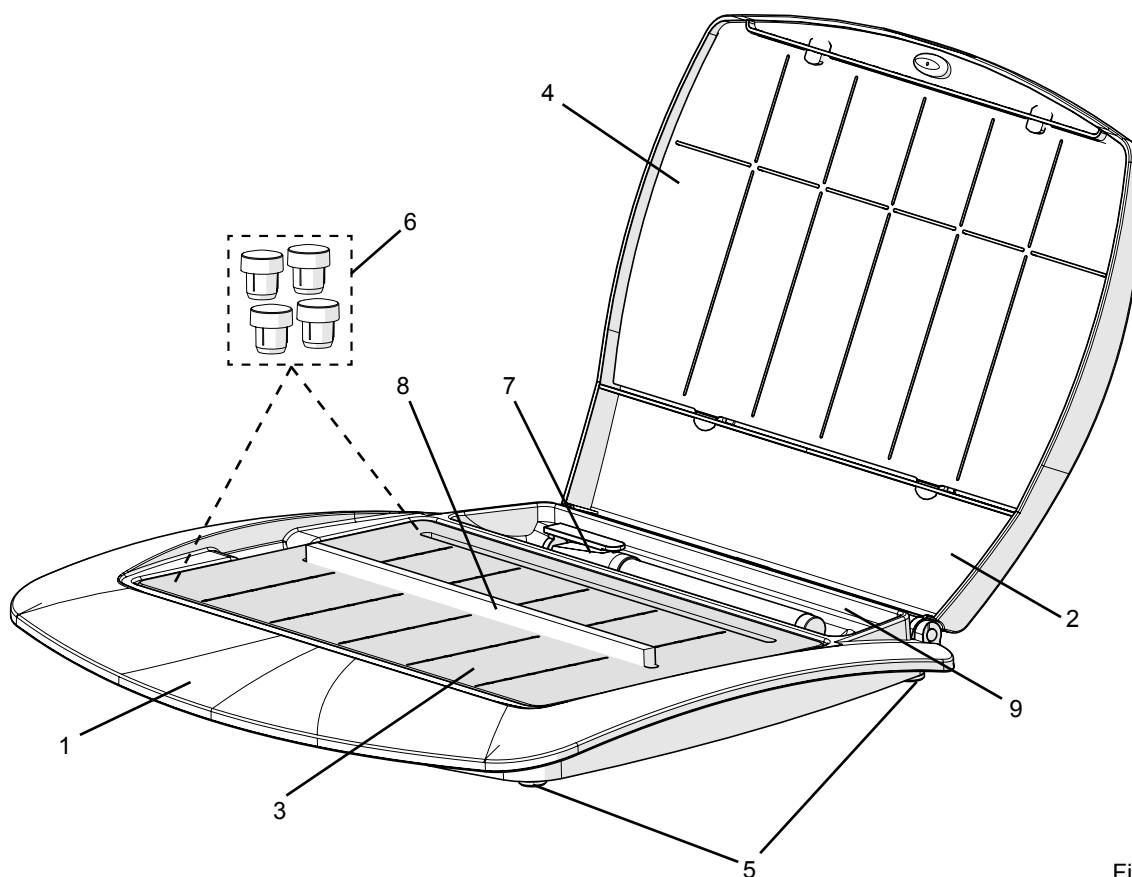
Fig. 1

2. Functional description of the case

When the lid is opened in a 90° angle it can be easily removed from the case base.

Gently press the onto the base (1) and pull the lid (2) upwards.

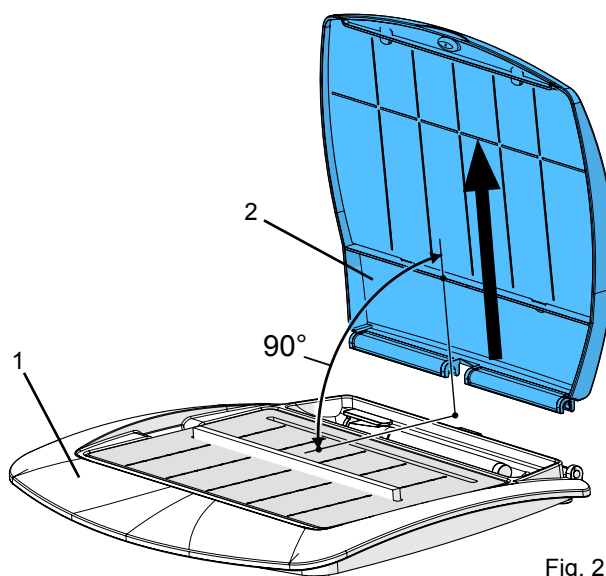
Fig. 2

At an angle of 115° there is a lid locking position, which will hold the lid open in a stable position.

Caution:

If this position is overstretched, the hinges could become damaged.

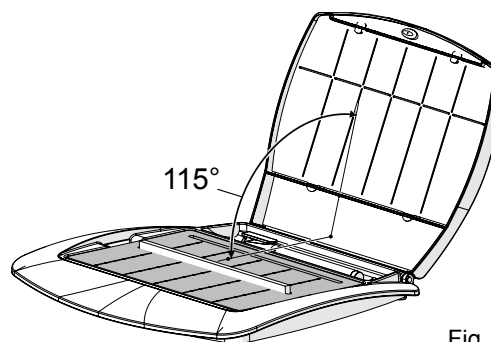
Fig. 3

3. Labelling foil

The non-permanent marker pen provided (7) can be used to write on the labelling foil (4). The foil can be removed from the lid in order to exchange or inscribe the label. Use the non-reflective side of the foil to write on.

When replacing the foil, clip behind the 4 mounting points which can be found on the lid. Firstly, bend the retaining flaps towards the matt surface (a).

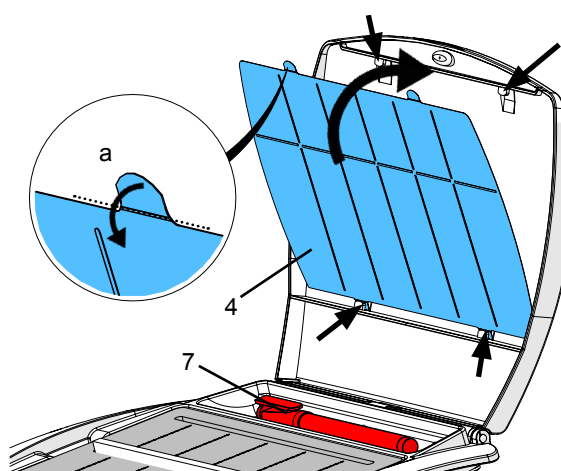
Fig. 4

4. Preparation for use

Lay the palette (3) into the case base with the unprinted side towards the top (1).

- Slide the moistening strips (8) down through the slot in the glass mixing palette until they have even contact with the case base. The strips should extend approx. 1 - 2 mm over the side of the glass palette.
- Fill **exclusively distilled water** into the holes at the side of the base. Tap water will have a detrimental effect on the function and will lead to calcification of the capillaries.
- The maximum fill level is approx. 2 mm below the upper edge of the glass mixing palette.

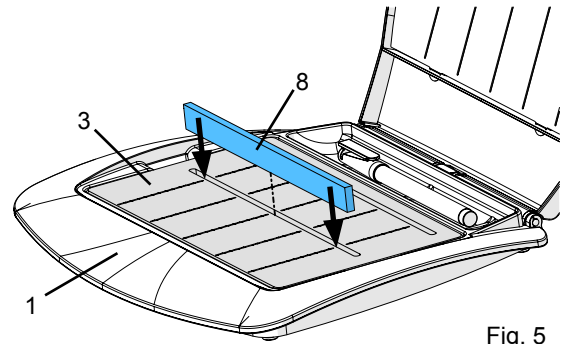
Fig. 5

Tip:

When using for the first time, in order for the strip to become moist quicker, swipe firmly over the upper edge of the strip after 5 minutes.

5. Use

Mixing with porcelain modelling liquid:

Lay the required amount of porcelain powder onto the glass surface. Mix the porcelain to the desired consistency by adding the modelling liquid. Push part of the mixed porcelain towards the moistening strips so that there is contact.

Mixing without modelling liquid:

Place the required amount of porcelain powder onto the glass surface. Push part of the porcelain towards the moistening strips. Saturate the porcelain with distilled water. You can control the saturation level by varying the size of the contact area between porcelain and moistening strip.

Even a small contact area will guarantee an optimal level of moisture for several days.

Note:

The water level will decrease in time, it is therefore necessary to refill distilled water into the case base occasionally.

The moistening strips can be re-used many times. These can also be cleaned under running water. After cleaning, squeeze out any moisture thoroughly and repeat point 4 to replace into the glass mixing palette. Depending on the amount of use, the moistening strips should be replaced from time to time.

6. Care instructions

In order to maintain the consistent quality of the mixing palette, it is essential to clean on a regular basis.

Take the glass mixing palette out of the case base and brush clean under running water or clean carefully using a steam cleaner.

If used inappropriately, the glass could break and cause a risk of injury.

Clean the case using conventional detergent and then rinse thoroughly under running water.

Do not clean using sharp edged, abrasive or solvent based materials. Please only use soft brushes or cleaning cloths.

When using Tricolor stains it may happen that vaporized stains fluid etches the plastic lid.

Do not clean the plastic lid with monomers or with liquids containing solvents!

7. Spare parts

1043 0100	Labelling foil crystal aqua xl, 3 pcs.
1043 0200	Labelling foil crystal aqua s, 3 pcs.
1043 1010	Moistening strips crystal aqua xl, 12 pcs., 18,3 cm / 7.2 inch
1043 2020	Moistening strips crystal aqua s ,12 pcs., 13,3 cm / 5.2 inch
9 1044 9991	Housing lay:art big
9 1045 9991	Housing lay:art small
9 0003 0021	Rest plugs tall lay:art, 4 pcs.
9 0003 0038	Case feet, 4 pcs.

7. REACH

Information on REACH and SVHC is available on our website www.renfert.com, in the Support area.

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Renfert GmbH • Untere Gießwiesen 2 • 78247 Hilzingen/Germany