making work easy


- de
- en
- fr
- it
- es
- pt
- tr
- ru
- zh
- ja
- ko
- ar

# POWER steamer water softener


Quick Start Guide

Made in Germany


21-6034 09062022

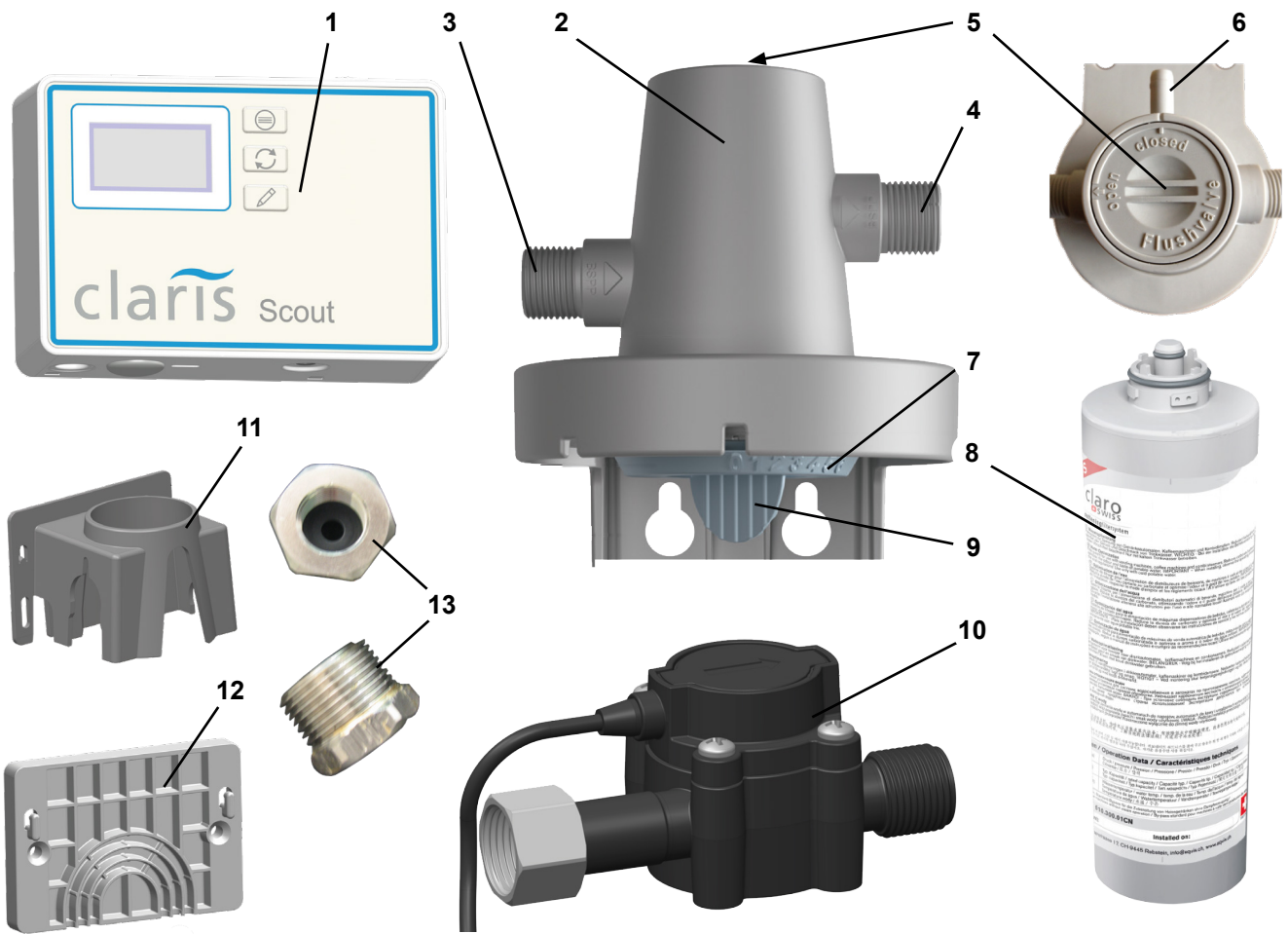**Fig. 1**

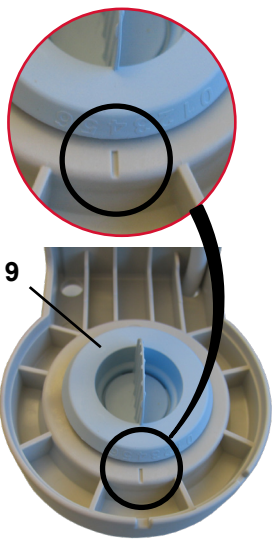
**Fig. 2**

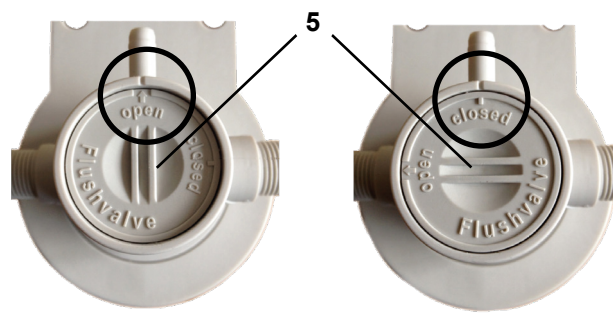**Fig. 3**

**Fig. 4**

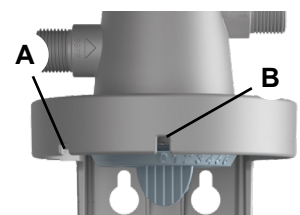**Fig. 5**

**Fig. 6**

# 1 Einleitung

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[www.renfert.com](http://www.renfert.com).


# 1 Introduction

This Quick Start Guide contains instructions on the operation of the unit and relevant safety information.

Detailed information about the unit is provided in the instructions for use available online at [www.renfert.com/p918](http://www.renfert.com/p918).

Enter the following item number: 18460100.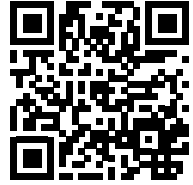

## 1.1 Symbols used

Symbols with the following meanings are used in these instructions or on the equipment:

### **Danger**

**Immediate danger of injury. Consult accompanying documents!**

### **Electrical voltage**

**Danger due to electrical voltage.**

### **Caution**

**Risk of damage to the equipment if the instruction is not followed.**

### **Notice**

**Provides the operator with useful information that eases use.**

**The equipment complies with applicable EU directives.**

**The equipment is subject to the provisions of the EU directive 2002/96/EC (WEEE directive).**

## 2 Safety

### 2.1 Intended use

The POWER steamer water softener is looped in between the water connection and the POWER steamer 2.

### 2.2 Improper use

Only spare parts and accessories supplied or authorized by Renfert GmbH may be used with this product. If other spare parts or accessories are used, this could have a detrimental effect on the safety of the equipment, increase the risk of serious injury and lead to damage to the environment or the product itself.

### 2.3 Ambient conditions for storage and transport

The following ambient conditions must be observed during storage and transport:

- Ambient temperature of -15 – +45 °C [-4 – +140 °F],
- Maximum relative humidity 80 %

### 2.4 Hazard and warning information

#### 2.4.1 General information

- ▶ If the unit is not used in accordance with the supplied operating instructions, the intended safety can no longer be guaranteed.
- ▶ Check connection cables (such as power cords), tubes and housing (such as the key pad) regularly for damage (such as kinks, cracks and porosity) or signs of ageing. Do not operate equipment with damaged connection cables, tubes or housing parts or other defects!
- ▶ Only operate the equipment under supervision.
- ▶ Please observe the national accident prevention regulations!
- ▶ Information on REACH and SVHC is available on our website at [www.renfert.com](http://www.renfert.com), in the Support area.

## 2.4.2 Specific information

- ▶ Only cold water of drinking water quality may be used to supply the system.
- ▶ When supplying with softened water, only use filters for drinking water with > 4° dH total hardness.
- ▶ Store components in a dry location.
- ▶ The system must not come into contact with chemicals, solvents, or any fumes.
- ▶ Before the filter system is put into operation, the consumer unit supplied must be free from limescale.
- ▶ No copper pipes or galvanized / nickel-plated pipes or connecting parts may be installed between the filter system and the consumer unit.
- ▶ The filter cartridge must not be opened or mechanically damaged.
- ▶ Water that is already microbiologically contaminated or of unknown microbiological quality may not be used under any circumstances without sufficient disinfection before or after the filter.
- ▶ The filter head and the wall bracket must be replaced after 5 years of use / at the latest 6 years after the production date. This also applies to hoses and seals. Pay attention to the date of the production stamp.
- ▶ All components must be installed in accordance with country-specific guidelines. DIN 1988 must be observed for the assembly and operation of the system.
- ▶ If the system pressure is greater than 8 bar, a pressure reducer must be installed upstream of the filter system.
- ▶ The sensor unit must not be exposed to any mechanical loads. Pay particular attention to the lever effect of hoses and bend radii. If necessary, provide support for supply lines!
- ▶ Only use commercially available AA batteries (2 x) for the sensor unit.
- ▶ Ensure that the battery cover of the sensor unit is properly closed. Otherwise, foreign objects or moisture could penetrate.

## 2.5 Authorized persons

Installation and maintenance of the filter systems may only be carried out by trained and authorized personnel.

Operation and maintenance of the equipment may only be performed by trained personnel.

Any repairs not described in these operating instructions may only be performed by a qualified electrician.

# 3 General description

## 3.1 Assemblies and functional elements

See Fig. 1 (on inside cover)

- |   |  |    |   |
|---|--|----|---|
| 1 | Programming and display unit           | 8  | Filter cartridge                              |
| 2 | Filter head with wall bracket          | 9  | Key for blending adjustment                   |
| 3 | Inlet                                  | 10 | Sensor unit                                   |
| 4 | Outlet                                 | 11 | Holder for programming and display unit       |
| 5 | Purge/relief valve                     | 12 | Wall bracket for programming and display unit |
| 6 | Flushing connection with flushing hose | 13 | Adapters (2 x)                                |
| 7 | Blending levels                        |    |   |

## 3.2 Scope of delivery

- 1 Filter head with flushing hose
- 1 Connection hose
- 2 Adapters incl. gaskets
- 1 Filter cartridge M for POWER steamer water softener
- 1 Flowmeter (items 1, 10, 11, 12) for POWER steamer water softener
- 1 Operating instructions
- Test strips for determining the water hardness

## 3.3 Accessories

Information on accessories and how to use them is provided in the instructions for use available online.

# 4 Startup

- ⇒ Remove the equipment and accessories from the package.
- ⇒ Check the delivery for completeness (compare with the scope of delivery).

**Required batteries (2x AA) are not included in the scope of delivery.**

## 4.1 Installing the filter system

**Only use the hoses and adapters provided.**

**A shut-off valve, e.g., a water faucet, must be installed upstream of the filter system.**

**The filter system can also be operated either free-standing or on its side without being secured. Vertical wall mounting is recommended.**

- ⇒ Before the filter system is put into operation, the consumer unit supplied must be free from limescale.
- ⇒ Determine a suitable location for installing the filter system:
  - The location of the system must be frost-proof and protected against direct sunlight.
  - There must be sufficient space under the filter head for installation of the filter cartridge.

### ► Determining carbonate hardness

- ⇒ Briefly immerse the test strips (1 sec.).
- ⇒ Gently shake off excess liquid from the test strip.
- ⇒ Read the result after 1 minute:
- ⇒ Determine the blend level in accordance with the table.

Test strips Result					
	4x green	1x red	2x red	3x red	4x red
Blend level	3	3	2	2	1
Capacity Filter cartridge M in liters	3570	3060	1440	990	510
Capacity Filter cartridge L in liters	6670	5710	2690	1840	940

### ► Setting the blend level

- ⇒ Press the key (9, Fig. 2) for blending adjustment and turn the blending adjustment dial to the corresponding level (Fig. 2).
- ⇒ Remove the blending adjustment key after setting has been completed and store the key in a safe place for possible future adjustments to the settings.

### ► Assembling the filter head

**When assembling the sensor unit, the adapter and the water hoses, make sure that you use the corresponding gaskets.**

- ⇒ Screw one adapter (13) onto the inlet of the filter head (3, Fig. 1).
- ⇒ Securely attach the filter head to a wall in a vertical position using appropriate screws (not included in the scope of delivery).
- ⇒ Screw one adapter (13) onto the connection of the sensor unit (10)
- ⇒ Screw the sensor unit (10) to the outlet of the filter head (4, Fig. 1) using the union nut.
- ⇒ Disconnect the POWER steamer 2 from the power source.
- ⇒ Turn off the water supply to the POWER steamer 2 and disconnect the water hose to the POWER steamer 2 from the water supply.
- ⇒ Connect the water hose for the POWER steamer 2 to the adapter of the sensor unit.

**The sensor unit must not be exposed to any mechanical loads.**

- ⇒ Connect the water hose provided to the adapter at the inlet of the filter head and to the water supply.
- ⇒ Note the flow direction – markings on the filter head and sensor unit!
- ⇒ Max. torque 10 Nm.

Fig. 6: Typical installation example

### ► Rinsing the filter cartridge

- ⇒ Run the flushing hose into a suitable container (e.g., a bucket) or into a drain.
- ⇒ Open the purge valve (5) (OPEN, Fig. 3).
- ⇒ Turn on the water supply.

**Water escapes from the flushing hose as soon as the filter cartridge has been inserted!**

- ⇒ Insert the filter cartridge into the filter head.

**Insert the filter cartridge with its marking at insertion mark A (Fig. 5) and screw it in as far as it will go, operating mark B (Fig. 5).**

- ⇒ Vent and rinse the filter cartridge.
  - Filter cartridge M: Flush with min. 5 liters of water
  - Filter cartridge L: Flush with min. 10 liters of water
- ⇒ Close the purge valve (5) (CLOSE, Fig. 4).
- ⇒ After the initial installation of the filter system, vent and flush the water drain hose and the POWER steamer 2 with at least 2 liters of water. To do this, open the service opening on the POWER steamer 2 and run the rinsing program 3 times (see operating instructions for the POWER steamer 2).

**After installing the system and inserting or replacing a filter cartridge, check all components for leak-tightness. Water must not leak out at any location.**

## 4.2 Installing the programming and display unit


**To operate the programming and display unit, you will need 2 batteries (type AA). These are not included in the scope of delivery!**

The programming and display unit can either be attached to the filter head (2) of the filter unit using the holder (11) or to a wall using the wall bracket provided (12).

- ⇒ Push the holder (11) onto the filter head (2) until it snaps into place.
- or
- ⇒ Screw the wall bracket securely to a wall using suitable flat-head screws, max. shank diameter 4 mm, or with the adhesive pads on the back.
- ⇒ Open the battery compartment and insert batteries, 2x type AA.
- ⇒ Pay attention to correct polarity.
- ⇒ Hang the programming and display unit in the holder.
- ⇒ Insert the connection cable of the sensor unit into the port (20).

In the case of assembly using the wall bracket, the connection cable can be pressed into the cable guide (21) to prevent the connection plug from being unintentionally pulled out.

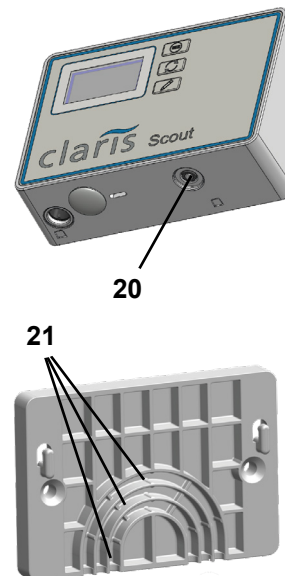### 4.2.1 Entering the filter capacity / resetting the counter

- ⇒ Determine the capacity of the filter cartridge used for the local carbonate hardness (see table in the installation section).
- ⇒ Enter the capacity:

Key		Display
	Press the "MENU" key until CAP SET is shown. - Display: Set filter capacity.	
	Press and hold the "MENU" key for approx. 5 seconds. - The active digit flashes.	
	Press the "MODIFY" key: - Set the active digit.	
	Press the "MENU" key: - Confirm the active digit and move to the next digit. - The active digit flashes (repeat the procedure until the last digit).	

⇒ Reset the counter

Key		Display
	The display is switched off.	---
	Press the "MENU" key: - Display: Filter status.	
	Press and hold the "RESET" key for approx. 5 seconds. - The display shows "reset 5s", the countdown runs down.	
	The filter capacity is now reset and the last value entered is shown.	

The system is now ready for operation.

## 4.3 Replacing a filter cartridge

Once 90 % of the filter capacity has been used up, at the latest after 360 days, you are prompted to replace the filter cartridge:

⇒ Run the flushing hose into a suitable container (e.g., a bucket) or into a drain.

⇒ Slowly unscrew the used filter cartridge anti-clockwise. This releases it from the filter head and it can be removed.

- ◆ During this procedure, the consumer unit and the water supply line are hydraulically disconnected from the filter head and the system is decompressed. In this case, it can happen that a minimal quantity of decompression water can escape from the flushing hose due to pressure peaks. Take this into consideration and place a suitable container underneath.

⇒ Open the purge valve (5) (OPEN, Fig. 3).

**Water escapes from the flushing hose as soon as the filter cartridge has been inserted!**

⇒ Remove the new filter cartridge from the packaging and check for damage.

⇒ Insert the filter cartridge into the filter head.

**Insert the filter cartridge with its marking at insertion mark A and screw it in as far as it will go, operating mark B.**

⇒ Vent and rinse the filter cartridge.

- Filter cartridge M: Flush with min. 5 liters of water
- Filter cartridge L: Flush with min. 10 liters of water

⇒ Close the purge valve (5) (CLOSE, Fig. 4).

⇒ Enter the date of installation and the time of replacement in the corresponding fields on the sticker on the filter cartridge (a ballpoint pen can be used).

**NOTICE: The rinsing water is initially milky and cloudy. This is due to the dispersed air and clears after a short time.**

**Check the correct position of the filter cartridge in the filter head using the markings on the filter head (B, Fig. 5) and filter cartridge. When inserting the filter cartridge, pay attention to the position of the sticker. This should face forward so that all the necessary information is visible.**

**If the filter cartridge is removed from the filter head without being replaced by a new one, the water supply to the filter head must be shut off.**

### 4.3.1 Resetting the counter / checking the filter capacity

If a filter cartridge of the same size was used when replacing the filter cartridge, only the counter needs to be reset.

If a larger filter cartridge is used, its capacity must be entered first.

See Chapter 4.2.1

## 4.4 Programming and display unit

Information on additional functions of the programming and display unit is provided in the instructions for use available online.

## 5 Cleaning / maintenance

Safe functioning of the system is only possible if the filter cartridge is replaced regularly. The replacement cycles depend on the carbonate hardness of the drinking water, usage, and the blend level set.

We recommend replacing the filter cartridge after 6 months, but no later than after 12 months.

The operator is required to check the system for leaks on a daily basis.

When replacing the filter cartridge, all parts must be checked for contamination and damage. Damaged parts must be replaced and contamination removed.

**If the system is not used for an extended period, the filter cartridge must be replaced.**

### 5.1 Flushing the system

The system must be flushed thoroughly after maintenance work or if it has not been used for an extended period.

⇒ Run the flushing hose into a suitable container (e.g., a bucket) or into a drain.

⇒ Open the purge valve (5) (OPEN, Fig. 3).

⇒ Turn on the water supply.

⇒ Allow the quantity of water specified in the table below to flow through.

⇒ Close the purge valve (5) (CLOSE, Fig. 4).

	Flush volume after 1 week of stagnation:	Flush volume after 4 weeks of stagnation:
Filter cartridge M for POWER steamer water softener	3 liters	15 liters
Filter cartridge L for POWER steamer water softener	5 liters	25 liters

⇒ Then carry out 3 rinsing cycles using the POWER steamer.

### 5.2 Spare parts

You can find wear parts and spare parts in the spare parts list available online at [www.renfert.com/p918](http://www.renfert.com/p918).

Enter the following item number: 18460100.

The components excluded from the warranty (wear parts, consumables) are marked in the spare parts list.

## 6 Disposal information

Equipment with this symbol may not be disposed of in unsorted domestic waste within the European Union.

Please contact your local authorities for more information on proper disposal.

You can find out about the current options for return online at [www.renfert.com](http://www.renfert.com)

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# Technical data

## Filter unit

Item no.	1846 0100 + 1846 0103	1846 0100 + 1846 0104
Dimensions		
Height of the entire system / mm [inch]	475 [18.7]	410 [16.1]
Height of the filter cartridge / mm [inch]	425 [16.7]	360 [14.2]
Diameter of the filter cartridges / mm [inch]	95 [3.7]	136 [5.6]
Minimum distance to the floor / mm [inch]	40 [1.6]	40 [1.6]
Weight of the filter cartridge / kg [lbs]	1.8 [4.0]	3.2 [7.1]
Operating data		
System pressure (without pressure surges) / hPa (bar) [psi]	2000 - 8000 (2 - 8) [29 - 116]	
Water temperature / ambient temperature / °C [°F]	4 - 30 [39 - 86]	

## Flowmeter

Item no.	1846 0101
Sensor unit	
Connection thread	Inlet 3/8" BSP IG / Outlet 3/8" BSP AG
Inlet pressure (min/max) / hPa (bar) [psi]	1000 - 8000 (1 - 8) [14.5 - 116]
Flow range / l/h [gal/l]	15 - 500 [3.96 - 132]
Pressure loss / hPa (bar) [psi]	≤ 500 (0.5) [7.25]
Measurement (horizontal installation) / %	+/- 5
Water temperature (min/max) / °C [°F]	4 - 30 [37.4 - 86]
Ambient temperature (min/max) / °C [°F]	4 - 40 [37.4 - 104]
Dimensions (W x H x D) / mm [inch]	80 x 39 x 42 [3.15 x 1.54 x 1.65]
Installation position	Horizontal recommended
Programming and display unit	
Graphic display	5 digits in liters and gallons
Counter	Decreasing from 99999 to -9999
Dimensions (W x H x D) / mm [inch]	80 x 54 x 29 [3.15 x 2.13 x 1.14]
Power supply, battery type	2 x AA

This unit corresponds to Part 15 of the FCC Rules.

Operation is subject to the following two conditions:

- (1) This unit may not cause harmful interference and
- (2) This unit must accept any interference received, including interference that may cause undesired operation.

The label with the FCC ID and logo is located inside the battery compartment.

**Hochaktuell und ausführlich auf ...  
Up to date and in detail at ...  
Actualisé et détaillé sous ...  
Aggiornato e dettagliato su ...  
La máxima actualidad y detalle en ...  
Актуально и подробно на ...**

**[www.renfert.com](http://www.renfert.com)**

Renfert GmbH • Untere Gießwiesen 2 • 78247 Hilzingen/Germany  
Tel.: +49 7731 82 08-0 • Fax: +49 7731 82 08-70  
[www.renfert.com](http://www.renfert.com) • [info@renfert.com](mailto:info@renfert.com)

Renfert USA • 3718 Illinois Avenue • St. Charles IL 60174/USA  
Tel.: +1 6307 62 18 03 • Fax: +1 6307 62 97 87  
[www.renfert.com](http://www.renfert.com) • [info@renfertusa.com](mailto:info@renfertusa.com)  
USA: Free call 800 336 7422

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c/o Renfert GmbH