making work easy

- FAQ - Deutsch
- FAQ - English
- FAQ - Français
- FAQ - Italiano
- FAQ - Español
- FAQ - Português
- FAQ - Русский
- FAQ - 中文
- FAQ - اللغة العربية
- FAQ - Suomi
- FAQ - Svenska
- FAQ - Українська

# Magma

FAQ - frequently asked questions

# FAQ - Deutsch

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# FAQ - English

For references to chapters, see the complete operating instructions on the Internet.

Error	Possible cause	Corrective action
Displays remain blank after the oven is switched on at the main power switch.	<ul style="list-style-type: none"> <li>• Power cord not plugged in.</li> <li>• Building fuse blown.</li> <li>• Device fuse blown.</li> </ul>	<ul style="list-style-type: none"> <li>• Plug the power cord into the wall outlet.</li> <li>• Check the building fuse and replace, as required.</li> <li>• Check the device fuse and replace, as required (refer to Sec. 6.3).</li> </ul>
The building fuse blows each time the heater is switched on.	<ul style="list-style-type: none"> <li>• Inadequate mains power protection.</li> </ul>	<ul style="list-style-type: none"> <li>• The building fuse must be at least 16A.</li> </ul>
Heating status display fails to go on even though the heater has been switched on with the Stop/Start key.	<ul style="list-style-type: none"> <li>• Oven door not closed.</li> <li>• Door sensor fallen off.</li> <li>• Faulty door sensor.</li> <li>• Faulty internal door open detection switch.</li> </ul>	<ul style="list-style-type: none"> <li>• Close the unit door.</li> <li>• Reinstall the door sensor (refer to Sec. 6.5).</li> <li>• Replace the door sensor (refer to Sec. 6.5).</li> <li>• Have the unit repaired.</li> </ul>
Heating status display indicates heating but the oven fails to warm.	<ul style="list-style-type: none"> <li>• Faulty thermocouple.</li> <li>• Faulty heater coil.</li> <li>• Faulty power electronics.</li> </ul>	<ul style="list-style-type: none"> <li>• Replace the thermocouple (refer to Sec. 6.4).</li> <li>• Have the unit repaired.</li> <li>• Have the unit repaired.</li> </ul>
Temperature display stays at a specific value even though the oven heats.	<ul style="list-style-type: none"> <li>• Faulty thermocouple.</li> </ul>	<ul style="list-style-type: none"> <li>• Replace the thermocouple (refer to Sec. 6.4).</li> </ul>
Extremely excessive temperature; oven overheats.	<ul style="list-style-type: none"> <li>• Faulty thermocouple.</li> <li>• Faulty power electronics.</li> </ul>	<ul style="list-style-type: none"> <li>• Replace the thermocouple (refer to Sec. 6.4)</li> <li>• Have the unit repaired.</li> </ul>
The programme immediately starts when the oven is switched on.	<ul style="list-style-type: none"> <li>• Oven was switched off before programme ran to completion. Switching the oven off was interpreted as a power failure.</li> </ul>	<ul style="list-style-type: none"> <li>• Wait until the programme concludes before switching the oven off or prematurely terminate the programme with the Stop/Start key.</li> </ul>
The wrong time is indicated in the standby mode.	<ul style="list-style-type: none"> <li>• Clock not set.</li> <li>• The clock backup battery is dead.</li> </ul>	<ul style="list-style-type: none"> <li>• Check the system time and reset, as required (refer to Sec. 2.3.1).</li> <li>• Replace the backup battery (refer to Sec. 6.6).</li> </ul>
The indicated casting time makes no sense	<ul style="list-style-type: none"> <li>• Faulty system time.</li> </ul>	<ul style="list-style-type: none"> <li>• Check the system time and reset, as required (refer to Sec. 2.3.1).</li> </ul>
The year indicator flashes each time the oven is switched on	<ul style="list-style-type: none"> <li>• Backup battery is dead.</li> </ul>	<ul style="list-style-type: none"> <li>• Replace the backup battery (refer to Sec. 6.6).</li> </ul>
Err 1 to Err 4	<ul style="list-style-type: none"> <li>• External fault</li> <li>• Faulty controller</li> </ul>	<ul style="list-style-type: none"> <li>• Switch the oven off, then on, and repeat the interrupted procedure (e.g., restart the heating programme).</li> <li>• Have the unit repaired.</li> </ul>
Err 5 and Err 6	<ul style="list-style-type: none"> <li>• Too much cold material placed in a warm oven.</li> <li>• Door left open too long.</li> <li>• Thermocouple cable loose.</li> <li>• Faulty thermocouple.</li> </ul>	<ul style="list-style-type: none"> <li>• Restart the heating programme.</li> <li>• Restart the heating programme.</li> <li>• Check that the heating cable is securely attached. Reattach as required.</li> <li>• Replace the thermocouple (refer to Sec. 6.4).</li> </ul>

Error	Possible cause	Corrective action
<b>Err 7</b>	<ul style="list-style-type: none"><li>• Faulty thermocouple.</li><li>• Oven cooled with compressed air and heating programme started (temperature increase due to the heat retained in the oven).</li></ul>	<ul style="list-style-type: none"><li>• Replace the thermocouple (refer to Sec. 6.4).</li><li>• Allow the oven to cool off.</li></ul>

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