making work easy

- FAQ - Deutsch**
- FAQ - English**
- FAQ - Français**
- FAQ - Italiano**
- FAQ - Español**
- FAQ - Português**
- FAQ - Türkçe**
- FAQ - Русский**
- FAQ - Polski**
- FAQ - Dansk**
- FAQ - Čeština**
- FAQ - 中文**
- FAQ - 日本語**
- FAQ - 한국어**
- FAQ - اللغة العربية**
- FAQ - Українська**

# Basic eco

FAQ - frequently asked questions

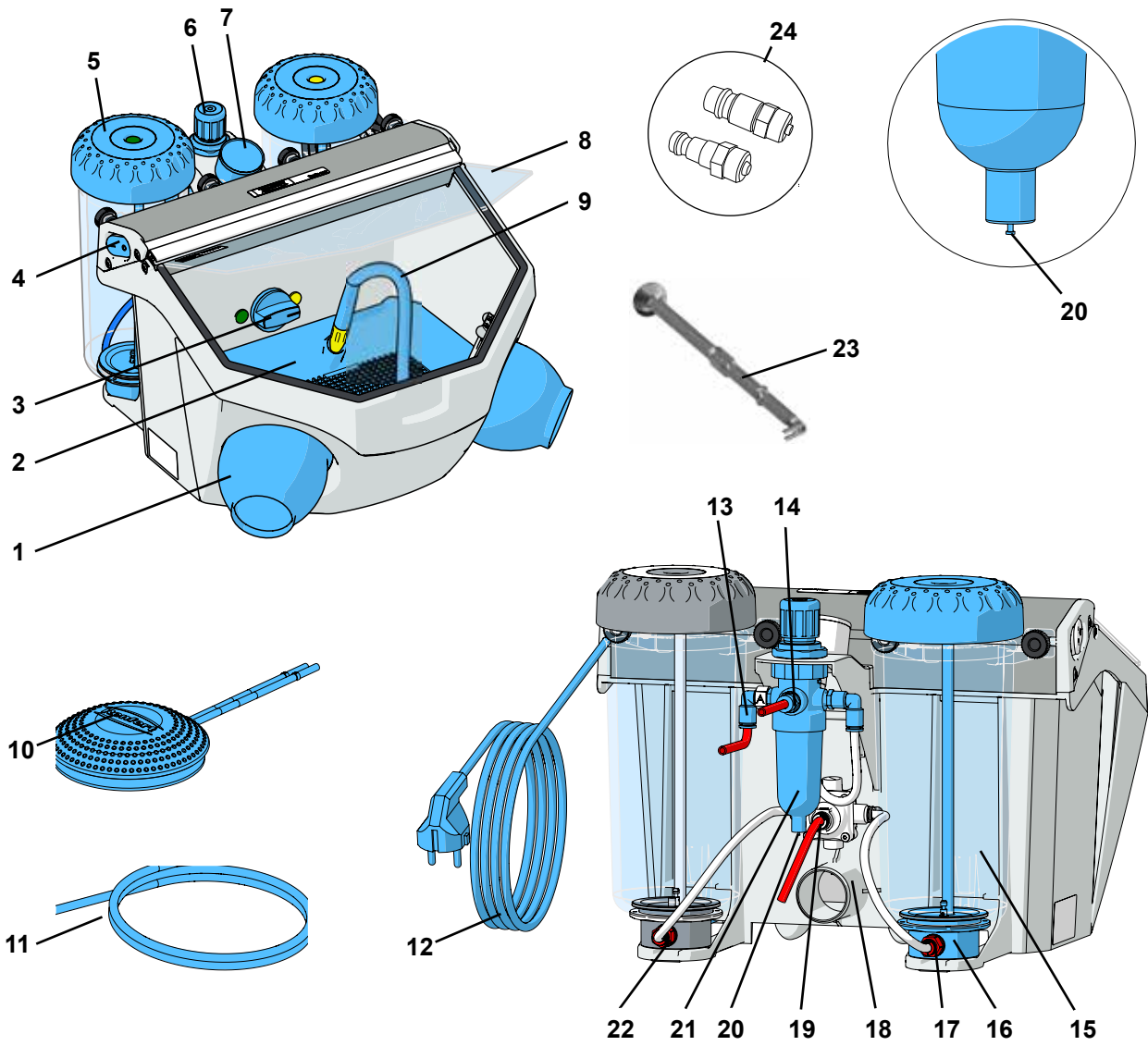
Fig. 1

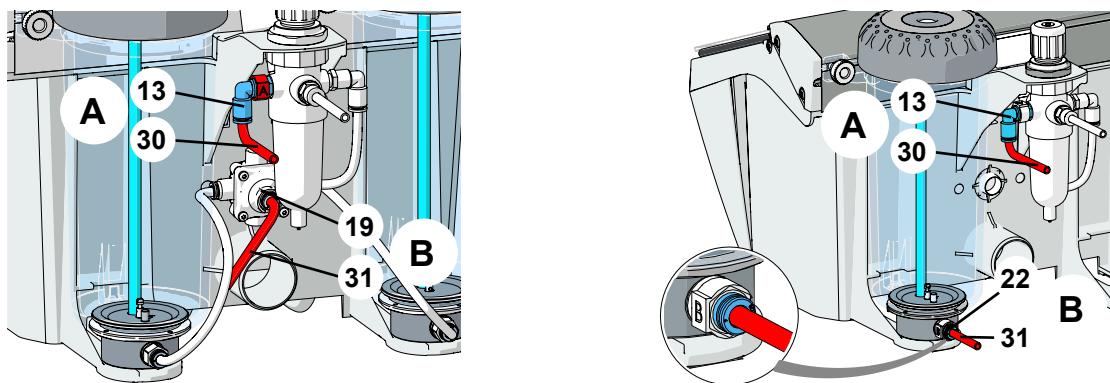
Fig. 2 / 3

# FAQ - Deutsch

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# FAQ - English

For information on operating elements, see [Fig. 1](#) and [Fig. 2](#) on page 2 of this document.

For chapter indications, refer to the complete instruction manual on the internet.

Problem	Possible cause	Corrective action
<b>No air and no abrasive.</b>	<ul style="list-style-type: none"> <li>Blasting nozzle blocked at the hand-piece.</li> <li>Foot switch connecting lines kinked.</li> <li>Tank selection switch in an intermediate position.</li> </ul>	<ul style="list-style-type: none"> <li>Unscrew the nozzle from the hand-piece and blow it out.</li> <li>With the nozzle removed, blow out the blasting hose by activating the foot pedal.</li> <li>Inspect / correct the connection lines.</li> <li>Turn the switch until you feel it click into place.</li> </ul>
<b>Only air, no or very little abrasive.</b>	<ul style="list-style-type: none"> <li>The dosage nozzle in the mixing chamber is blocked due to contaminated or moist sand.</li> <li>Mixing chamber filter combination blocked.</li> <li>Abrasive contains impurities or is moist.</li> </ul>	<ul style="list-style-type: none"> <li>Cleaning the dosing nozzle, see chapter 6.5.</li> <li>Change filter cartridge (<a href="#">17. Fig. 1</a>) (wrench SW14).</li> <li>Replace the abrasive.</li> </ul>
<b>Pressure control cannot be adjusted.</b>	<ul style="list-style-type: none"> <li>The pressure control button is locked.</li> </ul>	<ul style="list-style-type: none"> <li>Pull the pressure control button upwards to unlock.</li> </ul>
<b>Unit blasts continuously.</b>	<ul style="list-style-type: none"> <li>Foot switch connected incorrectly.</li> <li>Valve in foot switch is defective.</li> </ul>	<ul style="list-style-type: none"> <li>Inspect / correct the foot switch connection.</li> <li>Replace foot switch.</li> </ul>
<b>Lamp is not alight.</b>	<ul style="list-style-type: none"> <li>Power cord not plugged in.</li> <li>Illumination defect.</li> </ul>	<ul style="list-style-type: none"> <li>Plug the power cord into the wall outlet.</li> <li>Change light using the service or see spare part list.</li> </ul>
<b>Compressed air leaking from the foot switch.</b>	<ul style="list-style-type: none"> <li>Pneumatic hoses reversed.</li> </ul>	<ul style="list-style-type: none"> <li>Reverse the connections.</li> </ul>
<b>Foot switch fails to switch on.</b>	<ul style="list-style-type: none"> <li>No pressure at the compressed air connection.</li> <li>Pneumatic hoses are kinked.</li> </ul>	<ul style="list-style-type: none"> <li>Inspect the compressed air lines and/or the compressor.</li> <li>Reposition the hoses.</li> </ul>
<b>Foot switch hoses cannot be disconnected from the unit.</b>	<ul style="list-style-type: none"> <li>Push-in connection is not released (<a href="#">Fig. 2 / 3</a>).</li> </ul>	<ul style="list-style-type: none"> <li>Press the push-in connection ring on the hose side firmly while pulling on the hoses.</li> </ul>

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